

Components:

Pump / Solenoid
Assy. 1

Special Tools:

none



Caution:

Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool then drain reservoir.

Step 2: Disconnect wire harness from J2 and J4 from PC board.

Step 3: Disconnect boiler hose and pressure transducer tube.


Step 6: Slide pump plate back and lift up to remove.

Step 1: Disconnect tubing from Solenoid and External Condensing Tank tube.

Step 4: Remove hose from reservoir tank.

Step 5: Remove screw from base plate.

Installation: Reverse steps for installation.

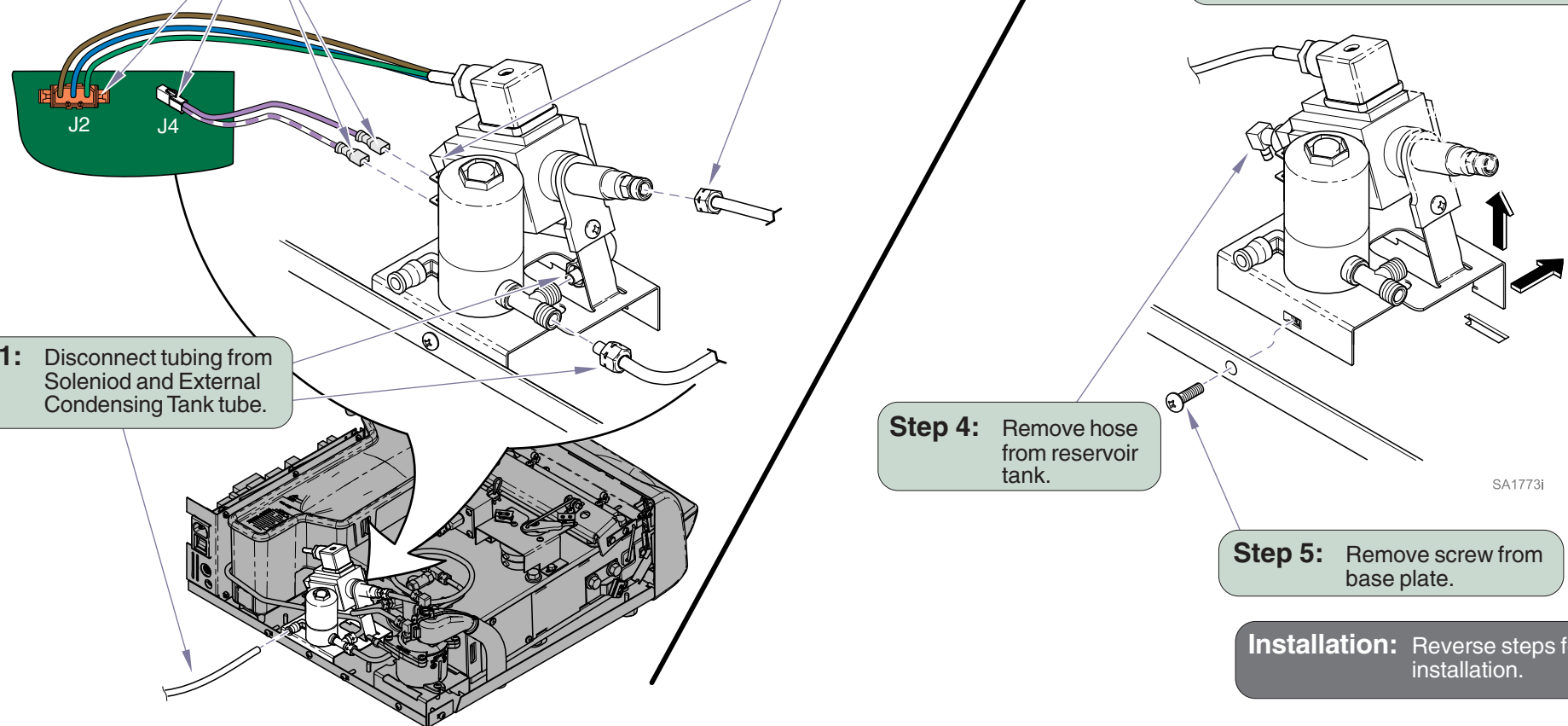

SA1773i