

Components:

Boiler assembly .. 1
Thermostat 1

Special Tools:


none



Caution:

Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool.

Step 3 : Remove 2 screws from boiler bracket.

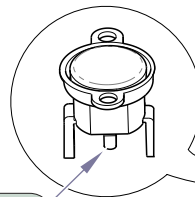

Step 4: Lift up boiler and unhook wiring from thermostat and remove.



Equipment Alert

Pull ring on relief valve to release any pressure in chamber.

Step 1: Check reset button on boiler thermostat. Push in to reset.

Step 2: Disconnect copper tubing, neoprene tube, and wires from boiler.

Boiler mounting bracket

Installation: Reverse steps for installation.

SA115300i