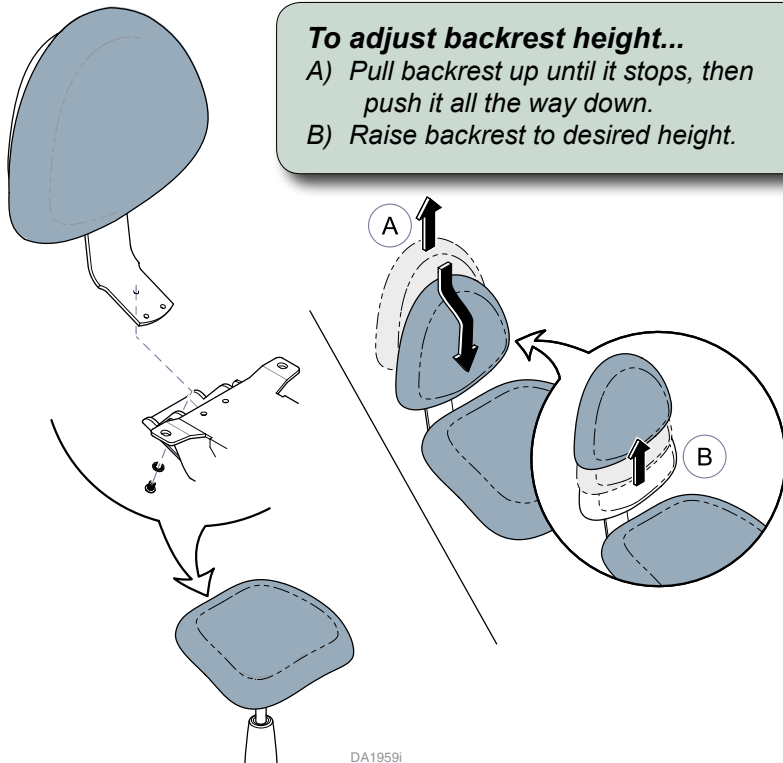
Cleaning:

- Clean upholstery with diluted bleach solution. Ratio - 10:1 (water:bleach)
Refer to Page 3 for more information.
- Wipe painted metal and plastic surfaces with a clean, soft cloth, and a mild cleaner.

Weight Capacity:
350 lbs (158 kg).

To install backrest...

- Position backrest on seat mount weldment as shown.
- Secure backrest with three star washers & screws.



To adjust backrest height...

- Pull backrest up until it stops, then push it all the way down.
- Raise backrest to desired height.


To install (optional) armrests...

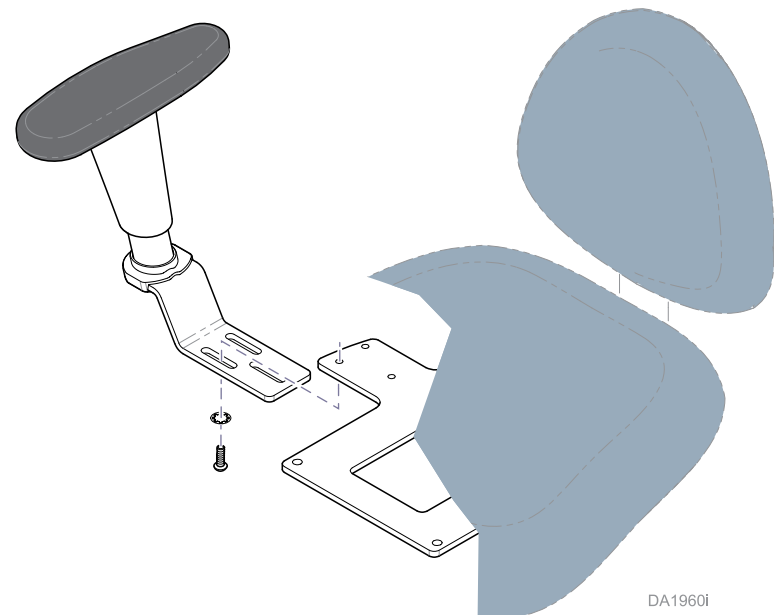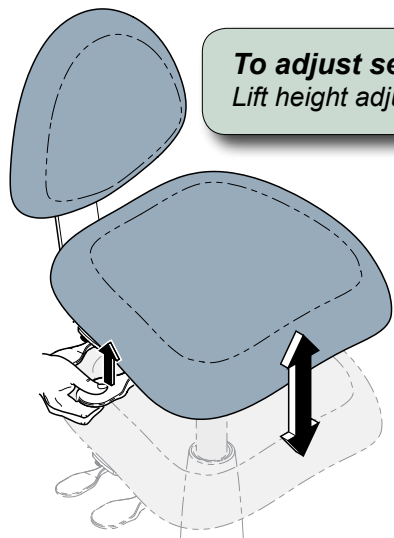- Align slots in armrest with holes in mounting plate.
- Secure armrest with three star washers & screws.
- Repeat for other armrest.


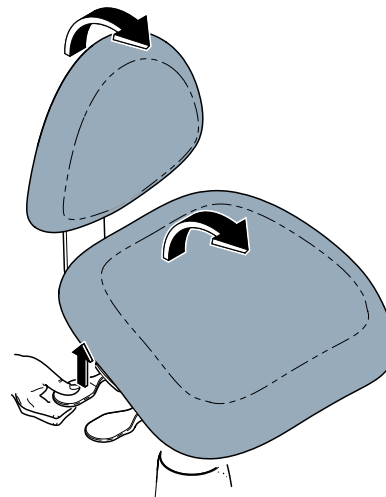
To adjust seat height...
Lift height adjustment lever. (front lever)

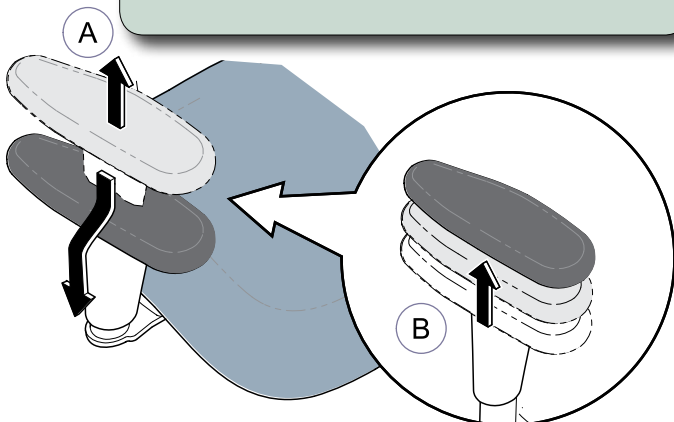
To adjust pivot angle of seat / back...
A) Lift pivot adjustment lever. (rear lever)
B) Lean forward / back to desired angle.

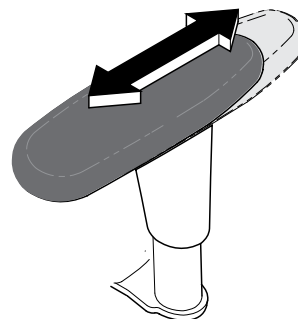To adjust armrest height...
A) Pull armrest up until it stops, then push it all the way down.
B) Raise armrest to desired height.

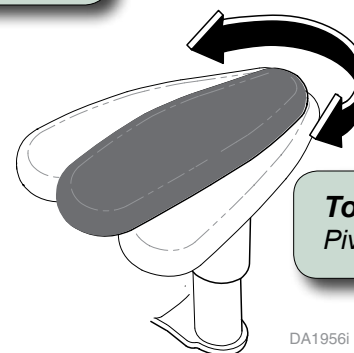To adjust armrest pad position...
Move armrest pad forward / back as desired.

To adjust armrest angle...
Pivot armrest pad in / out.

DA1956i

Specifications

Doctor's Stool (153811)	
Weight Rating	350 lbs (159 kg)
Seat Height: <i>Tall models (default)</i>	Min: 21 in. (53 cm) Max: 29 in. (74 cm)
<i>Medium models</i>	Min: 16.5 in. (42 cm) Max: 22 in. (56 cm)
<i>Short models</i>	Min: 15.25 in. (39 cm) Max: 18.25 in. (46 cm)
Seat Dimensions: <i>Standard Seat</i>	Width: 17.5 in (44.5 cm) Depth: 16.5 in. (42 cm)
<i>Small Seat</i>	Width: 16.25 in (41 cm) Depth: 16 in. (40.6 cm)
Base Dimension	24 x 24 in. (61 x 61 cm)

Cleaning

Every dental practice is different, and no single disinfectant is the best choice for every facility. Listed below are some organizations to assist you in choosing the best disinfectant for your practice

Organization for Safety & Asepsis Procedures:

<http://www.osap.org>

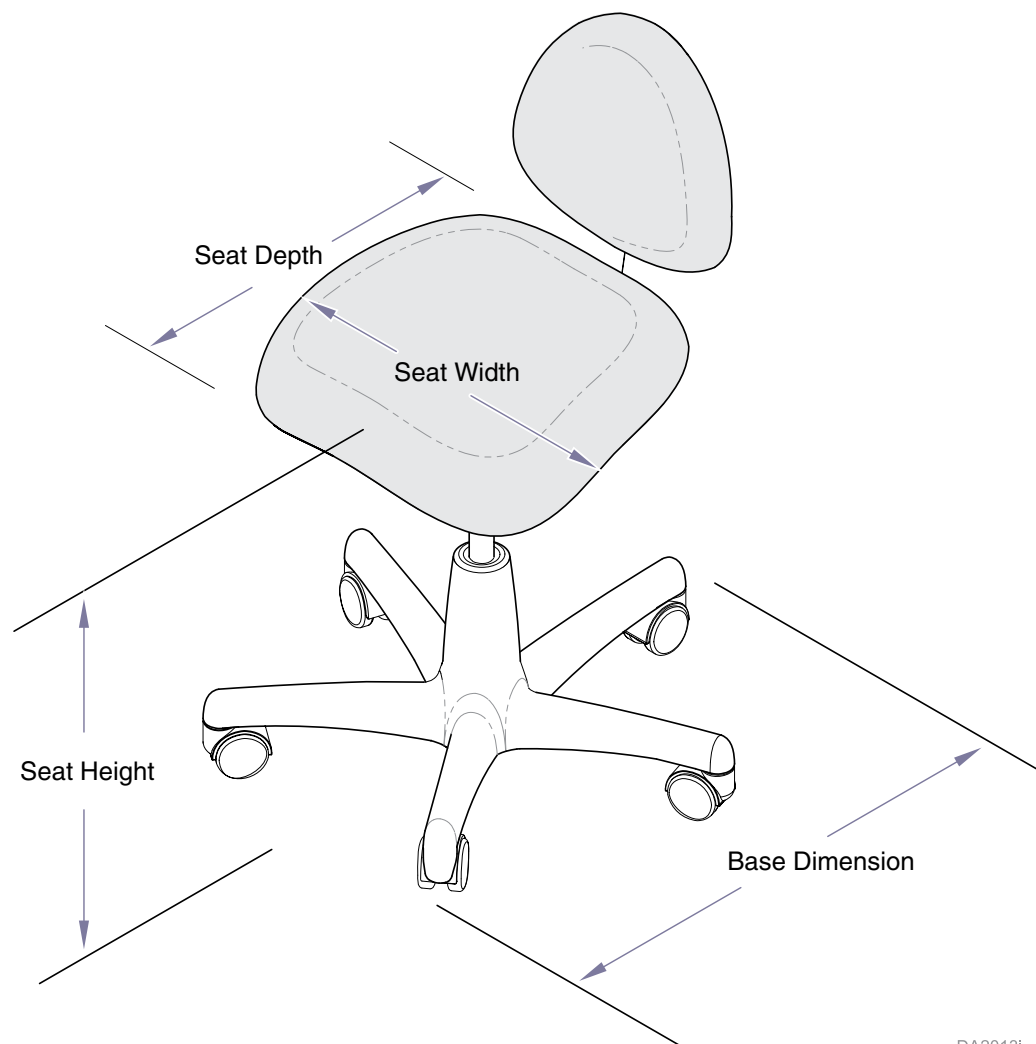
American Dental Association:

<http://www.ada.org>

Dept. of Health & Human Resources

Centers for Disease Control & Prevention (CDC):

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© Midmark Corporation 2008 Style A
Midmark Corporation