## Elevance® Delivery: LR Mounted Installation (Includes Elevance® Continental-Style Delivery)

Language of origin: English

### Equipment Alert

*Do not remove wraps or packaging from the delivery flex arm assembly until it is mounted onto the LR arm.*

### Attention!

*The electrical and plumbing supply connectors must be installed by an electrician and plumber in compliance with local building codes. All required hardware is to be supplied by contractor. Supply connections must be located to fit inside the J-box and be properly distanced, within reach of the chair umbilical [4 or 8 foot (1.2 or 2.4 m)].*

### Equipment Alert

*When installing 115~ (VAC) receptacle in the J-Box, position it so that all the other components needed in the box will also fit.*

*Grounding reliability can only be achieved if this equipment is connected to an equivalent receptacle marked Hospital Grade (NEMA 5-15R, HOSPITAL GRADE).*

## Electrical Supply Requirements

Have an electrician install a 1/2 in. (13 mm) conduit box with quad or equal receptacle. Top of box must be no higher than 4-1/2 in. (114 mm) above finished surface of floor.

## Water Supply Requirements (optional)

Use 1/2 in. (13 mm) pipe (supplied by contractor) protruding 2 in. (51 mm) above finished surface of floor. Manual shut-off valve is to be installed by a plumber.

Water pressure must be 30-50 PSI (206-344 kPa) maximum, at the shut-off valve. Flush water plumbing clean before making final connections to dental equipment.

Recommended water supply hardness level <60 ppm of calcium carbonate.

## Air Supply Requirements

Use 1/2 in. (13 mm) pipe (supplied by contractor) protruding 2 in. (51 mm) from finished surface of floor. Manual shut-off valve is to be installed by a plumber.

Air pressure must be 70-100 PSI (482 -689 kPa) maximum, at the shut-off valve. Flush air plumbing clean before making final connections to dental equipment.

Air supply to be filtered, dry and oil-free per NFPA 99 requirements or equivalent local codes.

## Drain Requirements (optional)

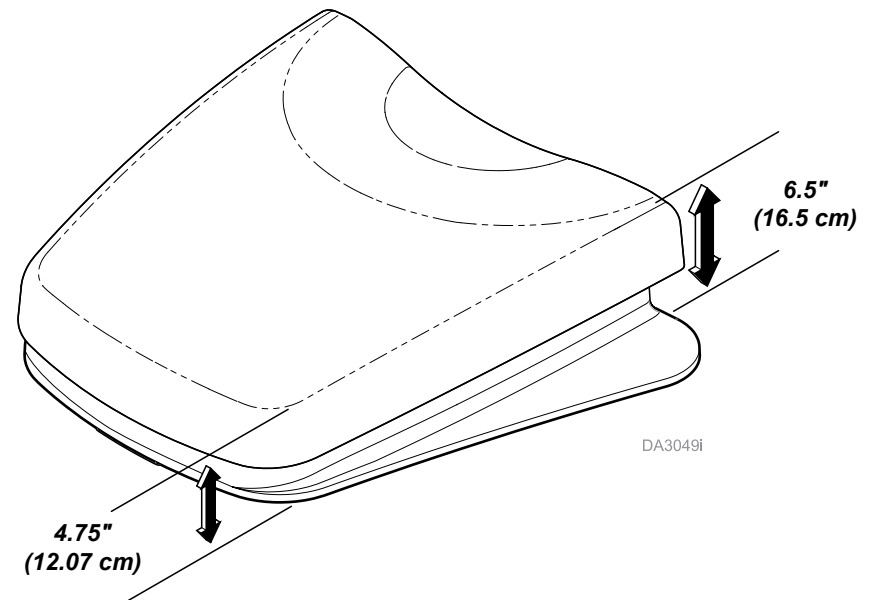
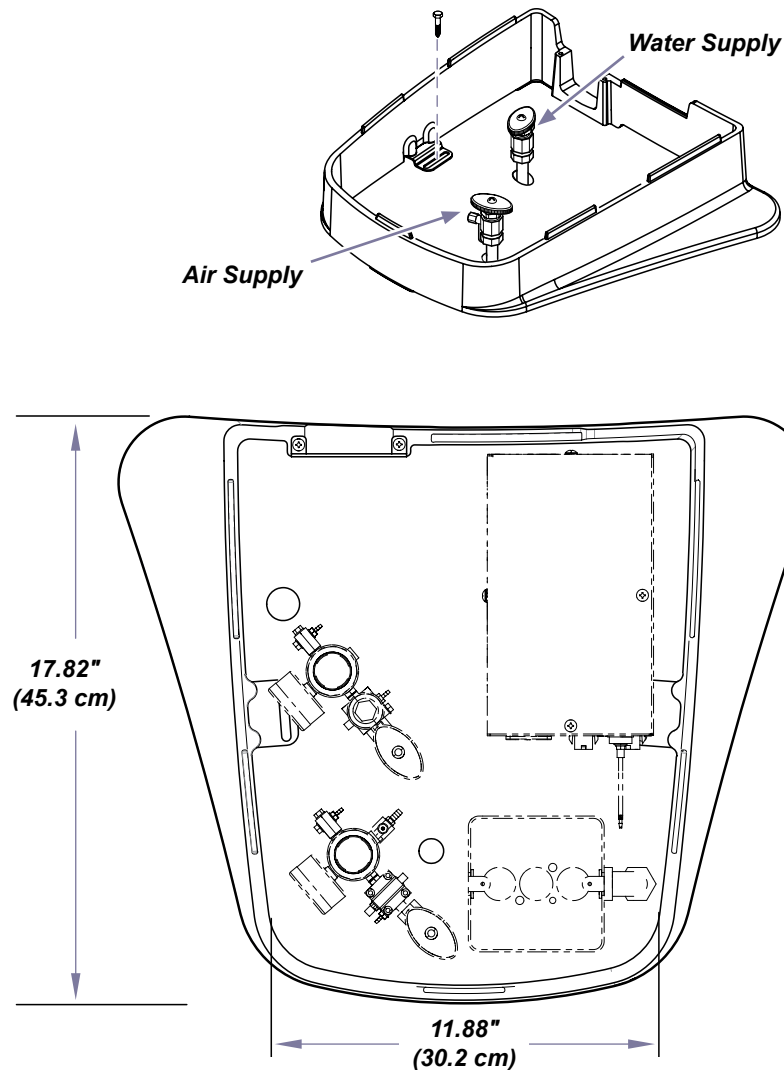
Not less than 1-1/2 in. (38 mm) I.D. pipe from downpipe to trap, decreasing to 3/4 in. (19 mm) pipe from trap to connection box. Terminate with 3/4 in. (19 mm) NPT pipe protruding 1 in. (26 mm) above finished surface of floor. Install adapter for 5/8 in. (16 mm) drain tubing. Place trap in line and vent to conform with local codes. [All hardware supplied by contractor].

## Vacuum Requirements (optional)

Plumbing to J-Box box should be specified by central vacuum supplier and terminated in J-Box with 3/4 in. (19 mm) O.D. tube protruding 1 in. (26 mm) above finished surface of floor.

**Step 1: Verify Proper Position and Types of Supply Connections Required for the J-Box.**

- A) Verify that you have the proper supply connections required for the components being installed.
- B) Verify that the connections fit inside the J-box and are located within reach of the umbilical provided (4 or 8 foot [1.2 or 2.4 m]).
- C) Reference the J-Box Installation Guide for more details.

**Step 2: Verify proper installation of Elevance chair.**

- A) Reference Elevance Chair Installation Guide packaged with chair.

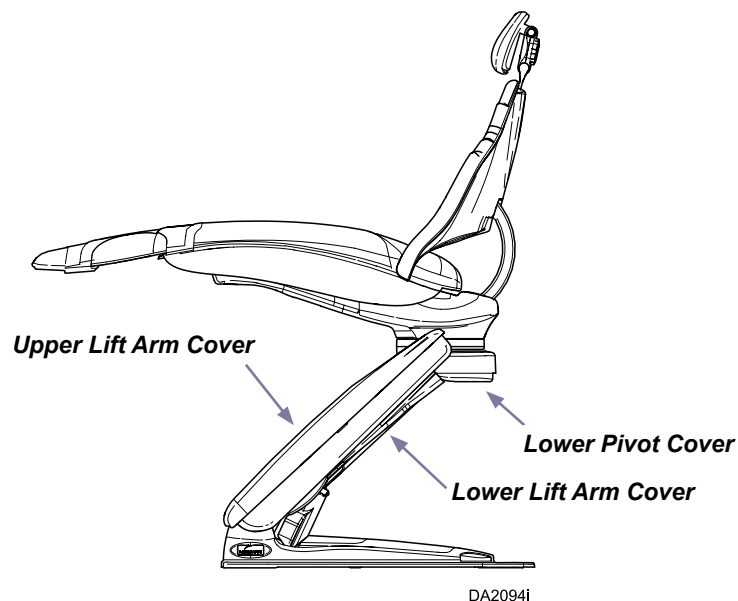
### Step 3: Position chair.

#### Equipment Alert

Tap a lift arm or lower pivot cover three times to raise the chair to the pre-programmed “home” position; a convenient height for installation or patient access.

The chair beeps until it reaches the home position.


#### WARNING

Always disconnect chair from the power source before removing any covers. Failure to do so may result in personal injury.

### Step 4: Remove seat upholstery.

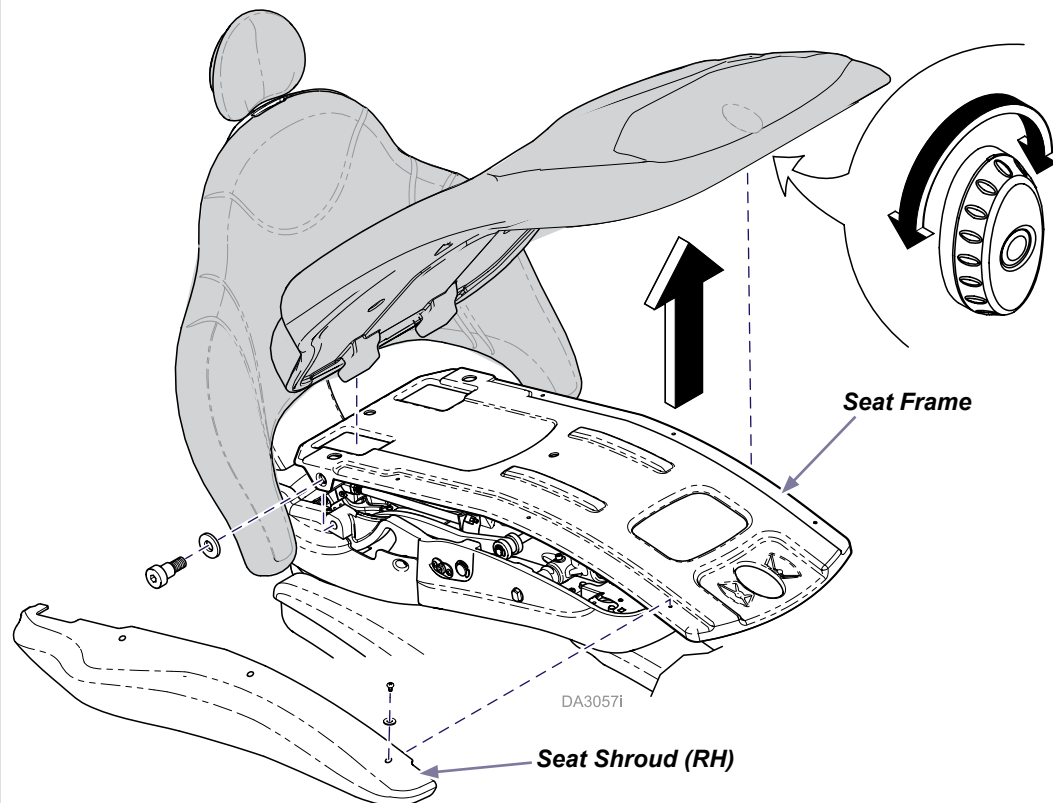- A) Disconnect the chair from power source.
- B) Rotate lock knob.
- C) Lift seat upholstery to remove tabs from slots.

### Step 5: Remove seat shrouds and seat frame.

- A) Remove three screws / washers and RH seat shroud.
- B) Repeat for LH seat shroud (not shown).
- C) Remove two shoulder bolts / washers and lift seat frame.

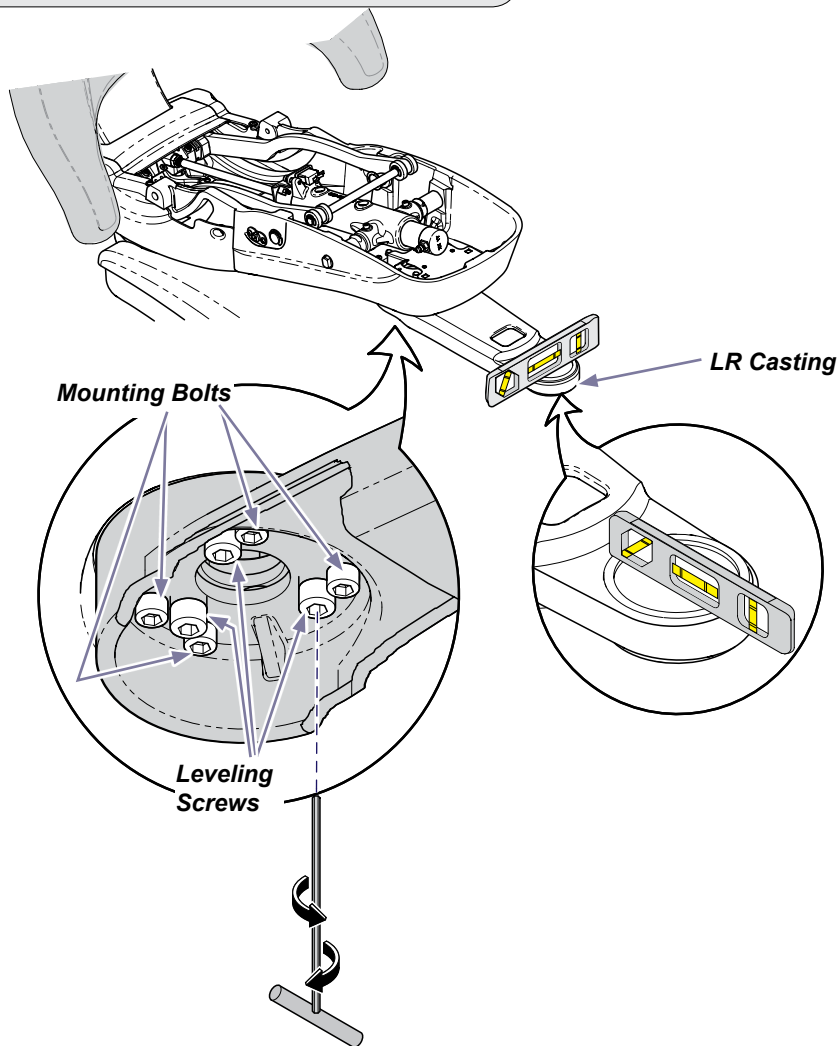
## WARNING

The Elevance chair must be anchored to floor before installing the delivery system components.

### Step 6: Level LR casting.

- A) Loosen four mounting bolts.
- B) Adjust three leveling screws as necessary.
- C) Tighten four mounting bolts.

Note: If LR casting is not level, delivery arm may drift.


## Note

Have an assistant help to mount the delivery unit on the LR casting.

### Step 7: Install LR delivery arm.

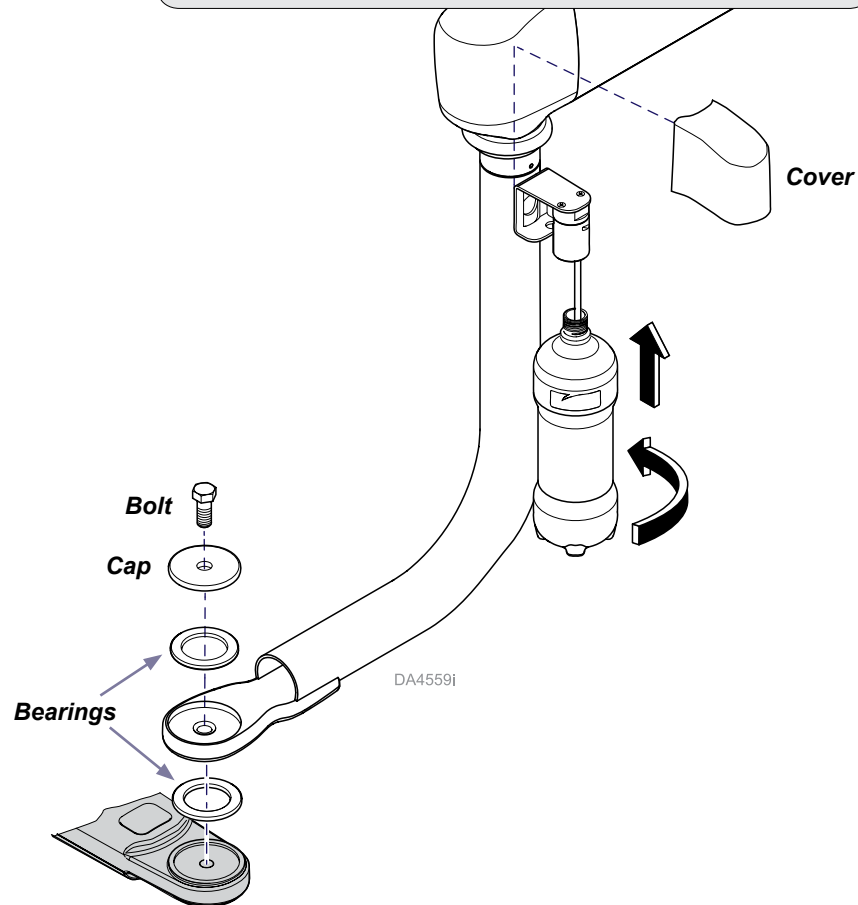- A) Position LR delivery arm / bearings / cap as shown.
- B) Secure to LR casting with bolt.
- C) Install water bottle (turn until tight) and cover.
- D) Adjust the drag on the delivery arm by loosening or tightening the bolt.


### Equipment Alert

Do not remove wraps or packaging from the delivery flex arm assembly until it is mounted onto LR arm.

### Step 8: Level delivery unit and adjust drag.

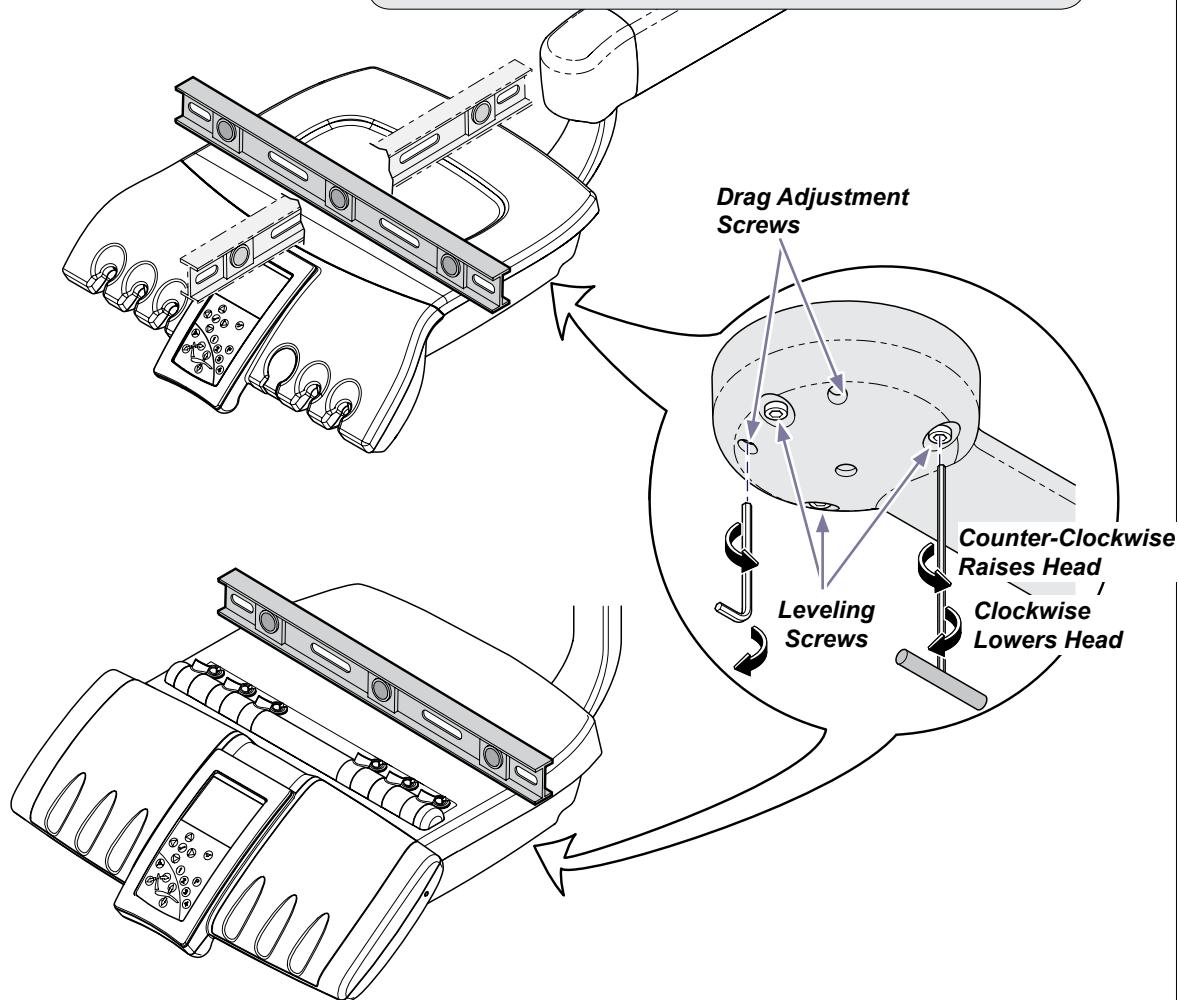
- A) Position flex arm horizontal to the floor.
- B) Place a level across unit and alternately turn the three leveling screws.
- C) Adjust drag with the drag adjustment screws.

### Step 9: Route delivery umbilical into chair.

- A) Route umbilical into opening and thru LR casting.
- B) Route umbilical up thru hole in chair casting.

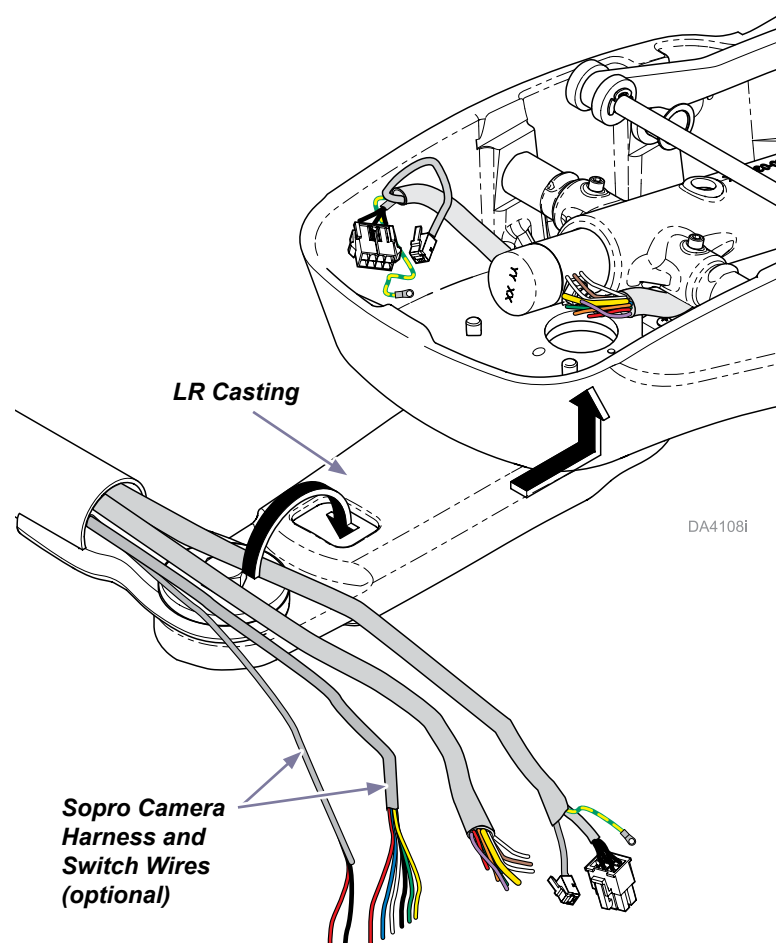
Note: Optional Sopro camera harness and switch wires need to be routed thru the chair to the J-box, following the same route as the chair umbilical.

### Step 10: Connect delivery umbilical to J-box umbilical.


- Connect two LIN lines to LIN coupler.
- Connect USB connectors to USB hub and place inside chair casting.
- Connect delivery power harnesses.
- Screw ground wires onto the chair casting using toothed-lock washer.
- Connect like colored tubing.
- Optional Sopro camera harnesses must be routed thru the chair to the J-box following the same path as the chair umbilical.

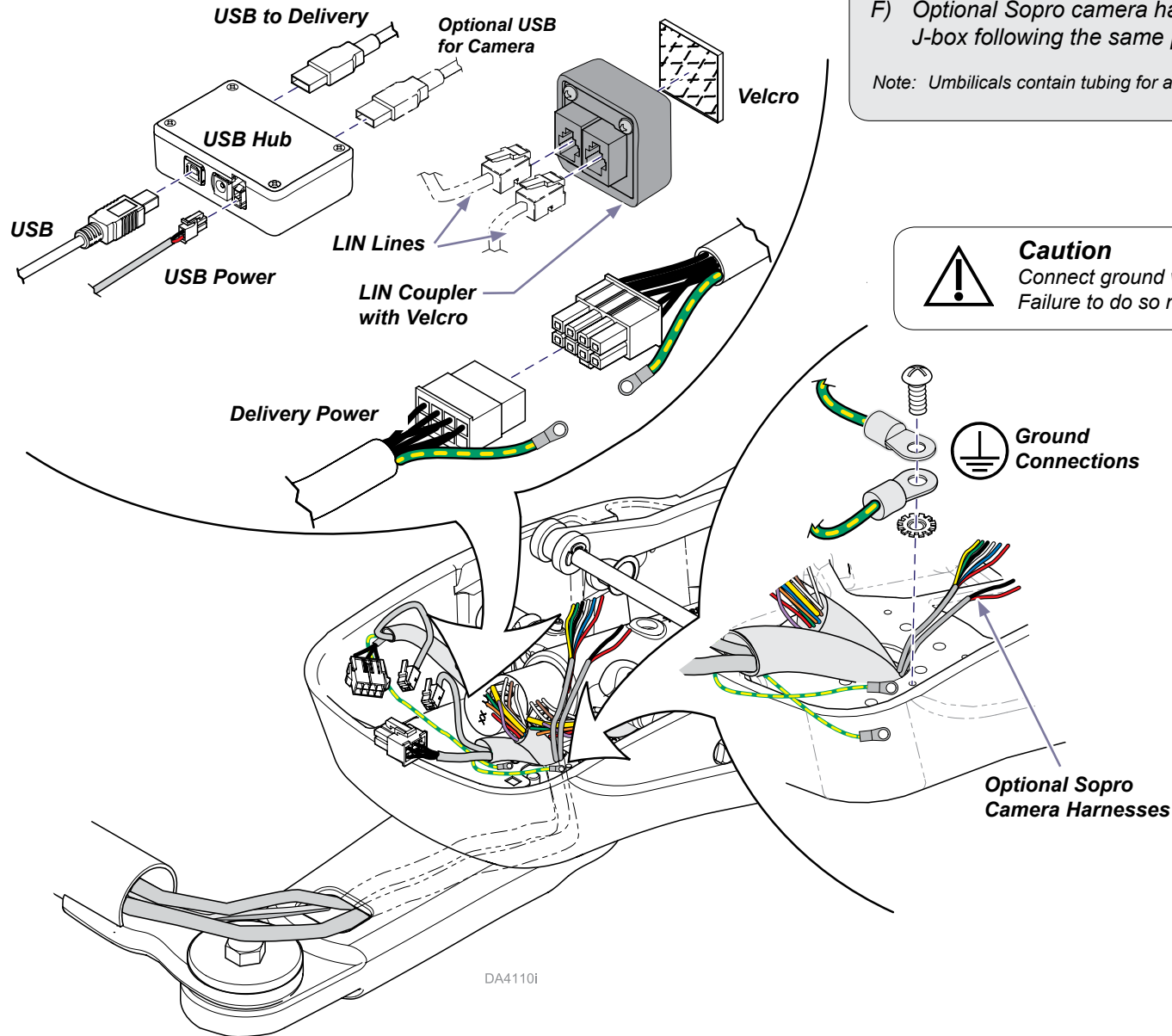
Note: Umbilicals contain tubing for all possible components. Leave unused tubing capped.

#### Caution

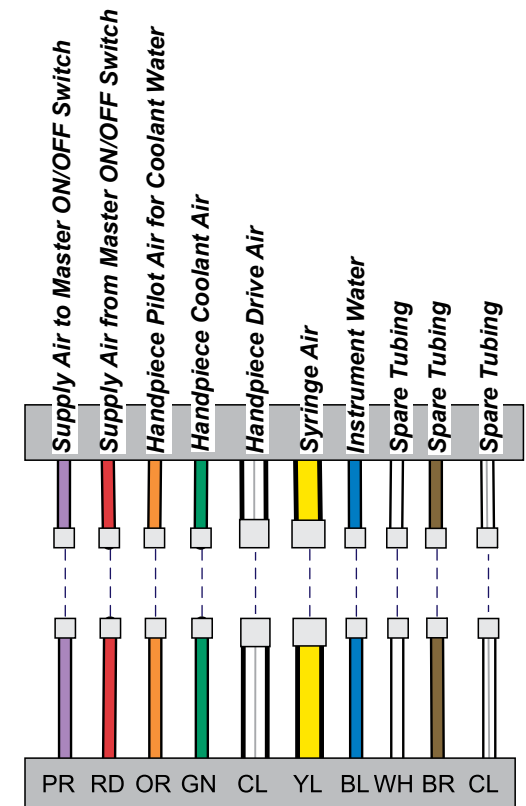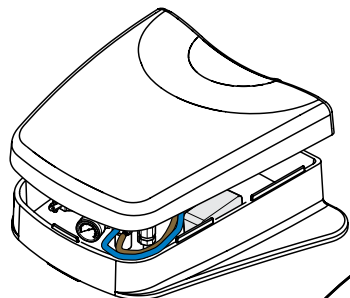
Connect ground wire to ensure system ground continuity. Failure to do so may result in electric shock.


**Step 11: Verify umbilical tubing and water supply connections.**

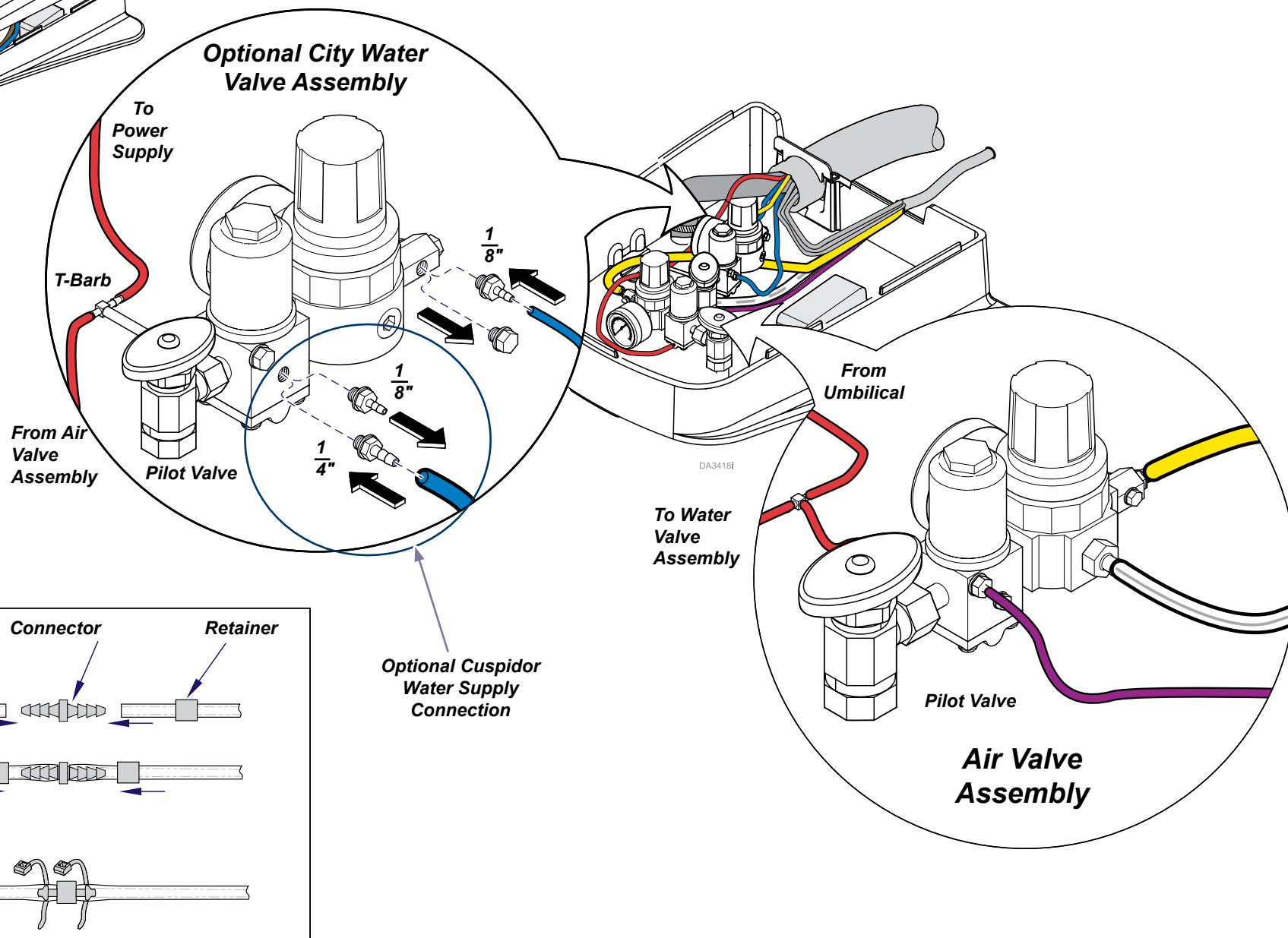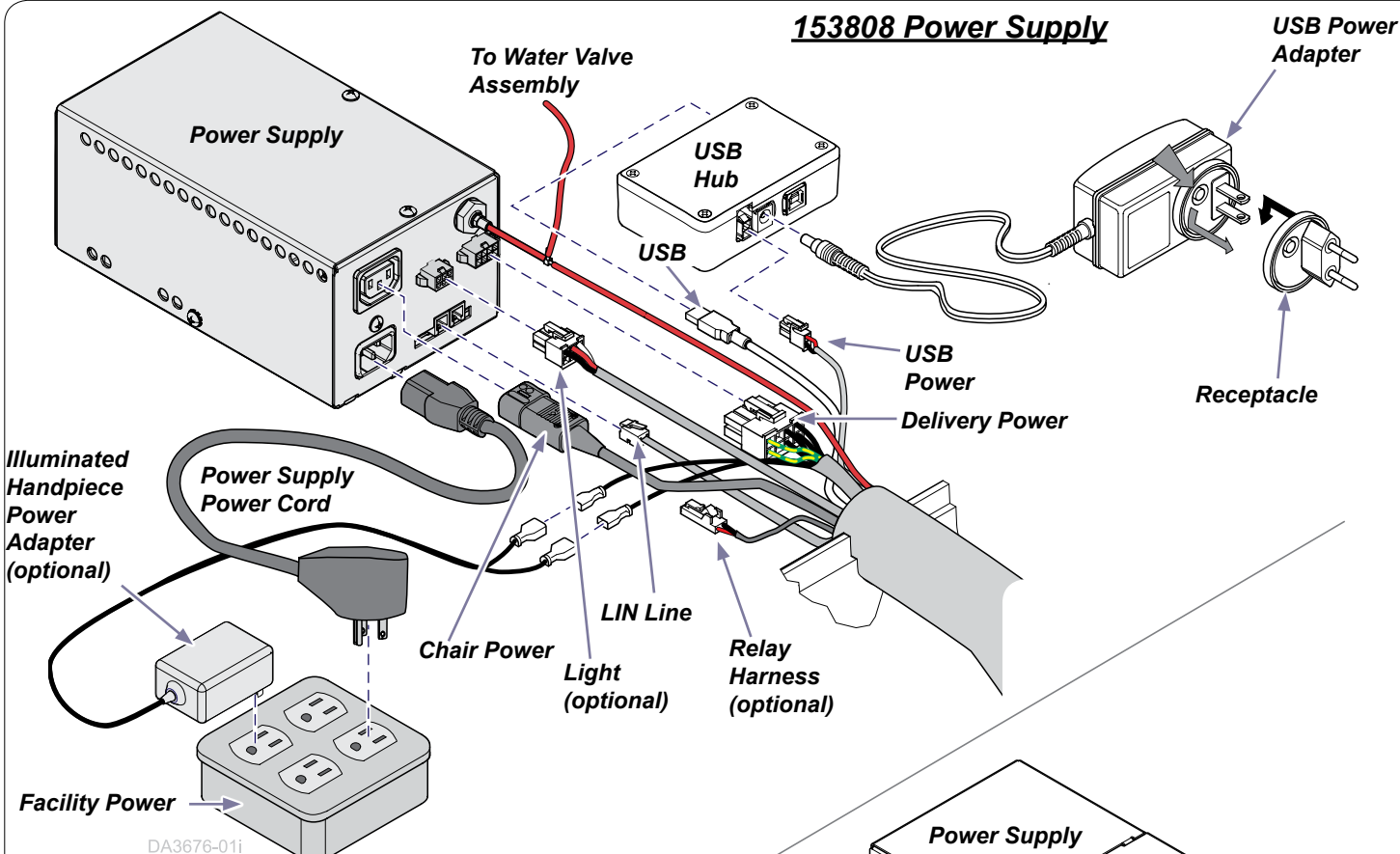
- Remove J-box lid.
- Verify proper yellow, clear and purple connections from umbilical to valve assemblies.
- Optional - Install city water supply fittings and connect blue tubing as shown.

**Equipment Alert**  
Once the delivery unit is powered ON, it must STAY ON for 30 minutes to charge the clock circuitry.

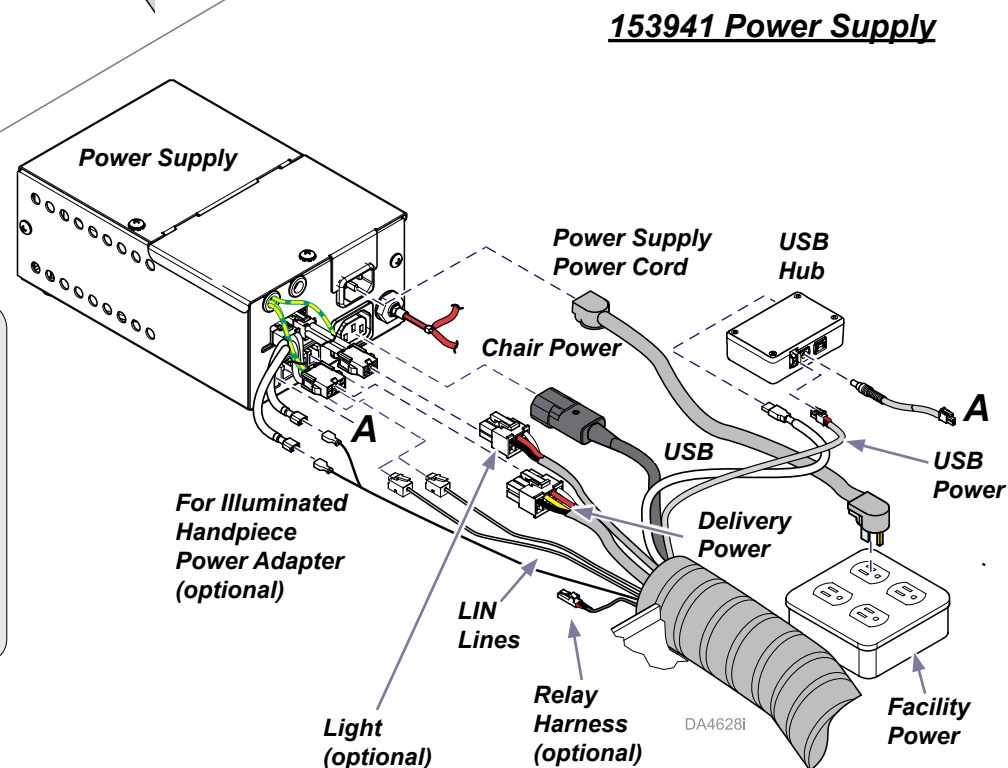**Equipment Alert**  
Chair umbilical includes connectors for optional components that may not be included in your installation.


### Step 12: Standard Power Supply Connections.


- Add T-fitting and connect red tubing to power supply.
- Install the appropriate receptacle onto the USB power adapter and connect the USB, USB power and the other end of the adapter to the USB Hub.
- Connect delivery power to power supply.
- Connect chair power to power supply.
- Connect LIN line to power supply.
- Install optional components (Steps 13 - 19) as required OR proceed to Step 20.

**Note**

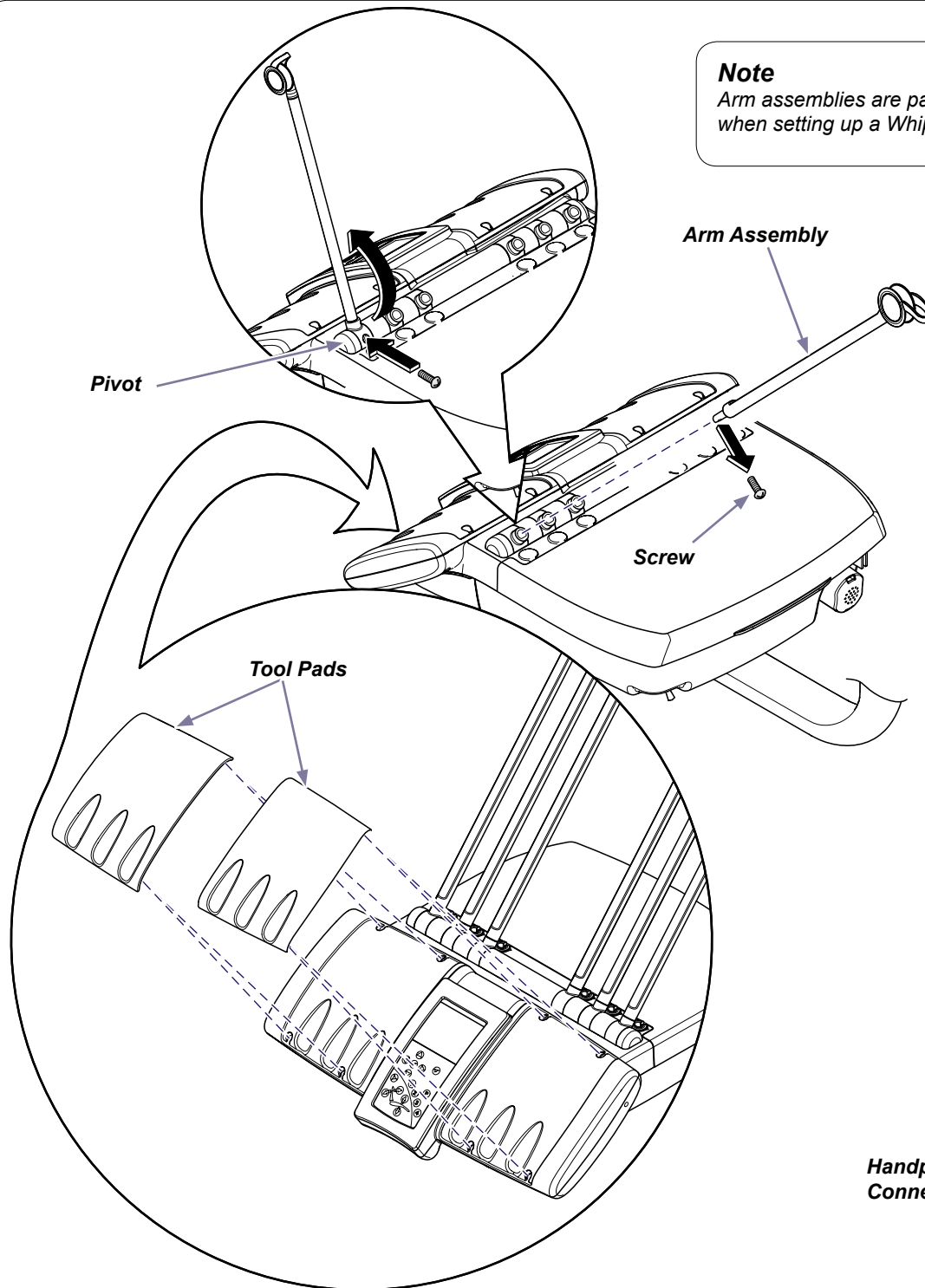Arm assemblies are packaged separately and must be installed when setting up a Whip delivery unit.

**Continental-Style Units Only****Step 13: Install Arm Assemblies and Tool Pads.**

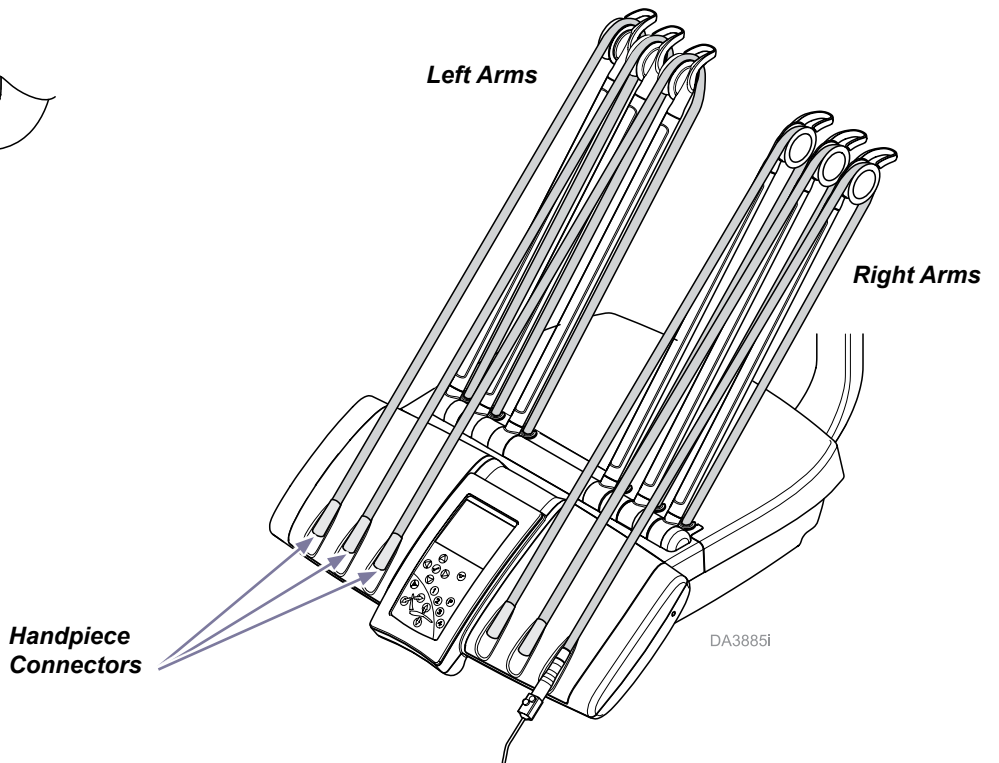- A) Remove screw from arm assembly.
- B) Insert arm assembly into the pivot and rotate forward.
- C) Install the screw thru pivot hole into arm assembly to secure.
- D) Install three left and three right arm assemblies.
- E) Install left and right tool pads onto the front of unit.
- F) Route tubing thru wheels and align handpiece connectors with tool pads as shown below.


Continental-Style Units Only



**Step 14: Install Tray Mount on Arm.**

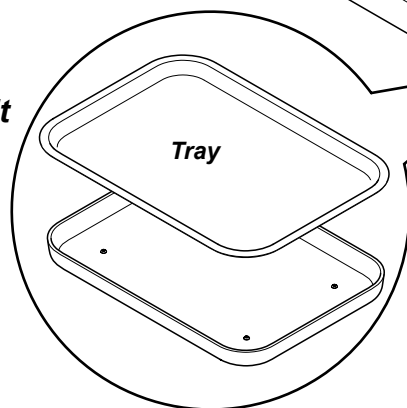- A) Align tray mount assembly onto arm post.
- B) Install set screw and tighten to secure tray mount to arm post.
- C) Adjust drag on tray rotation with set screw.

**002-1693-001 Kit**

**002-1693-002 &  
002-1693-003 Kits**

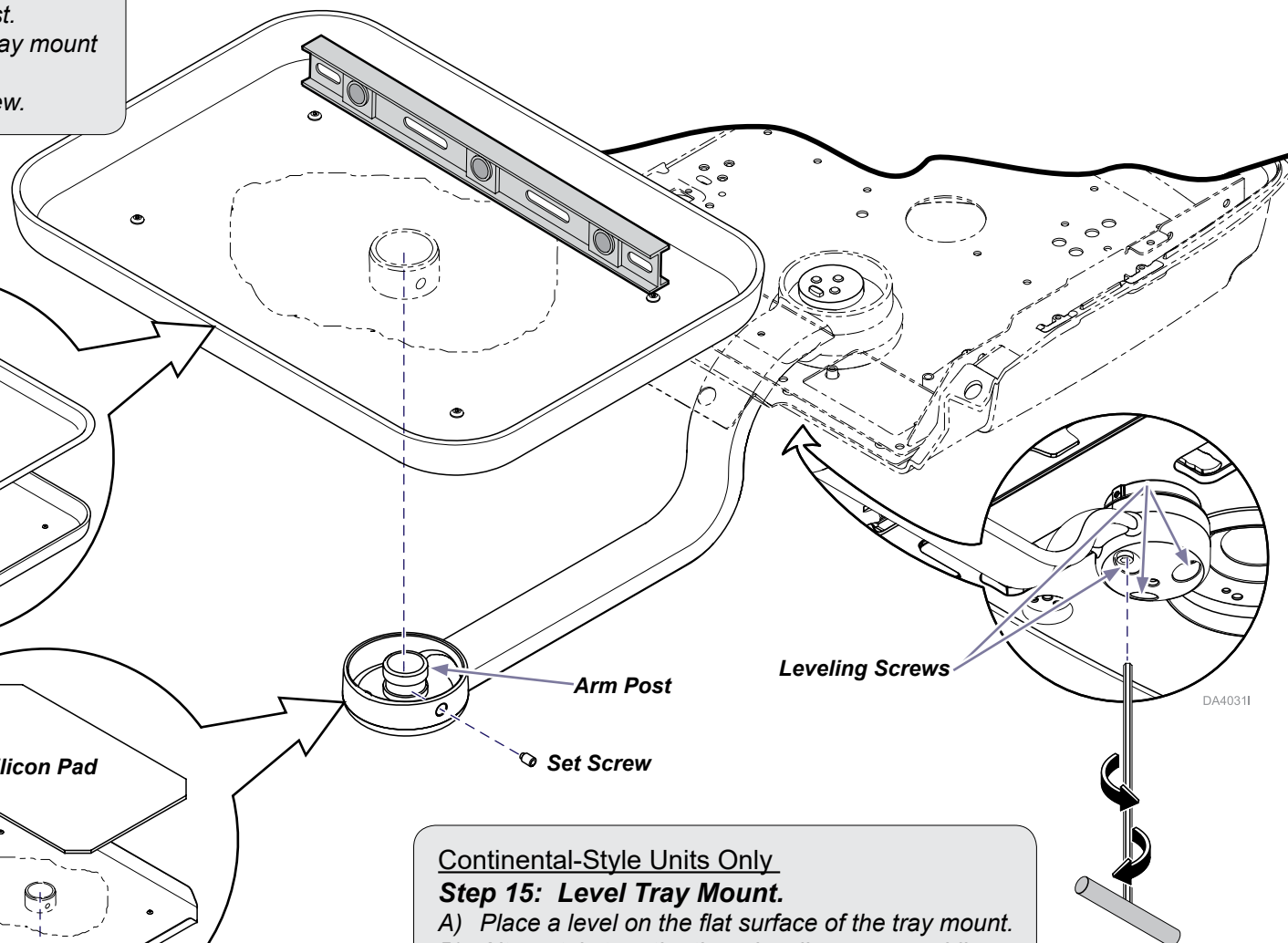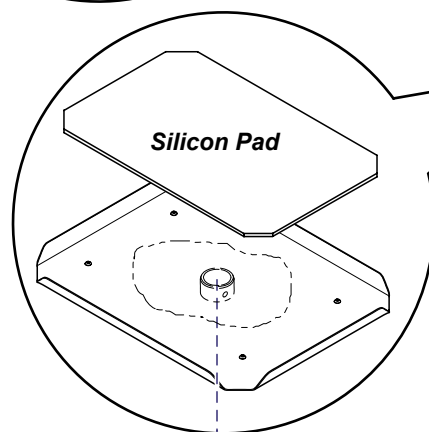Continental-Style Units Only

**Step 15: Level Tray Mount.**

- A) Place a level on the flat surface of the tray mount.
- B) Alternately turn the three leveling screws while watching level.

Continental-Style Units Only

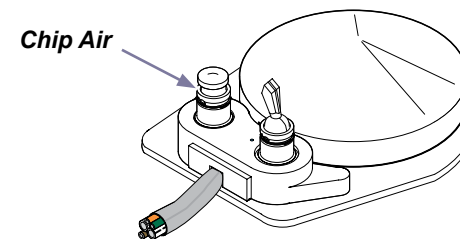
**Step 16: Install Pad and/or Tray.**

- A) For 002-1693-001 Kit: Set tray on tray mount
- B) For 002-1693-002 & 003 Kits: Place silicon pad onto tray mount.  
(Trays are not included in these kits.)


For Units Having Deluxe Rheostat with Chip Air Foot Controls Only:

***Make Chip Air Connection in Delivery Head.***

- A) Cut green 1/8" Coolant Air Supply tubing.
- B) Locate brown 1/8" tubing coming into the unit head.
- C) Use barbed tee and sleeves to tee the 1/8" brown tubing into the 1/8" green coolant air tubing.

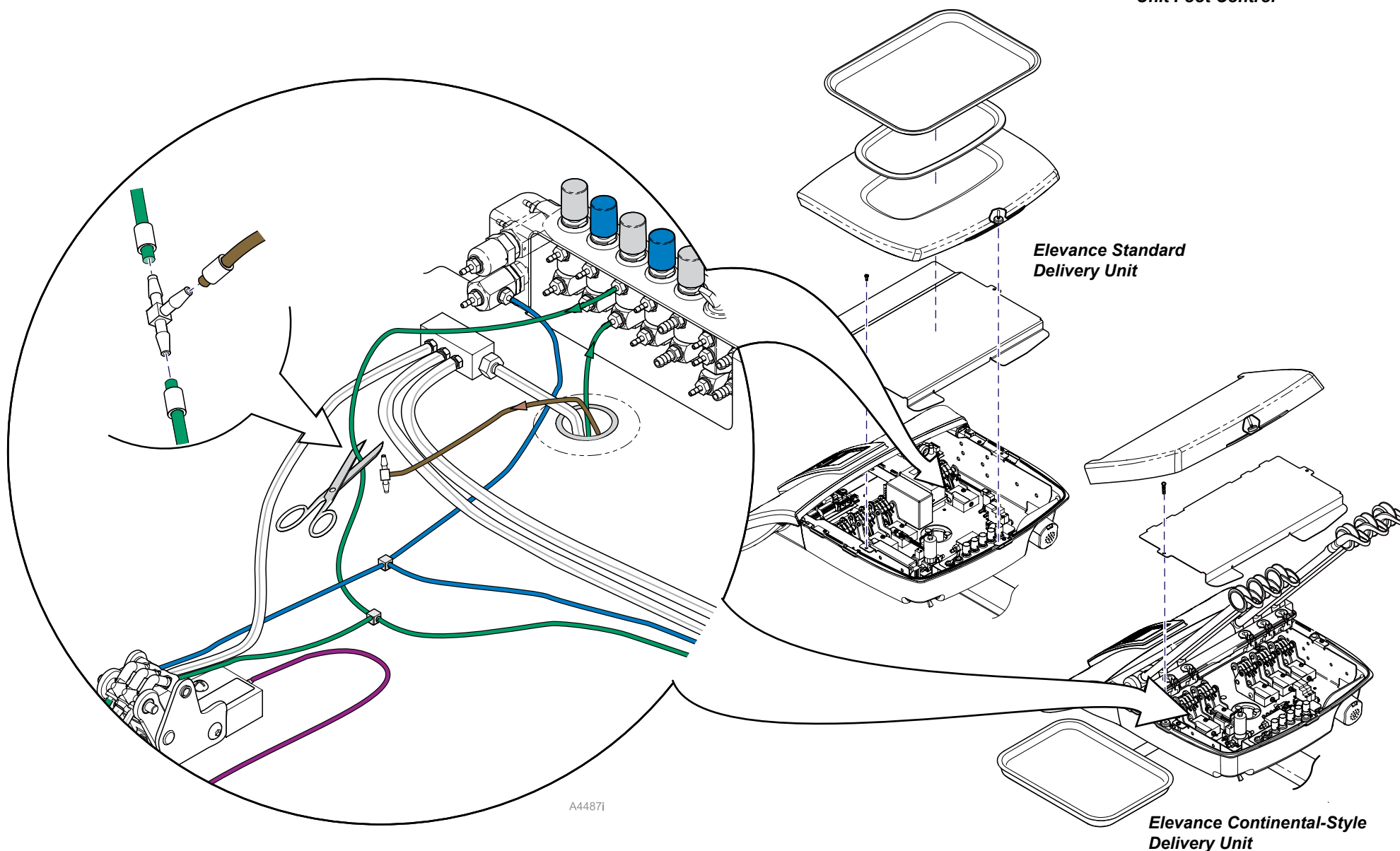
***Deluxe Rheostat with Chip Air  
Unit Foot Control***

**Perform Steps 17 thru 20 last, if you have additional components to install.**

**Step 17: Install seat shrouds and seat frame.**

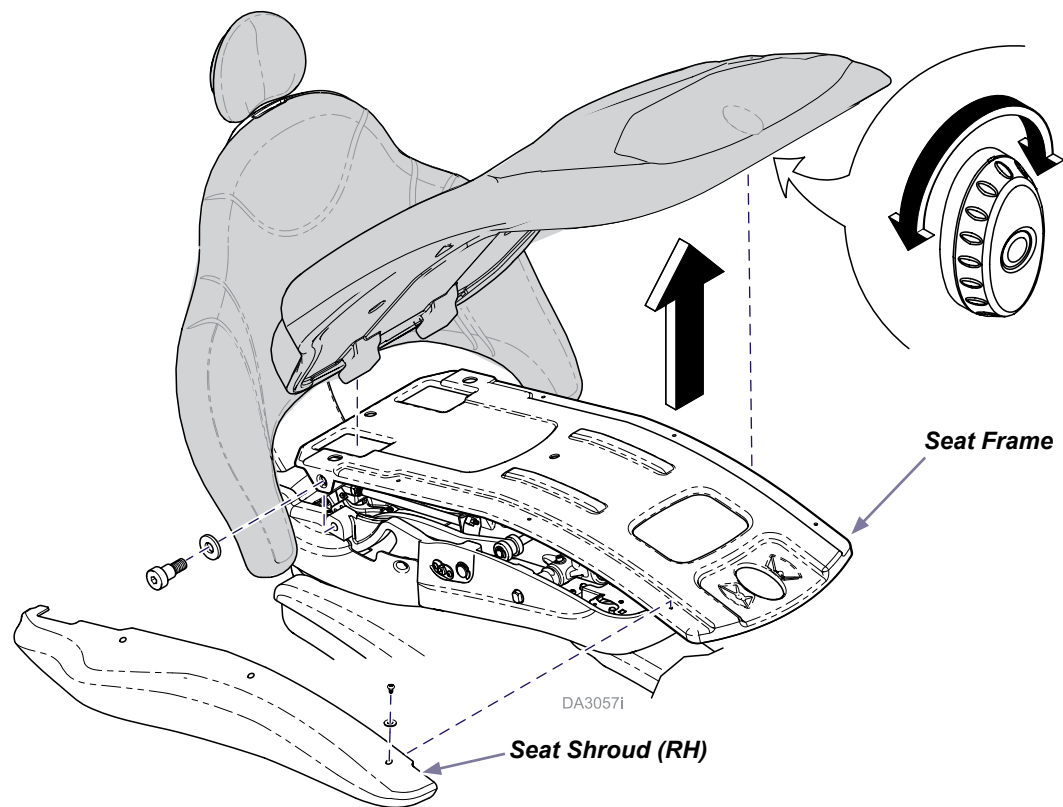- A) Position seat frame, then secure with two pivot screws / washers.
- B) Position RH seat shroud, then secure with three screws / washers.
- C) Repeat for LH seat shroud (not shown).

**Step 18: Install seat upholstery.**

- A) Insert seat upholstery tabs into slots.
- B) Lay upholstery flat. (Lock knob must align with hole)
- C) Rotate lock knob to secure upholstery.

### Step 19: Connect power and verify operation.

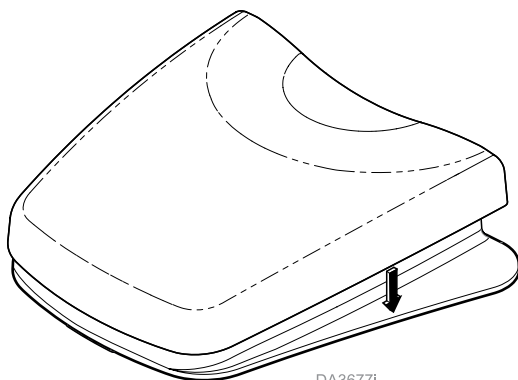
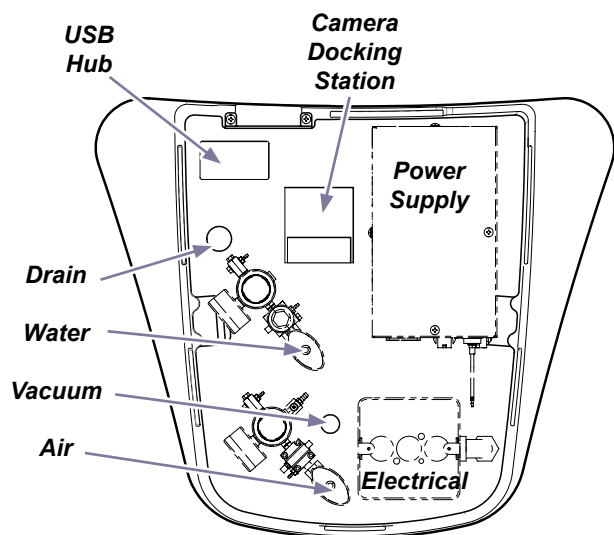
- A) Connect the power supply to facility power.
- B) Arrange components neatly inside J-Box.
- C) Connect power supply and optional power adapters into facility power.
- D) Install J-box lid.
- E) Verify operation of the delivery.

Note: Reference Elevance Delivery User Guide for more detailed operation information.

### Equipment Alert

Once the delivery unit is powered ON, it must STAY ON for 30 minutes to charge the clock circuitry.

DA3677i

## Installation Checklist

| Test | Verify the following:                                                                                                    | ✓ |
|------|--------------------------------------------------------------------------------------------------------------------------|---|
| 1    | LR casting and flex arm are level and secure.                                                                            |   |
| 2    | Flex arm and delivery unit rotate easily and stay where placed.                                                          |   |
| 3    | All covers are in place on all components and chair.                                                                     |   |
| 4    | All tubing is securely connected or capped if not used.                                                                  |   |
| 5    | Water and air supply regulator gauges in J-box read appropriate pressures.                                               |   |
| 6    | Electrical connectors and couplers are tightly connected, grounded and protected inside chair casting.                   |   |
| 7    | All electrical devices are properly connected to the power supply. Power supply is properly connected to facility power. |   |
| 8    | Delivery unit comes ON when master switch is turned ON.                                                                  |   |
| 9    | Control screens appear as handpieces are pulled from the delivery unit.                                                  |   |
| 10   | Control screens respond as expected when buttons are pressed.                                                            |   |
| 11   | Pressing footpedal delivers air/waster to handpieces as expected.                                                        |   |
| 12   | Wipe all components with a soft dry cloth to remove dirt and smudges.                                                    |   |

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