Assistant's Stool Installation / Operation Guide

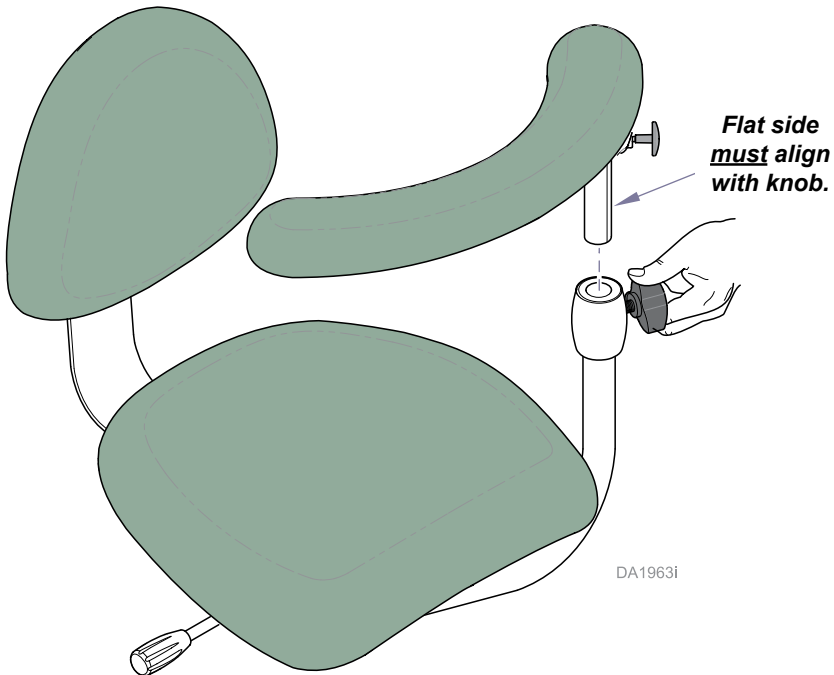
Applies to Models:
153812

Cleaning:

- Clean upholstery with diluted bleach solution.
Ratio - 10:1 (water:bleach) **Refer to Page 3 for more information.**
- Wipe painted metal and plastic surfaces with a clean, soft cloth, and a mild cleaner. Periodic applications of common furniture wax will ease cleaning and maintain the finished luster of the stool.

Weight Capacity:

350 lbs (158 kg).


DA1963i

To install body support...

- Install down tube of body support into collar as shown.
- Install / tighten adjustment knob.

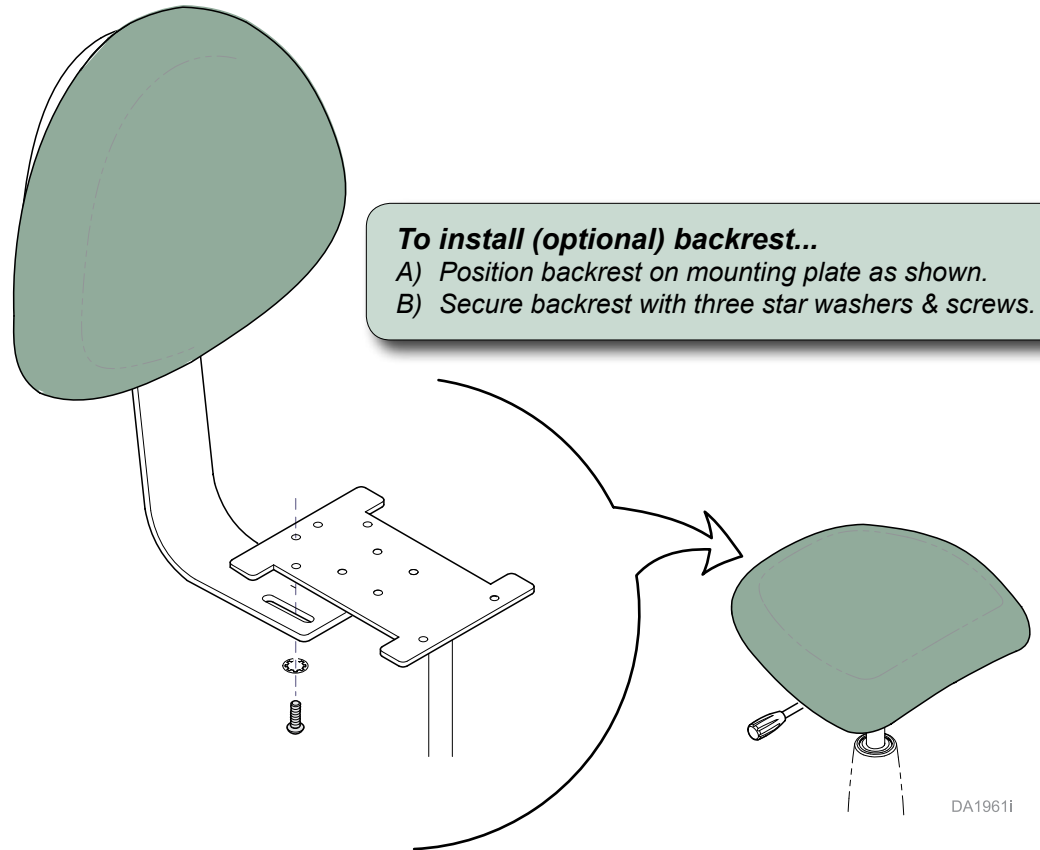
Note: Be sure flat side of body support shaft aligns with adjustment knob.


To install (optional) backrest...

- Position backrest on mounting plate as shown.
- Secure backrest with three star washers & screws.

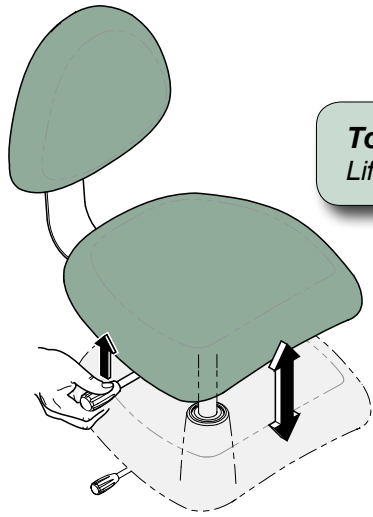DA1961i

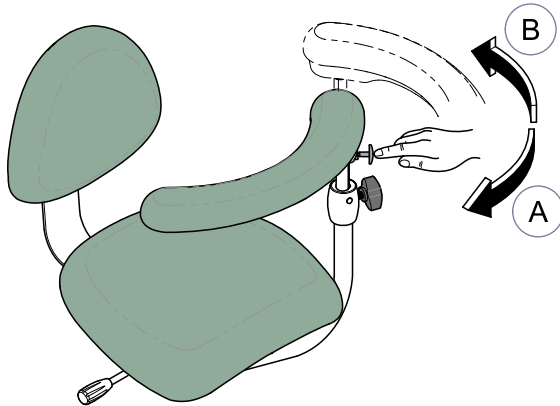To adjust seat height...
Lift height adjustment handle.


To pivot body support...

Inward:

A) Move body support toward back section.

Outward:

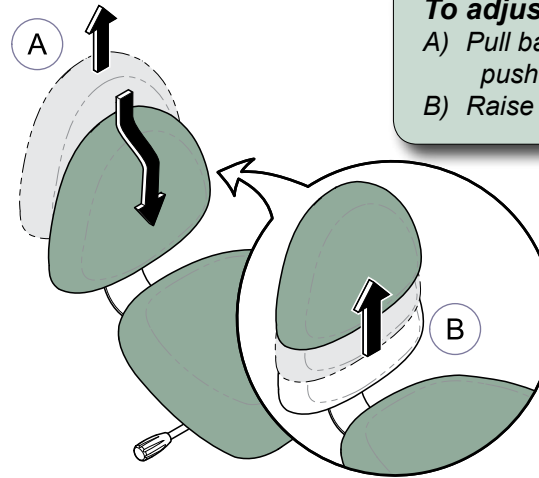
B) Push release knob, then move body support outward.

To adjust (optional) backrest height...

A) Pull backrest up until it stops, then push it all the way down.

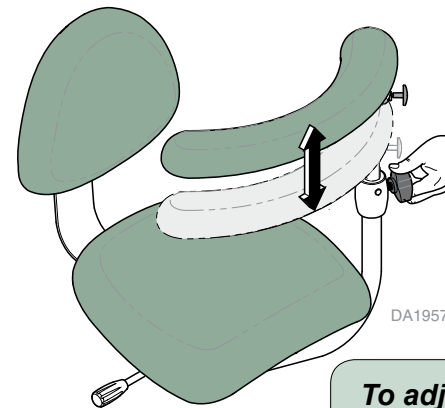B) Raise backrest to desired height.

To adjust body support height...

A) Loosen knob, then raise/lower body support.

B) Tighten knob to lock in position.

Note: Flat side of body support shaft must align with knob.

Specifications

Assistant's Stool (153812)	
Weight Rating	350 lbs (159 kg)
Seat Height: <i>Standard models</i>	Min: 21.5 in. (54.5 cm) Max: 27.5 in. (70 cm)
<i>Short models</i>	Min: 18 in. (45.5 cm) Max: 21 in. (53 cm)
Seat Dimensions	Width: 16.25 in (41 cm) Depth: 16 in. (40.6 cm)
Base Dimension	24 x 24 in. (61 x 61 cm)

Cleaning

Every dental practice is different, and no single disinfectant is the best choice for every facility. Listed below are some organizations to assist you in choosing the best disinfectant for your practice

Organization for Safety & Asepsis Procedures:

<http://www.osap.org>

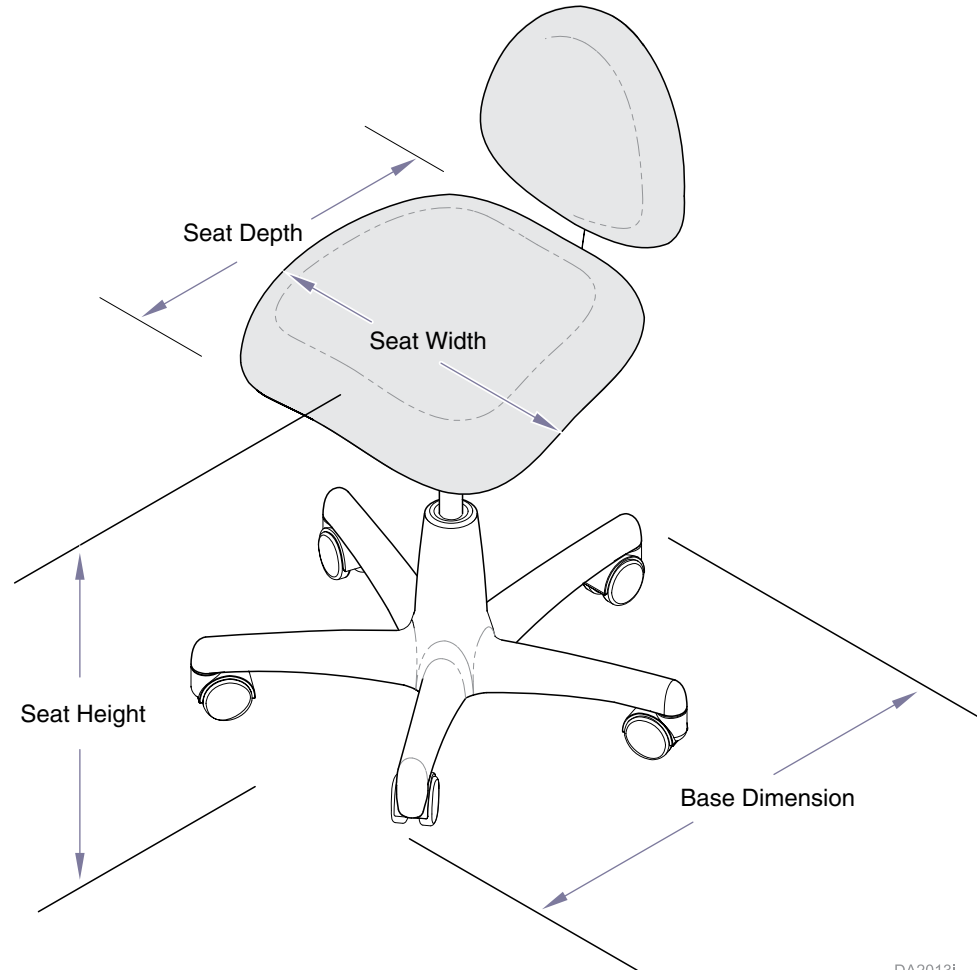
American Dental Association:

<http://www.ada.org>

Dept. of Health & Human Resources

Centers for Disease Control & Prevention (CDC):

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Midmark Corporation