




203 Drawer Kit [002-1128-xxx & 002-10252-xxx]

Components:

Drawer Assembly ...	2
Frame	1
Screws:	
(#10-24 x 3/8")	7

Step 1: Remove existing frame.
A) Carefully lay table upside down.
B) Remove frame and legs from table top.
Note: Keep hardware for reinstallation.


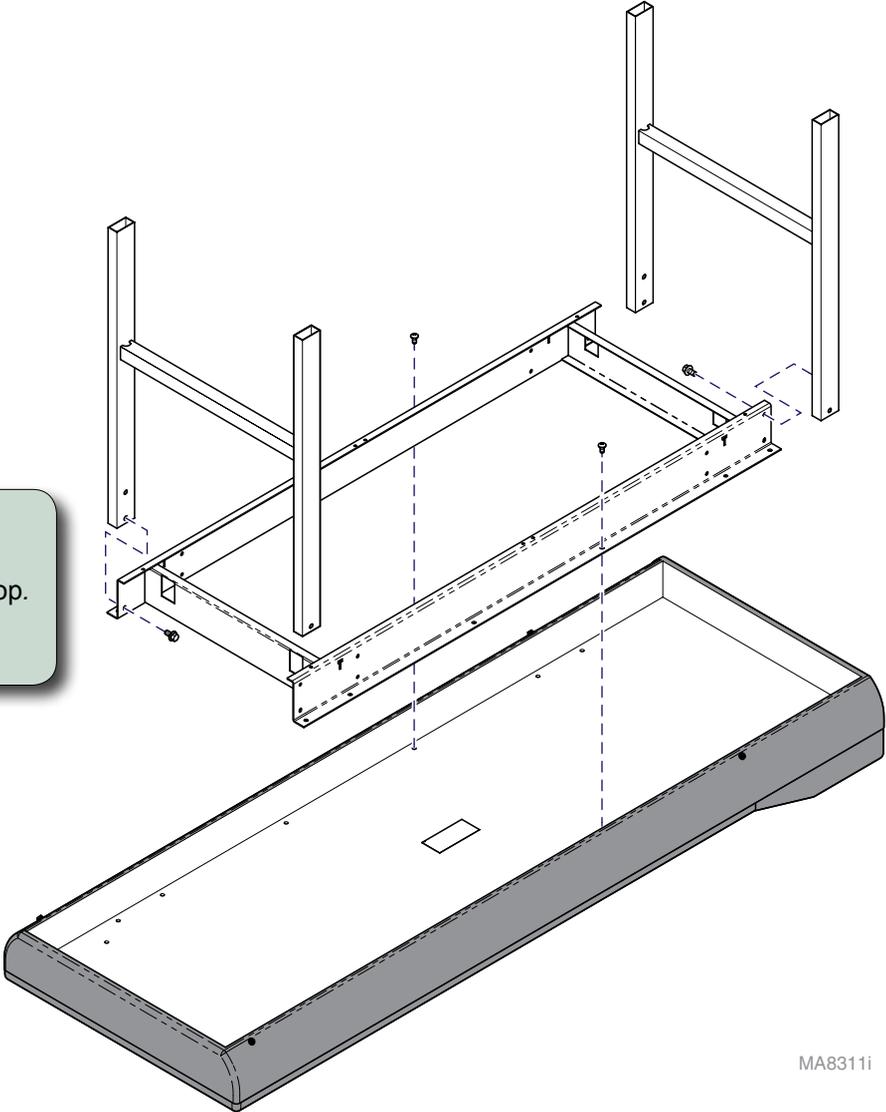


MA8311i

Step 2: Install frame / drawer with slots.

- A) Secure new frame to table top with hardware removed in **Step 1**.
- B) Secure drawer shell w/ slots to frame w/ four screws.


Note: Drawer can be mounted to face opposite direction.

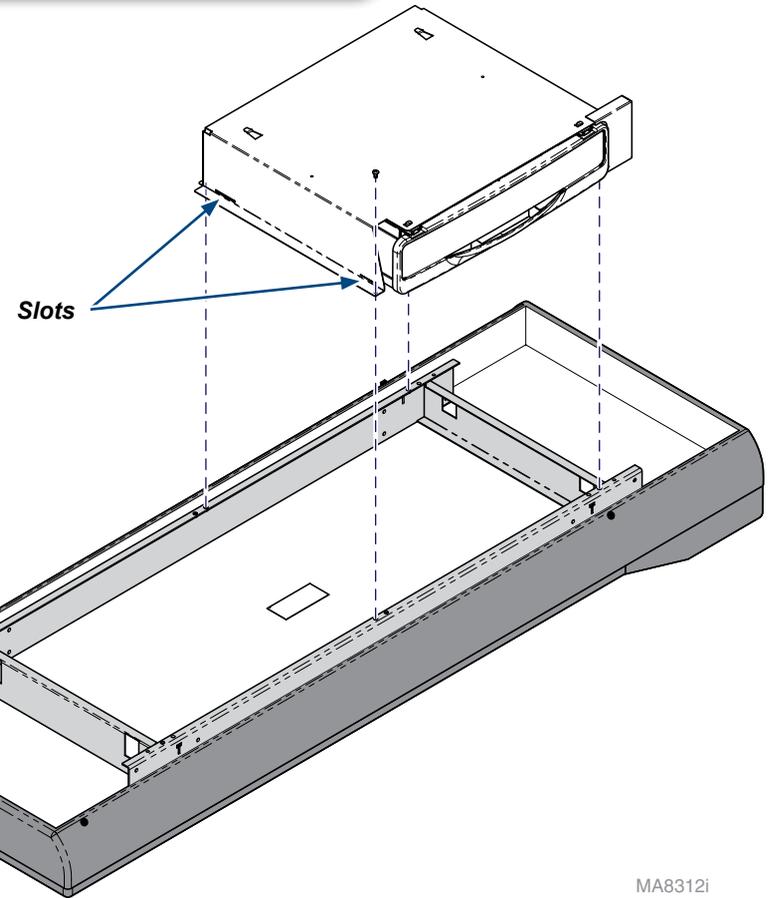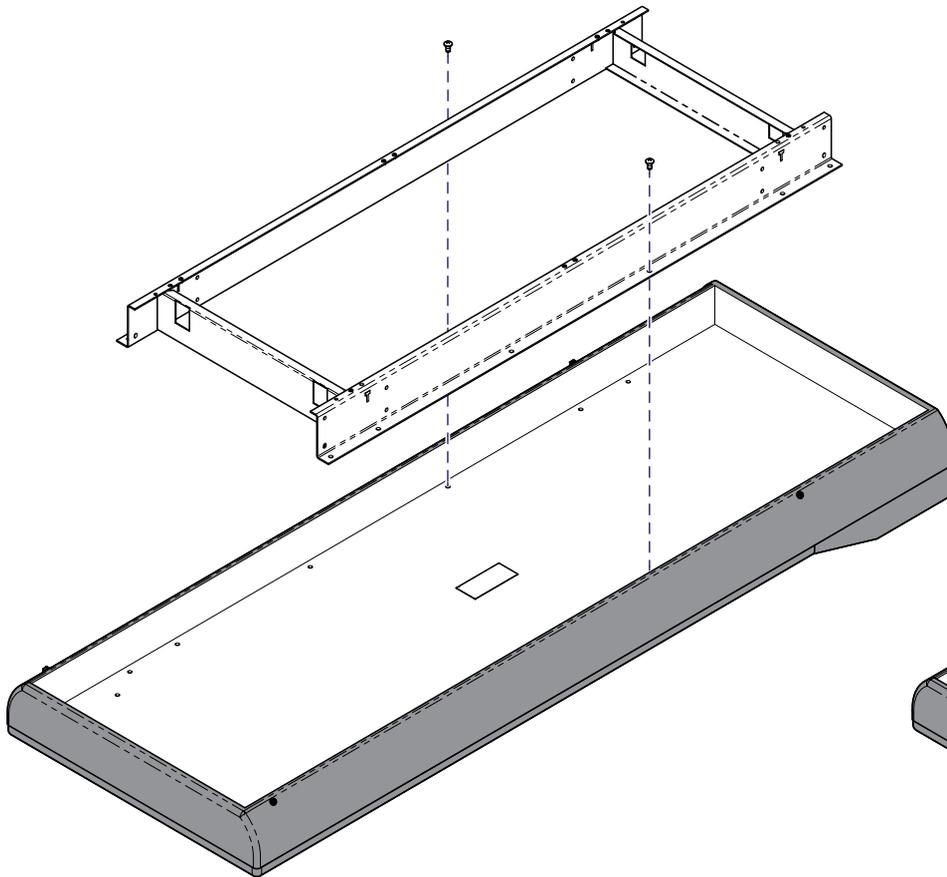

MA8312i


Step 3: Install drawer shell w/ tabs.

- A) Align drawer shell w/ tabs with slots in shell.
- B) Slide drawer shell to lock shells together and secure drawer shell w/ tabs to frame w/ three screws.

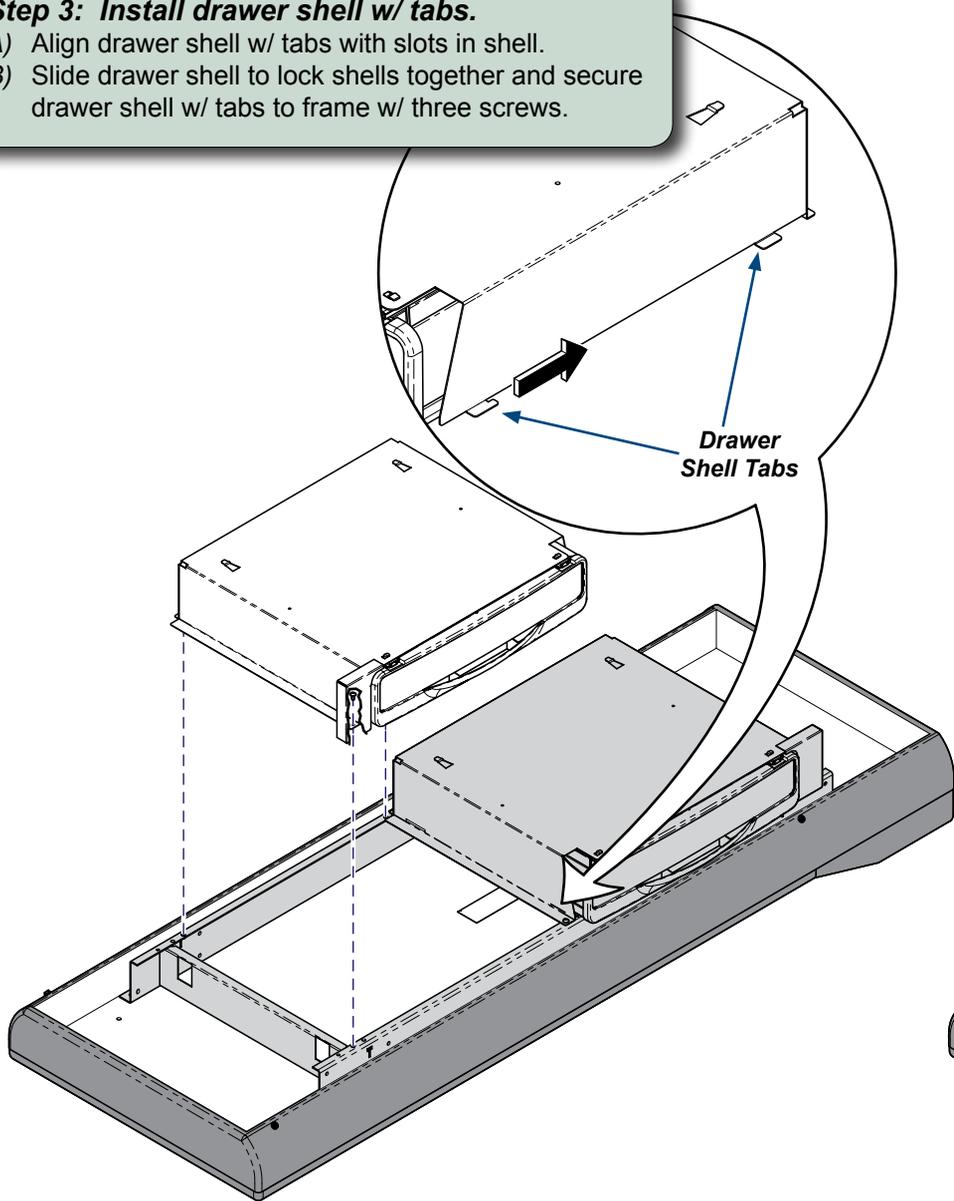

Step 4: Install legs.

- A) Slide legs inbetween table top and frame.
- B) Secure legs to new frame with hardware removed in Step 1.

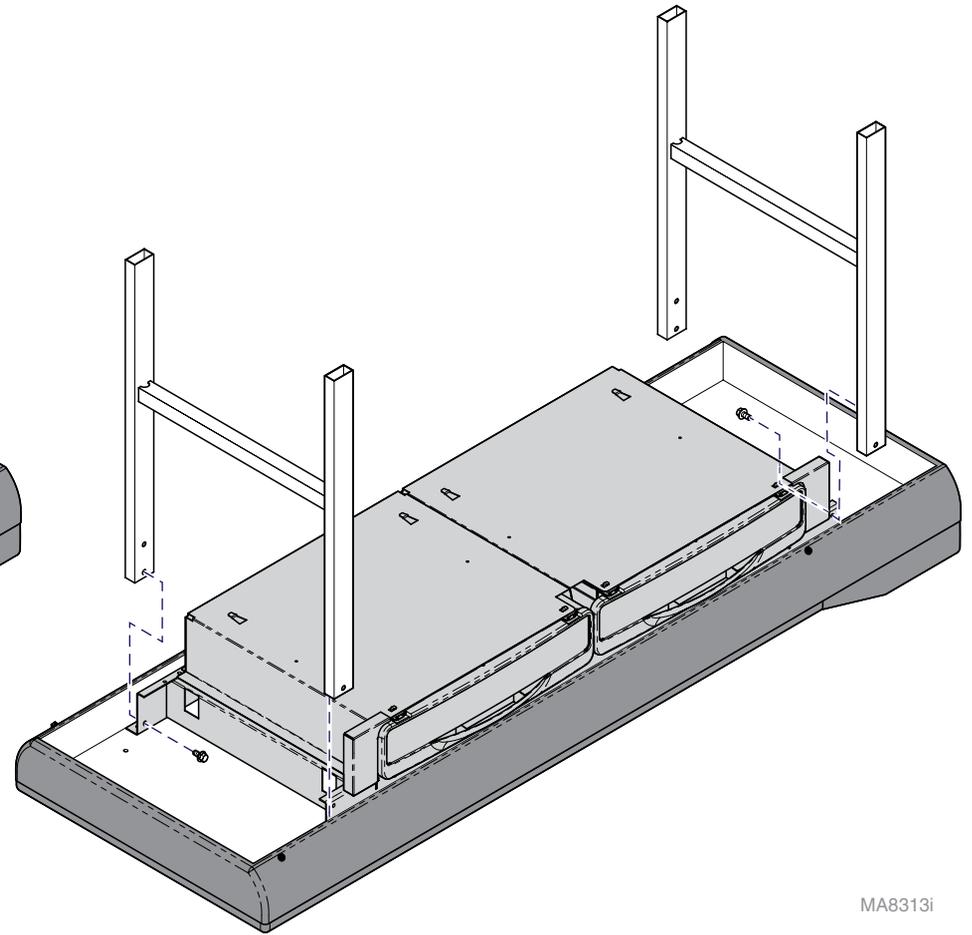

MA8313i



Because we care.