



**To extend / retract the foot section...**  
A) Squeeze handle on either side of foot section.  
B) Move foot section in / out to desired position.  
C) Release handle(s) to lock position.

Release Handles

**To access the debris tray...**  
Pull tray outward.



Brake Pedal

**Rotational Base (optional)**  
**To rotate the chair base...**  
A. Press and release brake pedal.  
B. Rotate the chair to desired position.

**To lock base in position...**  
Press and release brake pedal.

*Note: The base will lock automatically after two minutes.*

## Important Information

**Max. Patient Weight: 450 lb (204 kg)**

**Periodically inspect the following:**

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

**Cleaning**

**Upholstery**

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner buildup.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue buildup, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [midmark.com](http://midmark.com) in the Technical library under the User Information tab for your product.

**Painted Metal / Plastic Surfaces**

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

**Calling for Service**

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585  
Monday-Thursday 8:00 AM until 6:00 PM  
Friday 8:00 AM until 5:00 PM (ET)

Midmark Corporation  
60 Vista Drive  
Versailles, OH 45380-0286  
Phone: 937.526.3662  
Fax: 937.526.5542  
[midmark.com](http://midmark.com)



## Quick Reference Guide 646 / 647 Procedures Chair

**Model Number:**   
**Serial Number:**

**Dealer Information:**

**Purchase Date:**

**Upholstery Color:**

003-1893-00 Rev AA3 4/4/24


This guide outlines basic operation of the chair. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



**To operate the Base / Back / Tilt functions...**

Press and hold the appropriate button.

Example: The Base Up button is called out in the illustration.

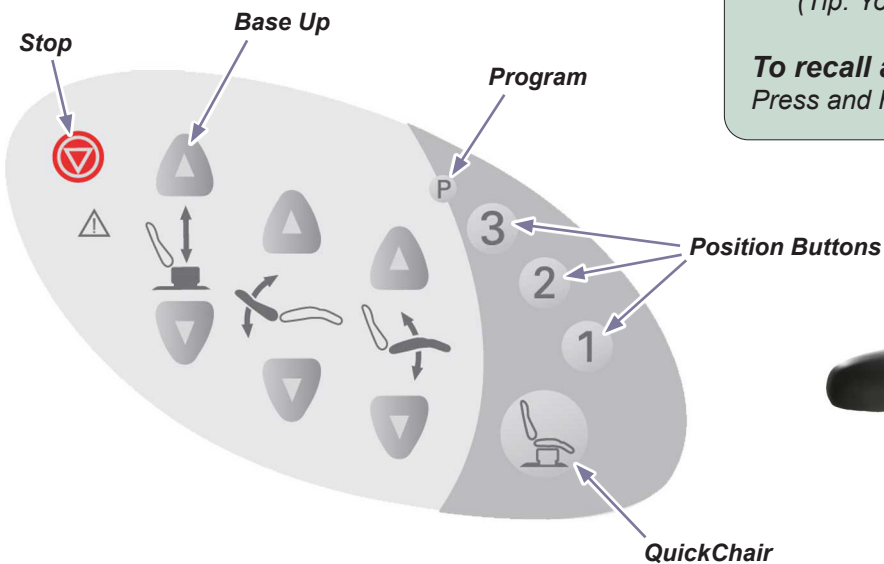

**Note:** The 3-function, programmable controls are shown. The applicable symbols used on the 2-function non-programmable controls are the same.

**Programming Chair Positions (647 programmable models only)**

This feature allows you to easily recall frequently used chair positions.

**To program a chair position...**

- Move chair to desired position.
- Press the Program button. [single “beep”]
- Press the desired Position button (1, 2, 3 or 4). [three “beeps”]  
(Tip: You must press the Position button within 5 seconds)


**To recall a programmed position...**

Press and hold the desired Position button (1, 2, 3 or 4).



**Attention**

To Stop movement of any automatic chair function:  
Press the Stop button on the touch pad or foot control.

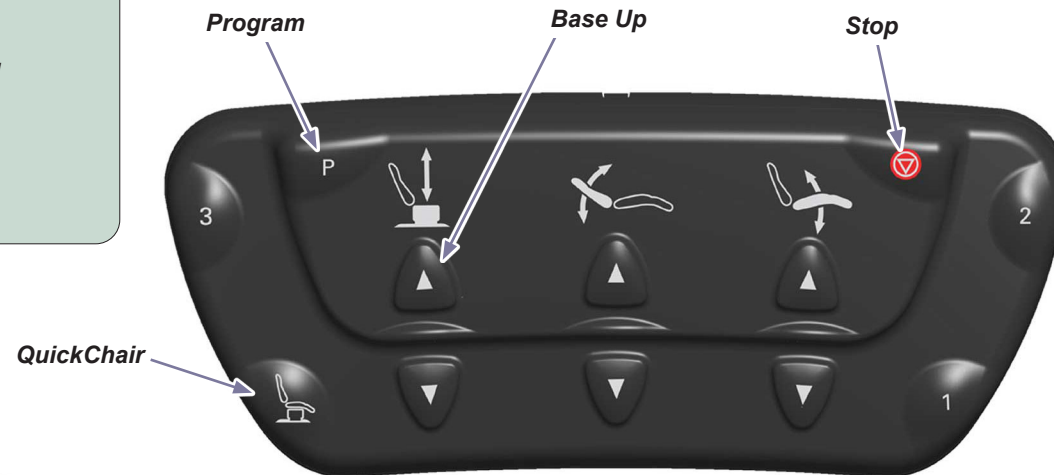

**QuickChair Function**

This function activates the Back Down, Tilt Down and Base Down (647 only) functions simultaneously.

**To activate the QuickChair function...**

Press and hold the QuickChair button.

**QuickExam® Function (647 non-programmable models only)**

This function positions the chair for an exam by activating the Base Up and Tilt Up functions simultaneously.

**To activate the QuickExam function...**

- Press and hold the QuickExam button.
- Release button when chair reaches desired position.

