Midmark Dental Light Setup Guide

(Console Mtd. / Elevance and Ultra Chair)


Applies to Models:
 153901 (-001, -004 thru -008)
 153818 (-001, -004 thru -006)
 153914 (-001, & -003)
 153915 (-001, & -003)

Note


This setup guide applies to both Elevance and Procenter model consoles, mounted on Elevance and Ultra Chairs.

Build the Curved Arm and Flex Arm Assembly

NOTE: This setup guide applies to Halogen and LED-style lights.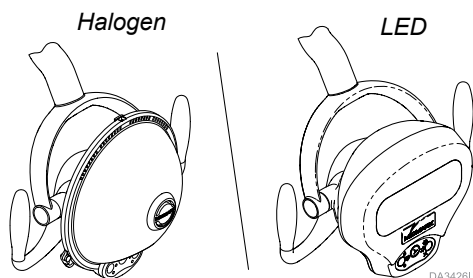

Step 1: Place the stop ring on the wire harness, face the tab towards the light head and insert flex arm into the curved arm assembly. Secure assembly with the set screw.

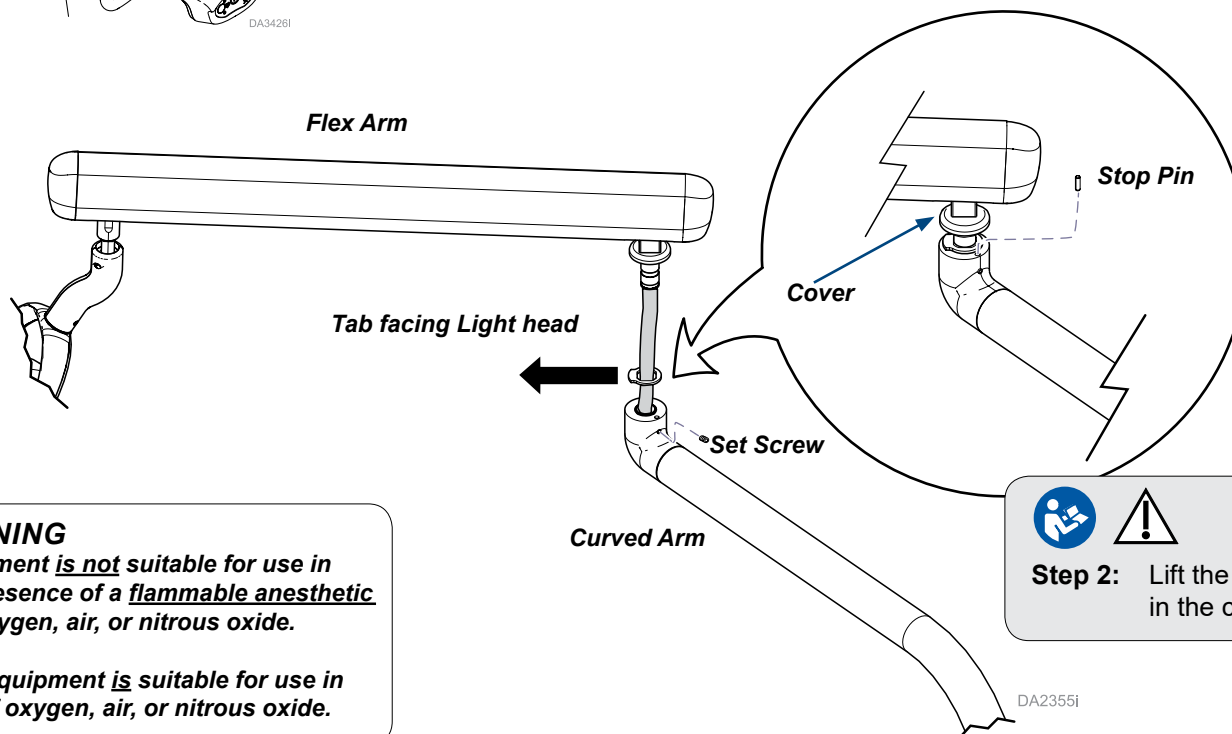WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture with oxygen, air, or nitrous oxide.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Step 2: Lift the cover and place the stop pin in the hole in the curved arm.

Install the Arm Assembly on the Procenter Console


WARNING

Always disconnect power before removing covers. Failure to do so may result in personal injury.

Step 3: Remove the console covers.


Procenter Console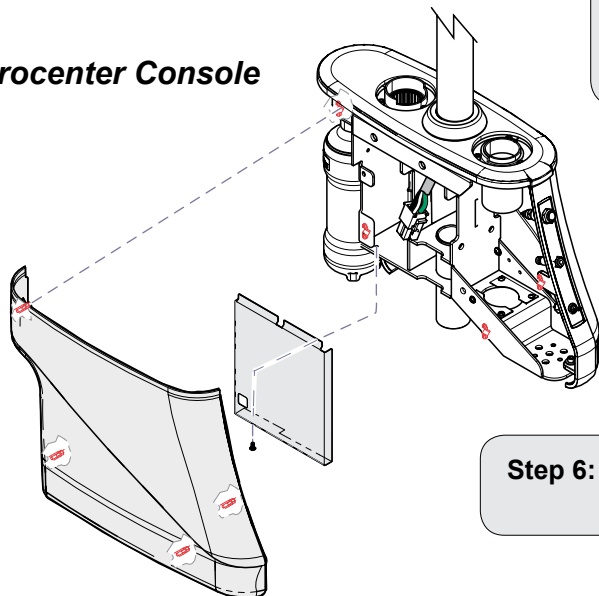


Step 4: Feed wire harness through trim ring, bearing washer and console bearing assembly.

Step 6: Secure bearing to console with mounting screws.

Elevance Console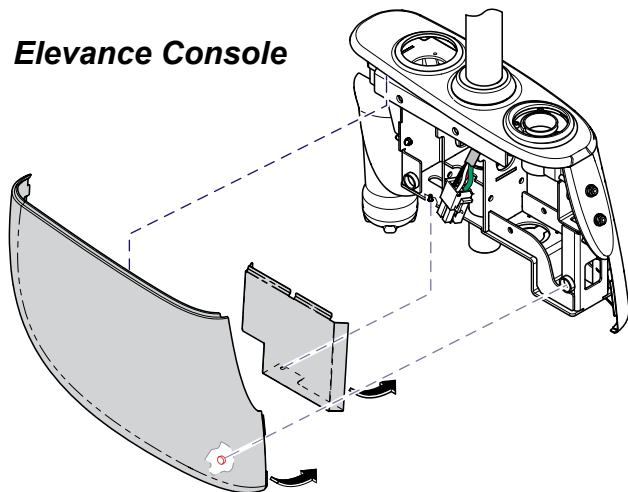

Step 5: Install console bearing assembly in center opening of console, positioned so that the two set screws (2) can be accessed through holes on either side of console frame. Lift the trim ring above the rotation stop screw hole and insert the rotation stop screw for left or right chair orientation.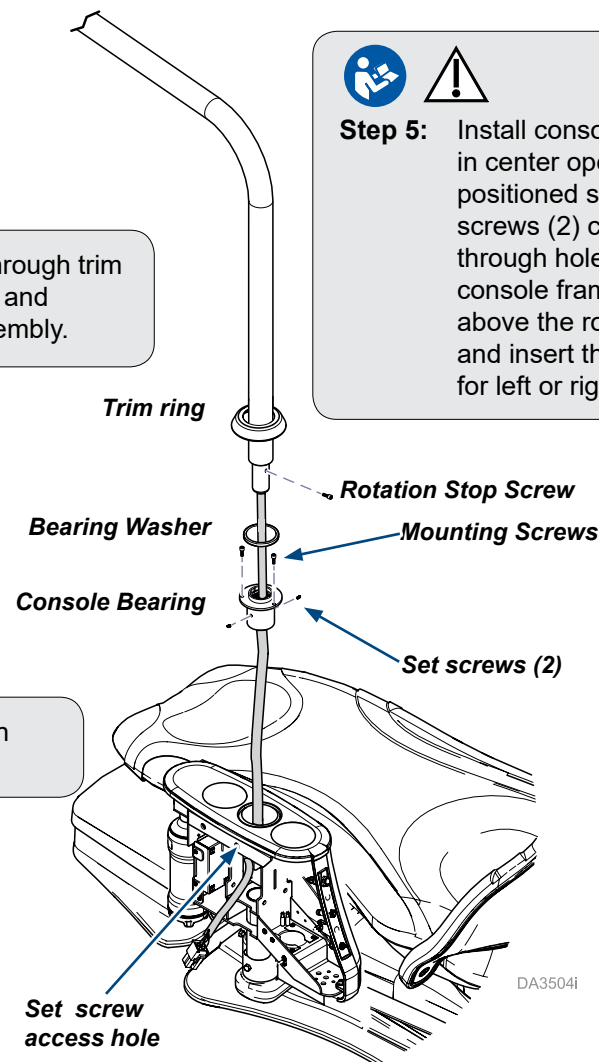

Equipment Alert

You can adjust the drag (rotational tension) on the **Light only assembly** by tightening or loosening the two set screws in the console bearing .


Connect Light to the Chair Power Supply

CAUTION


Connect Ground Wire to ensure system ground continuity.
Failure to do so may result in electric shock.

Step 7: Connect the light power from the arm assembly to power connector inside the console.
Use the screw and star washer to fasten ground wire to the console frame.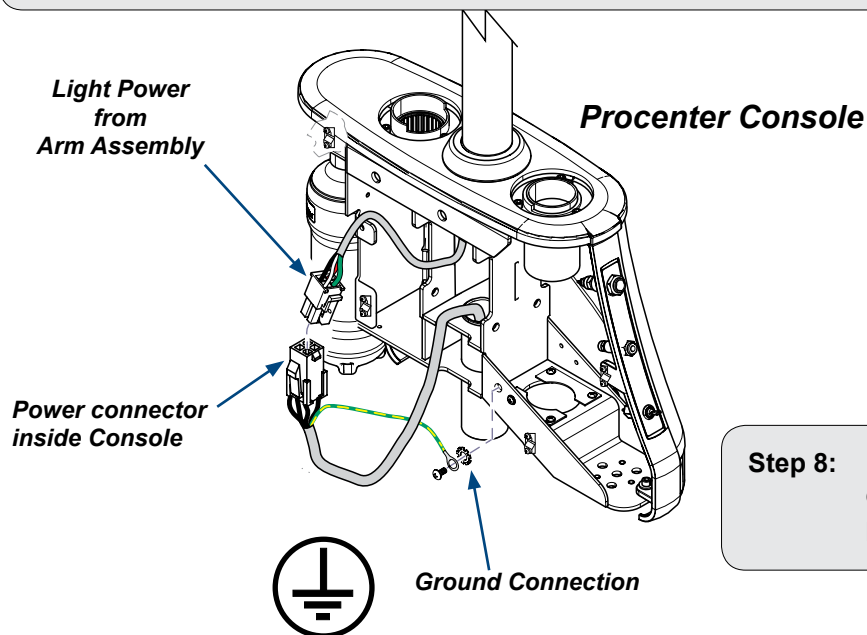


Step 8: Route ground wire through the frame cutouts so that the metal cover will not pinch it when fastened in place.

Step 9: Reinstall covers on console.

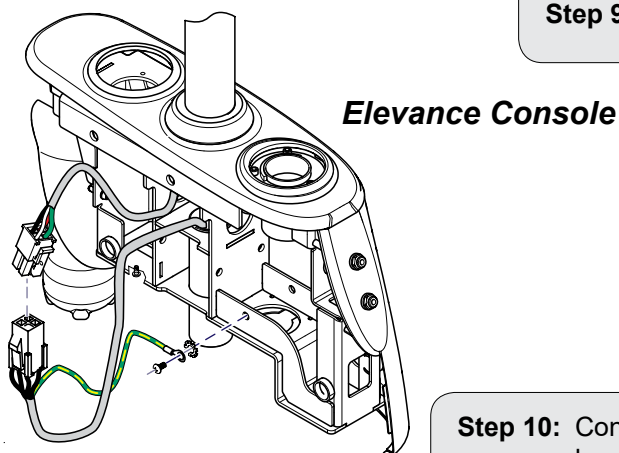
Step 10: Connect chair light harness to power supply.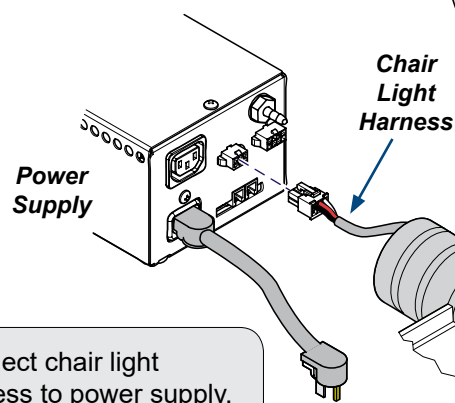

Step 11: Verify Operation of Light.

Note: If necessary, refer to User's Manual for operation.

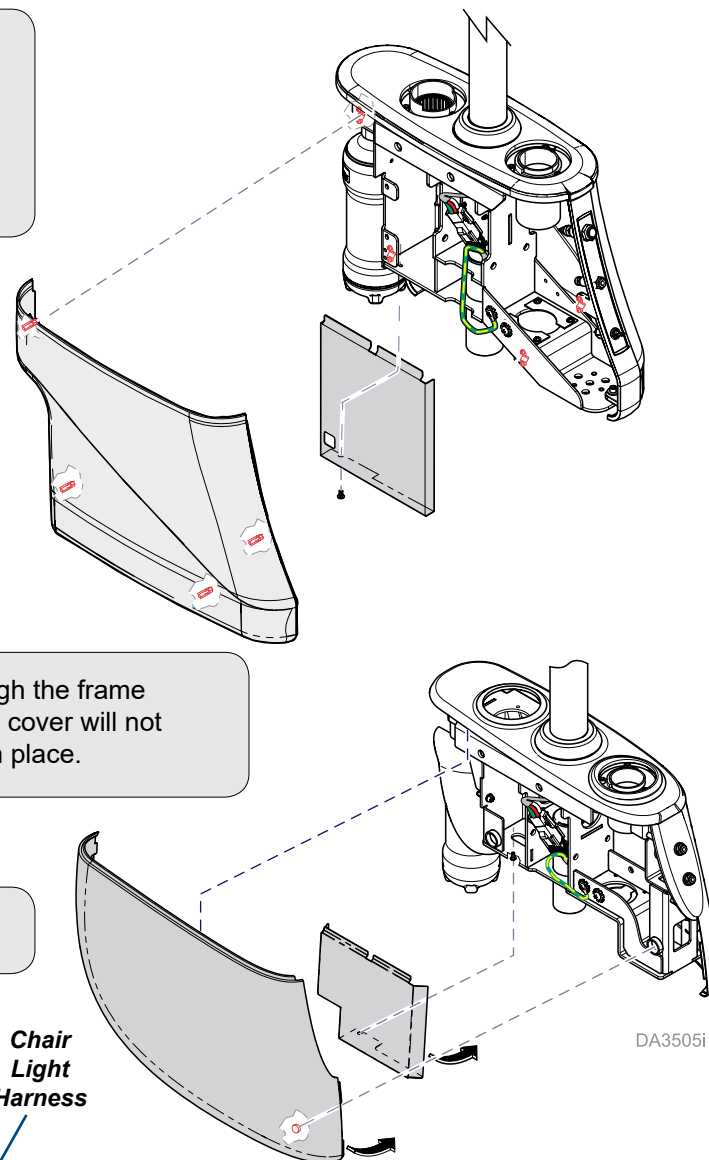Bypassing the Air Pressure Switch

Equipment Alert

If the installation location does not have a pressurized air supply, it will be necessary to BYPASS the air pressure switch and still operate the equipment.

Bypassing the Air Pressure Switch is not necessary if a pressurized air supply line is present.


WARNING

To prevent the possibility of electrical shock, always disconnect electrical power to the Power Supply before removing the cover. Failure to comply with these instructions could result in severe personal injury or death.

To Bypass the Air Pressure Switch...

- Disconnect power.
- Remove screws, lock washers and cover.
- Disconnect brown wire from the NO terminal on the air pressure switch.
- Connect brown wire to the NC terminal on the air pressure switch.
- Install cover and secure with screws and lock washers.
- Reconnect power to the power supply.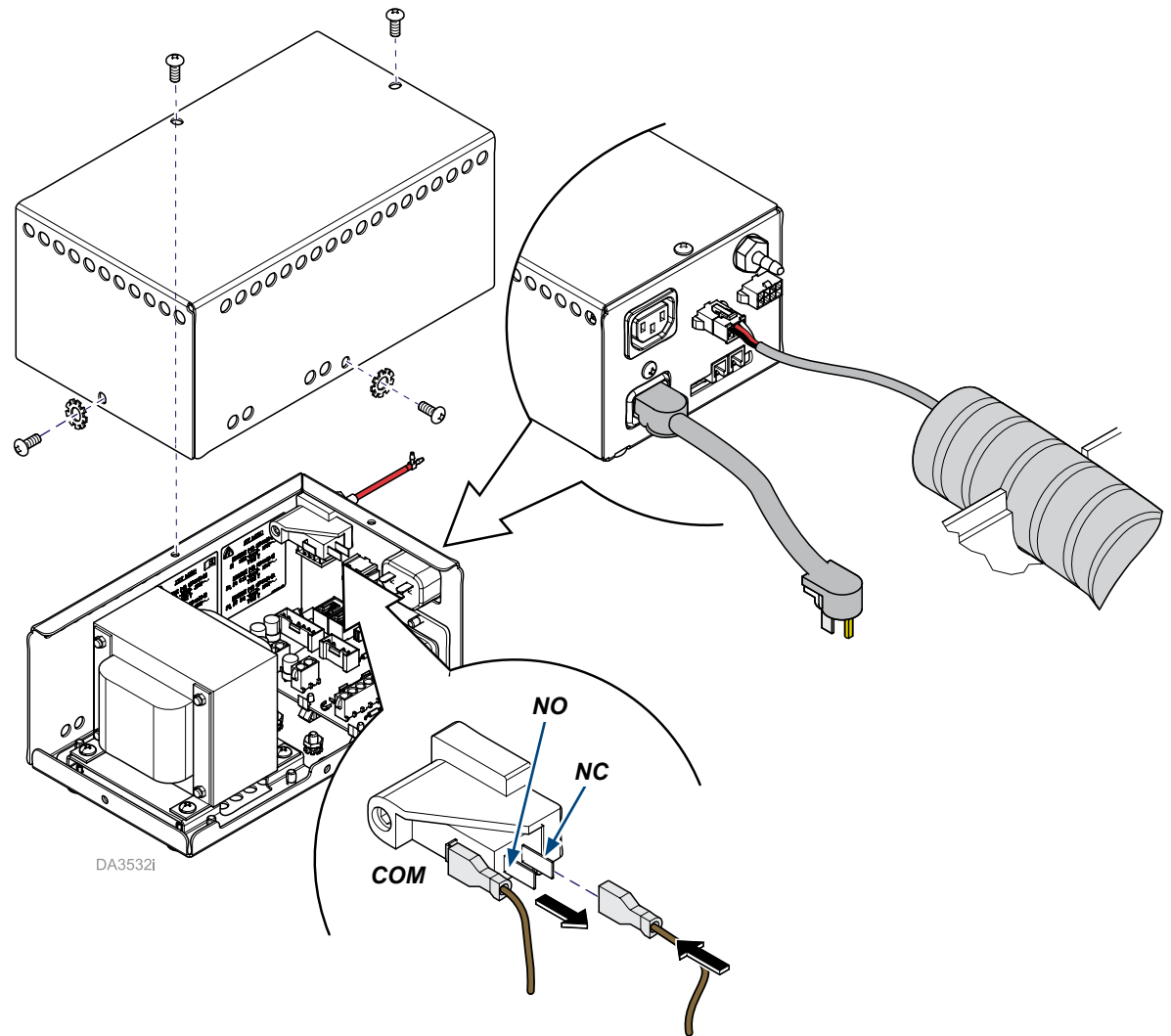

Tension Adjustments



To adjust tension at Light pivot joint...

- Remove screw cover as shown.
- Loosen set screw.
- Loosen or tighten light pivot adjustment screw.
- Tighten set screw.

To adjust tension at Arm / Yoke pivot joints...


- Remove neck cover as shown (see Note).
- Loosen or tighten appropriate screws to adjust tension.

Note: Some halogen models have a short neck on the lighthead. It is not necessary to remove the neck cover on these models.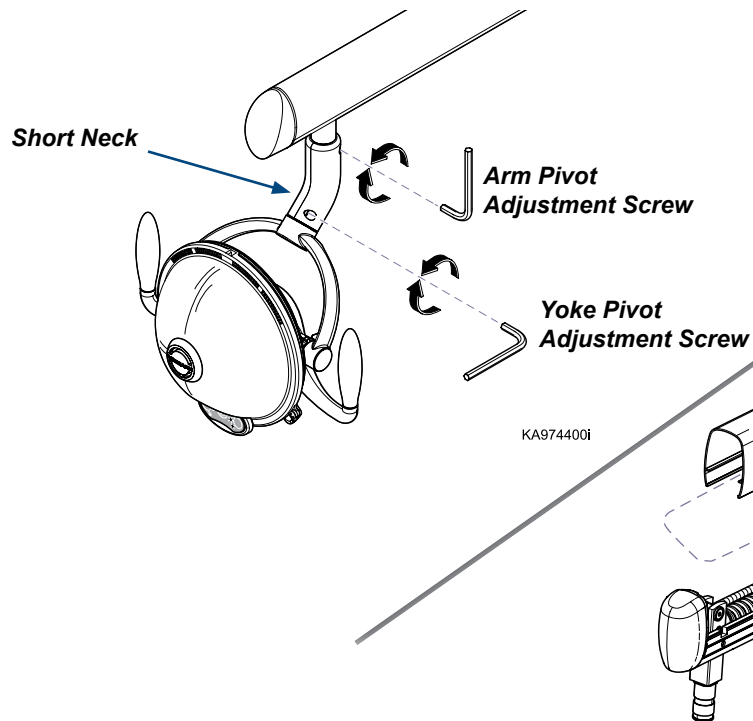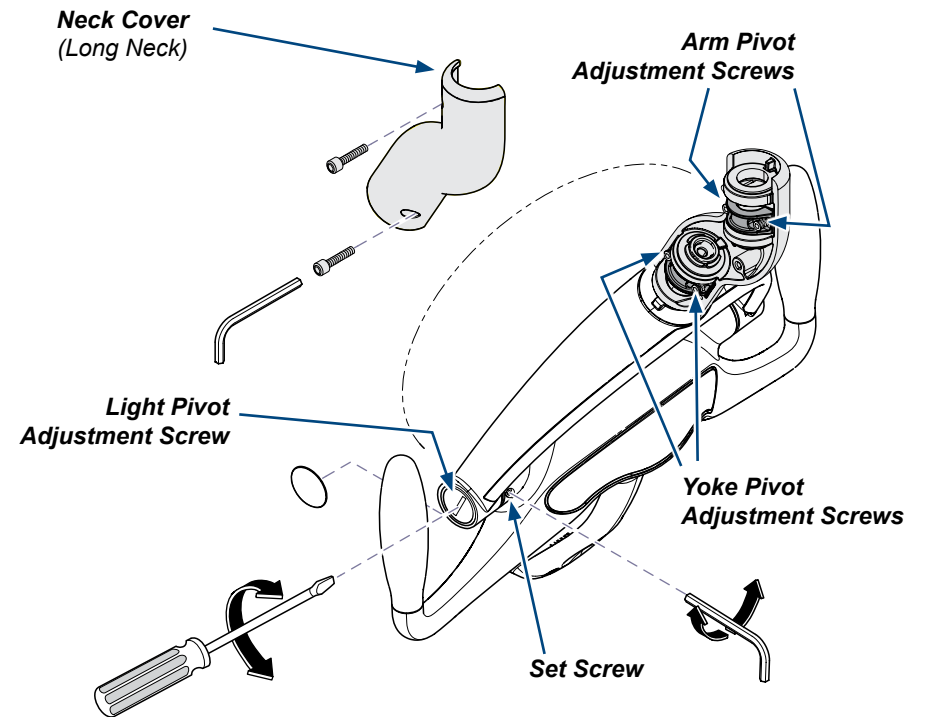

To adjust Flex Arm tension...

- Remove end cap and top cover as shown.
- Loosen or tighten adjustment nut as desired.

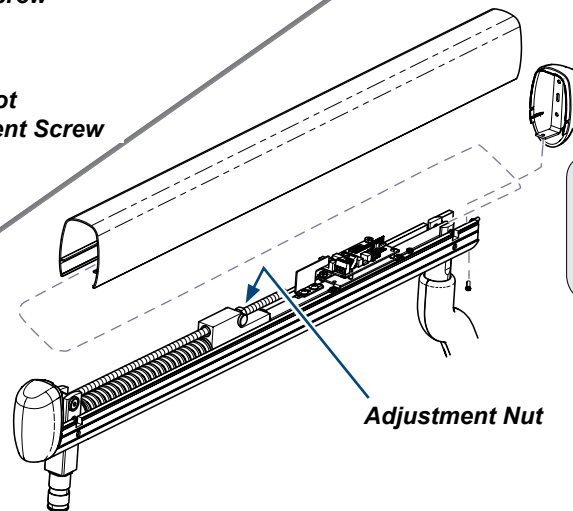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