



Pressure Relief Valve [002-1197-00]

Intended Use:

To provide replacement for the pressure relief valve.

Requirements / Restrictions:

- 1. Power off, unplug unit, allow to cool to ambient / room temperature and remove cover.
- 2. Release pressure by pulling ring of pressure relief valve if necessary.

Components:

- Pressure relief valve w/ washer 1
- Enclosure (RH) for Pressure relief valve 1
- Enclosure (LH) for Pressure relief valve 1
- Reducer 1/2 NPT to 1/8 NPT 1


Special Tools:

none

NOTE: The Pressure Relief Valve and Reducer have a seal on the threads to prevent leakage.

Step 3: Remove Pressure Relief Valve.

Step 2: Remove Pressure Relief Valve Enclosures.

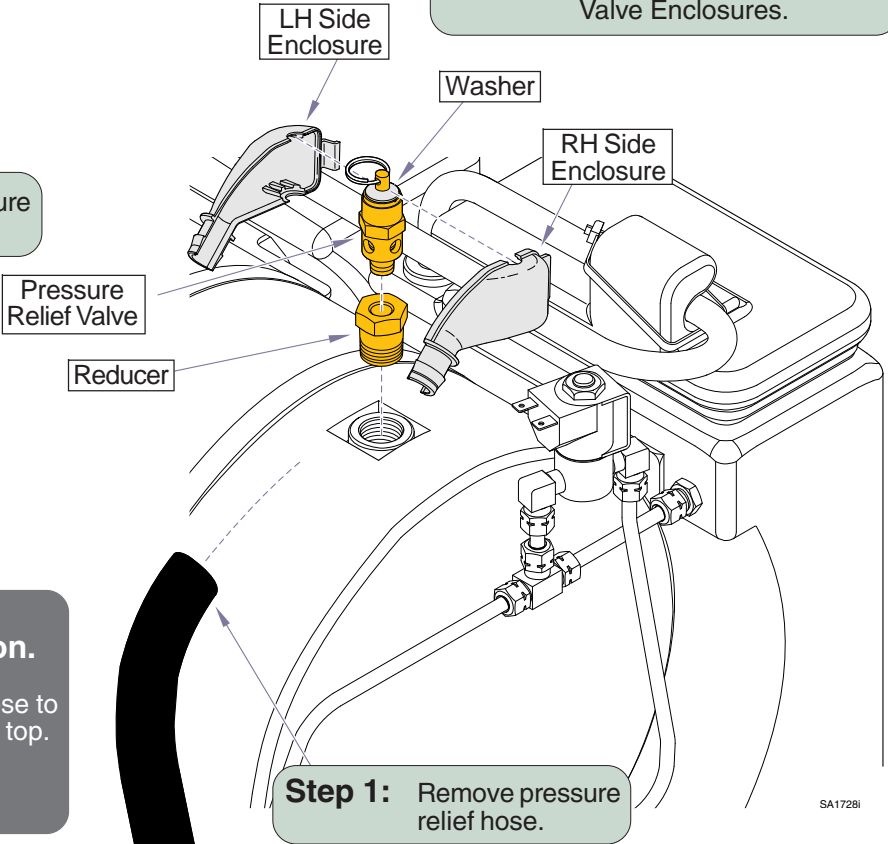

NOTE: Assure the washer is located between the top of the enclosure and the ring of the PRV.



Installation Reverse steps for Installation.

Note: If necessary, trim the PRV Hose to prevent it from touching the counter top.

- M9....13 3/4" (35 cm)
- M11..16 1/4" (41.25 cm)