Installation: Power Supply Model 153808

Applies to Models:

153808-001 (115V~)
153808-002 (230V~)

Special Tools:

none


Language of origin: English

Note

Power Supply units are available in 115V~ and 230V~ models. Make certain the one you are installing is the right one for the electrical mains supply power.

Power Supply

The Power Supply centralizes air and electrical supply power inputs and distributes that power as needed to existing operatory components.

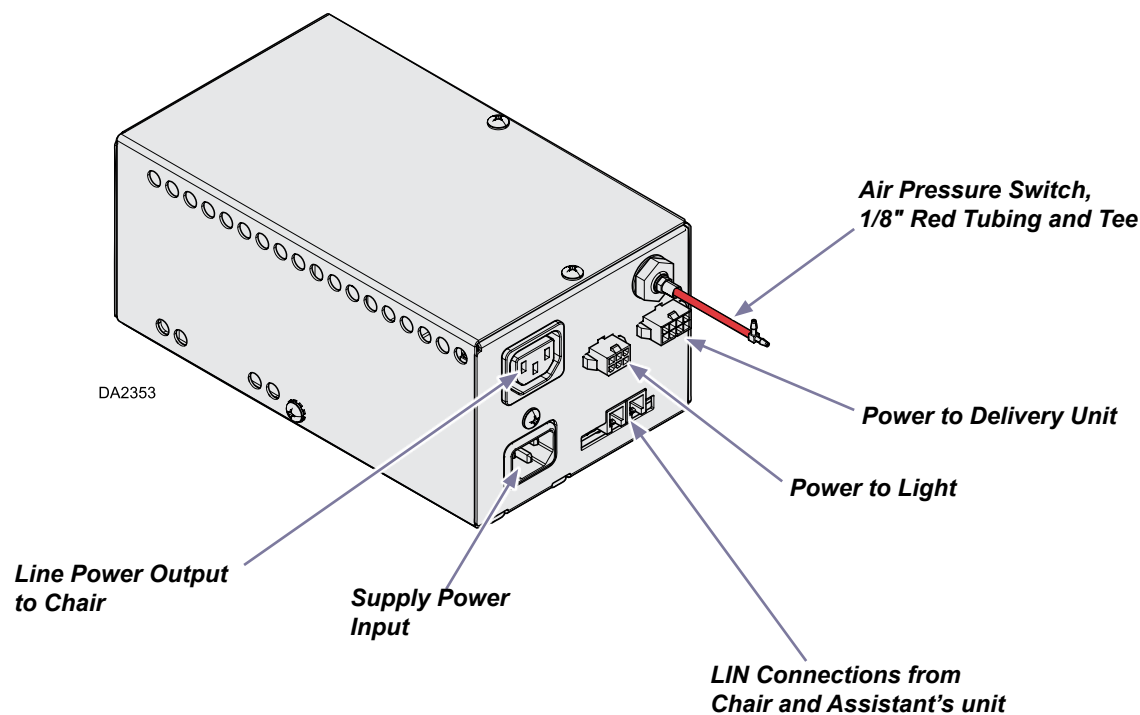Intended Use

The Power Supply is intended to be a non-user-accessible part. It is designed to deliver electricity at predetermined voltage levels (AC and/or DC) to a Midmark Dental Chair, Unit, and/or Light.

Electromagnetic Interference

Midmark dental operatory components are designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this operatory, remove the interfering device from room and / or plug product into an isolated circuit.

Transportation / Storage / Operating Conditions

Transportation / Storage Temperature:.....23°F to 100°F (-5°C to 38°C)

Relative Humidity..... 10% to 90% (non-condensing)

Atmospheric Pressure 7.2 PSI to 15.3 PSI (50 kPa to 106 kPa)

Operating Temperature Range:59°F to 95°F (15°C to 35°C)

Disposal of Equipment

At the end of this product's life, the unit, accessories and other consumable goods may be contaminated from normal use. Consult local codes and ordinances for proper disposal of this equipment and other consumable goods.

WARNING

Do not modify this equipment without authorization of the manufacturer.

Authorized EU Representative

Countries in the EU should direct all questions, incidents, and complaints to Midmark's Authorized EU representative listed below.

Midmark EMEA Ltd
Beech House
First Floor, East Wing
Ancells Business Park
Fleet
Hampshire GU51 2UN
United Kingdom

Tel: + 44 (0) 1252 360 940

Fax: + 44 (0) 1252 360 941

Calling For Service

Direct all service inquiries to your authorized Midmark dealer. When calling for service, you must provide the following information:

Model / serial number
Date of purchase
Symptom(s) of malfunction

Ordering Parts

The following information is required when ordering parts:

Serial number & model number
Part number for desired part
(Refer to Exploded Views and Parts at midmark.com)


Non-warranty parts orders may be faxed to Midmark using the Fax Order Form in the back of this manual.

For warranty parts orders, call Midmark's Technical Service Department with the required information.

Hours: 8:00 am to 5:00 p.m. EST (Monday thru Friday)

Phone: 1-800-Midmark

Specifications

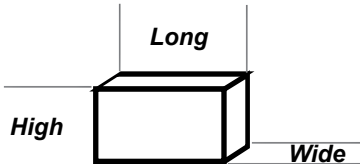Electrical Requirements:	115 V~ , 60Hz, 11A
	Fuse F1: T3.15AH, 250V~
	Fuses F2, F3, F6 and F7: T6.3AL, 250V~
	Fuses F4 and F5: T10.0AH, 250V~
	230V~ , 50 Hz, 7A
	Fuse F1: T1.6AL, 250V~
	Fuses F2, F3, F6 and F7 : T6.3AL, 250V~
	Fuses F4 and F5: T6.3AH, 250V~
Air Requirements:	Maximum 80 -100 PSI (5.52 - 6.9 bar)
Classifications:	Class I, Ordinary Equipment (IPX0), Continuous Operation
Certifications:	ES/IEC/EN 60601-1, CAN/CSA C22.2 No. 60601-1 (Safety Standards) EN/IEC 60601-1-2 (EMC Standards)
Dimensions:	8.5 in.(20.3 cm.) Long 4.0 in. (10.2 cm.) Wide 5.0 in. (12.7 cm.) High 

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture with oxygen, air, or nitrous oxide.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury if not avoided.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury if not avoided. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

NOTE

Amplifies a procedure, practice, or condition.

Consult Manuals

Type T
Fuse


Protective
Earth Ground

Tubing Connections


Always use the retainers provided to make secure tubing connections.

Step 1: Cut 1/8" red tubing between the air and water master pilot shut-off valves. Connect both ends to the Pressure Switch tee fitting from the Power Supply.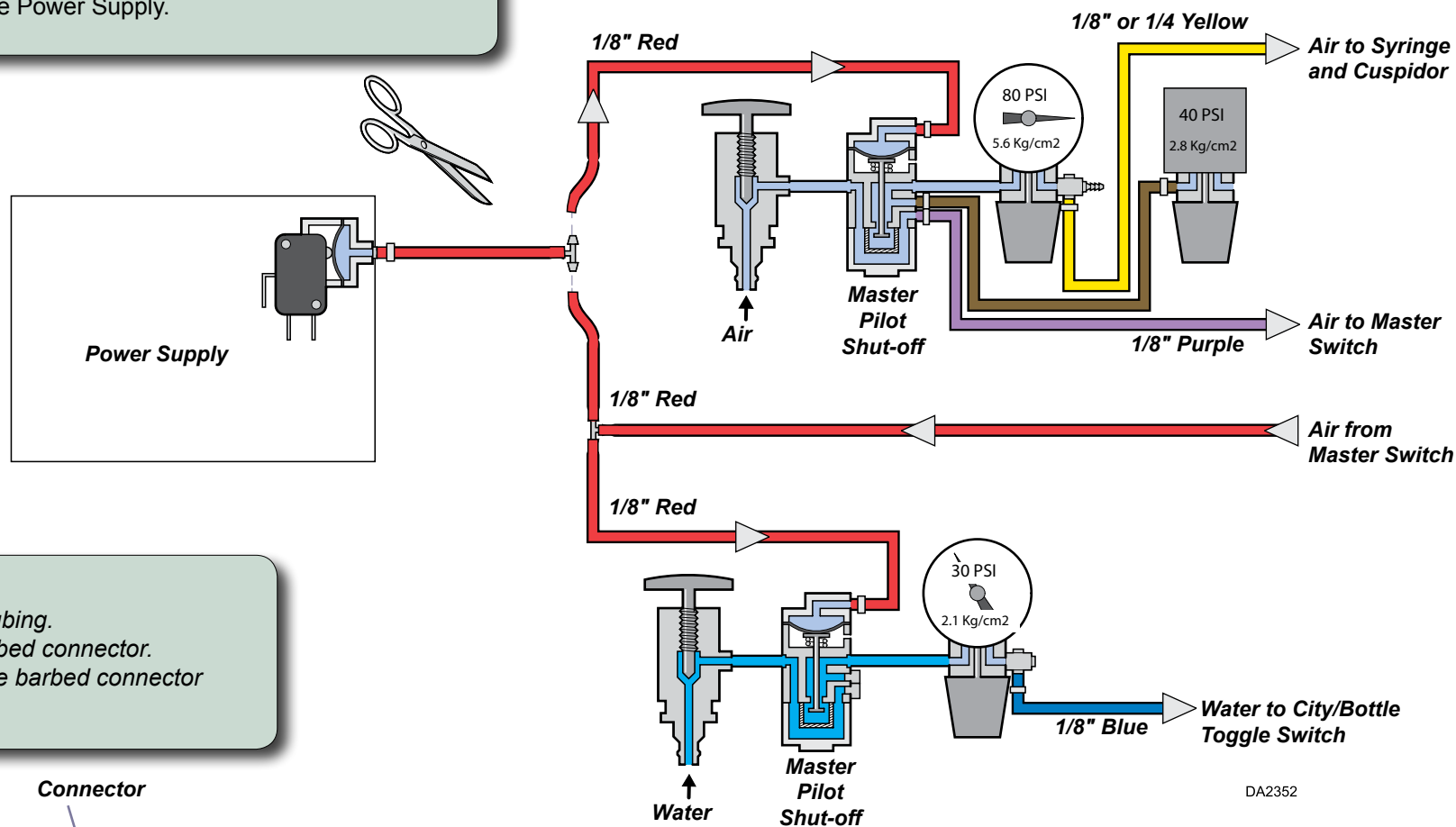

To connect tubing...

- Place retainer onto tubing.
- Push tubing onto barbed connector.
- Slide retainer over the barbed connector beneath the tubing.

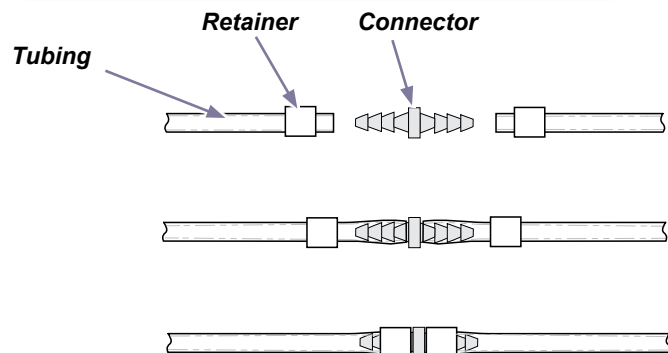DA2362

DA2352

Electrical Connections

WARNING

Electrical power must be supplied thru an electrical receptacle compliant with local codes. Install one that meets these requirements if one does not exist:

115 VAC Model: 115V~, 60 Hz, 11A

230 VAC Model: 230V~, 50HZ, 7A



Step 1: Plug Light and/or Delivery Unit power connectors (from umbilical, console or L/R) into Power Supply.

Step 2: Plug Chair power cord into Power Supply.

Step 3: Make LIN connections to the Power Supply.

Step 4: Plug supply power cord into receptacle.

Step 5: Position the Power Supply on floor, in J-box or cabinet being careful not to pull or pinch any wires.

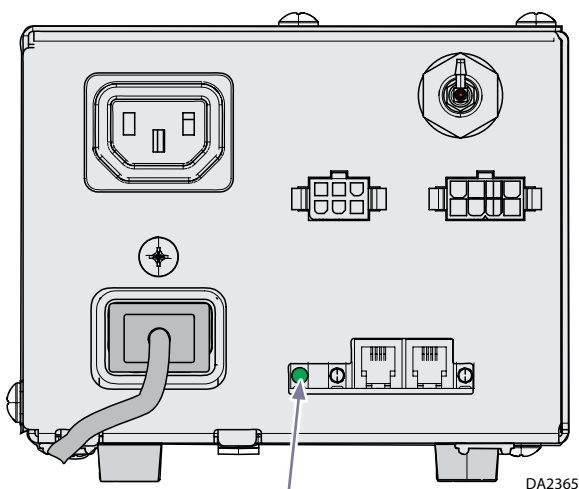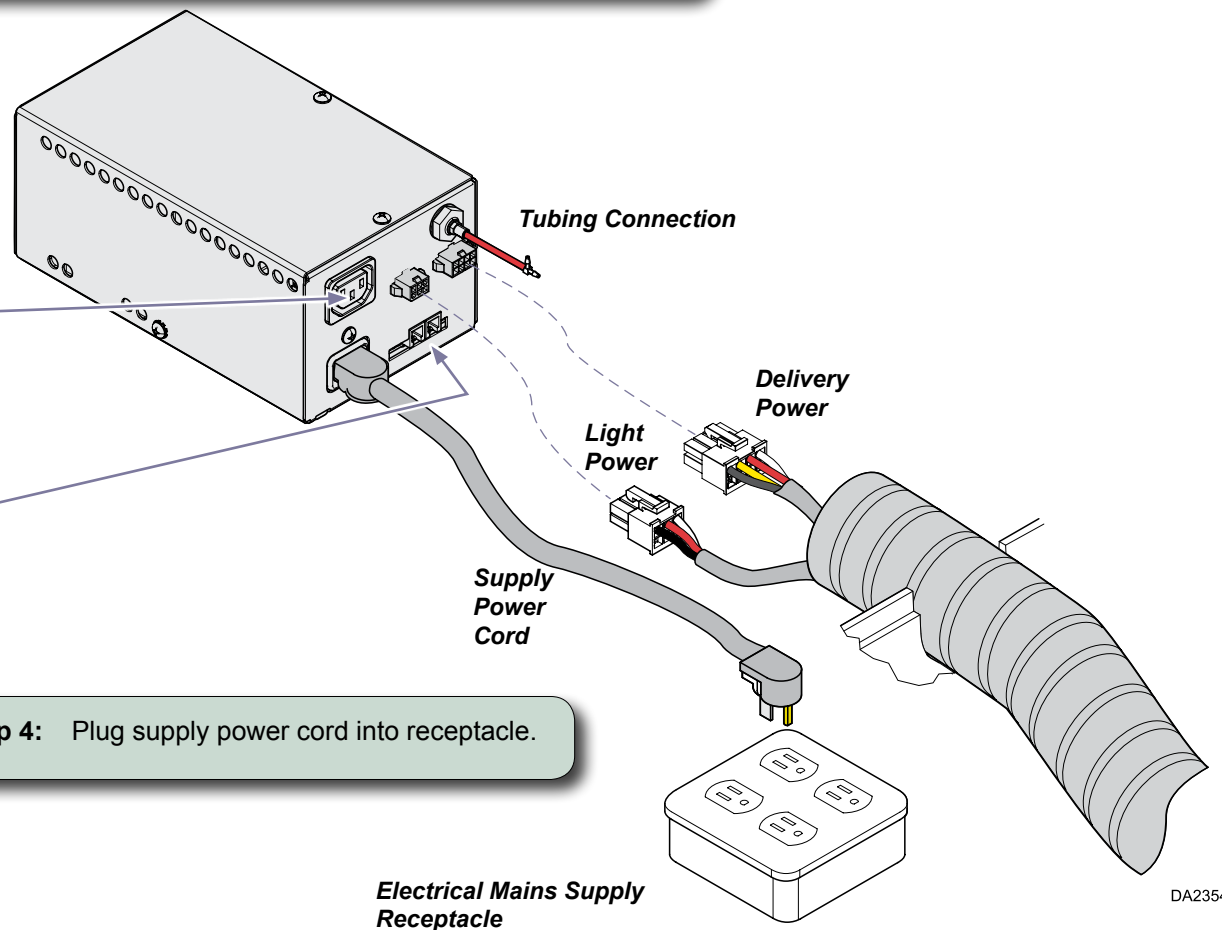Power ON LED

DA2365

DA2354

Warranty Information

SCOPE OF WARRANTY

Midmark Corporation ("Midmark") warrants to the original purchaser its Dental products and components (except for components not warranted under "Exclusions") manufactured by Midmark to be free from defects in material and workmanship under normal use and service. Midmark's obligation under this warranty is limited to the repair or replacement, at Midmark's option, of the parts or the products the defects of which are reported to Midmark within the applicable warranty period and which, upon examination by Midmark prove to be defective.

APPLICABLE WARRANTY PERIOD

The applicable warranty period, measured from the date of delivery to the original user, shall be as follows:

1. DENTAL PRODUCTS are warranted against defect in material and workmanship for the period indicated below from the time of delivery:
 - (a) Two (2) years for all products and components (except for the periods in b thru e).
 - (b) "KINK-VALVE" module carries a ten (10) year warranty.
 - (c) The original light bulb on a new light carries a one (1) year warranty.
 - (d) Accessories not manufactured by Midmark are excluded.
 - (e) Replacement parts and accessories carry a ninety (90) day warranty.
2. Oral Surgery PRODUCTS are warranted against defect in material and workmanship for a period of one (1) year from the time of delivery.
3. STERILIZERS PRODUCTS are warranted against defect in material and workmanship for a period of one (1) year from the time of delivery.
4. ULTRASONIC CLEANER PRODUCTS when used in accordance with manufacturer's instructions and under normal use, are guaranteed for two (2) years after date of shipment. Within the period guaranteed, the manufacturer will repair or replace free of charge, at its sole discretion, all parts that are defective because of material or workmanship including costs for removing or installing parts. Repairs are made on a FOB repair depot basis. Manufacturer's liability, whether based on warranty, negligence or other cause, arising out of and/or incidental to sale, use or operation of the transducer elements, or any part thereof, shall not in any case exceed the cost of repair or replacement of the defective equipment, and such repair or replacement shall be the exclusive remedy of the purchaser, and in no case shall manufacturer be responsible for any and/or all consequential or incidental damages including without limitation, and/or all consequential damages arising out of commercial losses.

5. INTEGRA™ DENTAL CASEWORK AND ARTIZAN™ DENTAL FURNITURE PRODUCTS are warranted against defect in material and workmanship for the period indicated below from the time of delivery:

(a) Five (5) years for all products and components (except for the shorter periods in (b and c), including plastic laminate top surfaces, door and drawer fronts, casters, and electrical components such as task lights and cords. (b) Three (3) years for task light ballast. (c) One year (1) for plastic laminate, solid surface countertops and accessories.

EXCLUSIONS

This warranty does not cover and Midmark shall not be liable for the following;

(1) repairs and replacements because of misuse, abuse, negligence, alteration, accident, freight damage, or tampering; (2) matching of color, grain, or texture except to commercially acceptable standards; (3) changes in color caused by natural or artificial light; (4) products which are not installed, used, and properly cleaned as required in the Midmark "Installation" and or "Installation/Operation" Manual for this applicable product; (5) products considered to be of a consumable nature; (6) accessories or parts not manufactured by Midmark; (7) specially manufactured products; (8) charges by anyone for adjustments, repairs, replacement parts installation, or other work performed upon or in connection with such products which is not expressly authorized in writing in advance by Midmark.

EXCLUSIVE REMEDY

Midmark's only obligation under this warranty is the repair or replacement of defective parts. Midmark shall not be liable for any direct, special indirect, incidental, exemplary or consequential damages or delay including, but not limited to, damages for loss of profits or loss of use.

NO AUTHORIZATION

No person or firm is authorized to create for Midmark any other obligation or liability in connection with the products.

THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

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