

Software Chip Replacement [002-1236-00]

Components:
 Software Chip..... 1
 Extraction Tool..... 1

Special Tools:
 none




Equipment Alert

Use standard precautions when handling the software chip.
 Ground yourself (by touching the metal surface of the chair),
 before removing the chip from packaging.



Caution


Unplug power cord before removing PC board cover.

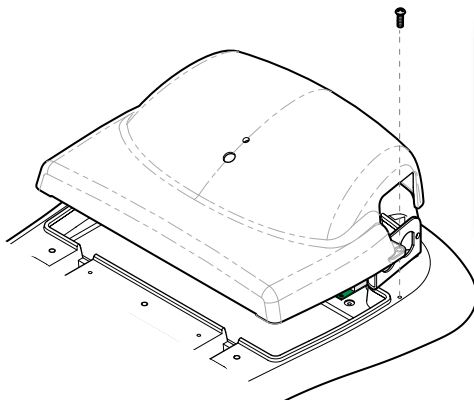

Equipment Alert

You must perform Step 2 before
 cover can be completely removed.

Step 1: Remove two screws, then
 partially separate cover.

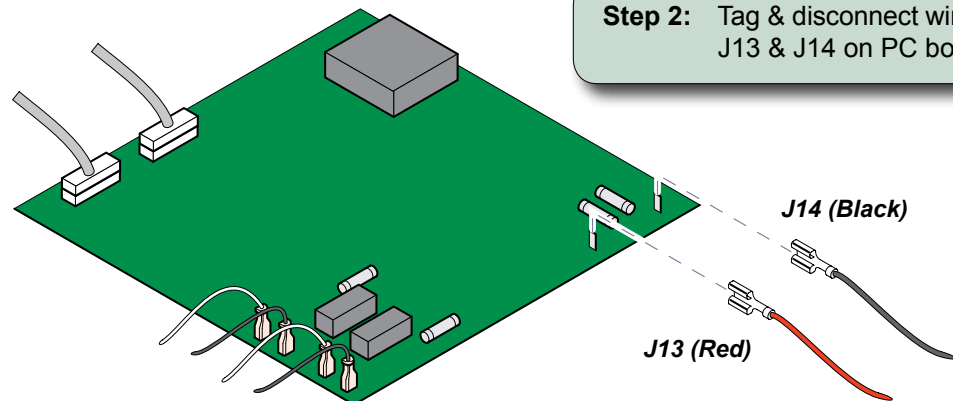

Step 2: Tag & disconnect wires from
 J13 & J14 on PC board.

MA7925i



Equipment Alert

Be sure the hooks of the extraction tool are under the software chip, not the chip holder.


Step 3: Use the extraction tool (provided) to remove the software chip.



Equipment Alert

Position the chip with the notch as shown. Incorrect orientation will cause the chair to malfunction.

Step 4: Position the new chip as shown.

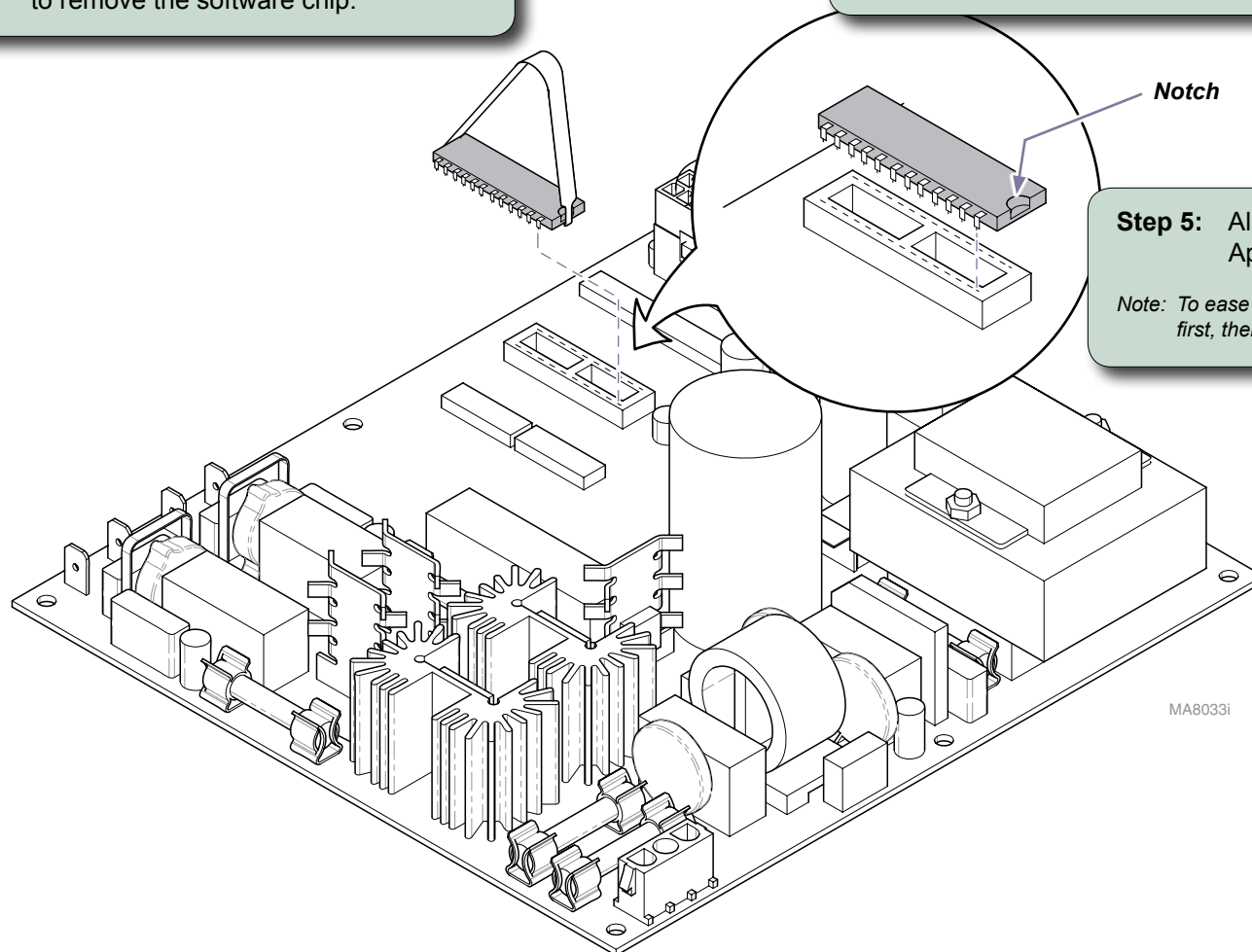

Step 5: Align the pins on the chip with the socket. Apply gentle pressure to fully seat all pins.

Note: To ease installation, insert the pins on one side of the chip first, then guide the other side into the socket.



Because we care.