

Procenter Delivery Lid Repair Kit [002-0711-00]

Components:

Delivery Lid with 3 Standoffs	1
L & R Bracket set with 4 screws	1


Step 1: Replace Lid on unit.

- Turn the Master Switch OFF.
- Open the Delivery unit (Lid release button on side).
- Remove 3 screws fastening the lid sub-panel to the standoffs in lid.
- Remove 6 screws from hinges.
- Align new lid and insert 6 screws into hinges, loosely.
- Lower lid sub-panel slots onto center hinge screws.
- Install 3 screws thru lid sub-panel into standoffs, then tighten all hinge screws to a torque of 13 in. lbs..

Note: Reuse all screws from existing lid.

Step 2: Install Brackets.

- Rotate lid and sub-panel assembly up 90°.
- Place a clean cloth over the unit to keep debris out of the tubing and controls.
- Measure and mark drill location points.
- Drill mounting holes for L & R Brackets.
- Mount the brackets using the four screws provided in kit.
- Close lid, turn on the Master Switch and test handpieces to verify functionality.

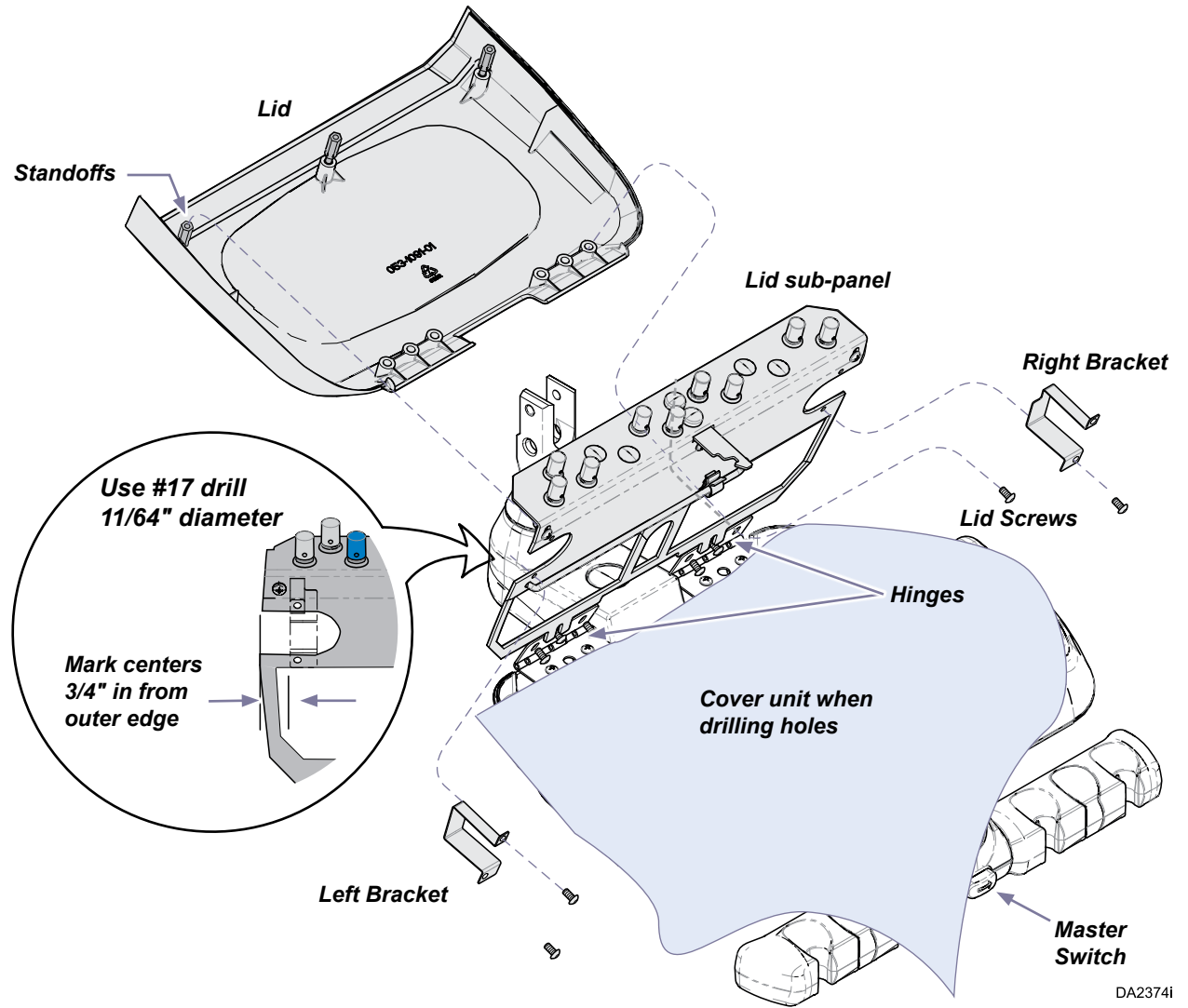

DA2374i