



Monitor Back Cover [Kit #002-1332-00]


Components:
Cover 1
Bracket 1
3/8" Pan head Phillips
Screws 4

No Special Tools


Note

Used with serial number before V1144327.

Step 2: Remove and **KEEP** the two screws* holding the mounting plate to the monitor back.

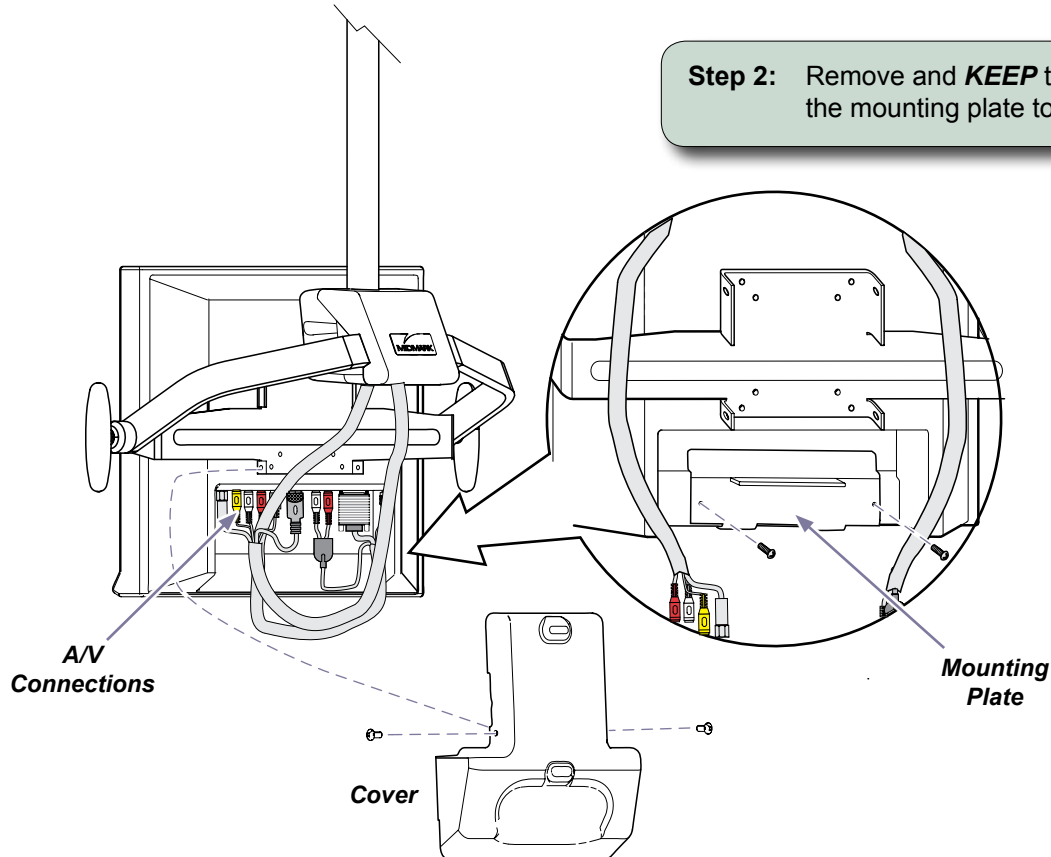

**Cover
Bracket**

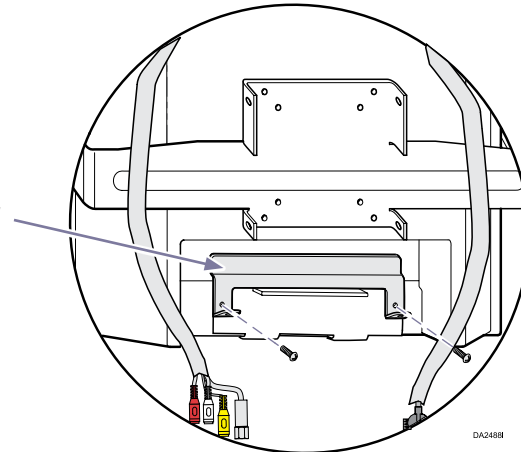


Step 3: Mount the new cover bracket to the mounting plate using the same two screws.

Step 1: Remove monitor cover (two screws) and unplug all the A/V connections to the monitor so you can access the mounting plate on the back of the monitor.

* Note

If a replacement screw is required, use an M3 x 14mm long screw.

Step 1: Plug all the A/V connections back into the monitor, looping the harnesses below the connections as shown here.

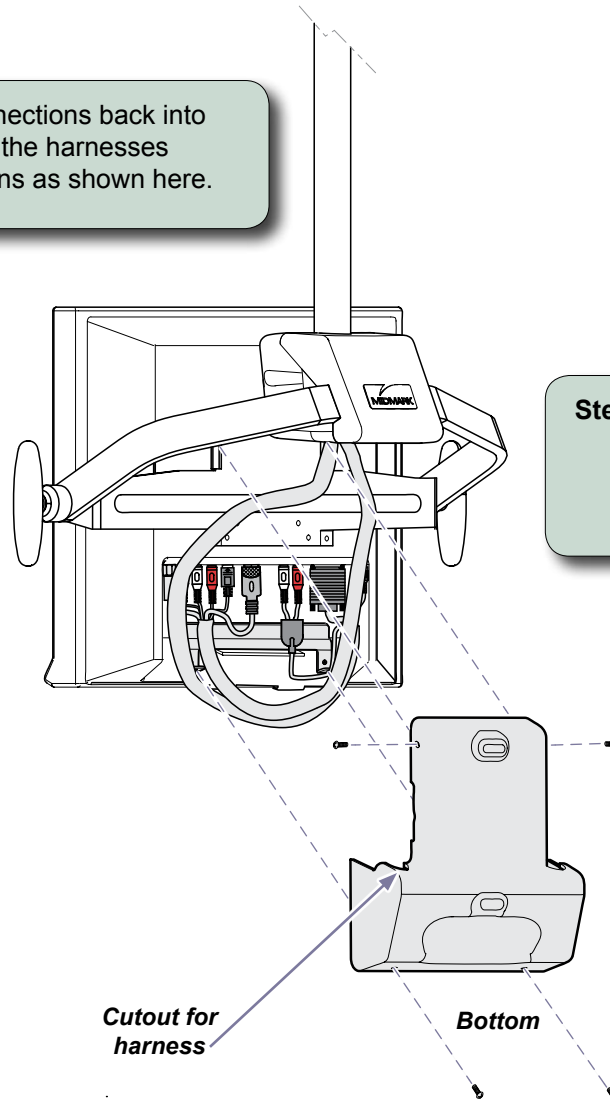

Step 2: Install the new Cover, using the screws supplied in the kit. Starting at the bottom, cover harnesses and install 2 screws into the previously installed Cover Bracket.

Step 3: Align harnesses with cutouts in Cover and press in gently to install top 2 screws.

Note: Be careful not to pinch harnesses or restrict their ability to go thru the monitor's full range of motion when installing the cover.

DA2493I