

## Note

Earlier built units will take the same solenoid as present day built units, but earlier units did not have a ground wire. Install ground wire for new solenoid on all units. For SN breaks refer to Midmark Compressor Service Manual (10540900) on [www.Midmark.com](http://www.Midmark.com)

## To install Solenoid...

- A) Turn compressor power off.
- B) Remove electrical box cover.
- C) Replace solenoid.
- D) Install ground wire on ground stud in electrical box.
- E) Install new wire diagram label over old one inside cover of electrical box.

Note: Refer to new wire diagram label.

