253

Ritter®

LED Examination Light

[English](#)

[Español](#)

[Français](#)

MA8209p

User Guide

Style P TP202 20-42-FO-00014 Rev A1 C2169 003-2181-99 Rev AA6 (8/20/25)

Product Information


Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

**Model / Serial
Number Location**

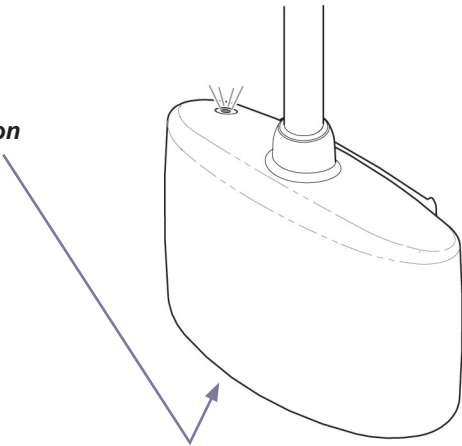MA8173i

Table of Contents

Important Information

- [Safety Symbols](#)..... 4
- [Transportation & Storage Conditions](#)..... 4
- [Before First Use](#)..... 5
- [Intended Use](#)..... 5
- [Electromagnetic Interference](#)..... 5
- [Disposal of Equipment](#)..... 5

Operation

- [Power ON / OFF](#)..... 6
- [Automatic Shutoff Feature](#)..... 7
- [Light Pattern Adjustment](#) 8

Maintenance

- [Fuse Replacement](#) 9
- [Troubleshooting](#) 10
- [Calling for Service](#) 10
- [Cleaning](#) 10
- [Periodic Maintenance](#) 10

Specifications

- [Specifications Chart](#)..... 11
- [Model Identification / Compliance Chart](#)..... 12

Warranty Information

- [Limited Warranty](#) 13

Important Information - Safety Symbols

These symbols may appear on your equipment and/or in the manuals.

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Proper Shipping Orientation

Consult User Guide

Humidity Limit

Fragile

Fuse Rating Specification

Pressure Limit

Keep Dry

Power ON / OFF

Temperature Limit

Type B Applied Part

Bright Light
(Do not look directly into light)

Catalogue Number

Manufacturer

Serial Number


Transportation / Storage Conditions

Ambient Temperature Range.....	-22°F to 140°F (-30°C to +60°C)
Relative Humidity.....	10% to 90% (non-condensing)
Atmospheric Pressure	500 hPa to 1060 hPa

Before First Use...

Remove protective film from lens...
Pull tab to peel protective film from lens.

Protective Film

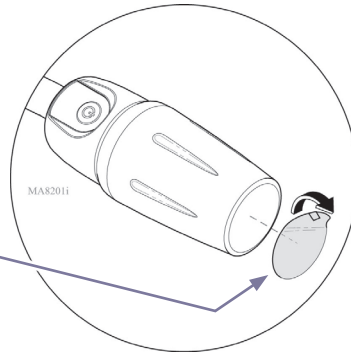Intended Use

The Ritter 253 provides focusable illumination for general examinations and procedures performed by trained professionals in a healthcare environment.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room
- Plug light into isolated circuit
- Increase separation between light and interfering device
- Contact Midmark if interference persists

Disposal of Equipment

At the end of product life, the light, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

Operation

Power ON / OFF

To connect power to the light...


- A) Connect power cord to light housing.
- B) Plug cord into properly rated receptacle.

Note: When power is connected, the lighthouse will flash several times, and the indicator light will illuminate.


To turn the light ON / OFF...

Press the power button.

Indicator Light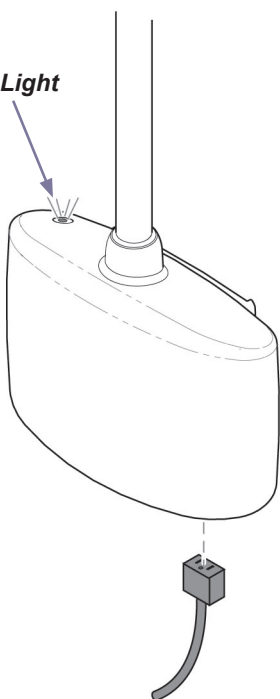

Power Button

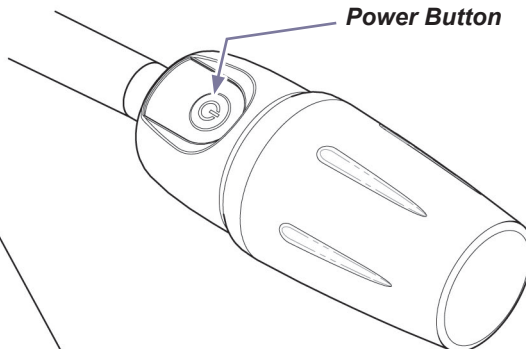MA8171i

Note

*The light has an automatic shutoff feature that turns the light OFF after two hours. Refer to **Automatic Shutoff Feature** section for more information.*

Automatic Shutoff Feature

The Automatic Shutoff Feature turns the light OFF two hours after the last power button press.


Note

Lights are shipped from the factory with this feature **disabled**.

To enable the Automatic Shutoff Feature...

- A) Press and hold the power button for 5 seconds.
- B) Observe. The lighthouse will flash **three** times indicating the feature has been enabled.


Note: To disable this feature, repeat the procedure.
The lighthouse will flash **four** times when this feature is disabled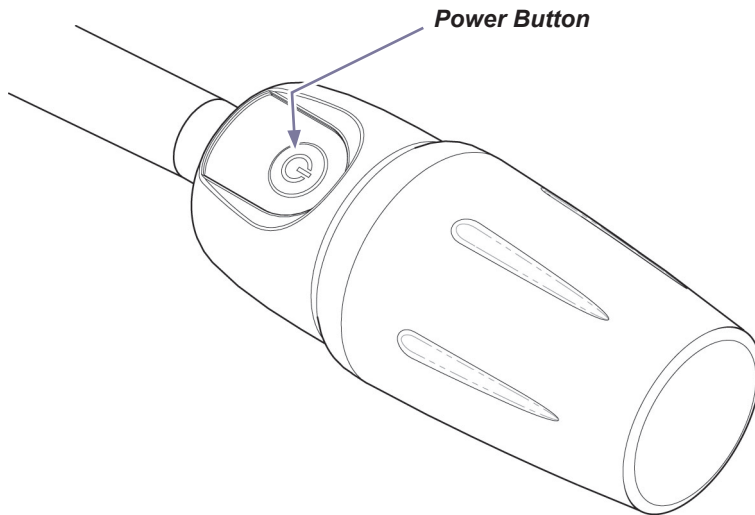

MA8170i

Light Pattern Adjustment

To adjust the size of the light pattern...
Rotate the lighthead clockwise / counterclockwise.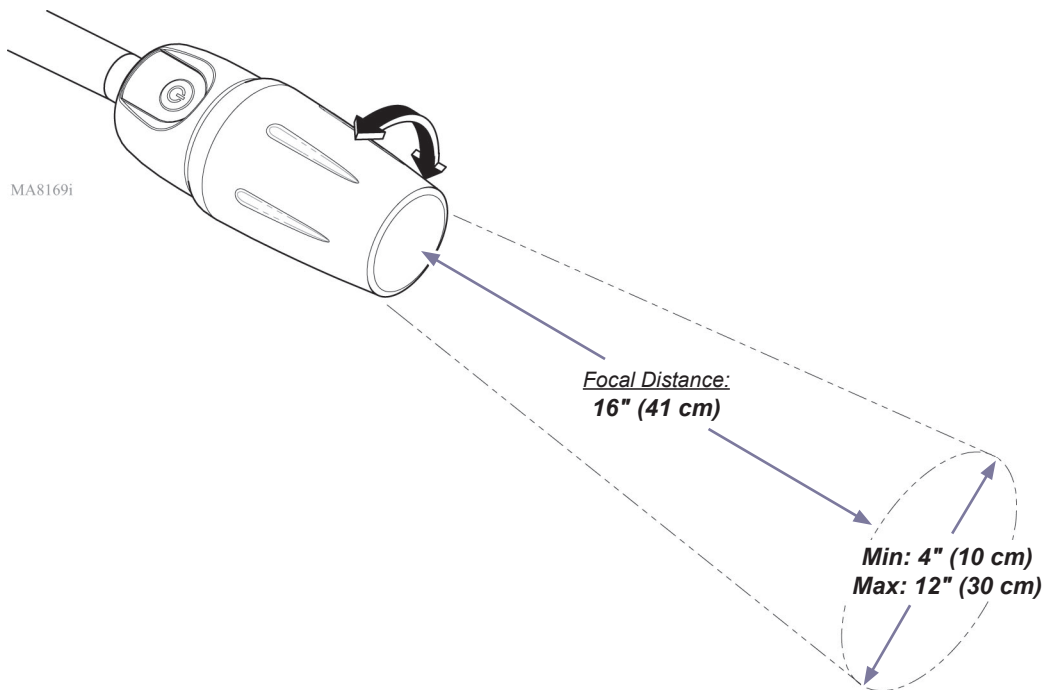


Maintenance

Fuse Replacement

To replace fuses...

- A) Disconnect power cord from light housing.
- B) Press tab and pull fuse holder out of cord inlet.
- C) Replace fuses.
- D) Push fuse holder into inlet until it locks in place.

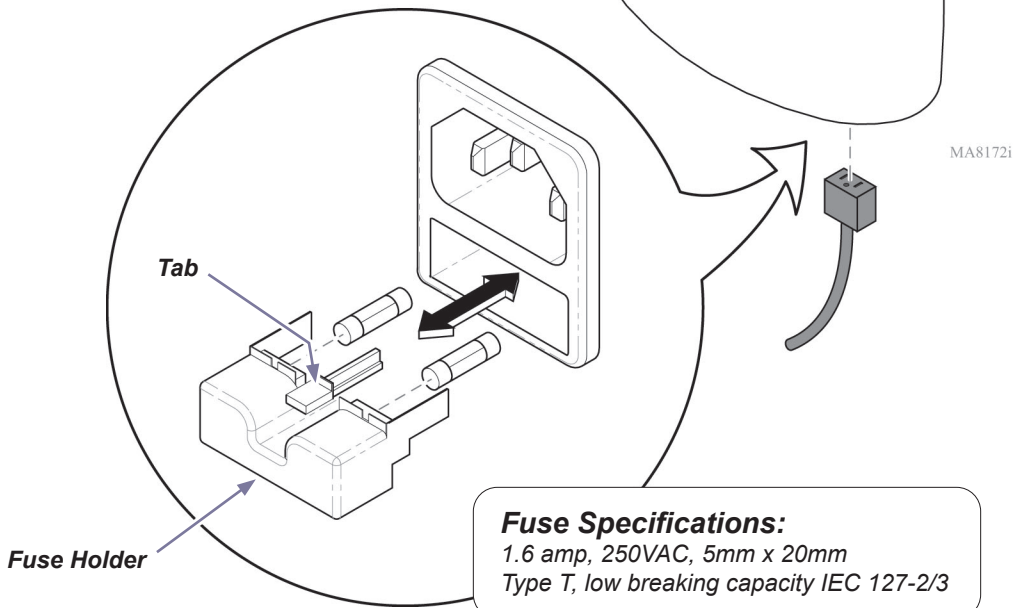Troubleshooting

Problem	Possible Cause	Correction
Light will not turn ON. (Power indicator light: OFF)	Power cord is not connected.	Secure power cord connections.
	Faulty fuse(s).	Replace fuses. (Refer to procedure in this manual)
Light will not turn ON. (Power indicator light: ON)	Lighthouse or flex arm malfunctioning.	Contact authorized Midmark service technician.
Light pattern will not adjust.	Lighthouse malfunctioning.	Contact authorized Midmark service technician.
Light turns OFF by itself.	Automatic Shutoff Feature	Disable automatic shutoff feature. (Refer to procedure in this manual)

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM
Friday 8:00 AM until 5:00 PM (ET)
midmark.com

Cleaning

Clean the painted metal and plastic surfaces using a clean soft cloth, and mild cleaner. Disinfect your product with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

Periodic Maintenance

Periodically inspect the following:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.

Have an authorized service technician inspect your equipment periodically.

Specifications

Specifications Chart	
LED	Life: approximately 50,000 hr* (<i>non-replaceable</i>) * <i>per bulb manufacturer</i>
Light Pattern	Circular
Adjustable Spot Size @ <i>Focal distance: 16" (41 cm)</i>	Diameter: 4" (10 cm) to 12" (30 cm)
Illumination @ <i>Focal distance: 16" (41 cm)</i> <i>Spot size: 4" (10 cm)</i>	Minimum 1,150 foot-candles / 12,379 lux
Color Temperature	4,500K
Power Cord Length	8 ft. (2.5 m)
Flex Arm Reach <i>[from power source enclosure]</i>	54.5" (138 cm)
Flex Arm Rotation <i>[at power source enclosure]</i>	140°
Electrical Rating	115 VAC, 50/60 Hz, 100mA
	230 VAC, 50/60 Hz, 60mA
Fuses	1.6 amp, 250V, 5mm x 20mm, Type T low breaking capacity IEC 127-2/3
Regulatory Compliance	UL 60601-1 Rev. 26-Apr-06 CAN/CSA C22.2 No. 60601-1:08 Canadian Medical Device Regulations 50R/98-282 NFPA99 (<i>applicable to product requirements</i>)
Classification	Class 1, Type B Applied Part

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture with oxygen, air, or nitrous oxide.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Model Identification / Compliance Chart

Model	Description	Complies To:				Electrical Ratings:		
		UL 60601-1 Rev 26-Apr-06	CAN / CSA C22.2, #60601.1:08	Canadian Medical Device Regulations 50R/98-282	NFPA 99	VAC +/- 10%	Amps	Cycles (Hz)
253-001	LED Exam Light only	•	•	•	•	115	100mA	50/60
253-002	Includes 253-001 and 9A454001 (Spline Mount)	-	-	-	-	230	60mA	50/60
253-003	Includes 253-001 and 9A455001 (Extended Spline Mount)	-	-	-	-	-	-	-
253-005	Includes 253-001 and 9A457001 (Base Rail Mount)	-	-	-	-	-	-	-
253-006	Includes 253-001 and 9A458001 (Caster Base)	-	-	-	-	-	-	-
253-007	Includes 253-001 and 9A474001 (Wall Mount)	-	-	-	-	-	-	-
253-011	LED Exam Light only	•	•	•	•	115	100mA	50/60
253-012	Includes 253-011 and 9A619001 (Standard Spline Mount)	-	-	-	-	230	60mA	50/60
253-013	Includes 253-011 and 9A620001 (Extended Spline Mount)	-	-	-	-	-	-	-
253-015	Includes 253-011 and 9A622001 (Base Rail Mount)	-	-	-	-	-	-	-
253-016	Includes 253-011 and 9A628001 (Caster Base)	-	-	-	-	-	-	-
253-017	Includes 253-011 and 9A623001 (Wall Mount)	-	-	-	-	-	-	-

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Midmark Corporation