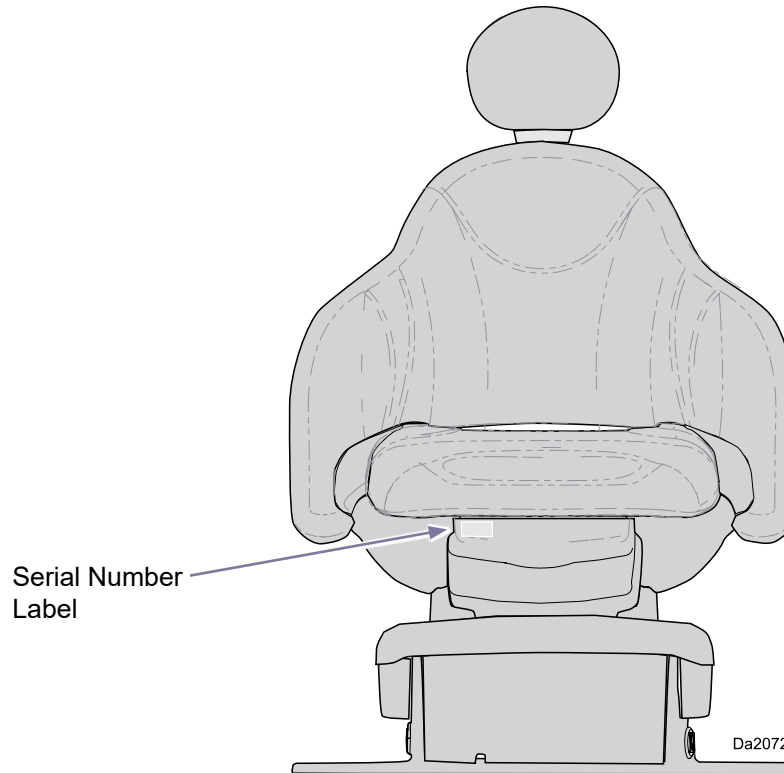
Elevance[®] Dental Chair

[English](#)
[Español](#)
[Français](#)

Model: 153810

Compliance Information:

Intended Use	2
Electromagnetic Interference	2
Transportation / Storage / Operating ...	2
Disposal of Equipment	2
Authorized EU Representative	2
FCC Compliance Statement	2
Calling for Service	2
Symbols	3

Product Features:

Power ON Light	4
Collision Protection System	4
Brake Release Pedals	4
Chair Controls	5
Wireless Remote	5
Footpad Control	5
Unit Control Pads	5

Operation and Adjustments:

Back Up/Down	6
Chair Up/Down	6
Programming Chair Positions	7
Adjusting the Headrest	8
Rotating the Chair	9
Chair Lockout Feature	9
Massage and Heat (optional)	10
Hand Control for Massage and Heat ..	11

Cleaning:

Barriers	12
Cleaning Chair Upholstery	12
Cleaning Procedures	12

Product Specifications	13
Elevance Chair Warnings	13
EMC - Manufacturer's Declaration and Guidance	14
Warranty Information	17

For installation information go to:
[Technical Library at Midmark.com](#)

Compliance Information

Intended Use

The Chair is intended for use by professional dental practitioners in providing treatment to dental patients in a dental operator. The system is designed to properly position the patient for said treatment.

Electromagnetic Interference

Midmark dental operator components are designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this operator, remove the interfering device from room and / or plug product into an isolated circuit.

Transportation / Storage / Operating Conditions

Transportation / Storage Temperature:.....23°F to 100°F (-5°C to 38°C)

Relative Humidity:..... 10% to 90% (non-condensing)

Atmospheric Pressure: 7.2 PSI to 15.3 PSI (50 kPa to 106 kPa)

Operating Temperature Range:59°F to 95°F (15°C to 35°C)

Disposal of Equipment

At the end of this product's life, the unit, accessories and other consumable goods may be contaminated from normal use. Consult local codes and ordinances for proper disposal of this equipment and other consumable goods.

Authorized EU Representative

Countries in the EU should direct all questions, incidents, and complaints to Midmark's Authorized EU representative listed below.

Newmed Srl.
Via Lenin 79/A
Quattro Castella (RE), 42020, Italy

Tel: +39.0522.875.166
Fax: +39.0522.243.096

FCC Compliance Statement

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

Any modifications to this product may violate the rules of the Federal Communications Commission and make operation of the product unlawful.

Calling For Service

Direct all service inquiries to your authorized Midmark dealer. When calling for service, you must provide the following information:

Model / serial number
Date of purchase
Symptom(s) of malfunction

Symbols

These symbols may appear on your equipment and/or in the manuals. Warnings and cautions are provided in the manuals where applicable

WARNING
Indicates a potentially hazardous situation which could result in serious injury if not avoided.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury if not avoided. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note

Amplifies a procedure, practice, or condition.

Refer to instruction manual/booklet

Refer to instruction manual/booklet

Duty Cycle

Non-ionizing Electromagnetic Radiation

Catalogue Number

Serial Number

Manufacturer

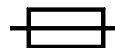
SerenEscape™ Heat and Massage back upholstery is a Type BF, Applied Part.
Do not penetrate upholstery with sharp objects, such as needles, knives, etc.

Maximum Patient Weight

Type B, Applied Part

Type T Fuse

Protective Earth Ground

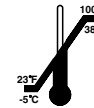
Fragile

Proper Shipping Orientation

Relative Humidity


Transportation & Storage Temperature

Atmospheric pressure

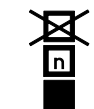Keep Dry


Maximum stacking height (Refer to “n” number on package.)

Transportation Orientation

POWER ON Light

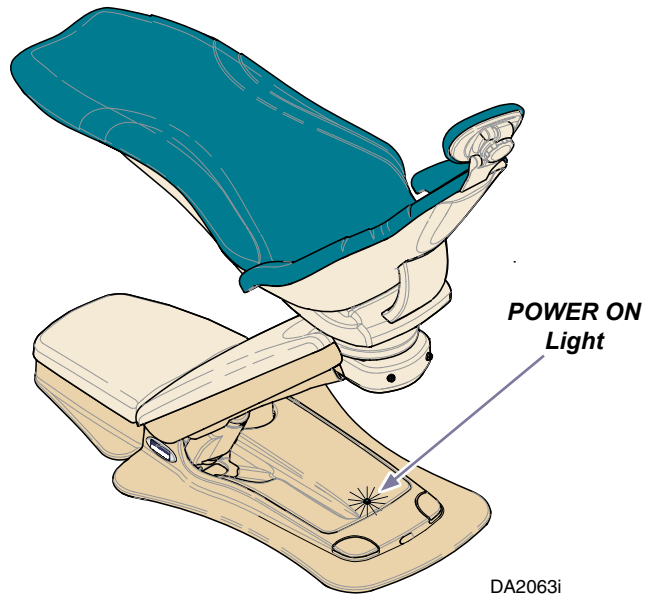The chair has a POWER ON light that illuminates when electrical power is supplied to the chair.

WARNING

During transportation, the chair must be at its lowest height and all attachments secured in their lowest and most central position possible. Failure to comply may result in injury and/or damage to equipment.

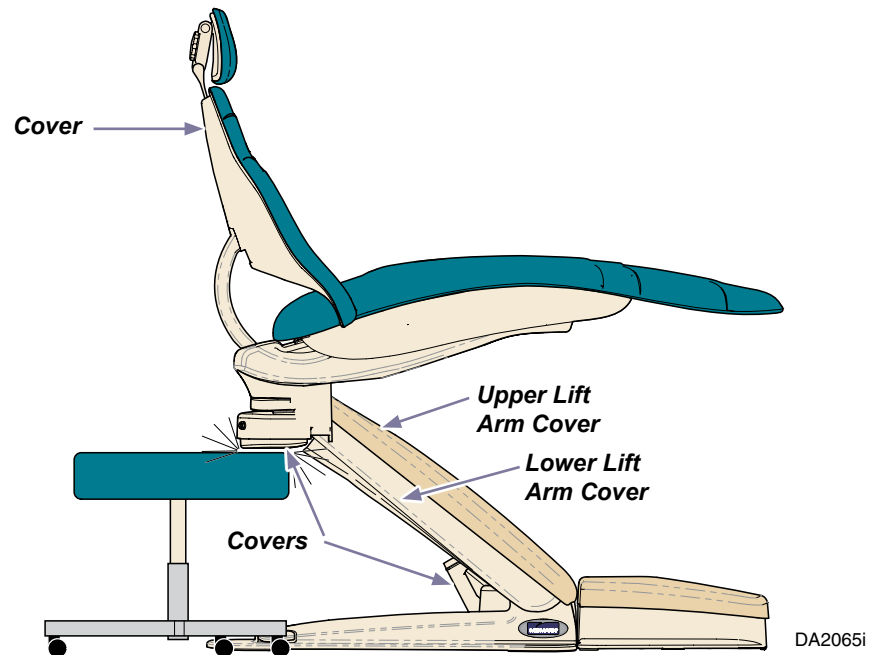
Equipment Alert

Tap any one of the lift arm covers 3 times to raise the chair to the pre-programmed "home" position, a convenient height for installation or patient access.

The chair beeps as it moves and stops beeping when it reaches the home position.

Collision Protection System

Dental Chairs are equipped with a Collision Protection System, designed to prevent damage to personnel or equipment. If a chair cover hits an object as it is lowering, it stops lowering and raises slightly to disengage contact; The chair also beeps until the control button is released. Normal operation can then be resumed.

Chair Controls

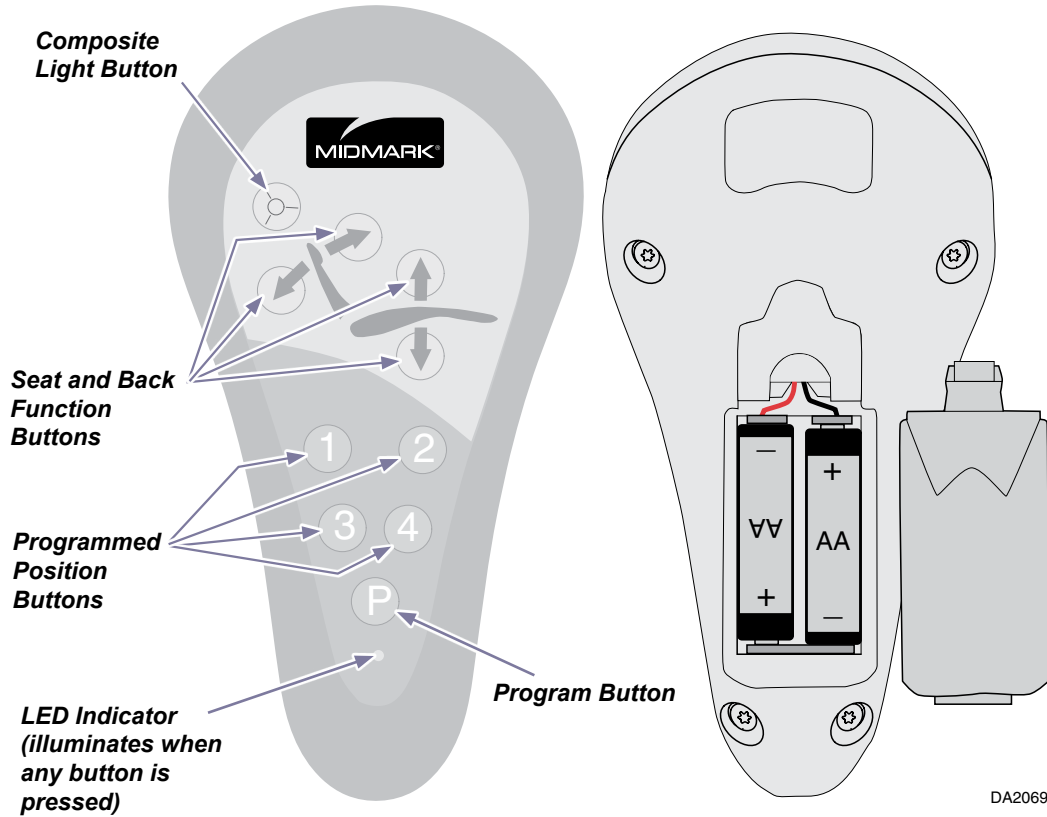
Control devices are available to quickly and accurately position the dental chair. All control devices are membrane covered, keypads with easy to understand graphics.

All control devices invoke the same chair movements.

Arrow buttons on all controls activate the Seat and Back Up and Down functions.


Numbered buttons on all controls are programmable position buttons

Wireless Remote Control


Footpad Control

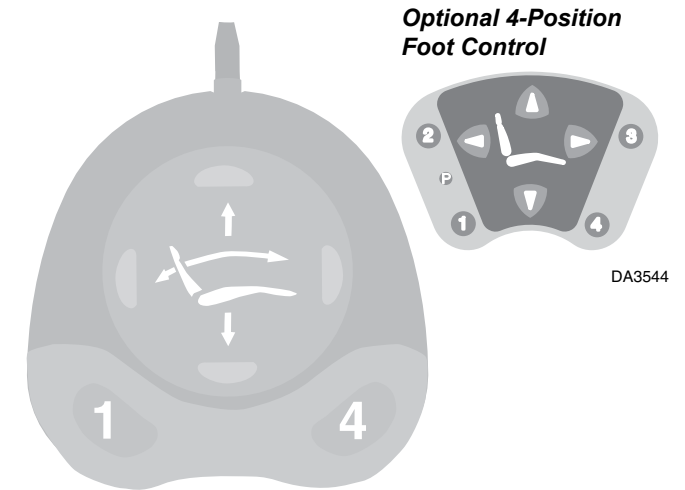Connects to the Chair by a cord.

Unit Control Pads

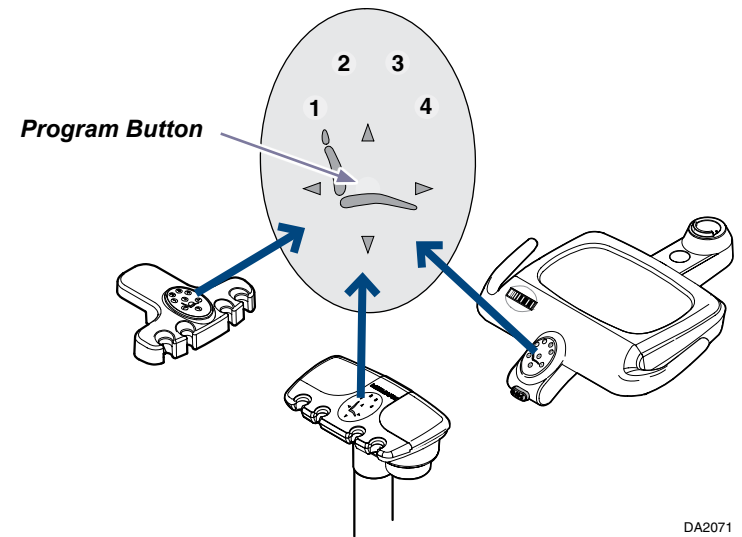Control pads are also found on Doctor's and Assistant's units

Operation and Adjustments

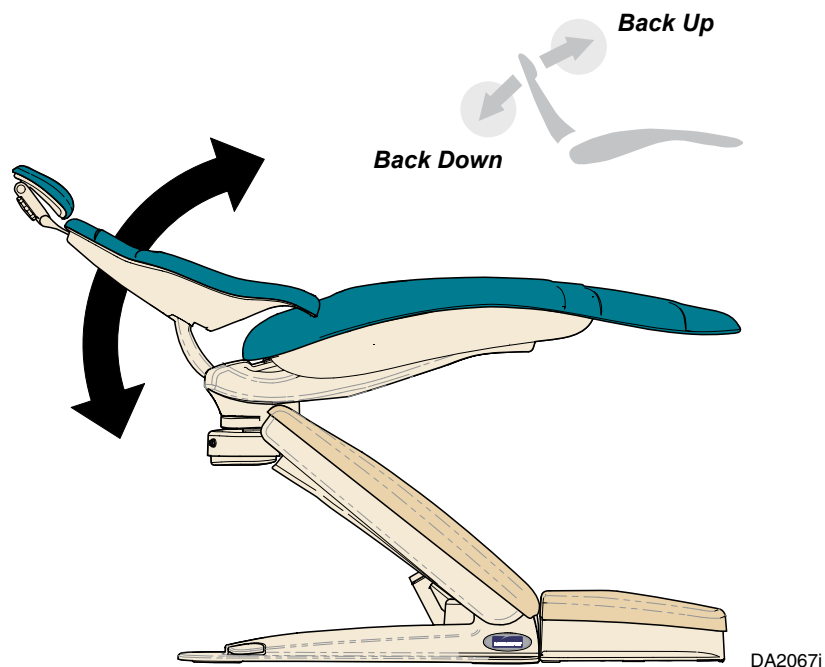
WARNING

Ensure patient is in a seated position while chair is moving.

Back Up/Down

To Raise or Lower the Back....

Press and hold the desired Back Up or Back Down button on any control device. The back stops moving when you release the button or it reaches the limit of its range of motion.


Equipment Alert

If pressing buttons on the wireless remote does not invoke any chair movements, replace the batteries. If new batteries do not remedy the situation, reference the Elevance Chair Service manual troubleshooting.

Chair Up/Down

To Raise or Lower the Chair....

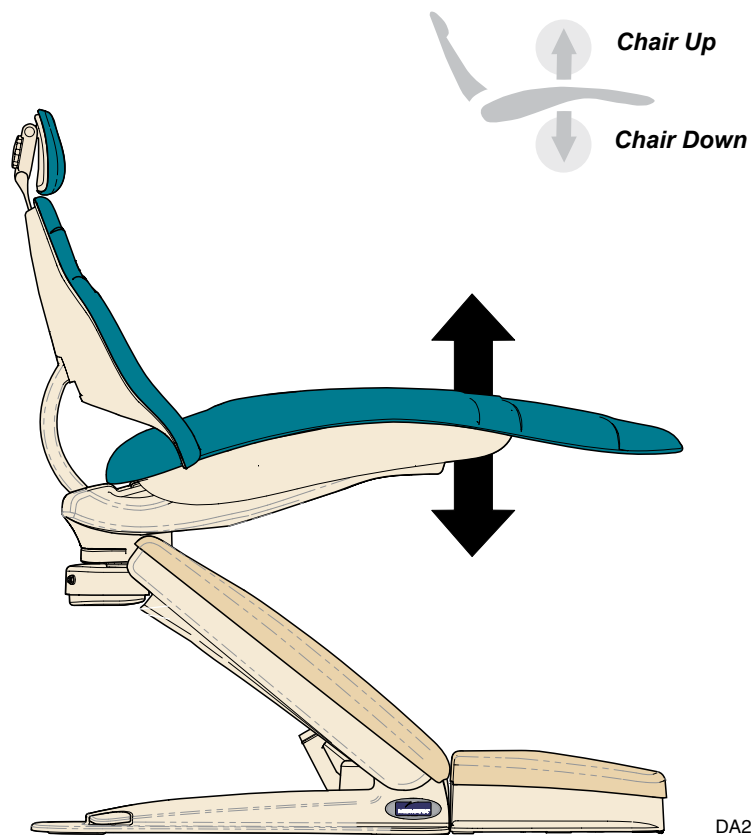Press and hold the desired Chair Up or Chair Down button on any control device. The chair stops moving when you release the button or it reaches the limit of its range of motion.

Programming Chair Positions

To program a button to a specified chair position....

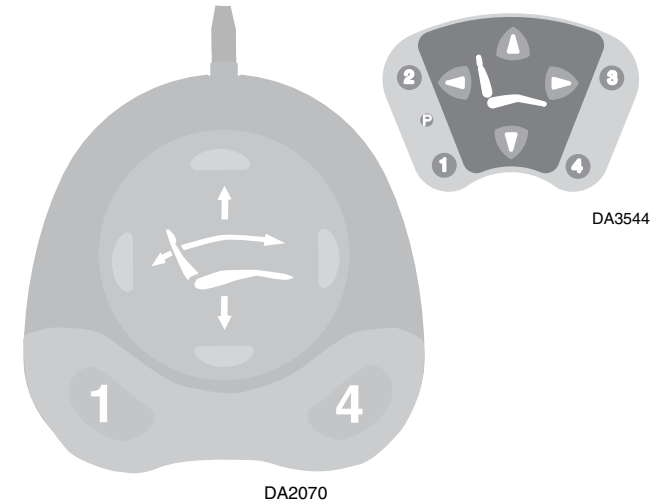
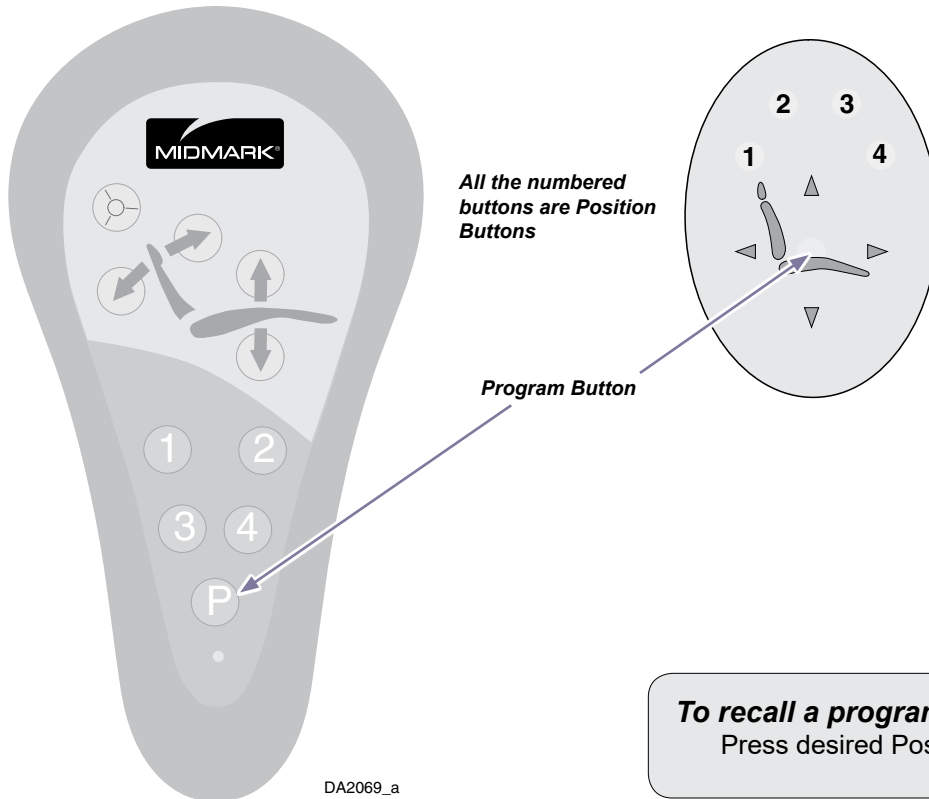
- A) Use the arrow buttons to move the chair to desired position.
- B) Press the Program Button. You will hear a single beep to indicate you are in the program mode*.
- C) Press the desired Position Button (1, 2, 3, or 4) to set the programmed position to that button. You will hear three beeps to indicate the button is programmed.

Alternate/Foot Control Programming Method: With the chair in the desired position, you can just press and hold a position button 2 seconds. The chair will beep three times to indicate the button is programmed.

* The Control device gives you 3 seconds to press (program) a Position button after you are in the Program mode. After 3 seconds the control returns to the normal operating mode.

Note

The chair can be stopped any time during a programmed positioning sequence by pressing any other button on the control device.

Special Programming Features

The number 4 Position Button is capable of being programmed to invoke any one of these three different functions.

- A) The Cuspidor Return Function.
- B) The Return to the Last Position Function.
- C) Function the same as any other Position button.

Which of these functions the 4 button invokes is dependant on a dipswitch setting on the PC Board and should only be changed by a service technician.

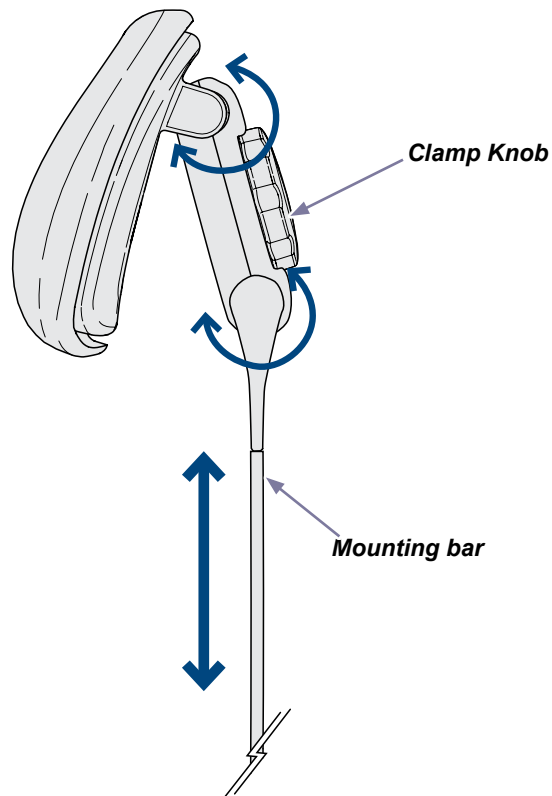
To recall a programmed position....

Press desired Position Button (1, 2, 3, or 4).

Adjusting the Headrest


To adjust the headrest

- A) Loosen (turn counterclockwise) the Clamp Knob and rotate the headrest at the two pivot points to achieve the desired position..
- B) Tighten (turn clockwise) the Clamp Knob to lock the headrest in the desired position.
- C) Pull up or push down on the headrest, sliding the mounting bar to the desired height.

Using the optional magnetic headrest

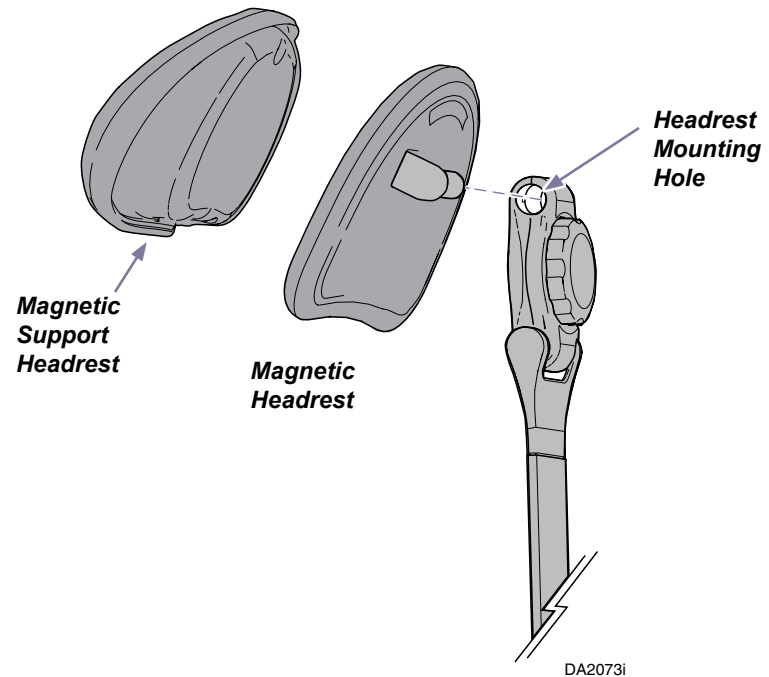- A) Loosen the Clamp Knob and slide the standard headrest out of the mounting hole on the top of the headrest mount..
- B) Insert the magnetic headrest into the mounting hole, affix the magnetic support headrest to it if desired, and adjust to the desired position before tightening the Clamp Knob.
- C) Pull up or push down on the headrest assembly, sliding the mounting bar to the desired height.


DA2073i

Rotating The Chair


To rotate the chair

Step on a brake release pedal and hold it down to dis-engage the rotation brake pin. As long as a brake release pedal is depressed you can rotate the chair to the desired position. Once the brake release pedal is no longer depressed, the rotation brake pin will engage in the closest slot location.

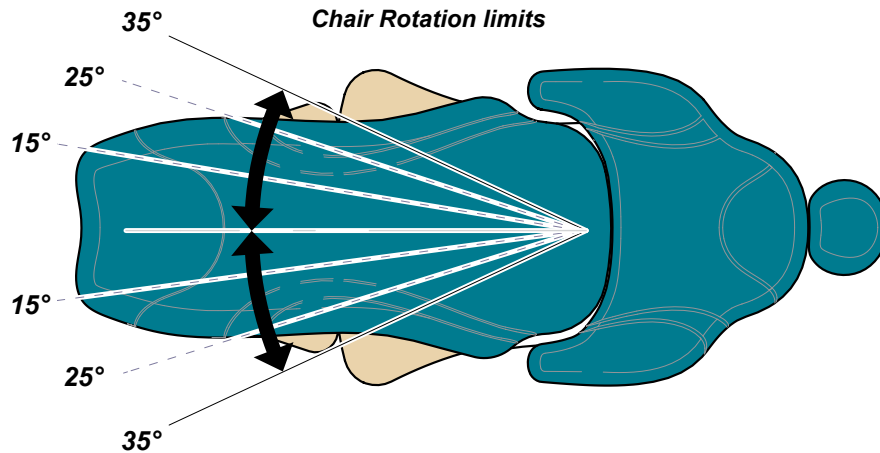Note: Do not attempt to rotate the chair prior to depressing the brake release pedal. The rotation brake pin must be dis-engaged, prior to rotating chair. If not done correctly the pin could stay engaged and the chair will not rotate.

Rotation limits differ for different Chair Models		
Model	° of rotation from center line	
Chair Only	35° Left and 35° Right Rotation Lock Positions also located at 15° and 25°	shown
L/R Chair	15° Left and 15° Right	not shown
Console Chair		

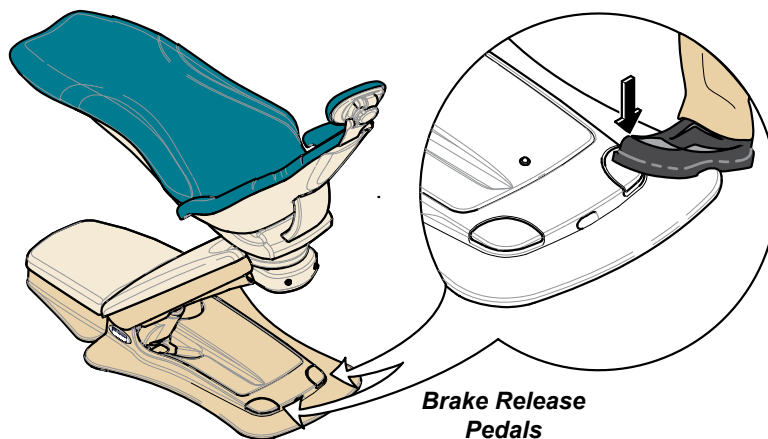Chair Lockout Feature

To prevent unintentional chair movements or rotation a Chair Lockout feature is provided. Simply press either of the brake release pedals 5 times in succession to activate the Chair Lockout feature. The chair beeps 5 times to indicate it is locked. (Buttons on control devices and the brake release pedals no longer work - they are locked out.)

To release the Chair Lockout feature simply press either of the brake release pedals 5 times in succession. The chair again beeps 5 times to indicate it is **not locked**. (Buttons on control devices and the brake release pedals will function normally again.)

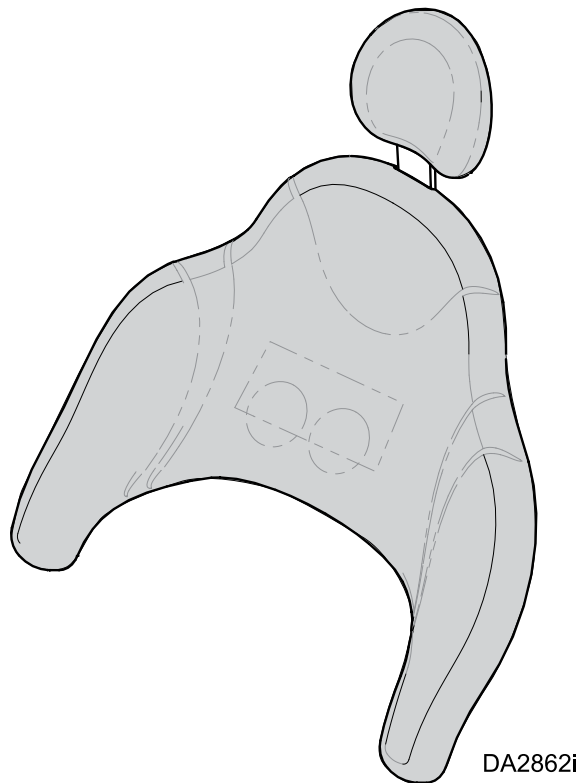
Massage and Heat (optional)

Caution

- Ensure that the surface of the SerenEscape™ Back upholstery is free from mechanical damage prior to each application.
- Do not penetrate SerenEscape™ Back upholstery with sharp objects, such as needles, knives etc.
- Do not use the SerenEscape™ feature in conjunction with high frequency electrosurgical equipment, such as hyfrecators, etc.
- The SerenEscape™ Back upholstery heats from 20°C (68°F) to a range of 85 to 90° F (29-32°C) in approximately 10 minutes.

Chair Back


The SerenEscape™ Heat / Massage components are embedded in the Back upholstery and are operated by a separate hand control.

Heat or Massage can be used separately or together, at the same time.

All buttons on the hand control activate with Touch and Release operation. Just touch and release, do not hold down a button.

SerenEscape™ heat and message functions each have timers that turn off automatically, after 20 minutes without user input.

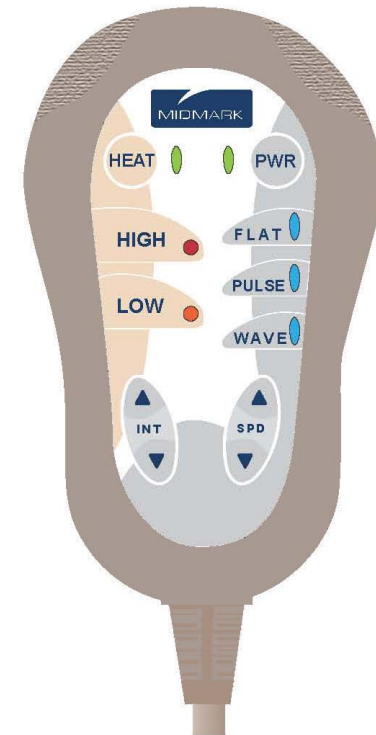Hand Control

Hand Control Operation for Massage and Heat

Note:

Indicator lights beside each button illuminate when the function is ON.

HEAT Button

- Press to toggle heat ON and OFF
- Green HEAT light illuminates when ON.

HIGH and LOW Buttons

- Press HIGH or LOW button to select.
- Orange light illuminates beside the active selection.

PWR (Power) Button

- Press to toggle massage motors ON and OFF.
- Green PWR light illuminates when ON.

Massage mode selections:

Press FLAT button to select flat massage mode.

Flat massage constantly runs both left and right massage motors at the same time.

- Blue light beside the FLAT button illuminates when flat mode is active.
- Use INT arrows to increase or decrease the depth of the massage.
- SPD arrows are not applicable to the flat massage mode.

Press PULSE button to select pulse massage mode.

Pulse massage turns both left and right massage motors ON then OFF together.

- Blue light beside the PULSE button illuminates when pulse mode is active.
- Use INT arrows to increase or decrease the depth of the massage.
- Use SPD arrows to increase or decrease how fast both massage motors alternate ON and OFF.

Press WAVE button to select wave massage mode.

Wave massage alternately turns the left and right motors ON and OFF independently.

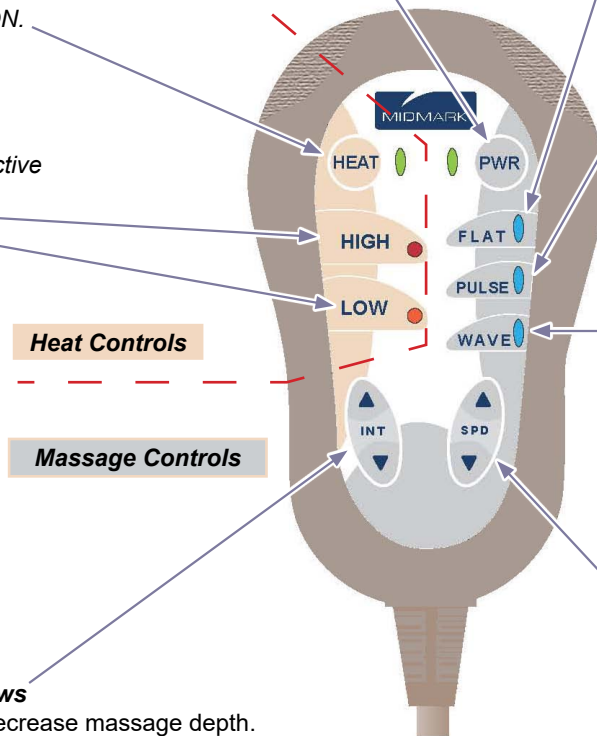
- Blue light beside the WAVE button illuminates when wave mode is active.
- Use INT arrows to increase or decrease the depth of the massage.
- Use SPD arrows to increase or decrease how fast massage motors turn ON and OFF.

SPD (Speed) Arrows

Use to increase/decrease how fast massage motors turn ON and OFF.
Has no effect on FLAT massage mode.

INT (Intensity) Arrows

Use to increase or decrease massage depth.

Note:

After approximately 20 minutes with no user interaction, all SerenEscape™ heat and massage functions turn OFF automatically.

Cleaning

Barriers

Single-use barriers and disposable items significantly reduce the need for chemical cleaners, thus prolonging the life of the equipment. Barrier material must be impervious to moisture / fluids.

Examples of protective barriers:

- Plastic covers (available from your dealer or equipment manufacturer)
- Clear plastic wrap
- Plastic bags
- Plastic sheets
- Plastic tubing
- Plastic-backed paper
- Materials similar to those listed here

Cleaning Chair Upholstery

Spot Cleaning use mild soap and water solution

Difficult stains use a mild solvent (ex. Fantastic®, 409®)

Allow upholstery to air dry or use a hair dryer on the lowest setting!

Caution

Midmark assumes no responsibility or liability for any result, expressed or implied. These are suggested practices, based on the best information available at this time this is written.

Cleaning Procedures

Use cleaners that are appropriate for the situation, such as warm water and mild detergents, or a 10% solution of bleach with water.

NOTE

Every dental practice is different, and no single cleaner is the best choice for every facility. Listed below are some organizations to assist you in choosing the best cleaners available for your practice.

- **Organization for Safety & Asepsis Procedures:**

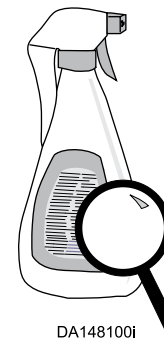
<http://www.osap.org>

- **American Dental Association:**

<http://www.ada.org>


- **Dept. of Health & Human Resources
Centers for Disease Control & Prevention (CDC):**

<http://www.cdc.gov>

DA148100i

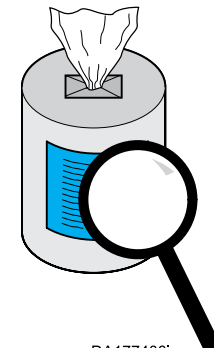**Read all labels
carefully!**

DA177400i

Product Specifications

Elevance Chair

Base Weight of Chair:	426 lbs / 193 kg
Shipping Weight:	506 lbs / 230 kg
Patient Weight (max):	450 lbs / 204 kg
Power Cord Length:	4 or 8 feet / 1.22 or 2.44 meters
Electrical Rating:	115 V~
	60Hz, 9A
	230 V~
	50 Hz, 6A
Duty Cycle:	Intermittent operation
	(Pump Motor and Brake: 30 seconds ON / 5 minutes OFF)
Fuses - 115 VAC Models:	F1 & F2 (line input) 250 V~, T6.3AL
	F3 (transformer) 250 V~, T0.16AL
Fuses - 230 VAC Models:	F1 & F2 (line input) 250 V~, T3.15AL
	F3 (transformer) 250 V~, T0.080AL
Classifications:	Class I, Type B Applied Part, except as noted for optional heated back upholstery, Ordinary Equipment, Intermittent Operation
Certifications:	ES/IEC/EN 60601-1, CAN/CSA C22.2 No. 60601-1 (Safety Standards)
	EN/IEC 60601-1-2 (EMC Standards)
	Optional heated back upholstery, Type BF Applied Part, complies with IEC 60601-2-35
	IPX0 Dental Chair IPX1 Foot Control IPX2 Heated Back Upholstery

Elevance Chair Warnings

WARNING

No modification of this equipment is allowed.

WARNING

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

WARNING

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 inches (30 cm) to any part of the Dental Chair, including cables specified by the manufacturer. Otherwise, degradation on the performance of this equipment could result.

Caution

Federal law restricts this device to sale by or on the order of a licensed dental practitioner.

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture with oxygen, air, or nitrous oxide.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Guidance and manufacturer's declaration – electromagnetic emissions

The Midmark Elevance® Dental Chair is intended for use in the electromagnetic environment specified below. The customer or the user of the Midmark Elevance® Dental Chair should assure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment – Guidance
RF emissions CISPR 11	Group 1	The Midmark Elevance® Dental Chair uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The Midmark Elevance® Dental Chair is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Recommended separation distances between portable and mobile RF communications equipment and the Midmark Elevance® Dental Chair

The Midmark Elevance® Dental Chair is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Midmark Elevance® Dental Chair can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Midmark Elevance® Dental Chair as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = [3.5/V1]\sqrt{P}$	80 MHz to 800 MHz $d = [3.5/E1]\sqrt{P}$	800 MHz to 2.5 GHz $d = [7/E1]\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.67	11.67	23.33

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Guidance and manufacturer's declaration – electromagnetic immunity

The Midmark Elevance® Dental Chair is intended for use in the electromagnetic environment specified below. The customer or the user of the Midmark Elevance® Dental Chair should assure that it is used in such an environment.

<i>Immunity Test</i>	<i>IEC 60601 Test Level</i>	<i>Compliance Level</i>	<i>Electromagnetic Environment - Guidance</i>
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3V rms 150kHz to 80MHz</p> <p>3V/m 80MHz to 2.5GHz</p>	<p>3V</p> <p>3V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Midmark Elevance® Dental Chair, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> <p>$d = [3.5/\sqrt{P}] \sqrt{P}$</p> <p>$d = [3.5/E1]\sqrt{P}$ 80 MHz to 800 MHz $d = [7/E1]\sqrt{P}$ 800 MHz to 2.5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation in meters (m).</p> <p>Field strength from fixed RF transmitters, as determined by the electromagnetic site survey ^a should be less than the compliance level in each frequency range. ^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> <div style="text-align: right;">  </div>

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Midmark Elevance® Dental Chair is used exceeds the applicable RF compliance level above, the Midmark Elevance® Dental Chair should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Midmark Elevance® Dental Chair.

^b Over the frequency range 150kHz to 80MHz, field strengths should be less than 3 V/m.

Guidance and manufacturer's declaration – electromagnetic immunity

The Midmark Elevance® Dental Chair is intended for use in the electromagnetic environment specified below. The customer or the user of the Midmark Elevance® Dental Chair should assure that it is used in such an environment.

<i>Immunity Test</i>	<i>IEC 60601 Test Level</i>	<i>Compliance Level</i>	<i>Electromagnetic Environment - Guidance</i>
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 2, 4, 6 kV contact ± 2, 4, 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast Transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1kV line(s) to line(s) ±2kV lines(s) to earth	± 0.5, 1 kV ± 0.5, 1, 2 kV	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 s	0 % U_T (100 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Midmark Elevance® Dental Chair requires continued operation during power mains interruptions, it is recommended that the Midmark Elevance® Dental Chair be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m, 50 / 60 Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: U_T is the a.c. mains voltage prior to application of the test level.

Warranty Information

Midmark Limited Warranty - Dental Products

SCOPE OF WARRANTY

Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will at Midmark's option repair or replace components of the dental products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. Midmark's obligation under this limited warranty is limited to the repair or replacement of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which, upon examination by Midmark, prove to be defective. This warranty extends only to the first retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

APPLICABLE WARRANTY PERIOD

The applicable warranty period, measured from the date of delivery to the original user, shall be as follows: Effective March 1, 2018 these applicable warranty periods are measured from the date of invoice to the original user, shall be as follows:

1. OPERATORY PRODUCTS

- a. Five (5) years for all products (except for the items in (b) through (e)).
- b. Two (2) years for upholstery (chairs and stools).
- c. "KINK-VALVE" module carries a ten (10) year warranty.
- d. The original light bulb on a new light carries a one (1) year warranty.
- e. Accessories not manufactured by Midmark are excluded including but not limited to Bien Air handpiece systems, Dentsply Cavitron scaler, Satelec scaler and curing light, and Sopro cameras.

2. ORAL SURGERY PRODUCTS are warranted for a period of one (1) year.

3. STERILIZER PRODUCTS are warranted for a period of one (1) year.

4. ULTRASONIC CLEANER PRODUCTS are warranted for a period of two (2) years.

5. AIR AND VACUUM PRODUCTS

- a. PowerAir® oil-less compressors – Five (5) years or 3,500 hours of use, whichever occurs first.
- b. PowerVac® and PowerVac® G dry vacuums – Five (5) years or 10,000 hours of use, whichever occurs first (except that the vacuum pump warranty term is ten (10) years or 20,000 hours of use, whichever occurs first).
- c. Classic Series® wet-ring vacuums – Five (5) years or 10,000 hours of use, whichever occurs first.
- d. PowerMax surgical suction – Two (2) years.
- e. Hg5 Series Amalgam Separator - One (1) year. (f) Midmark manufactured accessories - One (1) year.

6. SYNTHESIS™ DENTAL CASEWORK AND ARTIZAN® EXPRESSIONS PRODUCT

- a. Five (5) years for all products and components including door and drawer fronts, casters and slides, except for the items in (b), (c) and (d).
- b. Three (3) years for electrical components such as task lights/LED lights, cords, controls and accessories.
- c. Two (2) years for sliding track monitor mount and components and upholstery. (d) One (1) year for countertops and resin, including accessories.

7. IMAGING PRODUCTS are warranted for a period of two (2) years except for the ClearVision CR reader which is warranted for a period of one (1) year.

8. MIDMARK Replacement Parts and Accessories carry a ninety (90) day warranty

EXCLUSIONS

This warranty does not cover and Midmark shall not be liable for the following;

1. defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
2. products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environment conditions and electrical power requirements;
3. products considered to be of a consumable or sterile nature;
4. accessories or parts not manufactured by Midmark;
5. charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark
6. costs and expenses of routine maintenance and cleaning;
7. representations and warranties made by any person or entity other than Midmark;
8. matching of color, grain or texture except to commercially acceptable standards;
9. changes in color caused by natural or artificial light;
10. custom manufactured products;
11. alterations or modifications to the product by any person or entity other than Midmark; and
12. Products that would otherwise be covered under Sections 1 and 2 of this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical, animal health or dental market, as the case may be, in which purchaser intends to use the product.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER

MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

WARRANTY DISCLAIMER

THIS LIMITED WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS

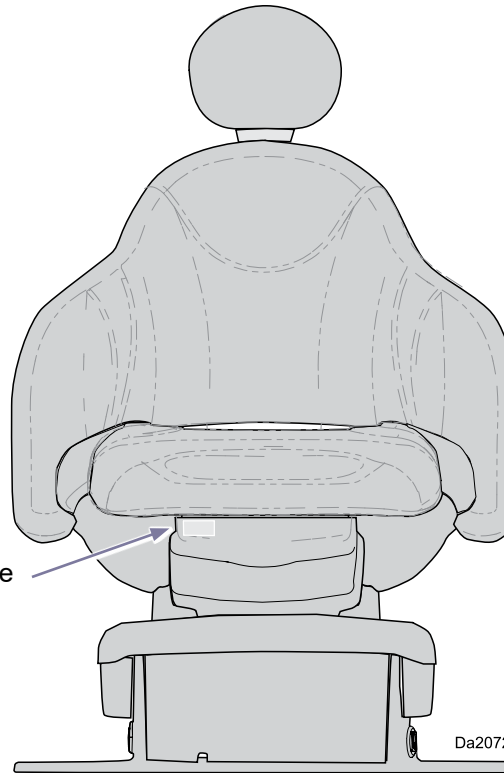
No action may be brought against Midmark for breach of this limited warranty, or implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

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