

# M9/M11 Fill / Vent Valve Repair [002-1360-0x / 002-1361-0x]

## Components:

Valve ..... 1  
Wire Harness..... 1

## Special Tools:

none



### Caution

Always disconnect power to sterilizer before removing covers or performing any service procedure.

### Note


Replaces old style Fill / Vent Valve Assemblies.

### Step 1: Drain water from reservoir.


Note: For fill valve replacement only.

### Step 2: Disconnect compression fitting from top of vent valve.

Note: For vent valve replacement only.

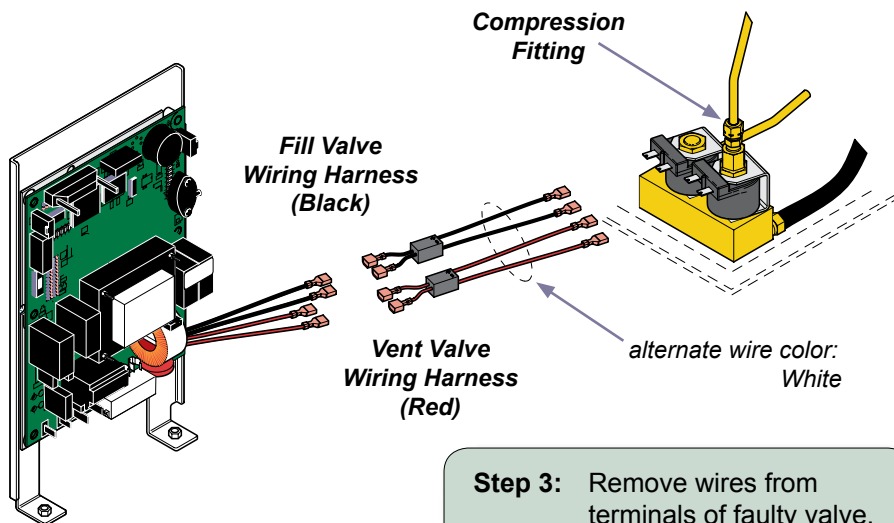

### Step 3: Remove wires from terminals of faulty valve.

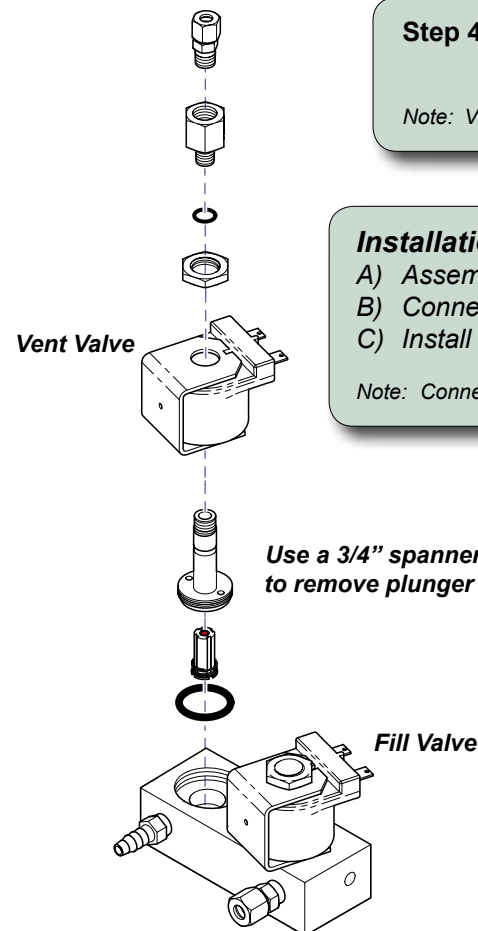

### Step 4: Disassemble valve and remove any debris.

Note: Vent valve shown.

### Installation...

- Assemble valve.
- Connect compression fitting. (Vent Valve)
- Install Fill / Vent valve wiring harness.

Note: Connect Red to Red - Black to Black

### Test...

- Refill reservoir.
- Run test cycle.
- Check for leaks.

SA1759i