

Elevance / Procenter Interface PCB Installation Kit and Parts List (002-1369-00)

Contents:


Installation	1
Harness Identification	2
Kit Parts List	3

Language of origin: English

Note

The Elevance / Procenter Interface Printed Circuit Board (PCB) Box enables cabinet and wall mounted Procenter delivery unit touch pads to control an Elevance Chair.


Delivery System Control Panel

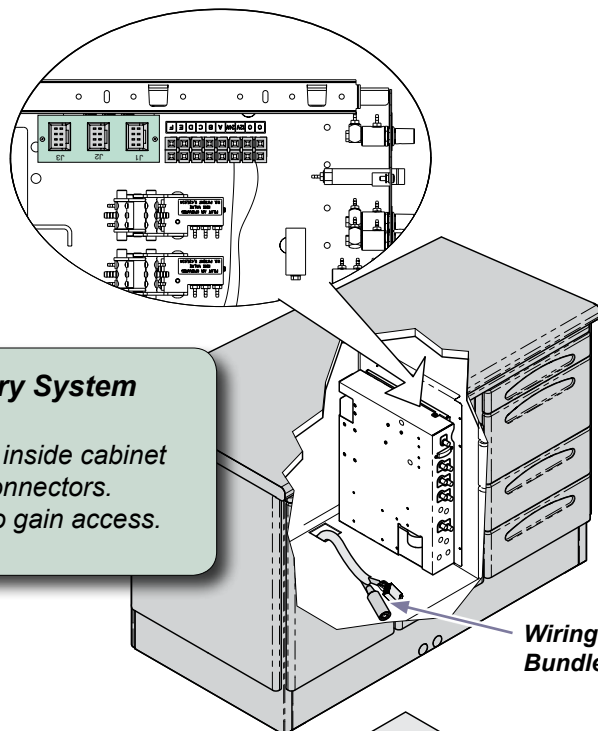


Step 1: Access Delivery System Control Panel

- Slide out control panel inside cabinet enclosure to access connectors.
- Lift wiring bundle out to gain access.

Elevance / Procenter Interface PCB Box

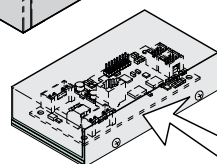

Wiring Bundle

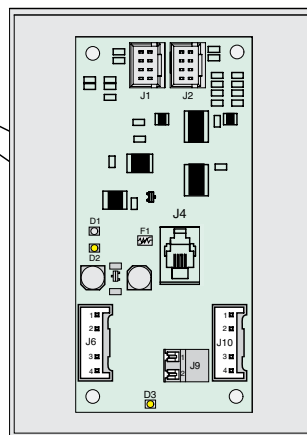

Step 3: Make Connections.

- Connect Modular Harness to Chair PCB and Delivery System Control Panel.
- Connect Delivery Control Harness to Elevance / Procenter Interface PCB and to Delivery System Control Panel.

Note: For Dr. or Dr. and Assistant Units - use NEW Delivery Control Harness from Kit.

For Assitant's Unit ONLY or Side Delivery Units - use Delivery Control Adapter with existing Delivery Control Harness.

Step 2: Install wiring harness(es).

- Install Modular Harness into the wiring bundle.
- ONLY for Dr. or Dr. and Assistant Units - replace existing Delivery Control Harness with New one from kit.


Note: See Wiring Harness Identification page.

Step 4: Install Elevance / Procenter Interface PCB Box and replace all covers.


- Install cover on Elevance / Procenter Interface PCB Box. Secure with four screws.
- Slide Delivery System Control Panel into cabinet.
- Return Wiring Bundle to cavity in cabinet.

Harness Identification


For all types of Delivery Units

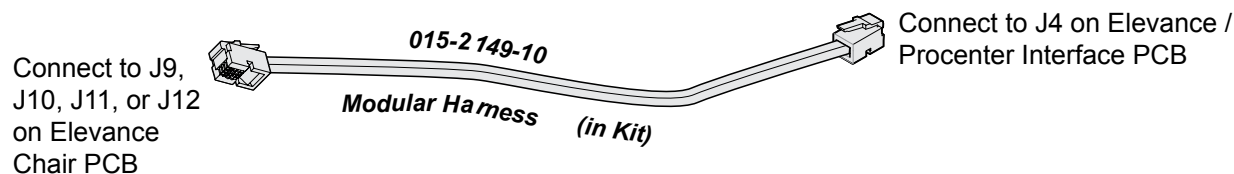

For Dr. or Dr. and Assistant Delivery Units

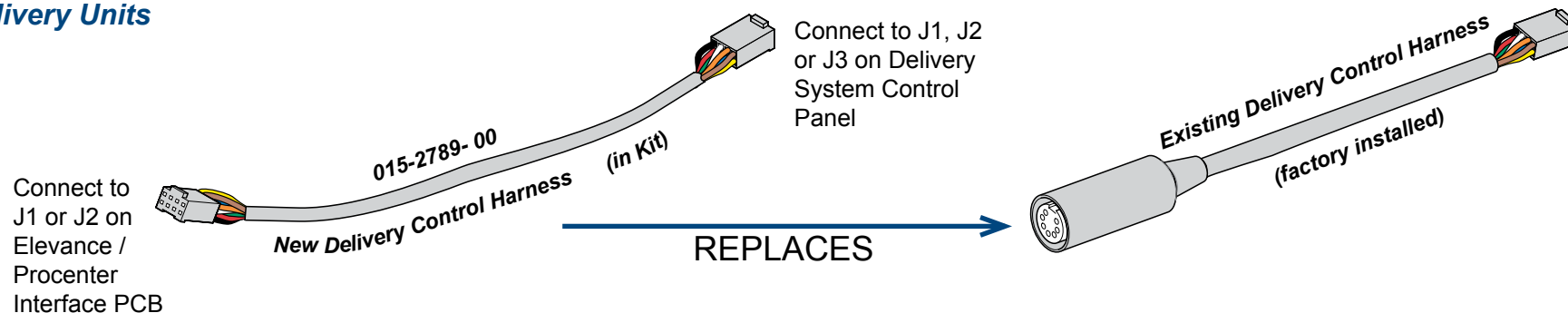


For Assistant's ONLY and Side Delivery Units

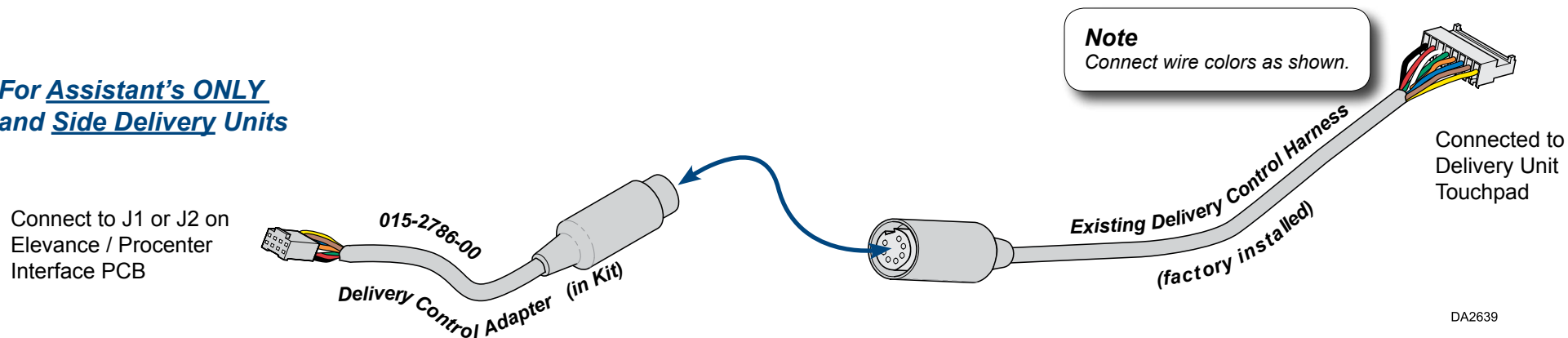

DA2639

Kit Parts List

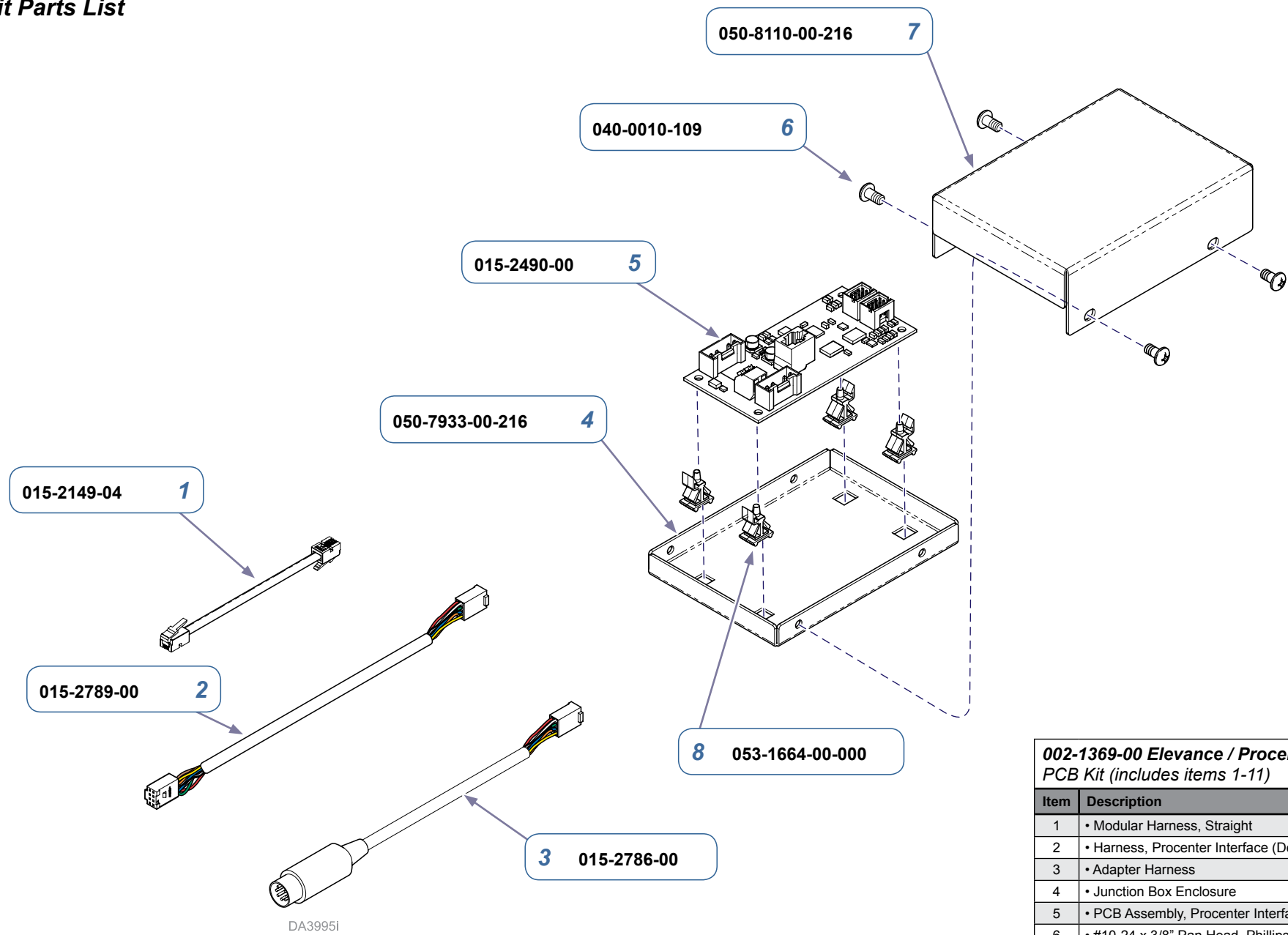

002-1369-00 Elevance / Procenter Interface PCB Kit (includes items 1-11)		
Item	Description	Qty.
1	• Modular Harness, Straight	1
2	• Harness, Procenter Interface (Delivery Control)	1
3	• Adapter Harness	1
4	• Junction Box Enclosure	1
5	• PCB Assembly, Procenter Interface	1
6	• #10-24 x 3/8" Pan Head, Phillips Screws	4
7	• Box, Interfaced PCB	1
8	• PC Board Standoffs	4
9	• 003-2225-00, Installation Instructions (not shown)	1
10	• Kem-Pac, 24-inch	1
11	• 5x7x7 inch Carton (not shown)	1

To Contact Customer Service for Information on Service Repair call 1-800-Midmark (1-800-643-6275).

Subject to Change Without Notice