Wireless Remote Installation [002-1090-xx]

Applies to Kits:

002-1090-01 Remote
002-1090-02 Remote &
Base Antenna

Special Tools:

none

Language of origin: English

WARNING

Disconnect chair from power during installation. Failure to do so may result in personal injury.

Step 1: Remove Covers.

- Disconnect chair from power supply.
- Push in with a screwdriver to release clips and remove upper lift arm cover.
- Remove two screws and the PC Board cover.
- Use a screwdriver to pry base cover up and remove.

Step 2: Install Wireless Base Station.

- Route communication wire through chair to the PC Board. Plug into a socket J9 through J12.
- Mount base station to chair with two screws.


Equipment Alert

Do not kink communication wire or apply stress to connection.

Step 3: Install Covers.

Repeat Step 1 in reverse.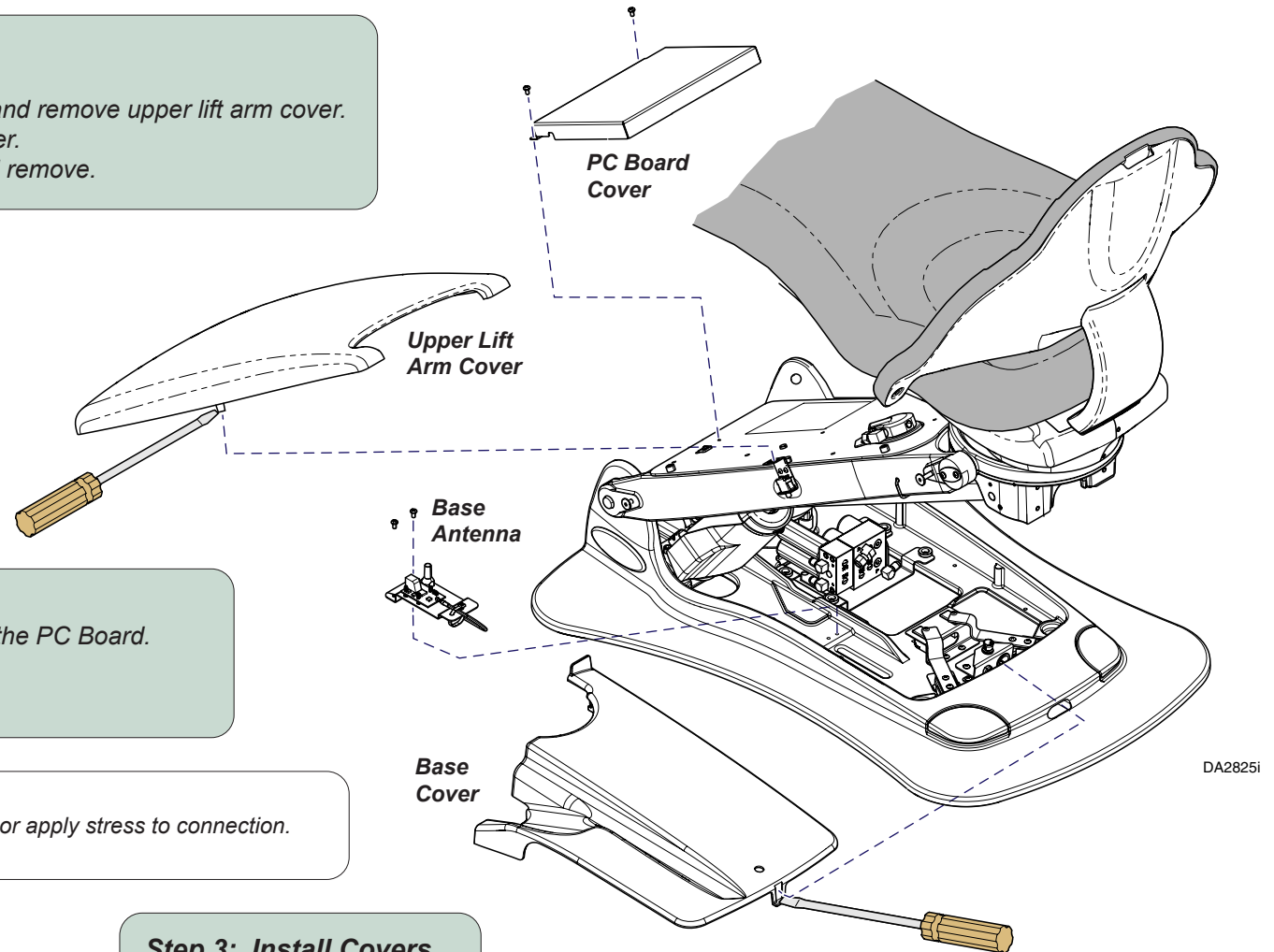

Step 4: Synchronize the Remote.



Synchronize the Wireless Remote

If nothing happens when any button on the Wireless Remote is pressed and no led lights on the PC board flash, the remote either needs new batteries or needs to be synchronized.

To synchronize the remote:

1. Chair must be Powered ON.
2. Verify that Chair has been calibrated.
(Refer to Chair Calibration procedure in Installation Guide 003-1923-99).
3. Remove Upper Arm Lift Cover* and the PC Board cover (two screws).
4. Locate the S2 PROG SW on the PC board and set the dipswitch 4 to the ON position. Listen for two beeps.
5. Press and hold buttons 2 and 3 on the remote within three seconds from moving dipswitch 4. Listen for three short beeps from the Chair then three beeps from the Remote.
6. Set dipswitch 4 to the OFF position to complete the synchronization process.
7. Replace PC Board & Upper Arm Lift covers.

* Push in with a screw driver to release clips and lift Upper Arm Lift Cover up to free front clips from shaft. Lift Cover must be removed to access PC Board cover.

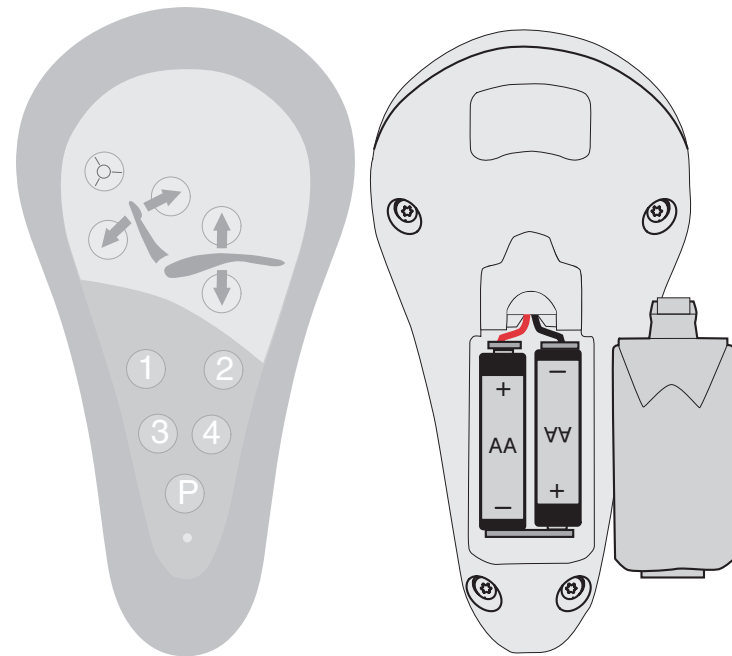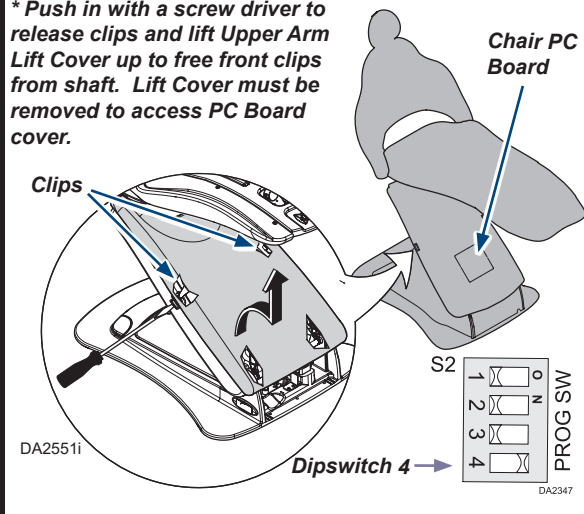Step 5: Press all remote buttons and verify operations.

FCC Compliance Statement

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

Any modifications to this product may violate the rules of the Federal Communications Commission and make operation of the product unlawful.

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Midmark Corporation