Elevance® Delivery: Console Mounted Installation (Includes Elevance® Continental-Style Delivery Unit)

Language of origin: English

Equipment Alert

Do not remove wraps or packaging from the delivery flex arm assembly until it is mounted on the console.

Attention!

The electrical and plumbing supply connectors must be installed by an electrician and plumber in compliance with local building codes. All required hardware is to be supplied by contractor. Supply connections must be located to fit inside the J-box and be properly distanced, within reach of the chair umbilical [4 or 8 foot (1.2 or 2.4 m)].

Equipment Alert

When installing 115~ (VAC) receptacle in the J-Box, position it so that all the other components needed in the box will also fit.

Grounding reliability can only be achieved if this equipment is connected to an equivalent receptacle marked Hospital Grade (NEMA 5-15R, HOSPITAL GRADE).

Electrical Supply Requirements

Have an electrician install a 1/2 in. (13 mm) conduit box with quad or equal receptacle. Top of box must be no higher than 4-1/2 in. (114 mm) above finished surface of floor.

Water Supply Requirements (optional)

Use 1/2 in. (13 mm) pipe protruding 2 in. (51 mm) above finished surface of floor. Manual shut-off valve is to be installed by a plumber.

Water pressure must be 30-50 PSI (206-344 kPa) maximum, at the shut-off valve. Flush water plumbing clean before making final connections to dental equipment.

Recommended water supply hardness level <60 ppm of calcium carbonate.

Air Supply Requirements

Use 1/2 in. (13 mm) pipe protruding 2 in. (51 mm) from finished surface of floor. Manual shut-off valve is to be installed by a plumber.

Air pressure must be 70-100 PSI (482 -689 kPa) maximum, at the shut-off valve. Flush air plumbing clean before making final connections to dental equipment.

Air supply to be filtered, dry and oil-free per NFPA 99 requirements or equivalent local codes.

Drain Requirements (optional)

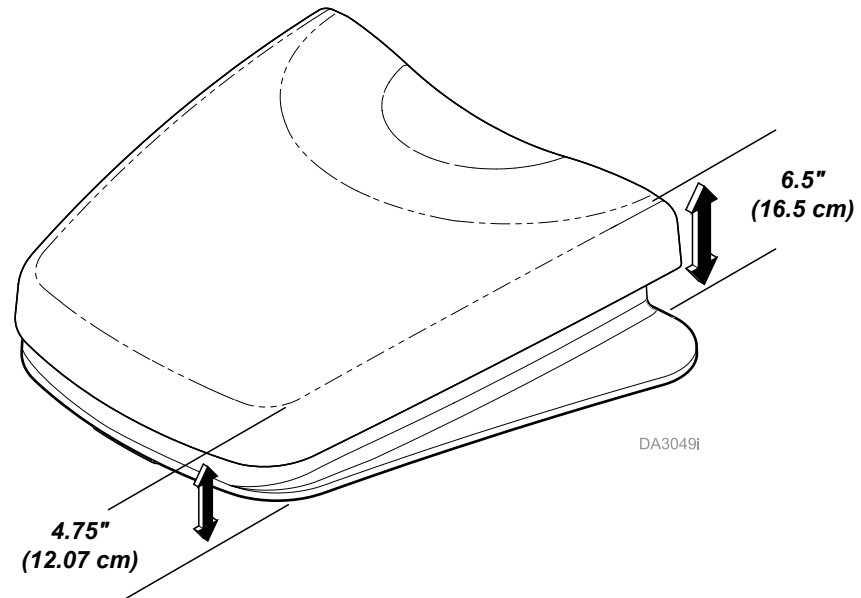
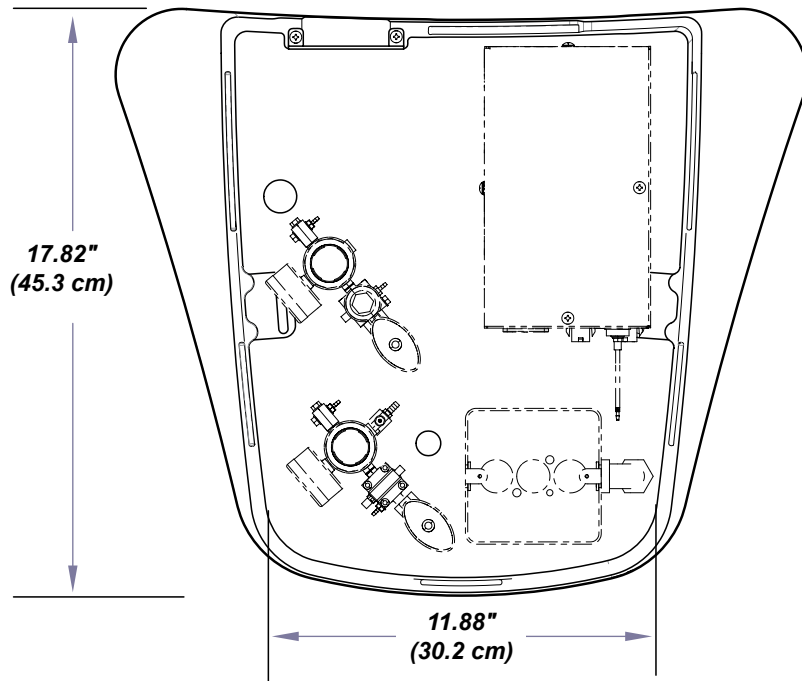
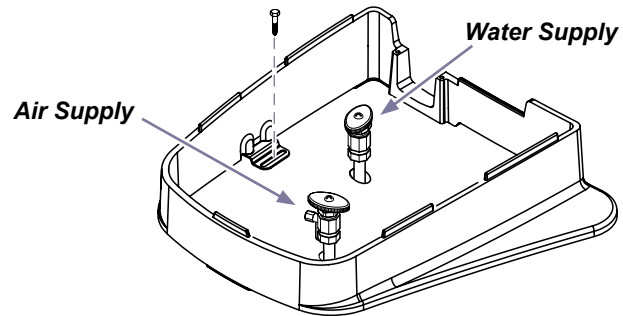
Not less than 1-1/2 in. (38 mm) I.D. pipe from downpipe to trap, decreasing to 3/4 in. (19 mm) pipe from trap to connection box. Terminate with 3/4 in. (19 mm) NPT pipe protruding 1 in. (26 mm) above finished surface of floor. Install adapter for 5/8 in. (16 mm) drain tubing. Place trap in line and vent to conform with local codes.

Vacuum Requirements (optional)

Plumbing to J-Box box should be specified by central vacuum supplier and terminated in J-Box with 3/4 in. (19 mm) O.D. tube protruding 1 in. (26 mm) above finished surface of floor.

Step 1: Verify Proper Position and Types of Supply Connections Required for the J-Box.

- A) Verify that you have the proper supply connections required for the components being installed.
- B) Verify that the connections fit inside the J-box and are located within reach of the umbilical provided (4 or 8 foot [1.2 or 2.4 m]).
- C) Reference the J-Box Installation Guide for more details.

Step 2: Verify proper installation of the Elevance chair.

- A) Reference Elevance Chair Installation Guide packaged with chair.

**WARNING**

The Elevance chair must be anchored to floor before installing the delivery system components.

Step 3: Position chair and disconnect power.

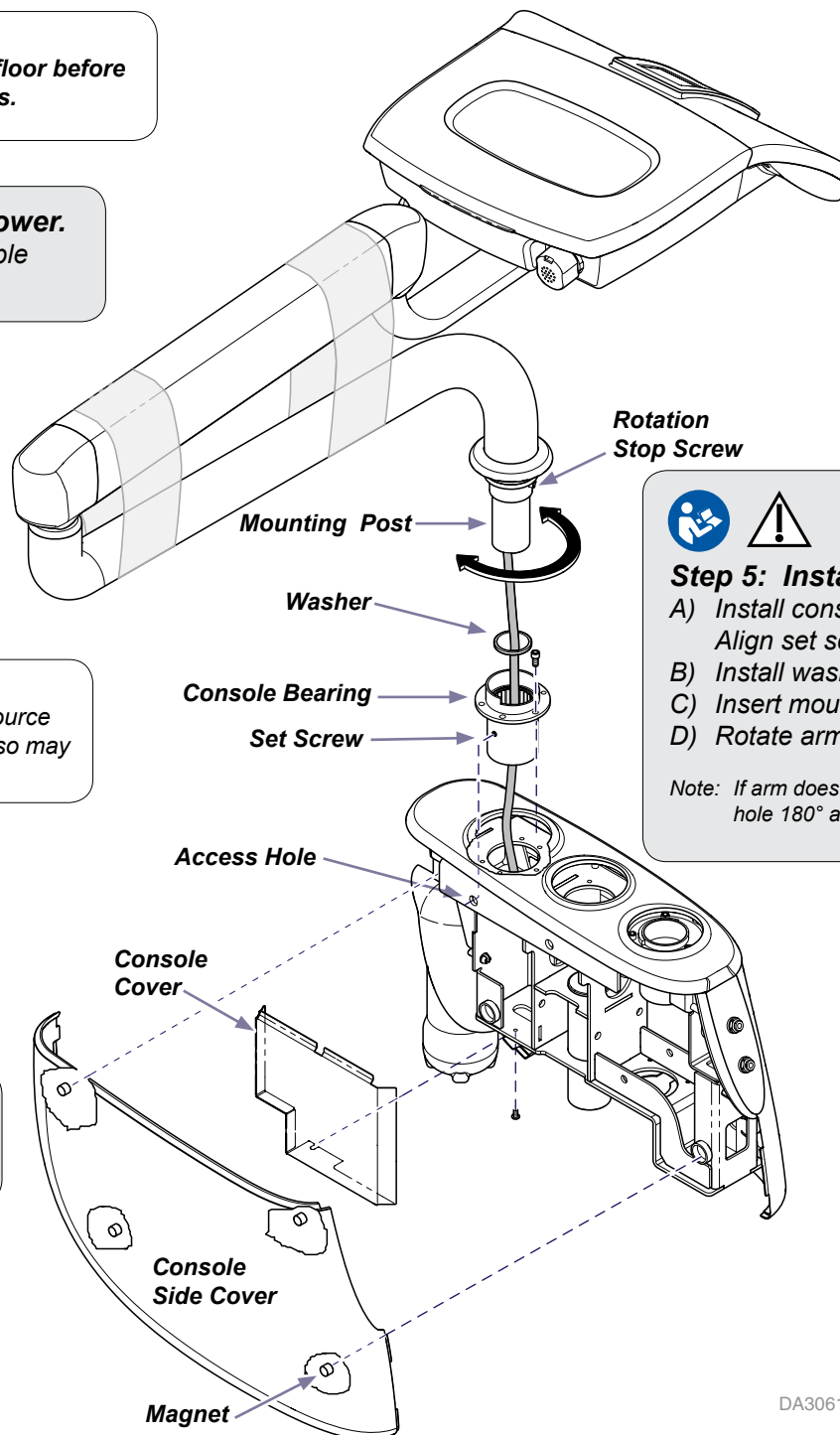
- A) Raise or lower chair / console to a comfortable working height.

**WARNING**

Always disconnect chair from the power source before removing any covers. Failure to do so may result in personal injury.

Note

Have an assistant help to mount the delivery unit on the console.

**Step 5: Install delivery.**

- A) Install console bearing in console using five screws. Align set screws with access holes in console frame.
B) Install washer on mounting post.
C) Insert mounting post into console bearing.
D) Rotate arm assembly to verify operation.

Note: If arm does not rotate over the chair, move rotation stop screw to hole 180° around mounting post.

Note


Tighten or loosen console bearing set screws to adjust drag if necessary.

DA3061i


Step 6: Setup flex arm.

- A) Remove bubble wrapping and cut the wraps.
- B) Lift up and rotate arm to left or right, to align rotation stop screw to the appropriate opening.
- C) Press flex arm fully into the lower arm.
- D) Lower trim ring over mounting opening.
- E) Rotate arm to verify proper rotation.

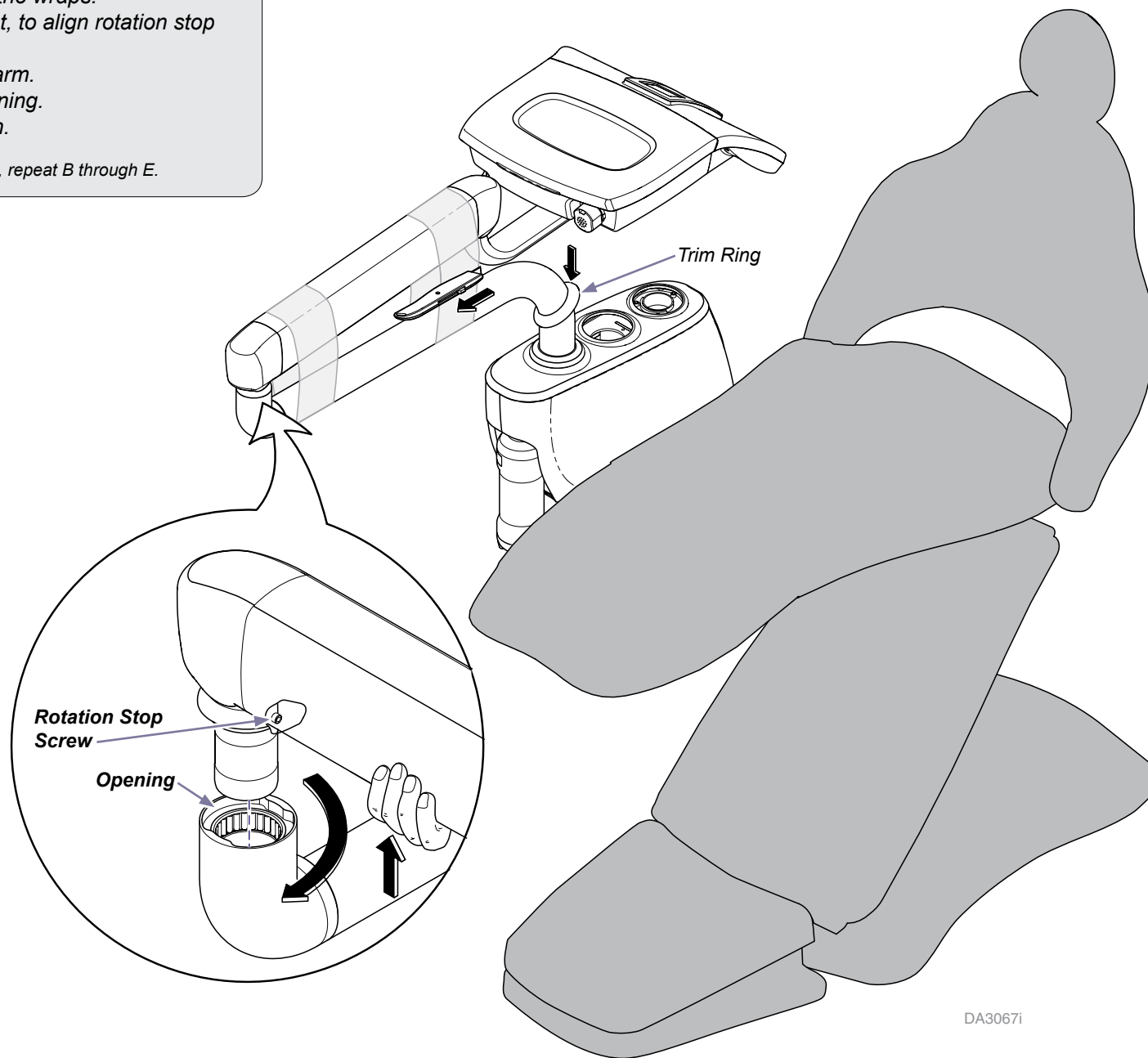Note: If arm does not rotate in desired direction, repeat B through E.

Standard Water Bottle System


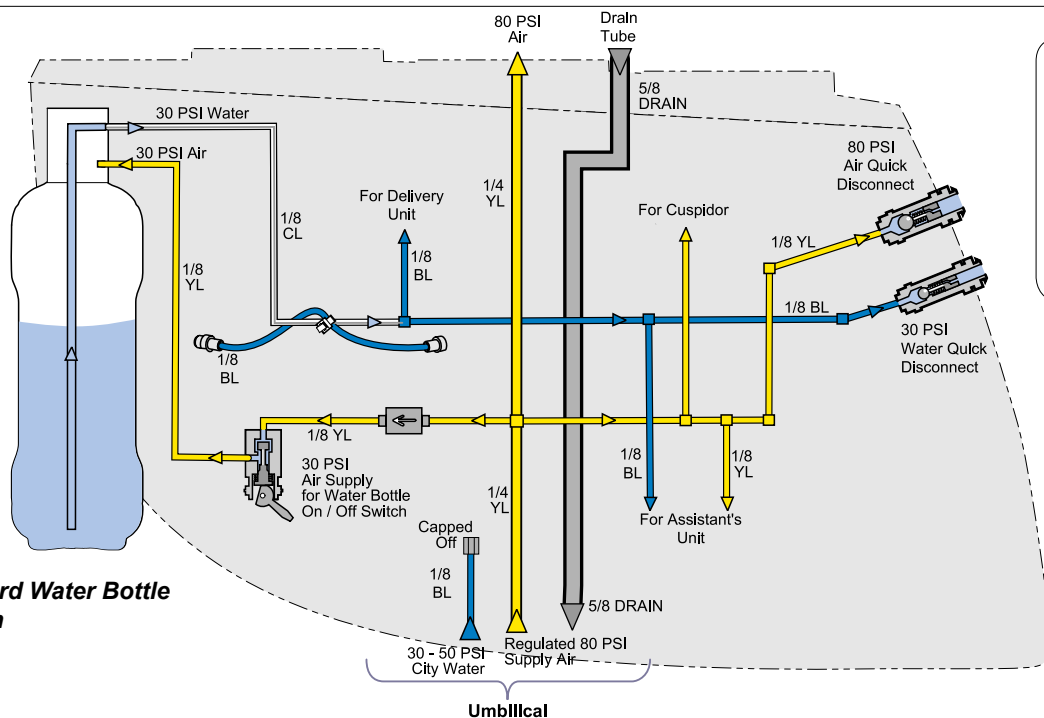
Note

Tubing and connectors are provided for connecting a delivery unit and an assistant's unit. Leave connectors capped on any unused tubing.

An additional 1/8" piece of blue tubing is provided in the standard console. This could be used to connect to a city water supply.

Step 7: Connect tubing by matching colors.

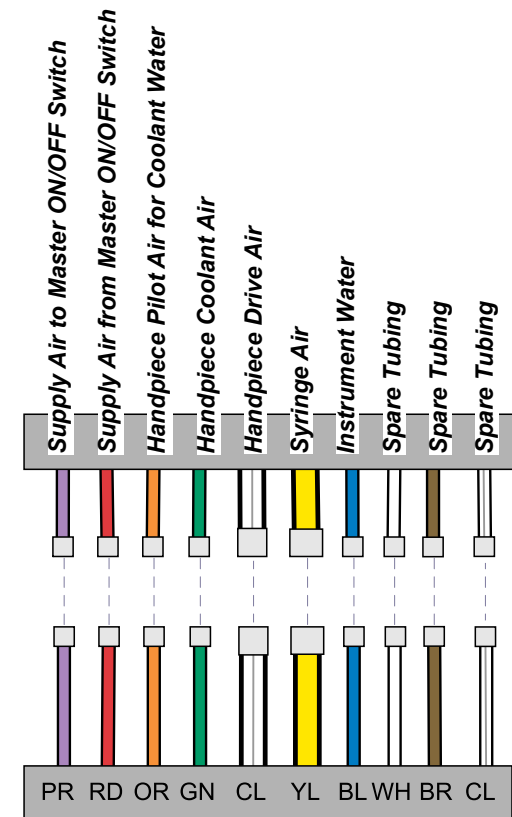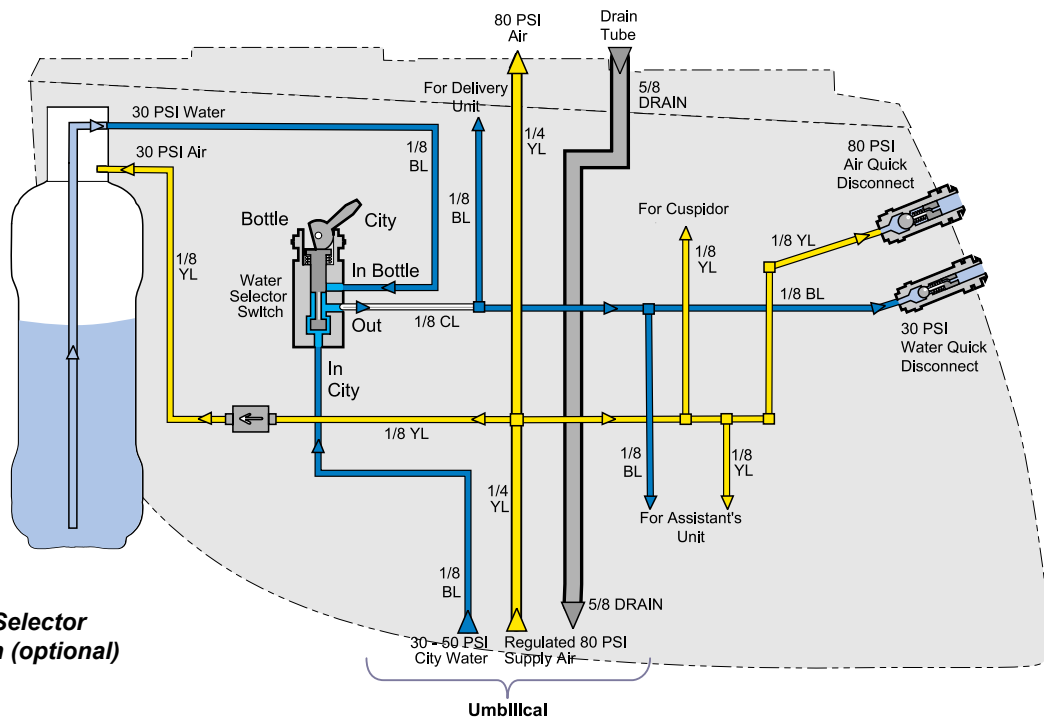Water Selector System (optional)


DA3379I

**Caution**

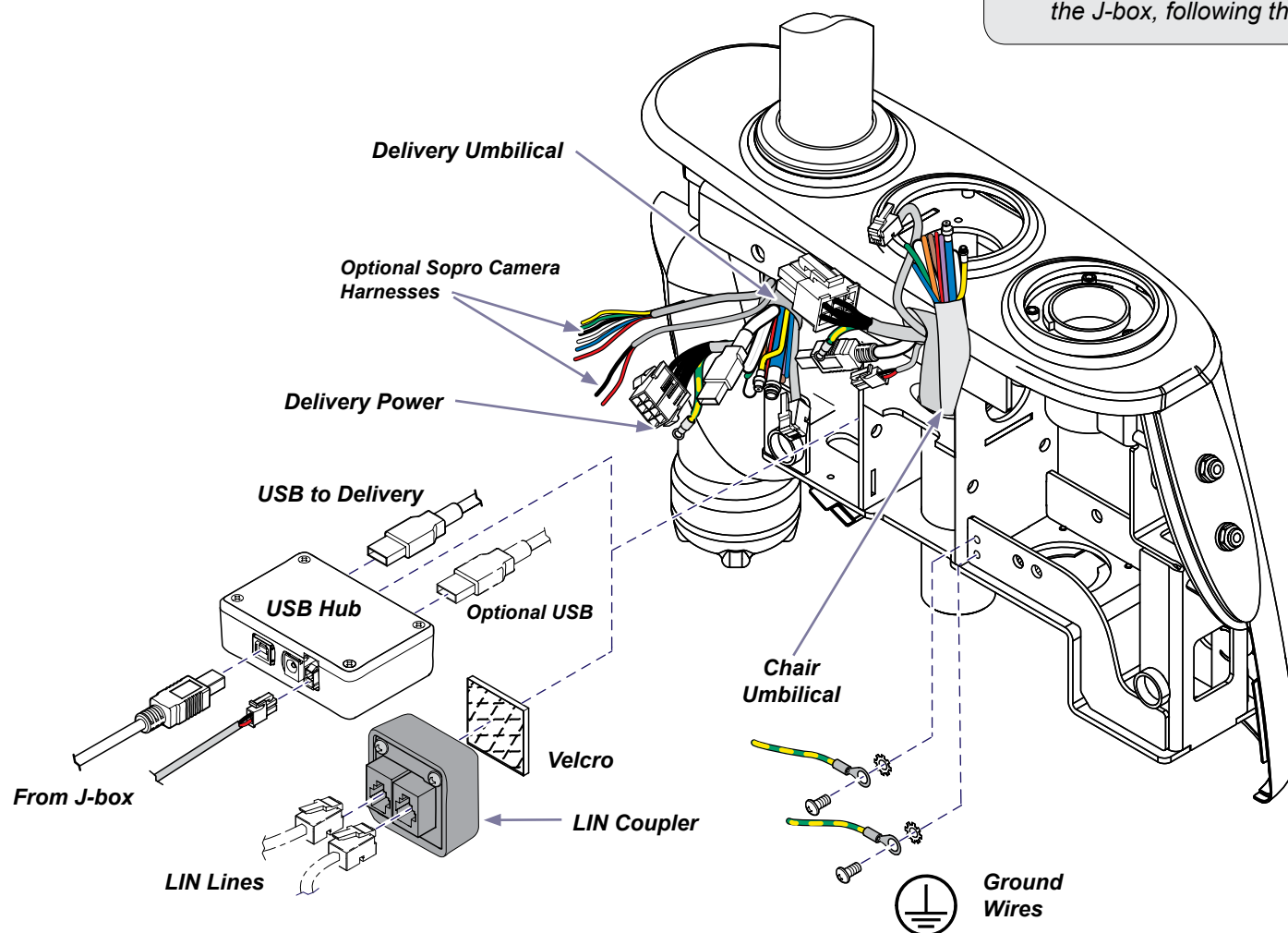
Connect ground wire to ensure system ground continuity.
Failure to do so may result in electric shock.

Step 8: Connect delivery umbilical to chair umbilical.

- A) Connect two LIN lines to the LIN coupler.
- B) Connect USB connectors to USB hub and place inside console.
- C) Connect delivery power harnesses.
- D) Screw grounds with washers onto console frame.
- E) Optional - Sopro Camera harnesses must be routed thru the chair to the J-box, following the same path as the chair umbilical.


**Step 9: Install covers.**

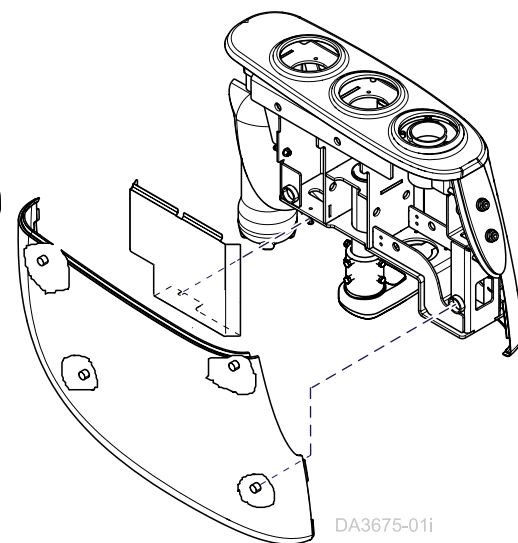
- A) Install console cover over all electrical connections.
- B) Install console side cover.

Step 10: Level delivery unit and adjust drag.

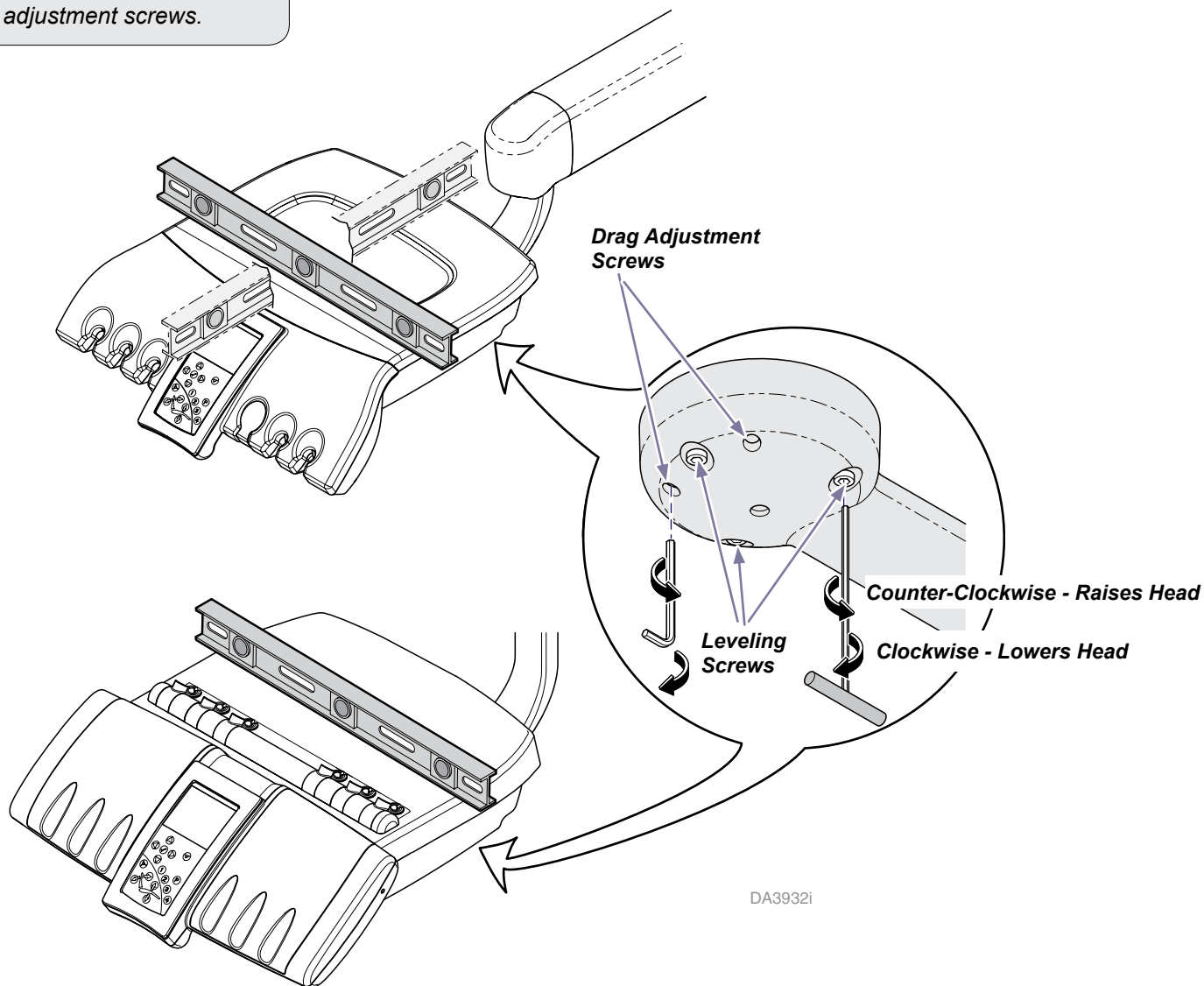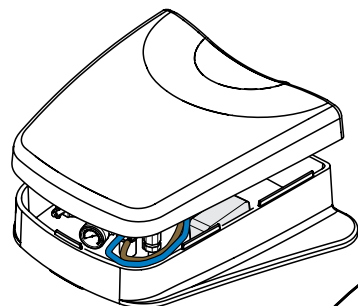
- A) Position flex arm horizontal to the floor.
- B) Place a level across unit and alternately turn the three leveling screws.
- C) Adjust drag using the drag adjustment screws.


Step 11: Verify umbilical tubing to water and air supply connections.

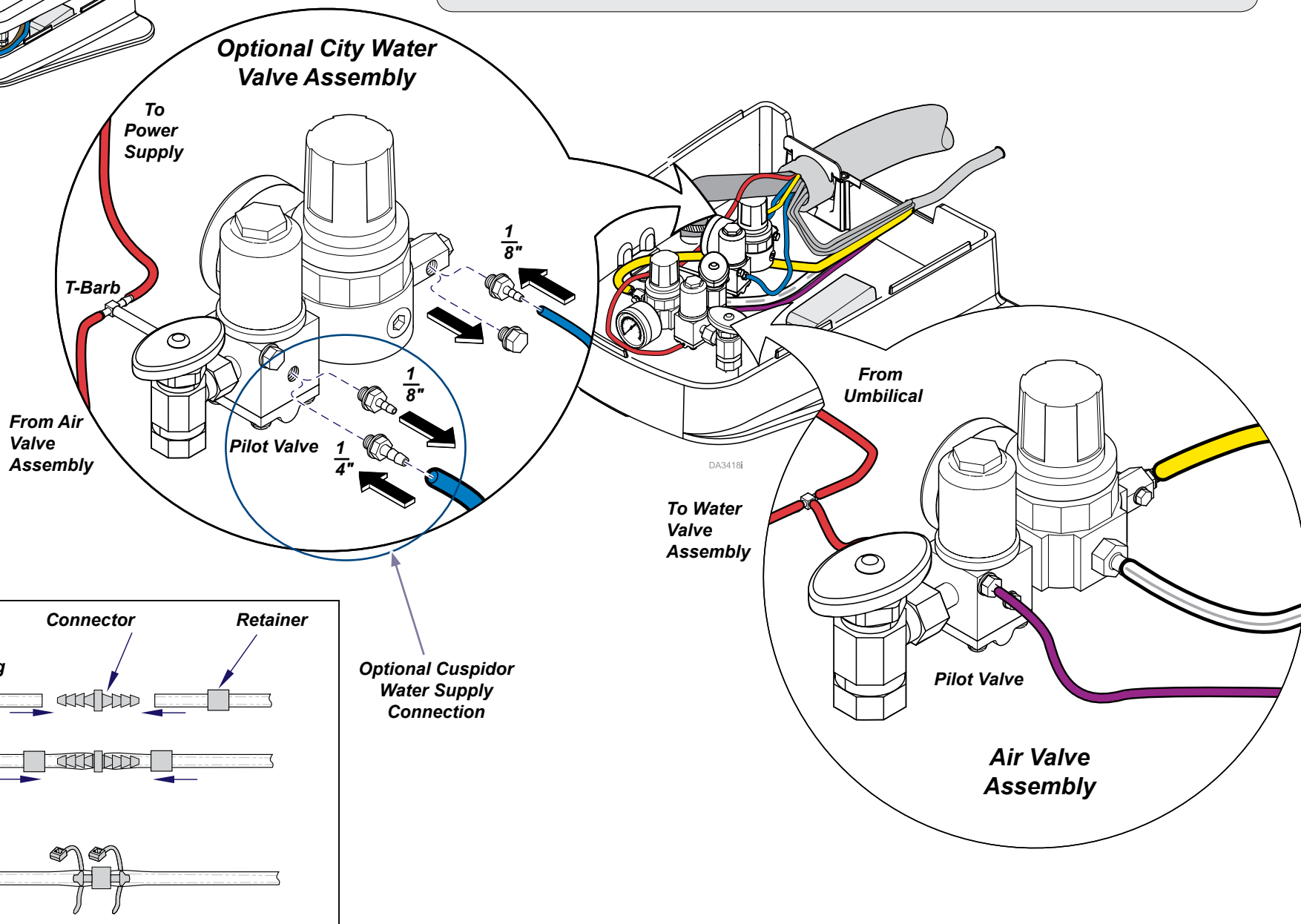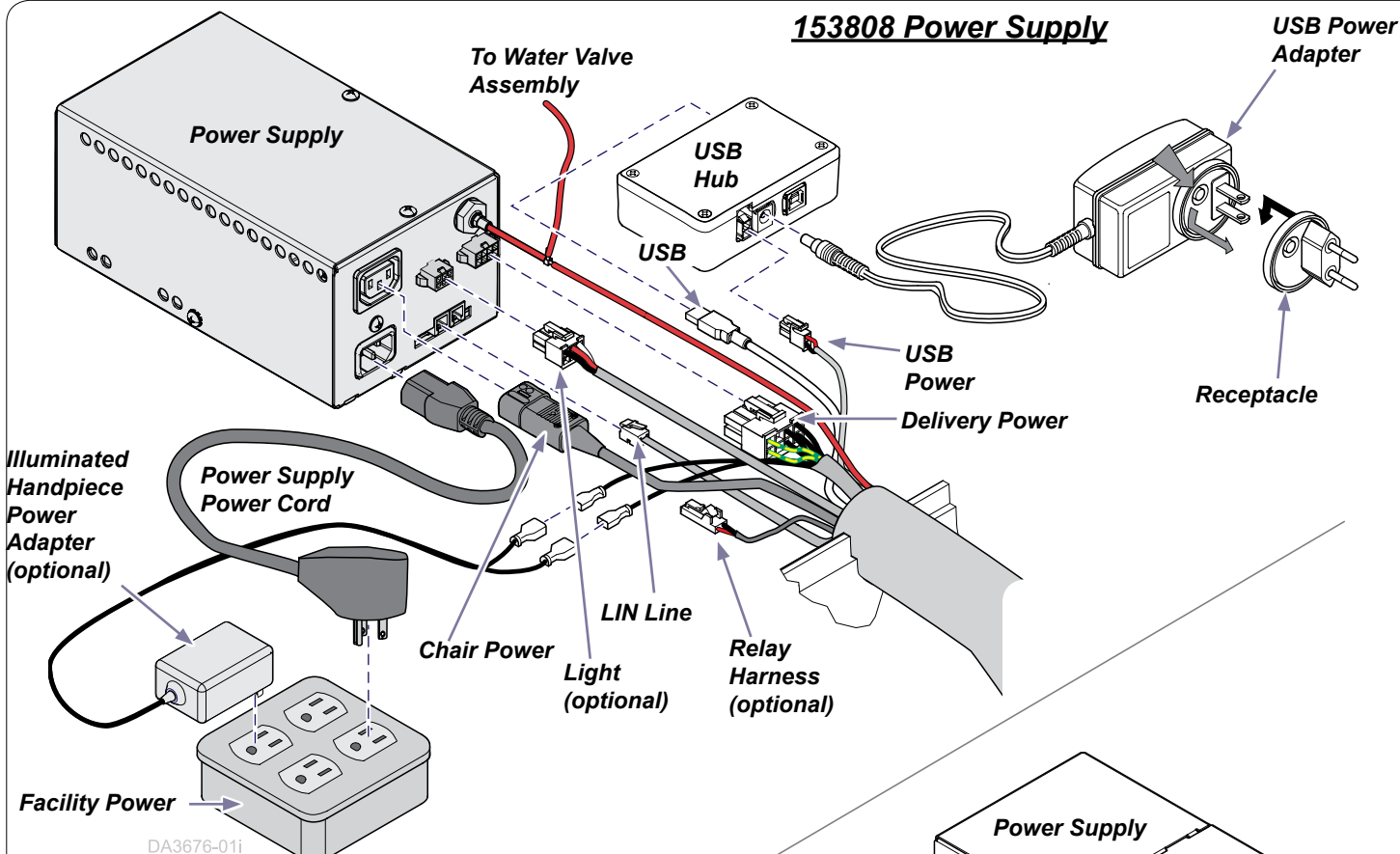
- A) Remove J-Box lid.
- B) Verify proper yellow, clear and purple connections from umbilical to valve assemblies.
- C) Optional - Install city water supply fittings and connect blue tubing as shown.

Equipment Alert
Once the delivery unit is powered ON, it must STAY ON for 30 minutes to charge the clock circuitry.

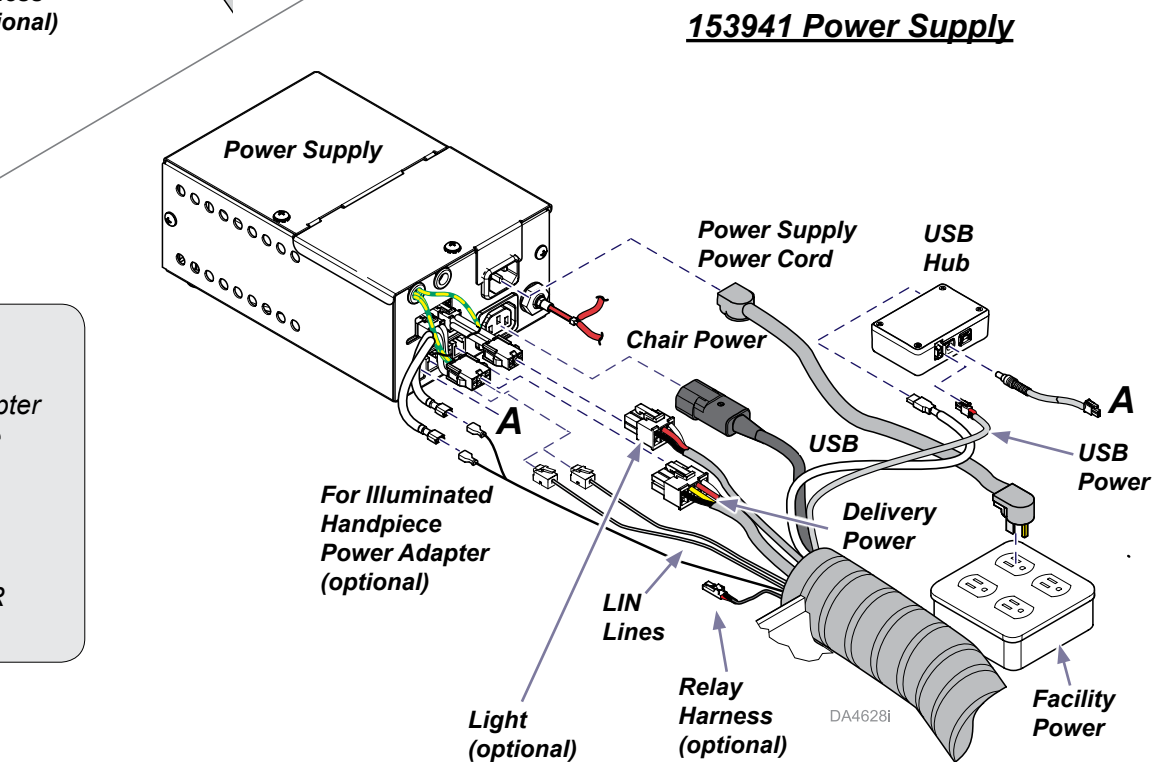Equipment Alert
Chair umbilical includes connectors for optional components that may not be included in your installation.


Step 12: Standard Power Supply Connections.


- Add T-fitting and connect red tubing to power supply.
- Install the appropriate receptacle onto the USB power adapter and connect the USB, USB power and the other end of the adapter to the USB Hub.
- Connect delivery power to power supply.
- Connect chair power to power supply.
- Connect LIN line to power supply.
- Install optional components (Steps 13 - 19) as required OR proceed to Step 20.

Note

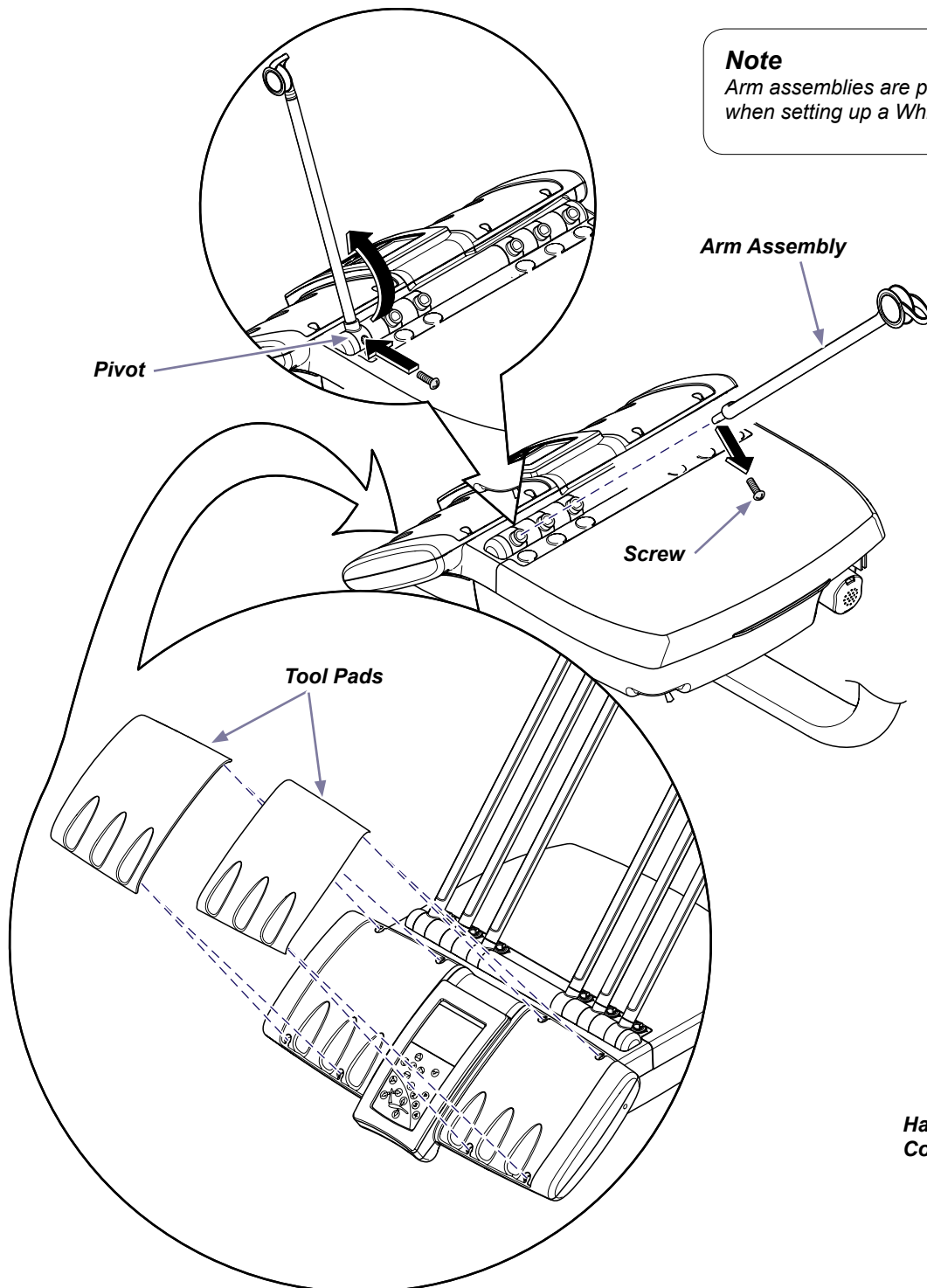Arm assemblies are packaged separately and must be installed when setting up a Whip delivery unit.

**Continental-Style Units Only****Step 13: Install Arm Assemblies and Tool Pads.**


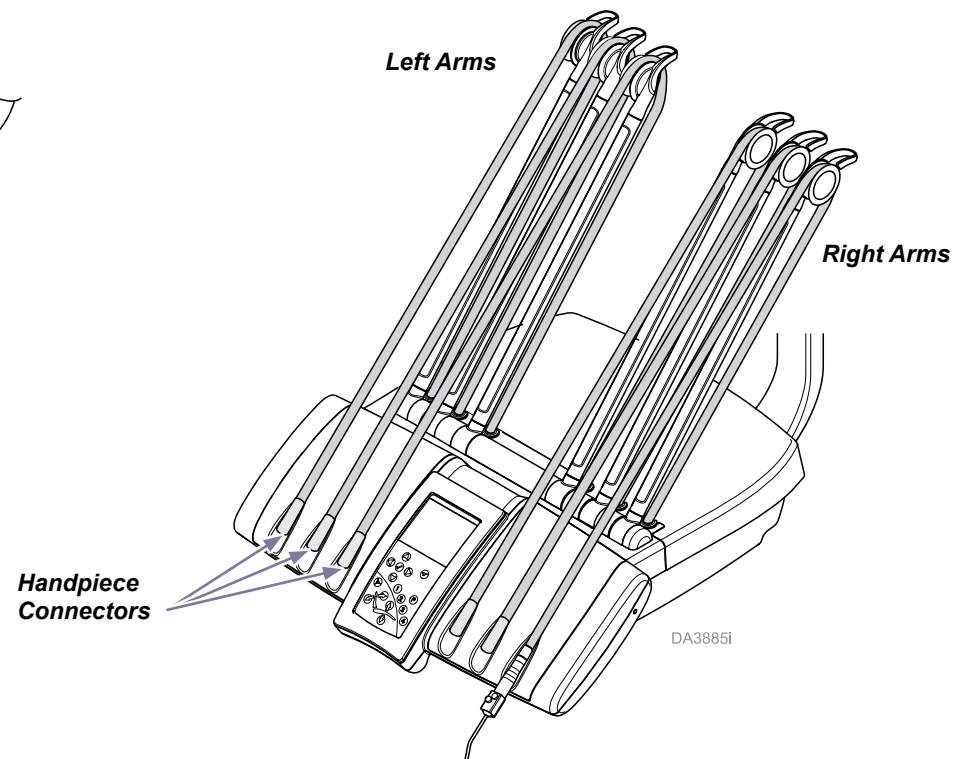
- A) Remove screw from arm assembly.
- B) Insert arm assembly into the pivot and rotate forward.
- C) Install the screw thru pivot hole into arm assembly to secure.
- D) Install three left and three right arm assemblies.
- E) Install left and right tool pads onto the front of unit.
- F) Route tubing thru wheels and align handpiece connectors with tool pads as shown below.

For Units Having Deluxe Rheostat with Chip Air Foot Controls Only:

Make Chip Air Connection in Delivery Head.

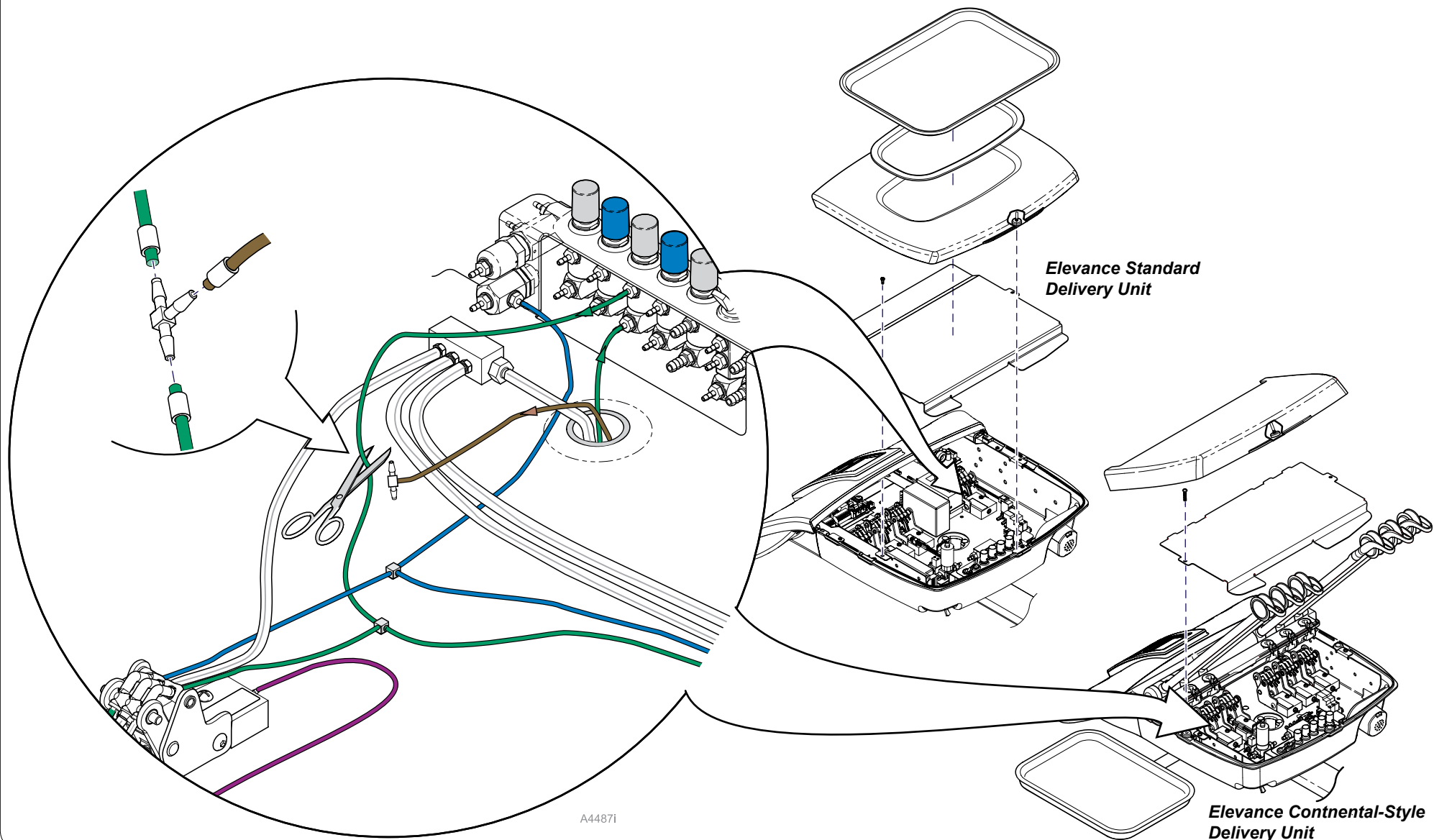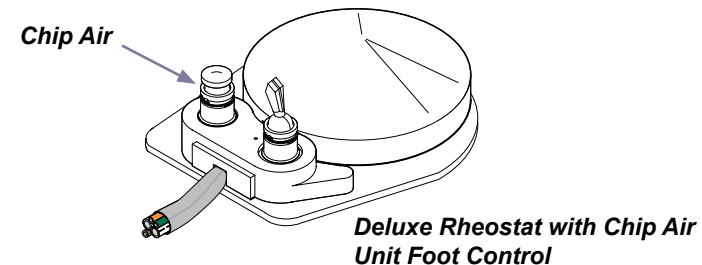- A) Cut green 1/8" Coolant Air Supply tubing.
- B) Locate brown 1/8" tubing coming into the unit head.
- C) Use barbed tee and sleeves to tee the 1/8" brown tubing into the 1/8" green coolant air tubing.

Continental-Style Units Only

Step 14: Install Tray Mount on Arm.

- A) Align tray mount assembly onto arm post.
- B) Install set screw and tighten to secure tray mount to arm post.
- C) Adjust drag on tray rotation with set screw.

002-1693-001 Kit

Tray

**002-1693-002 &
002-1693-003 Kits**

Silicon Pad

Continental-Style Units Only

Step 15: Level Tray Mount.

- A) Place a level on the flat surface of the tray mount.
- B) Alternately turn the three leveling screws while watching level.

Leveling Screws

Arm Post

Set Screw

DA4031i

Continental-Style Units Only

Step 16: Install Pad and/or Tray.

- A) For 002-1693-001 Kit: Set tray on tray mount
- B) For 002-1693-002 & 003 Kits: Place silicon pad onto tray mount.
(Trays are not included in these kits.)

If there are no additional components to install...

Step 17: Install covers and connect power supply.

- A) Install all covers.
- B) Arrange components neatly inside J-Box.
- C) Connect power supply and optional power adapters into facility power.
- D) Install J-box lid.
- E) Verify operation of the delivery.

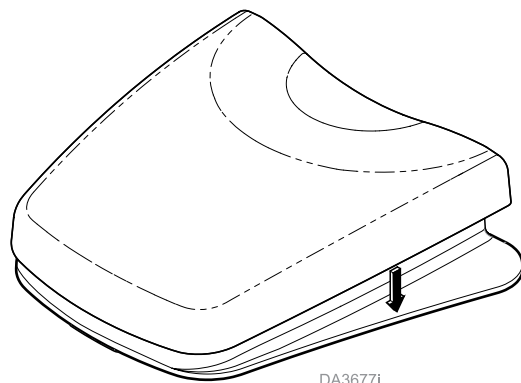
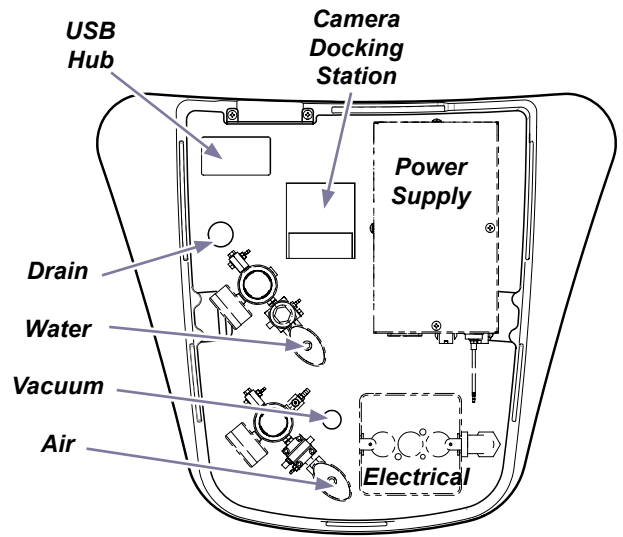
Note: Reference Elevance Delivery User Guide for more detailed operation information.

Equipment Alert

Once the delivery unit is powered ON, it must STAY ON for 30 minutes to charge the clock circuitry.

Installation Checklist

DA36771

Test	Verify the following:	✓
1	Flex arm and delivery unit are level. Trim ring is in place.	
2	Flex arm and delivery unit rotate easily and stay where placed.	
3	All covers are in place on all components and chair.	
3	All tubing is securely connected or capped if not used.	
4	Water and air supply regulator gauges in J-box read appropriate pressures.	
5	Electrical connectors and couplers are tightly connected, grounded and protected under the console cover.	
6	All electrical devices are properly connected to the power supply. Power supply is properly connected to facility power.	
7	Delivery unit comes ON when master switch is turned ON.	
8	Control screens appear as handpieces are pulled from the delivery unit.	
9	Control screens respond as buttons are pressed.	
10	Pressing footpedal delivers air/water to handpieces as expected.	
11	Wipe all components with a soft dry cloth to remove dirt and smudges.	

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