Anchor Bolt Installation Kit [002-1431-0x]

WARNING
Always disconnect power source before installation. Failure to do so may result in personal injury. Ensure appropriate floor thickness and strength before installation.


Components:

- Anchor 1
- Washer 1
- Lag Screw..... 1
- Anchor Bar..... 1


Special Tools:

- Concrete Floor
 - Roto-hammer drill
 - Hammer
 - 3/8" concrete drill bit
 - 9/16" socket and ratchet
- Non-concrete Floor
 - 1/4" drill bit

For Elevance Chairs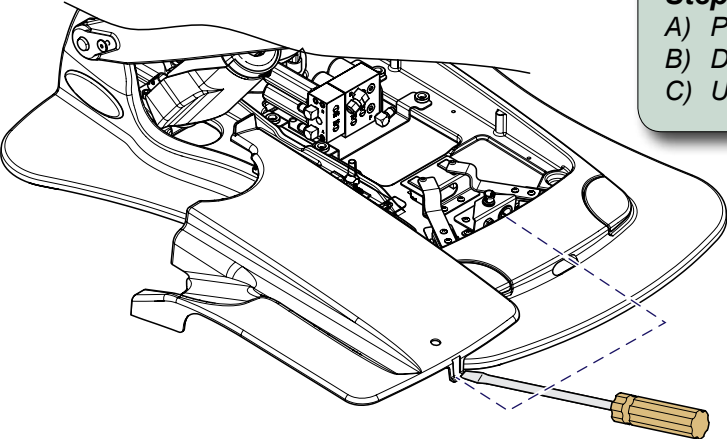

Hole Locations
(choose one)

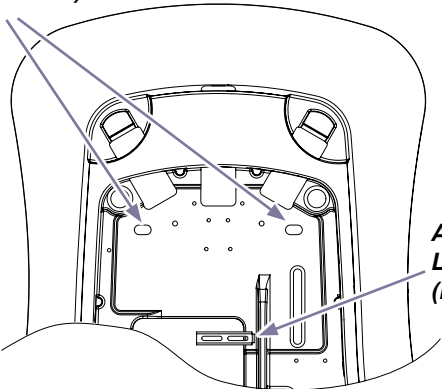Anchor Bar
Location
(if no holes)

DA2885i

Step 1a: Pre-installation...


- A) Position chair at highest height for maximum access.
- B) Disconnect power source.
- C) Use vacuum to keep work area clean and keep dust out.

For Elevance Chairs ONLY

Step 1b: Remove base cover.

- A) Release clip with a slotted screw driver and remove base cover.

For Ultra Chairs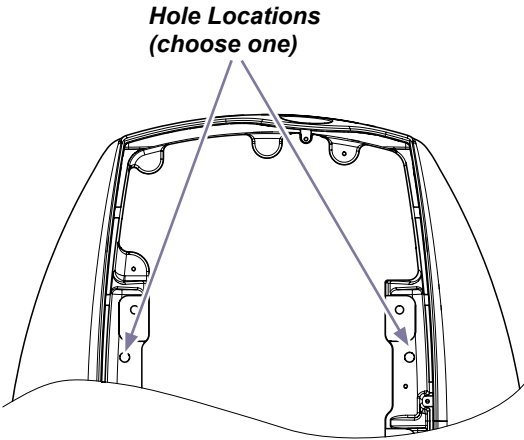


DA3716i

Step 2: Pre-drilling...

- A) Cover chair components near work area.
- B) Choose between anchor bar location or two hole locations (one hole location on one side).
- C) Mark one drill hole location.

Note: Choose an anchor (concrete floor) or lag screw (non-concrete floor).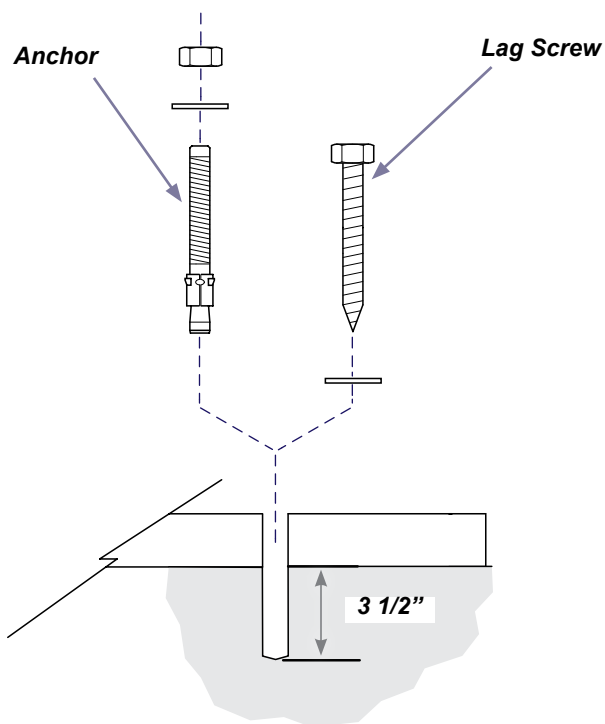

Step 3: Drill hole in floor.


- A) Use 3/8" concrete drill bit and roto-hammer drill (concrete) or 1/4" drill bit (non-concrete) to drill one hole 3 1/2" or 2 3/4" deep for anchor bar on marked location.
- B) Vacuum work area while drilling to reduce concrete dust.

Step 4a: Secure chair to concrete floor.

- A) Install one washer on anchor.
- B) Install nut on anchor even with anchor top.
- C) Tap anchor with hammer to install into hole.
- D) Tighten nut on anchor to secure.

Step 4b: Secure chair to concrete floor.

- A) Install one washer on lag screw.
- B) Install lag screw into hole.
- C) Tighten lag screw to secure.

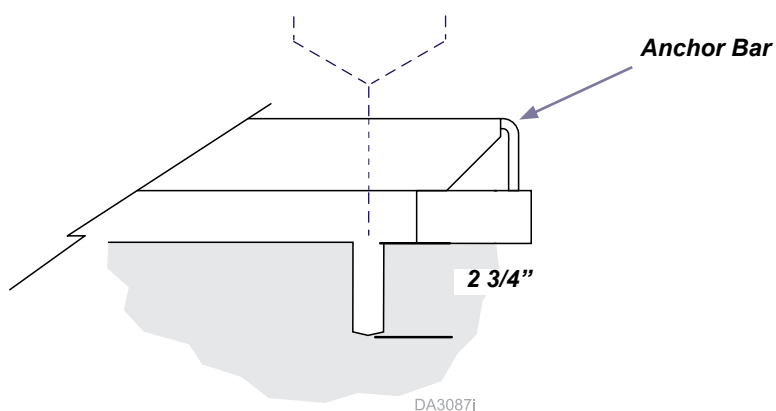Step 5: Complete installation.

- A) Clean work area.
- B) Install base cover - reverse Step 2.
- C) Connect chair to power source.
- D) Check chair operations.

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