

# V100 Drive Unit (VFD) Replacement [# 002-1447-00]

**Components:**  
VFD box..... 1

**Special Tools:**  
none





## WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

### To replace VFD box...

- Turn power off at on/off switch and main power supply box.
- Remove electrical panel from base unit.
- Remove VFD box covers and disconnect wires.
- Remove VFD box from base unit.
- Install new VFD box into base unit.
- Reverse steps A thru D to complete installation.

Note: Refer to wire diagram on next page when reconnecting VFD box.  
Also refer to base unit wire diagram on inside of electrical panel.

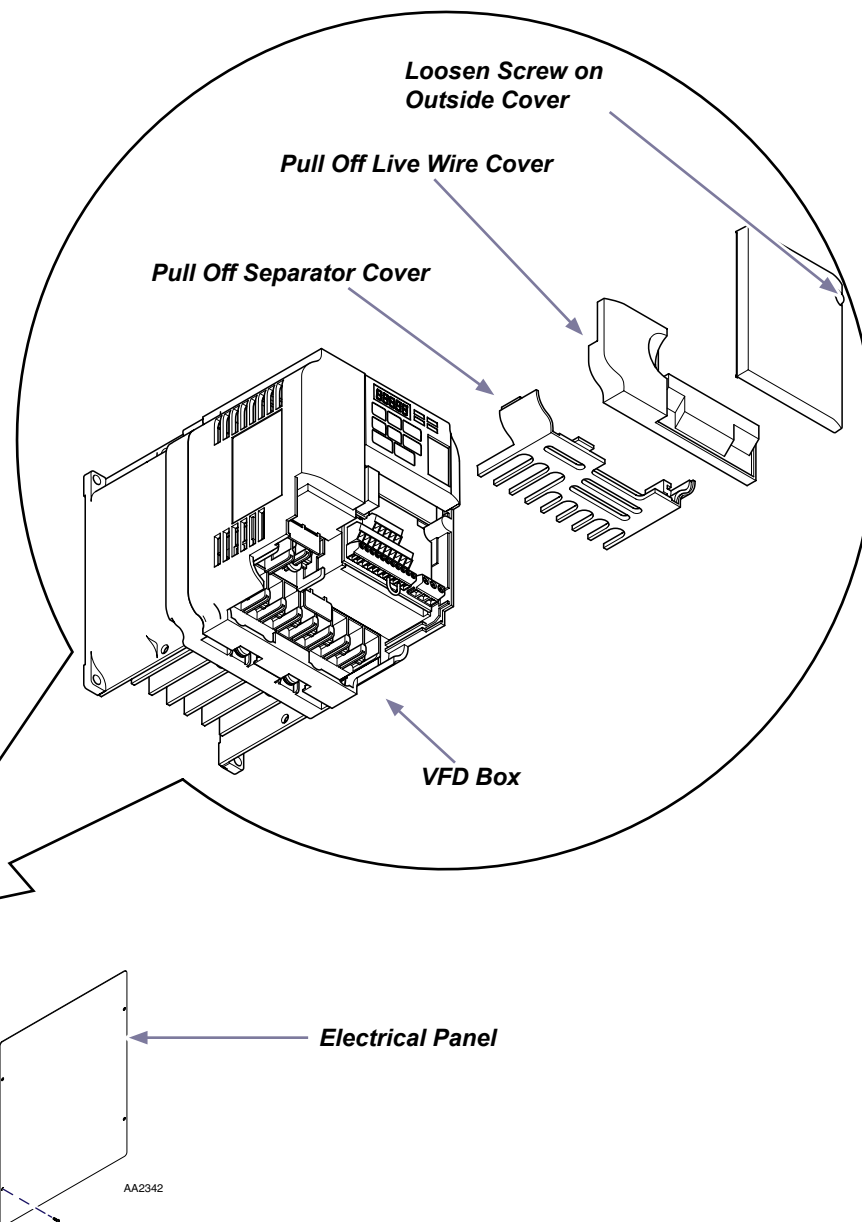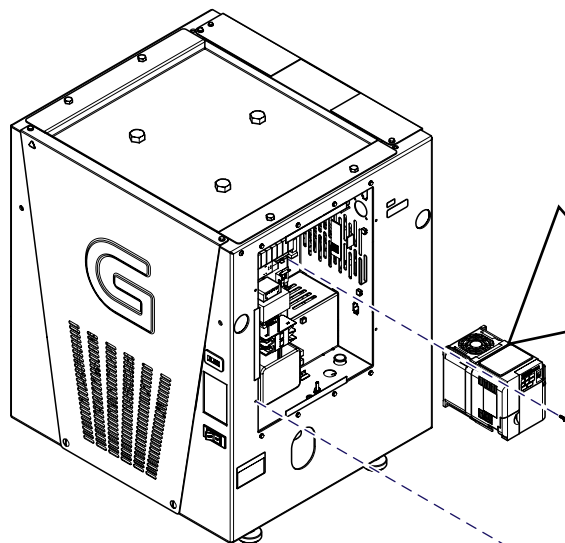

AA2342

## VFD Wire diagram

