

# PC Board & Daughter Card Replacement [# 002-1445-00]

**Components:**  
 PC board ..... 1  
 Daughter Card ..... 1  
 Timer board ..... 1  
**Special Tools:**  
 none



## WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

### To replace PC Board...

- Turn power off at on/off switch and main power supply box.
- Remove electrical panel from base unit.
- Unplug wire harnesses from PC Board.
- Squeeze standoff and raise up clips to release PC Board.
- Install new daughter card on PC Board.
- Install new Timer on PC Board.
- Install new PC Board into base unit.
- Reverse steps A thru D to complete installation.

Note: Refer to wire diagram on next page when reconnecting PC Board.  
 Also refer to base unit wire diagram on inside of electrical panel.



### Equipment Alert

Use care to prevent static discharge when handling PC Boards.



