

# Daughter Card Replacement [# 002-1461-00]

**Components:**  
Daughter Card..... 1

**Special Tools:**  
none




## **WARNING**

*Equipment is not suitable for use in the presence of a flammable anesthetic mixture.*

### **To replace PC Board...**

- Turn power off at on/off switch and main power supply box.
- Remove electrical panel from base unit.
- Unplug daughter card from PC board.
- Plug new daughter card into PC board.
- Install electrical panel and turn power on.

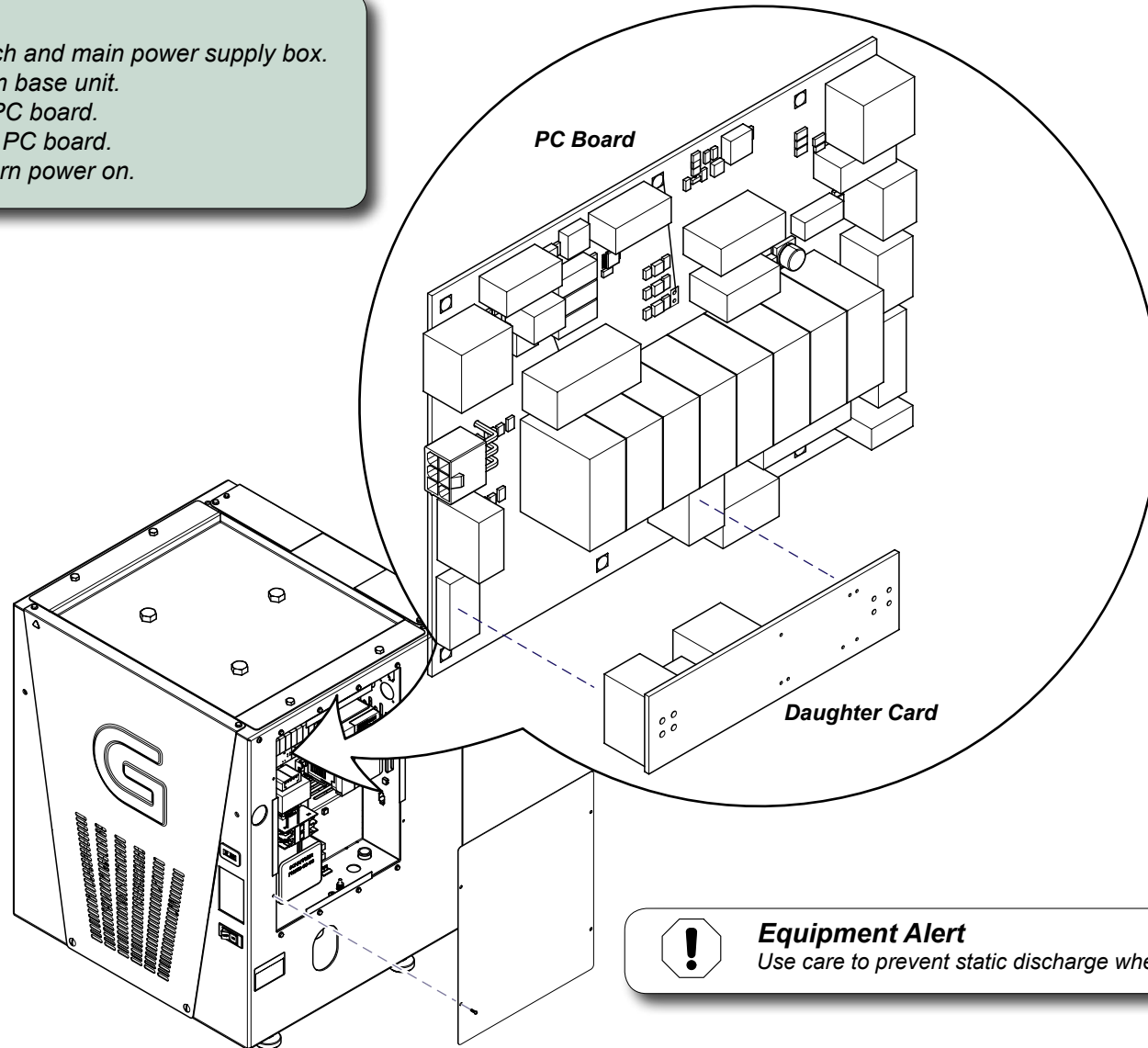

## **Equipment Alert**

*Use care to prevent static discharge when handling PC Boards.*