

Kit Components:

Massage Adapter 1
 Heat Adapter 1
 Connector Clips 3

Language of origin: English

Step 1: Position chair and disconnect power.

- A) Raise chair to a comfortable height to remove back.
- B) Disconnect chair from power supply.

Step 2: Remove back upholstery.

- A) Remove two screws in arms.
- B) Lift up on upholstered back to free from mounting pins.
- C) Disconnect cables and discard back upholstery.

Step 3: Connect and install new back upholstery.

- A) Connect adapter cables to new upholstered back and to chair.
- B) Route excess cables around top mounting pin.
- C) Install back upholstery. (see Step 2) Be careful not to pinch cables between chair and upholstered back.
- D) Connect chair to power supply.
- E) Test heat and massage operations to verify connections are good.

