



## Step 1: Remove back cover.

- Turn power off at on/off switch and disconnect box.
- Remove ground screw.
- Unfasten back cover by turning two fasteners 1/4 turn, counter clockwise.
- Lift back cover straight up to loosen from side panels.
- Unplug fan harness at electrical panel box.


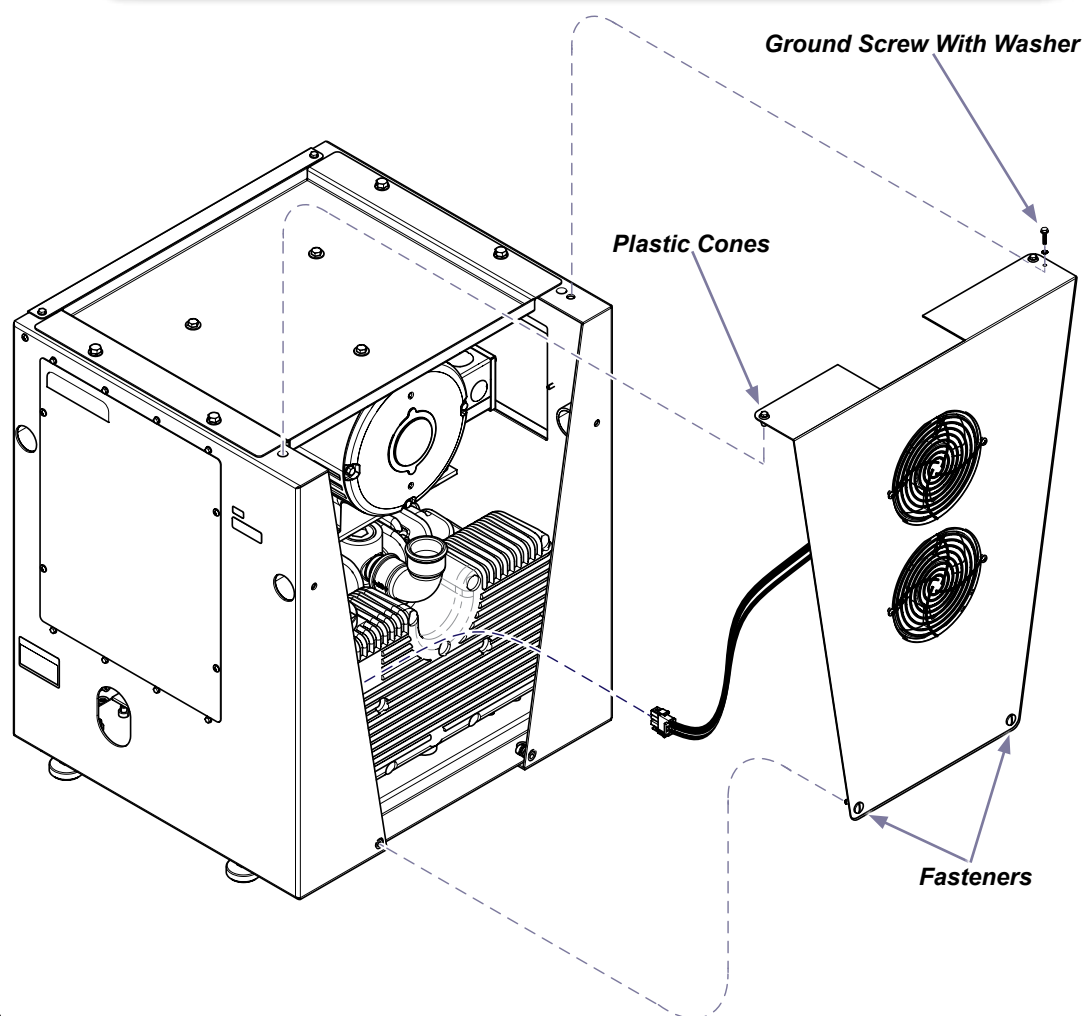


## Step 2: Remove electrical cover.

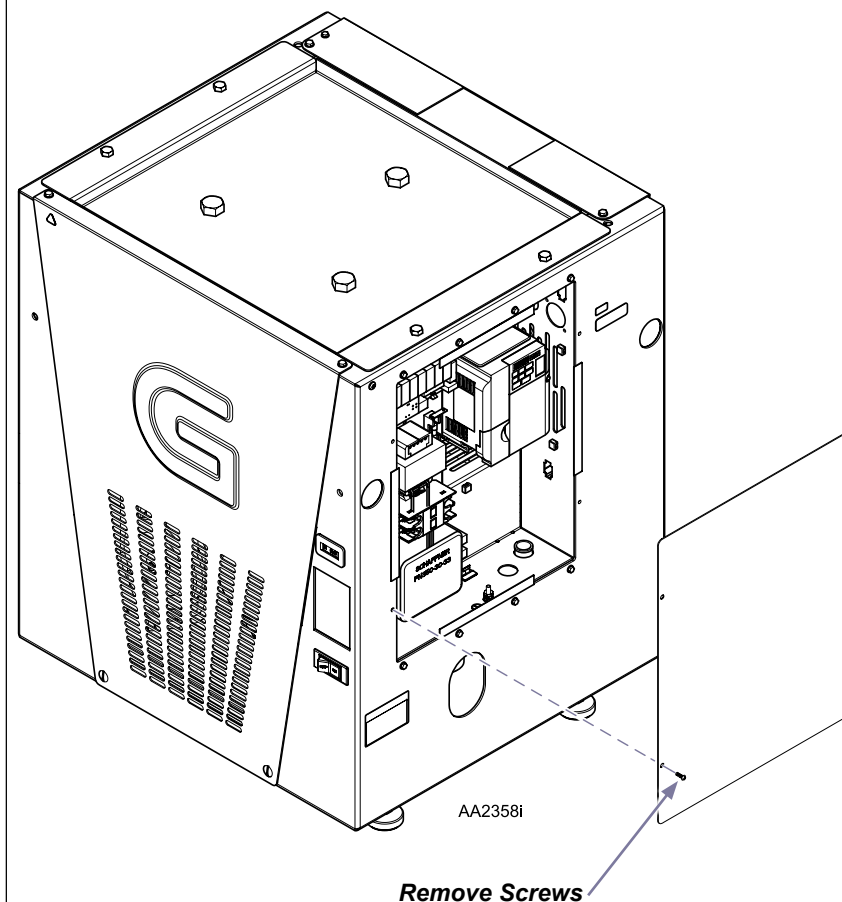

### Step 3: Replace jumpers.

- A) Remove panel mount connector from inside electrical panel box.
- B) Disconnect both jumpers at thermostat and contactor. Verify label part numbers on wires.
- C) Install new jumpers. **MUST** connect to opposite side of contactor. Refer to diagram on new label.
- D) Install new diagram label over old one on back cover.
- E) Reconnect fan jumpers from electrical panel to fans. Install electrical and back covers.

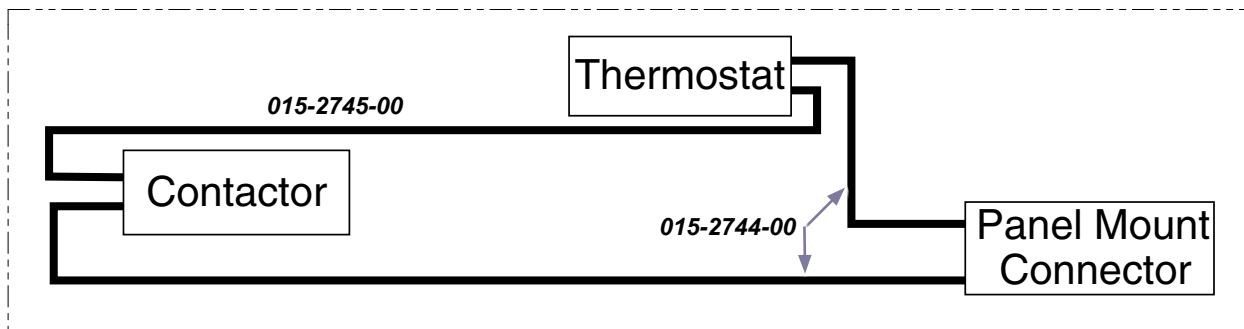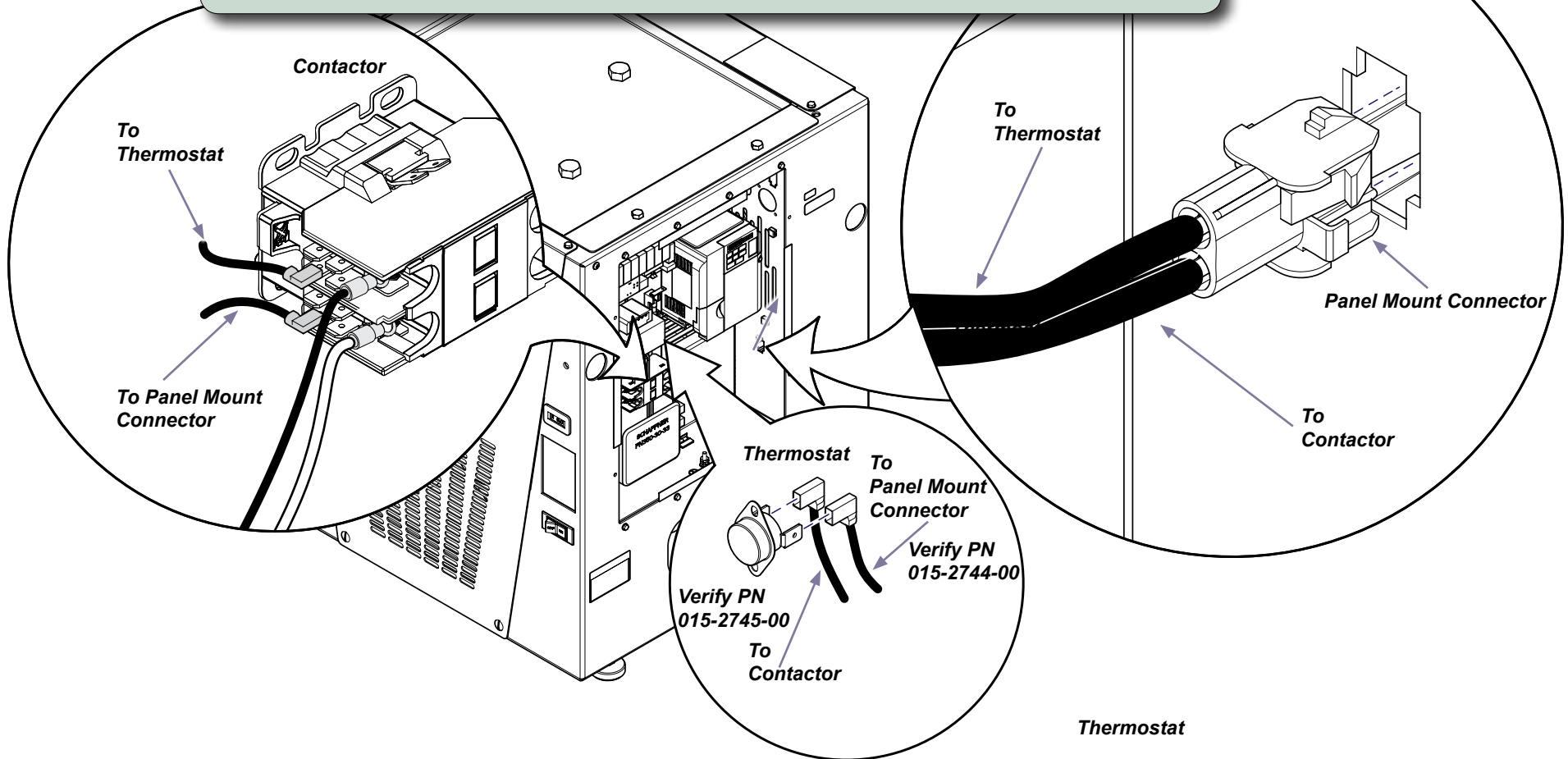

AA2359