

Filler Panel Installation [002-1504-00-xxx]


Components:

Filler Panel.....	1
Bracket	1
Screws.....	5

Step 1: Install mounting bracket.


- Position cabinet so side of unit is accessible.
- Attach mounting bracket as shown.

Note: For left hand filler, refer to Note before going to Step 3.

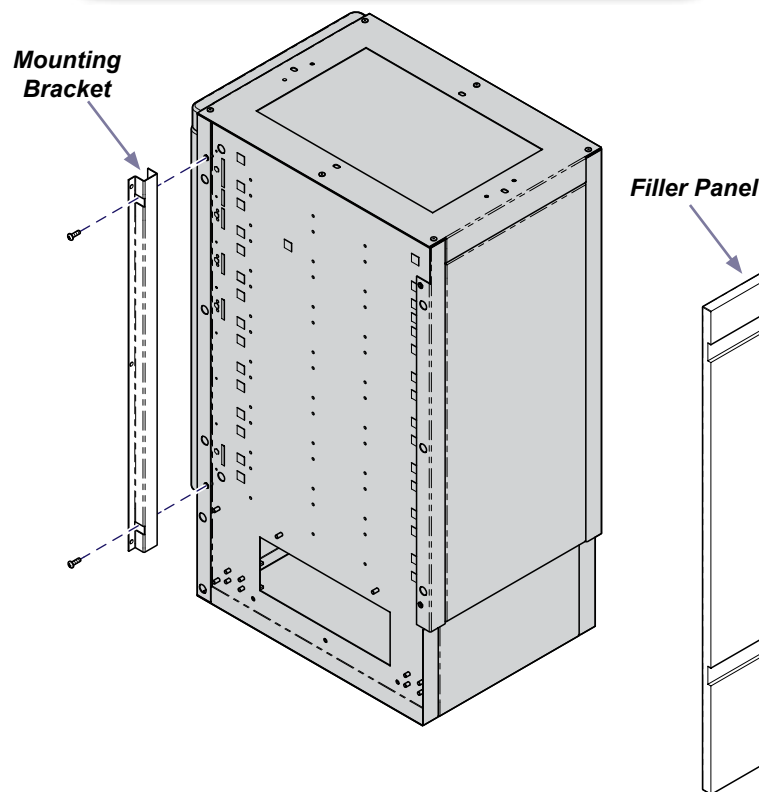


Step 2: Cut Filler Panel to size.

- Temporarily position cabinet in place.
- Determine size of filler panel.
- Cut filler panel to size.

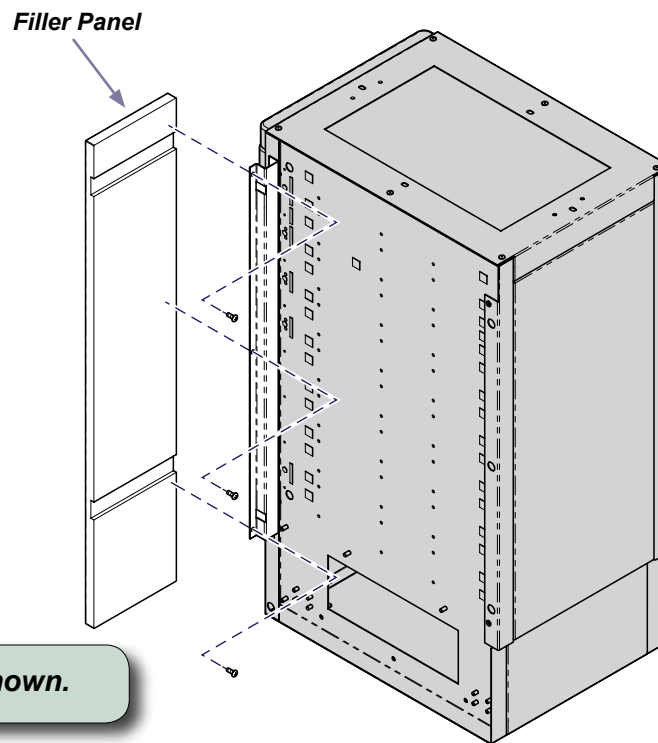

Step 3: Attach filler panel as shown.

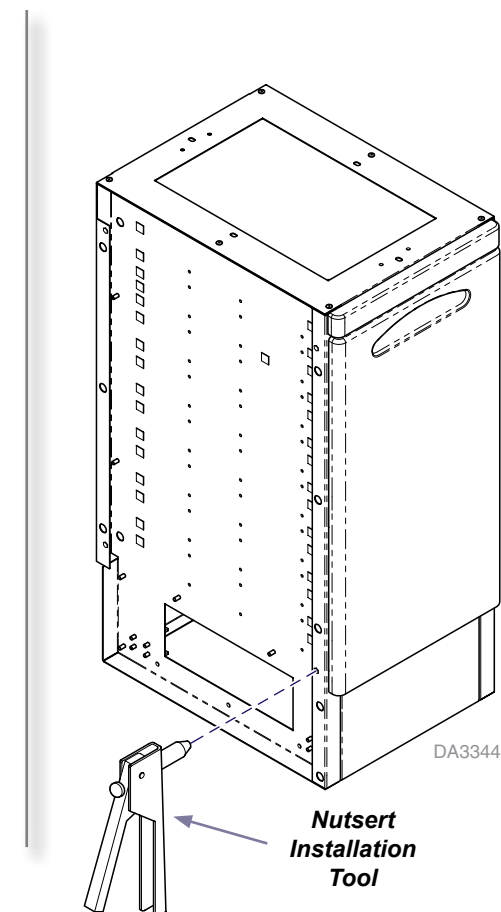

Note

If filler panel is attached to left side of cabinet, nutserts will need to be installed into cabinet. (Special nutsert installation tool necessary, contact Midmark for information.)