# Barrier-Free<sup>®</sup> Exam Table with IQscale<sup>®</sup> and IQhub<sup>®</sup>

[English](#)  
[Español](#)  
[Français](#)

For Model:  
625 (-004)

## User's Guide

Style P

TP202 20-42-FO-00014 Rev A1 C2169

003-2425-99 Rev.CA1 (2/7/18)

# Product Information

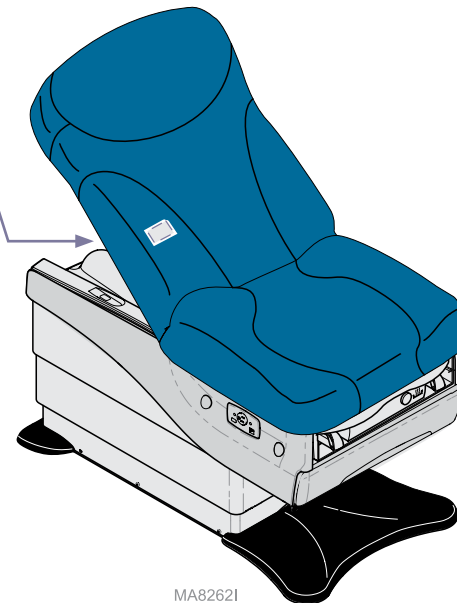
**Dealer :**

**Date of Purchase:**

**Model / Serial Number:**

**Midmark Authorized Service  
Company:**

**Model / Serial Number Label  
Located on the Back Mounting Frame**  
(exact location may vary)

MA8262I

# Table of Contents

## **Important Information**

<u>Safety Symbols</u> .....	4
<u>Transportation / Storage Conditions</u> .....	4
<u>Compliance Information</u> .....	5
<u>Intended Use</u> .....	6
<u>Electrical Requirements</u> .....	6
<u>Electromagnetic Interference</u> .....	6
<u>Disposal of Equipment</u> .....	6
<u>Digital Scale Accuracy</u> .....	7
<u>Safety Instructions</u> .....	7

## **Operation**

<u>IQscale® Operation</u> .....	9
<u>Base / Back Functions</u> .....	14
<u>Home Function</u> .....	15
<u>Quick Exam® Function</u> .....	16
<u>Exam Assistant® Drawer System</u> .....	17
<u>Paper Roll &amp; Tear Strap</u> .....	18
<u>Stirrups</u> .....	19
<u>Footrest &amp; Treatment Pan</u> .....	20
<u>Drawer Heater</u> .....	20
<u>Pelvic Tilt</u> .....	21
<u>Simplex Receptacles</u> .....	22
<u>IQhub® Operation</u> .....	23

## **Maintenance**

<u>Calling for Service</u> .....	27
<u>Cleaning</u> .....	27
<u>Preventative Maintenance</u> .....	27
<u>Troubleshooting</u> .....	28

## **Specifications**

<u>Weights, Dimensions, Electrical Specifications</u> .....	29
<u>Model Identification / Compliance Chart</u> .....	30
<u>Range of Motion &amp; Dimensions</u> .....	31

## **Warranty Information**

<u>Limited Warranty</u> .....	33
-------------------------------	----

# Important Information

## Safety Symbols

 **DANGER**  
Indicates an imminently hazardous situation which will result in serious or fatal injury. This symbol is used only in the most extreme conditions.

 **WARNING**  
Indicates a potentially hazardous situation which could result in serious injury.

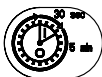
 **Caution**  
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

 **Equipment Alert**  
Indicates a potentially hazardous situation which could result in equipment damage.

**Note**  
Amplifies a procedure, practice, or condition.

 Consult User Guide for important information

 Protective earth ground

 Duty Cycle (motor run time):  
30 seconds ON, 5 minutes OFF

 Fuse rating specification

 Keep dry

 Type B, Applied Part

 Proper Shipping Orientation

 Maximum stacking height  
(palletted units)

 Fragile

 Emergency Stop

 Patient weight limit

## Transportation / Storage Conditions

Ambient Temperature Range:.....41°F to 104°F (5°C to 40°C)  
Relative Humidity.....10% to 90% (non-condensing)  
Atmospheric Pressure .....500hPa (.49 atm) to 1060hPa (1.04 atm)

# ***Compliance Information***

## ***FCC Compliance Statement***

This equipment has been tested and found to comply with the limits for a class A digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at their own expense.

Any modifications to this product may violate the rules of the Federal Communications Commission and make operation of the product unlawful.

## ***IC Compliance Statement***

### **Interference**

“This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.”

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### **Antennas**

“Under Industry Canada regulations, this radio transmitter may only operate using an antenna of a type and maximum (or lesser) gain approved for the transmitter by Industry Canada. To reduce potential radio interference to other users, the antenna type and its gain should be so chosen that the equivalent isotropically radiated power (e.i.r.p.) is not more than that necessary for successful communication.”

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## ***Intended Use***

The examination table is intended to be used as a chair / table to provide positioning and support of patients during general examination procedures conducted by medical professionals.

With the addition of the integrated digital scale, the examination table is additionally intended to be used for regular monitoring to assess patient weight change, as well as measuring patient weight to assist medical diagnosis. The integrated digital scale is not intended to be used for patients that weigh less than 30 pounds (13.6 kilograms) or more than 650 pounds (295 kilograms).

## ***Electrical Requirements***

*Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.*

### **Caution**

*When using high frequency surgical devices or endocardial catheters:*

- *Use non-conductive material to insulate patient from metal portions of the chair.*
- *Consult operating instructions for the device before using in conjunction with drawer or upholstery heaters.*

*Failure to comply may result in electric shock or burns to patient.*

## ***Electromagnetic Interference***

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room
- Plug chair into isolated circuit
- Increase separation between chair and interfering device
- Contact Midmark if interference persists

## ***Disposal of Equipment***

At the end of product life, the chair, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

## Digital Scale Accuracy

### Equipment Alert

Before using the digital scale, verify the scale is measuring accurately using a reference weight of known value. In addition, refer to the Preventive Maintenance Schedule for the recommended frequency of this check. If the scale is determined to be out of calibration, contact your authorized Midmark service provider before using the scale.

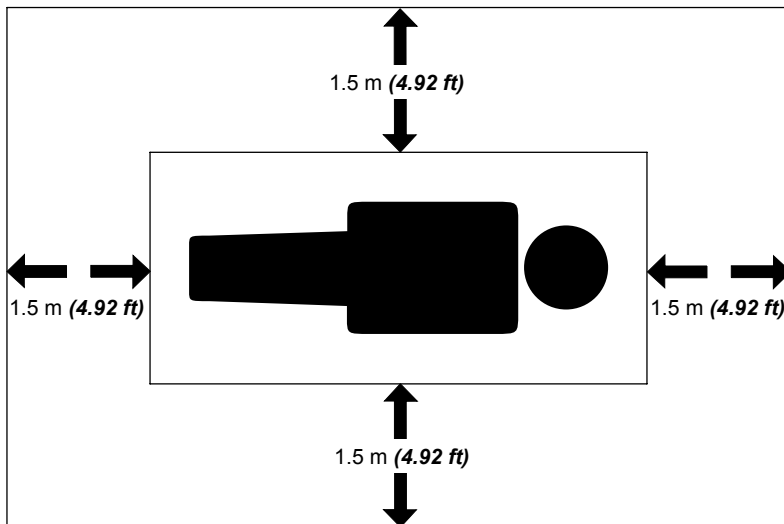
## Safety Instructions

### WARNING

**Risk of Electrical Shock Hazard: Accessory equipment connected to the SIP/SOP port must be certified according to the respective IEC standards, i.e., IEC 60950 for data processing equipment or IEC 60601-1 for electromedical equipment. All combinations of equipment must be in compliance with IEC 60601-1 systems requirements. Anyone connecting additional equipment to the SIP/SOP port configures a medical system, and therefore is responsible that the system complies with the requirements of the system standard IEC 60601-1-1.**

**IEC 60950 approved Information Technology Equipment must be placed outside the "patient environment." The patient environment is defined as an area 1.5 m (4.92 feet) from the patient.**

MA83171

**The following options are recommended if there is a need to remediate system leakage current:**

- **Redundant protective earth connection shall be made to either the medical device or the other equipment.**
- **The other equipment shall be powered from a medical grade (IEC 60601-1 compliant) separating transformer.**

## Safety Instructions - continued

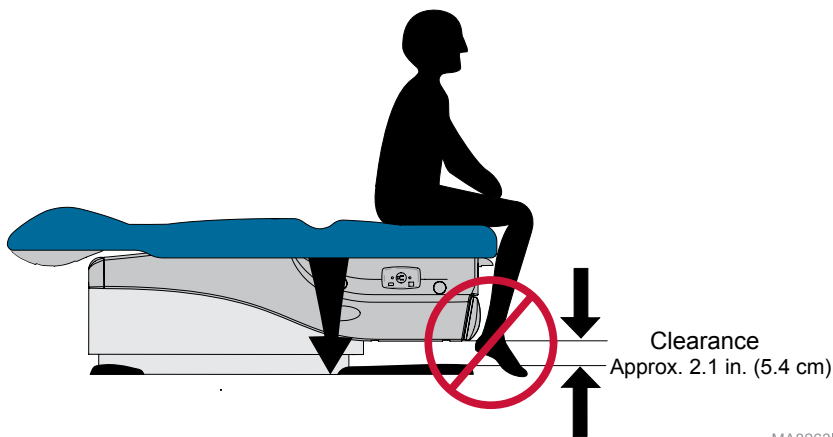
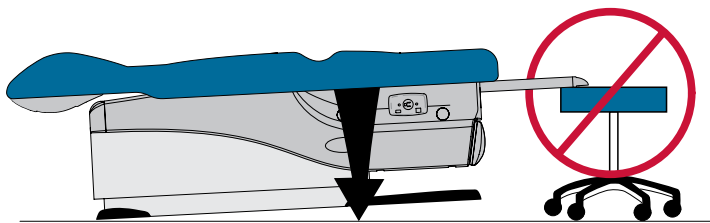
### Caution

Be sure that all personnel and equipment are clear of the table before activating any function. Failure to do so could result in personal injury.

At the table's lowest height [18 in. (45.7 cm)], the clearance between the bottom of the drawer and the floor is approximately 2.1 in. (5.4 cm). Make sure patient (esp. feet) and all objects are clear of this area before lowering table.

If the table malfunctions, immediately release foot pedal, unplug the table power cord, and assist the patient from the table. If table continues to malfunction, call for service.

Failure to comply with these instructions may result in injury to patient and/or damage to equipment.

MA8263I

# Operation

## IQscale® Functions

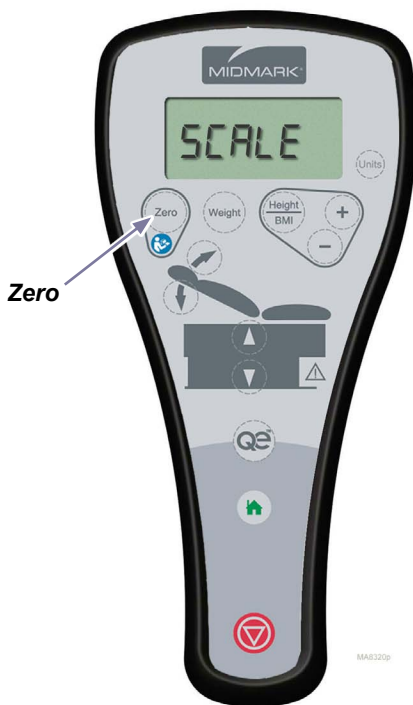
Best practice is to always zero the scale prior to weighing the patient. For details, please refer to the weighing guidelines in this manual.

### Note

*Before seating patient on exam table, ensure the scale has been zeroed. Failure to do so may affect the measurement of the patient's weight.*

### To zero the scale...

*Before seating patient, press and release the Zero button. The display will flash "Scale Ready"; the zero operation is complete and the patient may be seated.*

## IQscale® Functions - continued

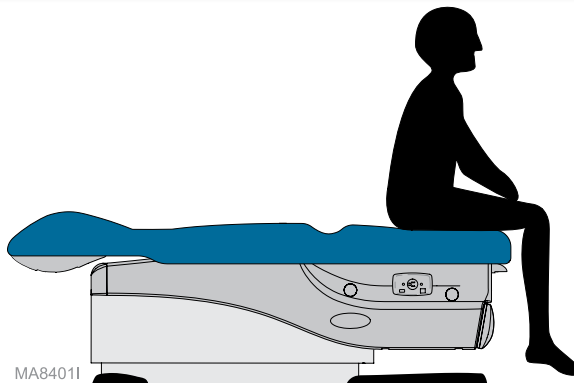
### Caution

The scale is intended for use with patients who weigh between 30 and 650 lbs (13.6 / 295 kg). For weights below 30 lbs (13.6 kg) or above 650 lbs (295 kg), the display will flash the weight reading to indicate the weight is out of range. Do not use an out of range (flashing) weight reading for patient measurements.

### To measure the patient's weight...

With patient in the seated position, press & release the Weight button.

Note: To prevent the patient's feet from touching the ground, the table will automatically rise to a height of twenty five inches prior to displaying patient weight.

### To calculate the patient's BMI...

- After patient weight is displayed, press and release the Height / BMI button.
- Press the (+) or (-) buttons to increase or decrease patient height on display.
- Press the Height / BMI button again to display patient BMI.

## Auto Zero Configuration

Best practice is to always zero the scale prior to weighing the patient. For details, please refer to the weighing guidelines on the next page.

In addition, at regular intervals the digital scale will zero itself automatically. This auto zero will only be performed when there are no patients present.

This auto zero function can be enabled or disabled if needed. The default factory setting is for auto zero to be enabled.

### Note

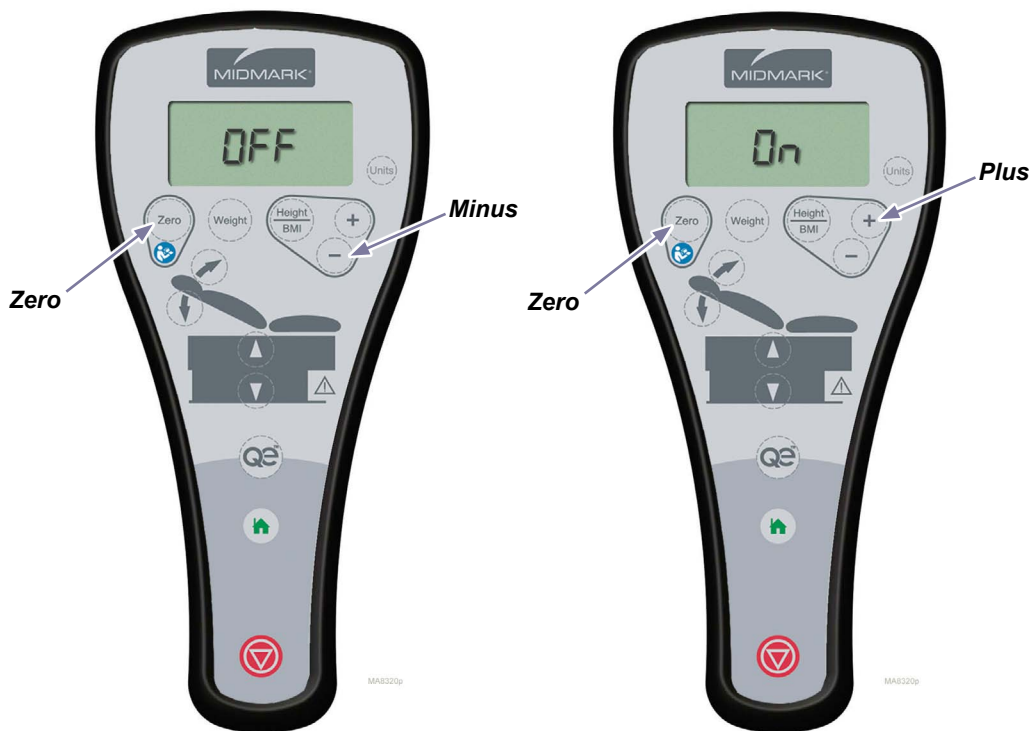
*Before seating patient on exam table, ensure the scale has been zeroed. Failure to do so may affect the measurement of the patient's weight.*

### To Disable:

- Press and hold the “Zero” and “Minus” buttons at the same time for five seconds.
- The display will flash “auto zero off”. Auto zero is now disabled and the scale is ready for use.

### To Re-enable:

- Press and hold the “Zero” and “Plus” buttons at the same time for five seconds.
- The display will flash “auto zero on”. The auto zero is now enabled and the scale is ready for use.

# Units Configuration

*(Applies to units with software version 141 and higher)*

## **Note**

To verify the software version, remove and reinstall the hand control batteries. The software version will be displayed momentarily on the hand control.

The Units key will be used to lock the units (English or Metric), and to unlock the units. The Units key will behave differently depending if the units are locked or unlocked.

### **If Units are unlocked:**

- When the display is blank releasing the Units key will show the current units indicator.
- When the display is showing only the units indicator then releasing the Units key will cause the Units indicator to toggle between displays showing inch/lb and cm/kg.
- When the display is showing only the units indicator then holding the Units key for 5 seconds will cause the LCD to display the following message 'UnitS','LOC',' ' repeat 3 times.
- When the display is showing Weight, pressing and releasing the Units key will toggle the units indicator between lb/kg and display the weight in the new units.
- When the display is showing Height, pressing and releasing the Units key will toggle the units indicator between in/cm and display the height in the new units.

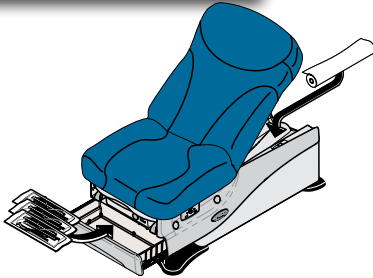
### **If Units are locked:**

- When the display is blank releasing the Units key will show the current Units indicator.
- When the display is showing only the units indicator then releasing the Units key will do nothing
- When the display is showing only the units indicator then holding the Units key for 5 seconds will cause the LCD to display the following message 'UnitS','UnLOC',' ' repeat 3 times.
- When the display is showing Weight, pressing the Units key will toggle the units indicator between lb/kg and display the weight in the new units. Releasing the Units key will toggle the units indicator back to the previous units.
- When the display is showing Height, pressing the Units key will toggle the units indicator between in/cm and display the height in the new units. Releasing the Units key will toggle the units indicator back to the previous units.


To ensure proper weighing of patients please review the following guidelines.


**Do** zero the scale after adding or removing supplies.

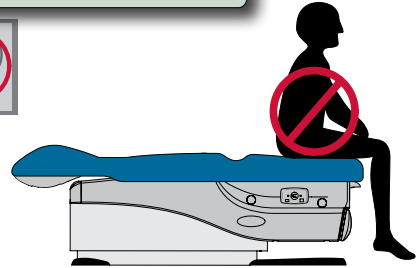
**Don't** zero the scale while the patient is on the table.

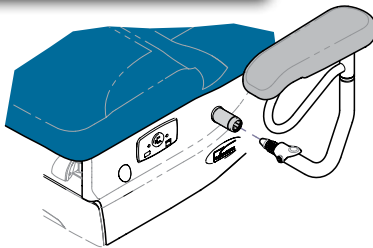
**Do** zero the scale after adding or removing equipment.

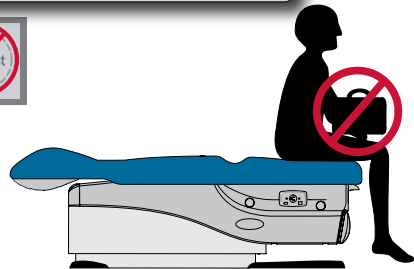
**Don't** weigh patients holding items such as purses, coats etc.

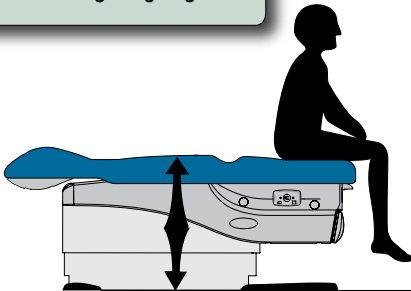**Do** ensure the patient's feet are off the floor during weighing.

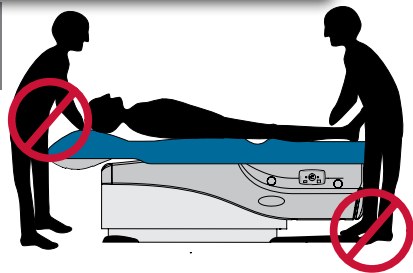**Don't** lean, step or otherwise apply load to table while measuring weight.

## Base / Back Functions

### To operate the Base / Back functions...

Press & hold the Up or Down arrow button for the desired function.

Note: The table automatically stops at its maximum and minimum height.  
In the event of a malfunction, press the Stop button to terminate movement.

### Equipment Alert

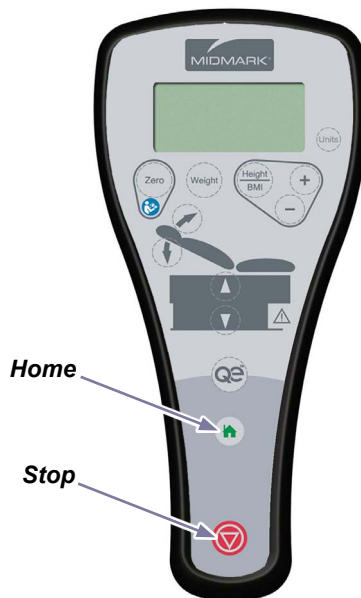
If the table's weight limit is exceeded, you will hear a "beep" and the function will be disabled. If the problem occurs with a normal load, contact your service technician.

## 'Home' Function

The Home function automatically lowers the table to approximately 20" (51 cm).

**To activate the Home function...**  
*Press & release the Home button.*

*Note: In the event of a malfunction, press the Stop button to terminate movement.*


## Quick Exam® Function

The Quick Exam® function automatically commands the table seat height to a programmed position.

### To program the Quick Exam® function...

- Press the Base Up button to move the table to the desired position.
- Press & hold the Quick Exam® & Base Up buttons simultaneously for three seconds.

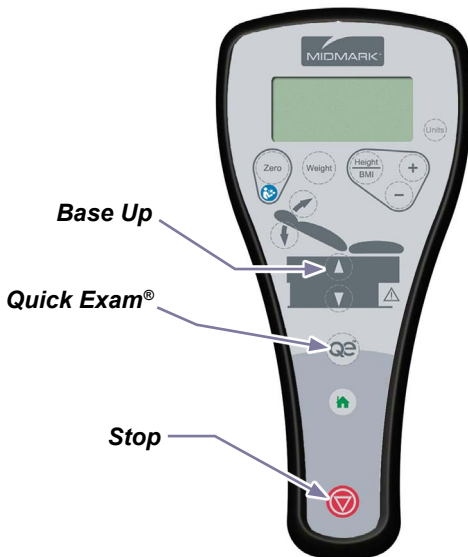Note: You will hear a brief chirp when the new Quick Exam® position is saved.

### To activate the Quick Exam® function...

Press & release the Quick Exam® button.

Note: In the event of a malfunction, press the Stop button to terminate movement.

### Equipment Alert

If the table's weight limit is exceeded, you will hear a "beep" and the function will be disabled. If the problem occurs with a normal load, contact your service technician.

# Exam Assistant® Drawer System

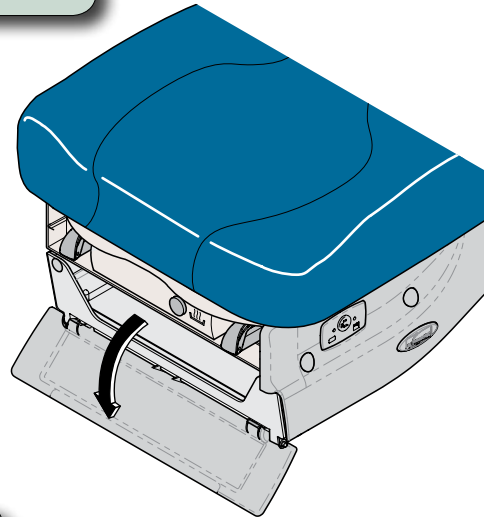
**To use the drawer front as a shelf...**

*Pull drawer front down to release the magnetic latch.*

**Equipment Alert**

*The maximum weight the shelf will support is 10 lbs (4.5 kg).*


**If drawer front folds past horizontal...**

*Pull drawer front upward until it snaps back into the horizontal position.*

**Equipment Alert**

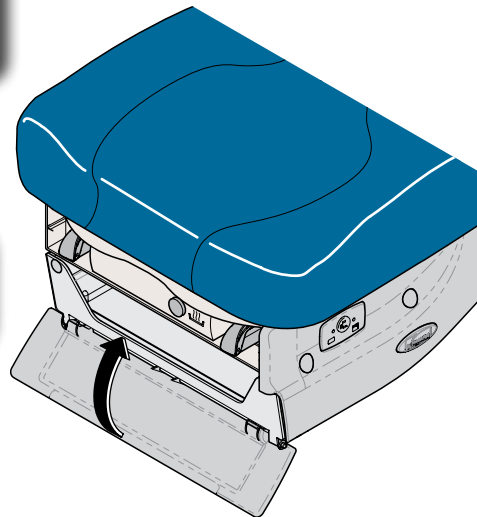*If drawer front folds past horizontal, do not operate table.*

MA8264I

## Paper Roll & Tear Strap

Upholstery sets are available in two widths: 28" (standard) & 32" (optional).  
Paper rolls of common size may be stored under the back section as follows:

<u>Uph. Width</u>	<u>Paper Roll Size</u>	<u>Max. Qty.</u>
28 in.	21 in. long x 3.0 in. diameter (53.3 cm x 7.6 cm).....	4
28 in.	21 in. long x 3.5 in. diameter (53.3 cm x 8.9 cm).....	2
32 in.	24 in. long x 3.0 in. diameter (70 cm x 7.6 cm).....	2
32 in.	24 in. long x 3.5 in. diameter (70 cm x 8.9 cm).....	1

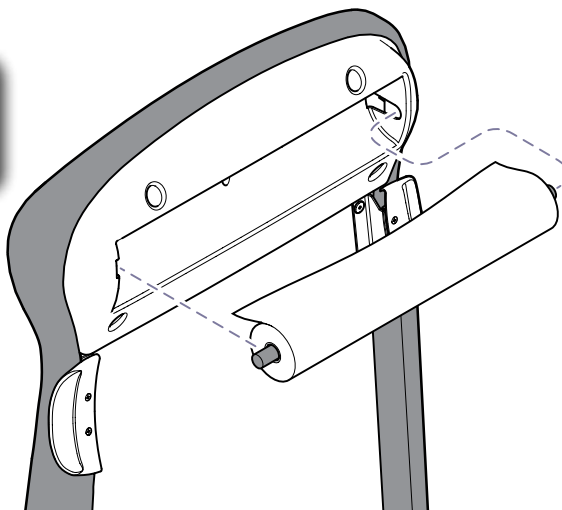
### Warning

To avoid injury, do not put head or extremities under the paper tear strap.


### To install paper roll...

Insert rod ends into sockets  
under head end of upholstery.

### To install paper tear strap...

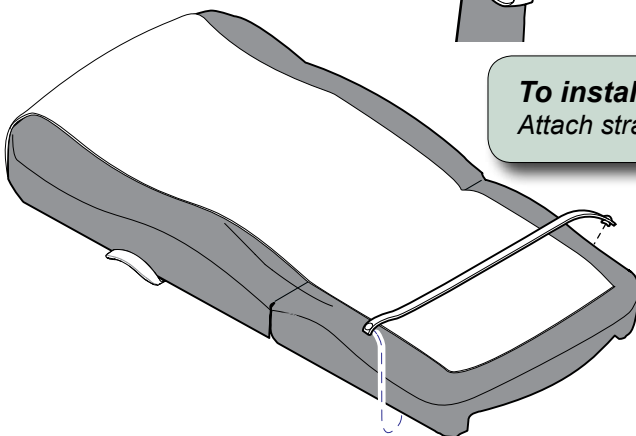Attach strap to snaps on each side of table.

MA8018i

# Stirrups

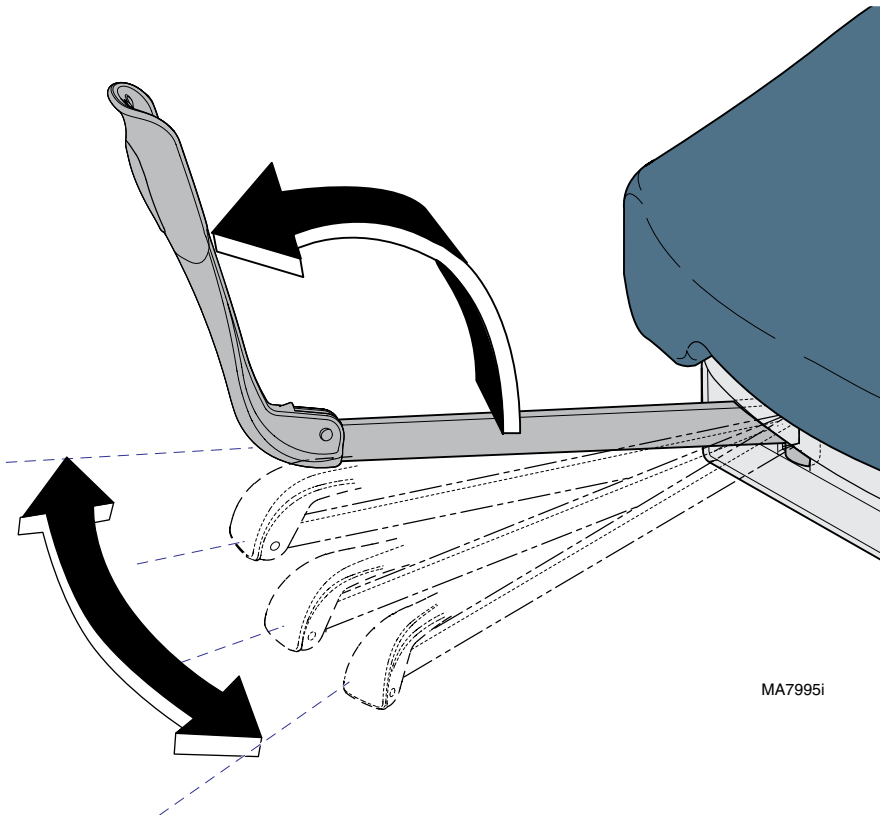
## **Caution**

*Be sure the stirrups are locked in place before using.  
The stirrups will not support the patient's entire weight.*

## **To position the stirrups...**

- A) *Pull the stirrup out, then unfold.*
- B) *Lift the stirrup slightly, then move it left or right as desired.*
- C) *Release stirrup to lock in position.*


MA7995i

## Footrest & Treatment Pan
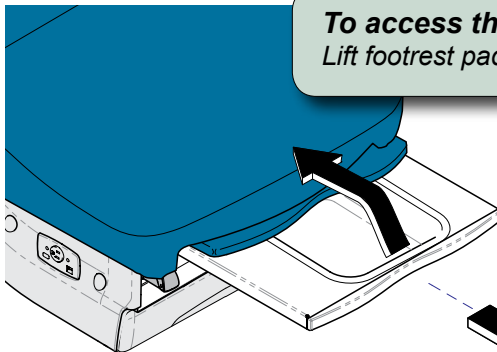
### **Caution**

*Do not use footrest to support patient's entire weight.*

### **To access the treatment pan...**

*Lift footrest pad slightly, then slide pad back into table.*

### **To extend footrest...**

*Pull footrest out to desired position.*


MA8265I

## Drawer Heater

The drawer heater warms the contents of the drawer to approximately 98°F (37°C).

### **To turn the drawer heater ON / OFF...**

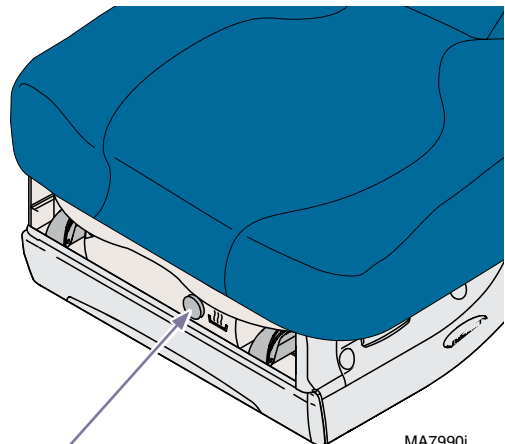*Press and release the drawer heater switch.  
(Switch will illuminate when heater is ON)*


MA7990i

**Drawer Heater  
Switch**

## **Pelvic Tilt**
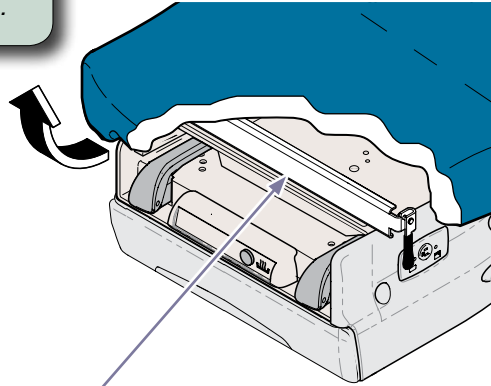
### **To elevate seat section...**


*Lift seat until pelvic support drops into place.*

**Pelvic Support**

### **To return seat section to flat position...**

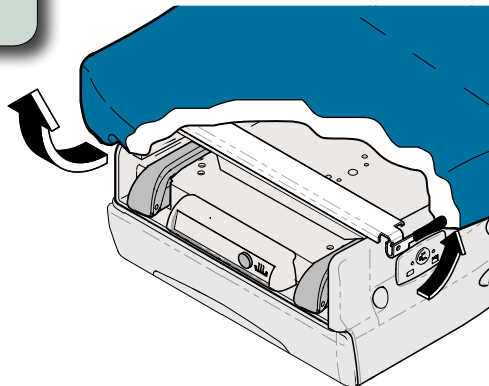- A) *Lift seat section slightly.*
- B) *Pivot pelvic support handle upward.*
- C) *Lower seat section.*


MA8266I

# Simplex Receptacles

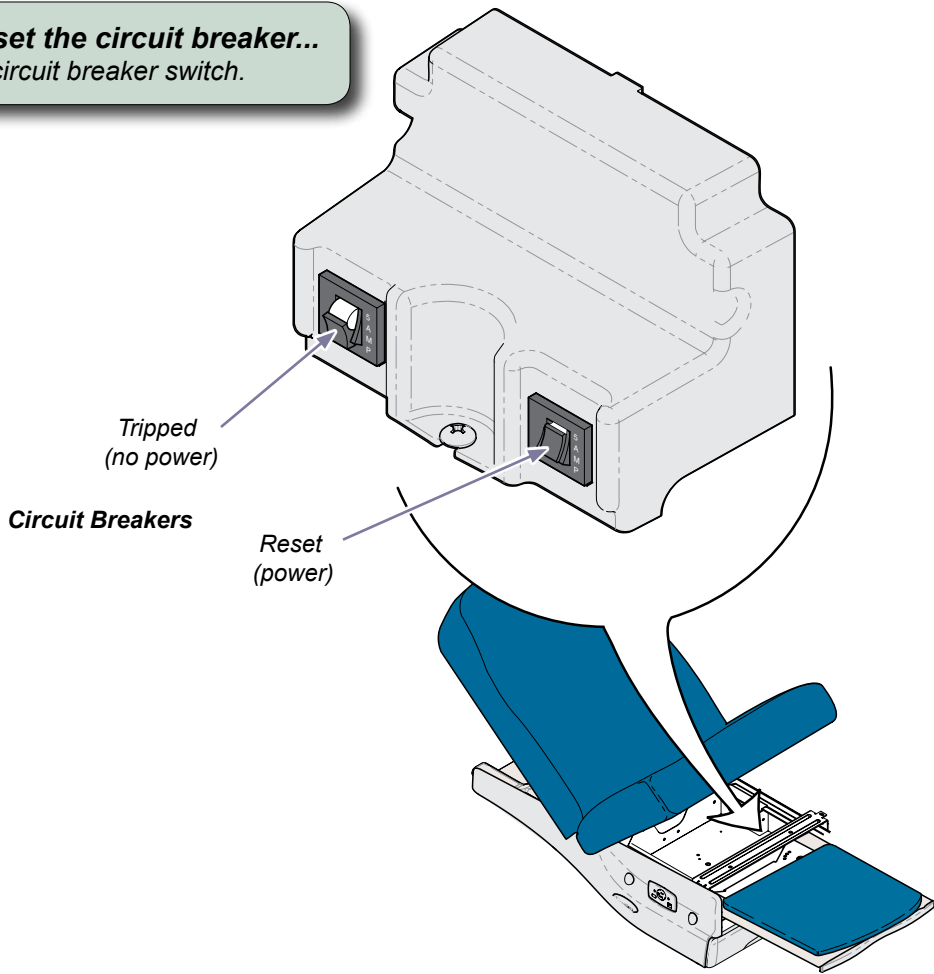Receptacles on each side of the table provide power for accessories used during medical procedures. There are two circuit breakers located under the seat section. If the receptacle's maximum load is exceeded, the circuit breakers interrupt power to the receptacle.

Maximum Load.....115 VAC, 5 amps

**To reset the circuit breaker...**  
Push circuit breaker switch.

MA82671

# ***IQhub® Operation***

***Data connections are located on each side of table and on rear base of table.***

**Ports 1, 2, 3, & 4:** USB connections.


**Ports 5, 6 & 7:** Vital Signs connections.

**Port 8:** Optional wired hand or foot control.

## **WARNING**

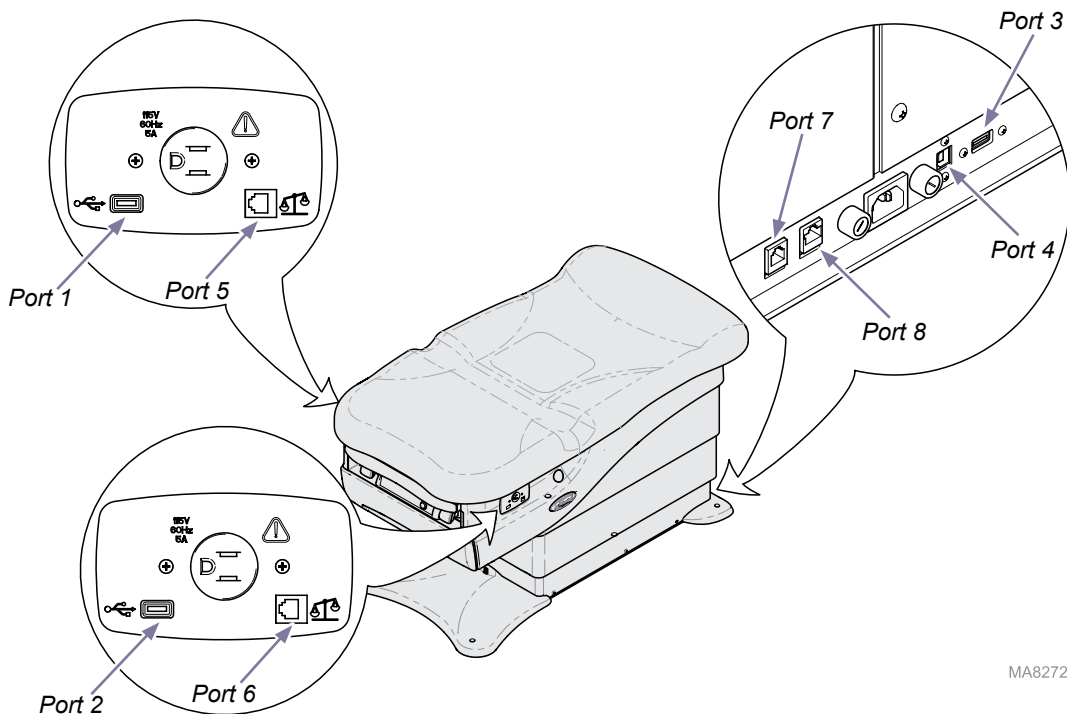***IEC 60950 approved Information Technology Equipment must be placed outside the "patient environment." Refer to safety instructions in this manual.***

MA8272i

# ***IQhub® Operation - continued***

## **USB Connections**

The four USB ports on the table work together as a high-speed USB hub, connecting up to three USB devices to a host computer at speeds up to 480 Mbit/s.

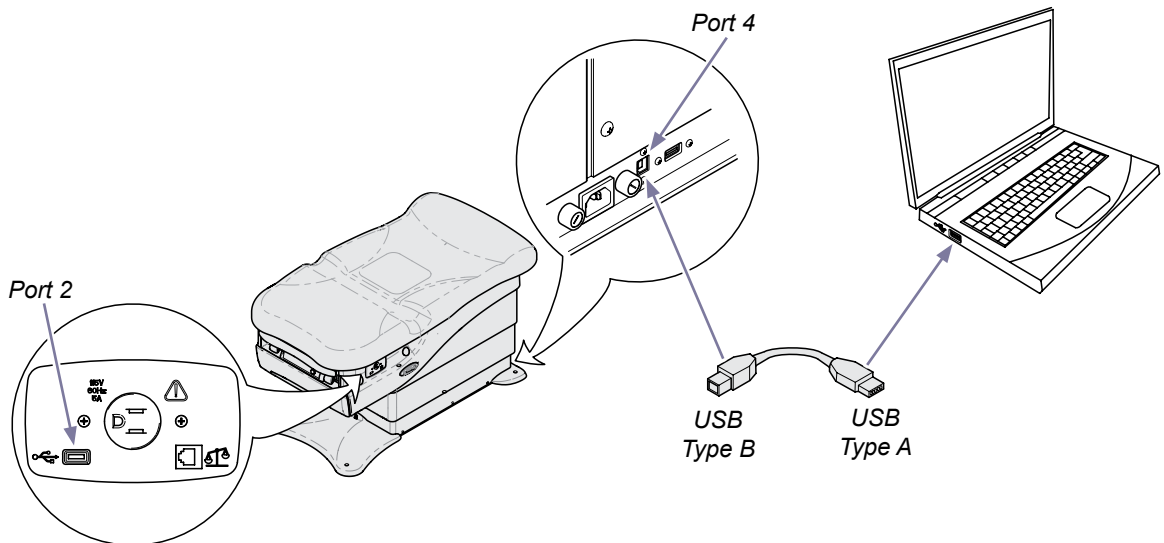
Once the USB hub is connected to the host computer, the digital scale will also connect. This will allow for digital transfer of measured weight to supported software.

### **Setup:**

- Using a standard USB cable\* (up to five meters in length), connect port 4 at the rear of the table to the host computer. A standard USB cable may also be ordered from Midmark (9A520001).
- If prompted, [click here](http://www.midmark.com/technicallibrary/Link-PDFs/625.html) to download the latest driver from [www.midmark.com](http://www.midmark.com). (<http://www.midmark.com/technicallibrary/Link-PDFs/625.html>)
- Connect additional USB devices to ports 1, 2, & 3 as needed. USB cables up to two meters in length may be used. Up to three additional devices may be connected simultaneously.

### **Note**

*Power is provided at the USB ports only when a host computer is connected to port 4. (e.g. Battery-powered devices will not charge via USB unless a host computer is first connected.)*

\* A standard USB cable has one Type A connector and one Type B connector. For best performance, use USB cables that are fully compliant with current USB specifications.

MA9943i

# ***IQhub® Operation - continued***

## ***Vital Signs Connections***

The serial ports on the table provide connection to a vital signs device. This allows for the patient weight to be captured from the digital scale and transferred to the vital signs device for display and capture.

Supported vital signs devices are:

- Midmark IQvitals®
- Welch Allyn® Spot Vital Signs® LXi \*

### **Setup:**

Purchase the appropriate serial cable for your equipment:

- Midmark IQvitals® Serial Cable, 6 foot length, coiled (9A478001).
- Midmark IQvitals® Serial Cable, 15 foot length, straight (9A478002).
- Midmark IQvitals® Serial Cable, 30 foot length, straight (9A478003).
- Midmark IQvitals® Serial Cable, 50 foot length, straight (9A478004).
- Welch Allyn® Spot Vital Signs® LXi Serial Cable, 6 foot length, coiled (9A479001).
- Welch Allyn® Spot Vital Signs® LXi Serial Cable, 15 foot length, straight (9A479002).

Connect one end of the vital signs connections (ports 5, 6, or 7) to one end of the serial cable, and the other to the corresponding port on your vital signs device.

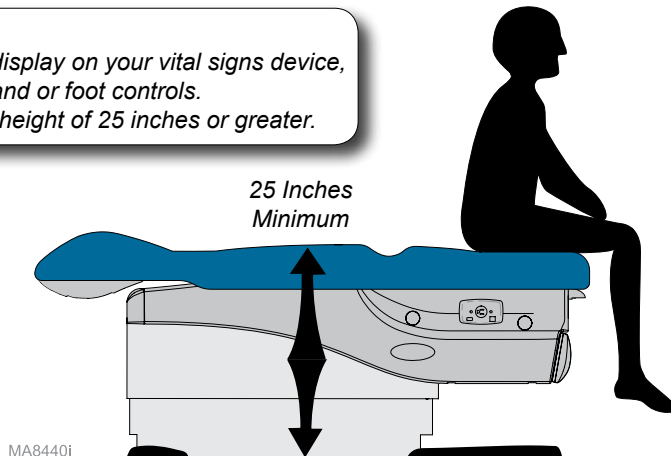
Patient weight will automatically be measured when the patient sits on the table.

### **Note**

*Before seating patient on exam table, ensure the scale has been zeroed.  
Failure to do so may affect the measurement of the patient's weight.*

### **Note**

*If weight does not display on your vital signs device,  
raise table using hand or foot controls.  
Table must be at a height of 25 inches or greater.*

\* Welch Allyn® and Spot Vital Signs® are registered trademarks of Welch Allyn, Inc.



# ***IQhub® Operation - continued***
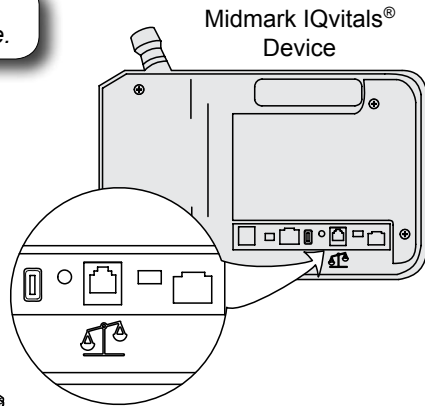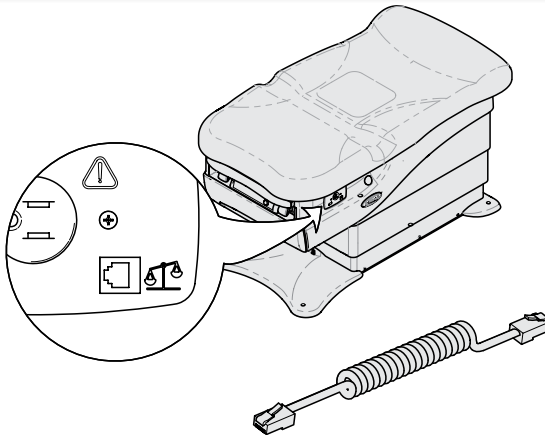
## ***Connecting Table to Vital Signs Device***



The serial cable may be connected to one of three ports located on the table.

**Note**

*Only one vital signs device may be connected at a time.*

MA84371

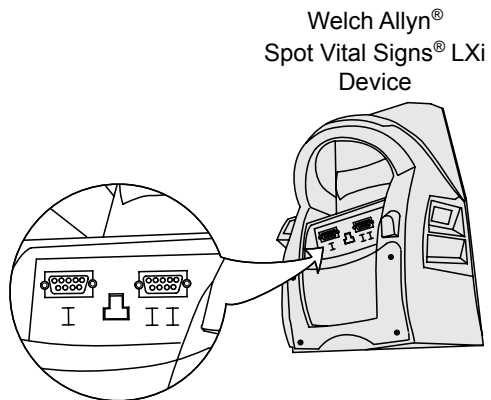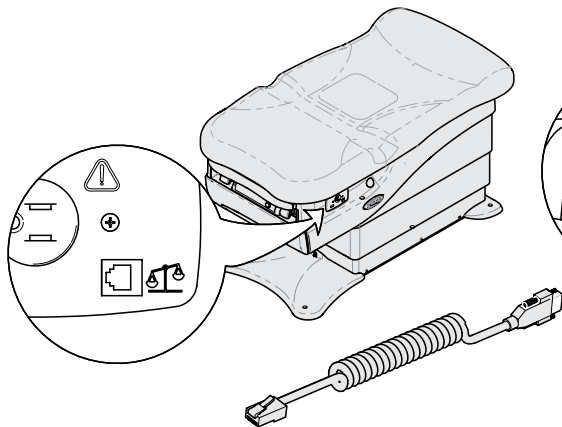MA84391

# Maintenance

## Calling for Service

### Note

Model / serial number information is required when calling for service.

If service is required, contact your authorized Midmark dealer.

To contact Midmark directly:

1-800-MIDMARK (1-800-643-6275)

8:00 am until 5:00 pm Monday - Friday (EST) [excluding standard U.S. holidays]

## Cleaning

### Upholstery

### Equipment Alert

*The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.*

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [www.midmark.com](http://www.midmark.com) in the Technical library under the User Information tab for your product.

### Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

## Preventative Maintenance

Periodically inspect the following areas:

- Power cord(s) should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

Periodically lubricate the back hinge to maintain quiet, smooth, operation (use light machine oil).

Have an authorized service technician inspect your equipment every six months.

Periodic calibration of the scale is not required. It is recommended to verify the scale is measuring accurately using a reference weight of known value every six months.

# Troubleshooting

Symptom	Probable Cause	Correction
No functions will operate.	Facility supply voltage.	Secure power cord connections. Reset facility circuit breaker.
Table stops and beeps.	Patient weight exceeded 650 lbs.	Inform staff that the maximum patient weight limit is 650 lbs.
No power at table receptacles. All other functions work.	Table circuit breaker(s) tripped.	Lift seat section to access circuit breaker(s), press to reset.
When any control button is pressed, that controller sounds a single beep. (All functions operate)	Low battery.	Replace batteries. ( <i>size: AA</i> )
When the weight button is pressed, the controller displays "Table Low".	Momentary power interruption.	Lower table all the way down.
Table does not move when pressing Quick Exam <sup>®</sup> button, or does not go to the correct height.	Quick Exam <sup>®</sup> height is user adjustable and may need to be reprogrammed.	See Quick Exam <sup>®</sup> instructions in this manual.

# Specifications

## Weights, Dimensions, Electrical Specifications

<b>Specifications Chart</b>	
<b>Patient Weight</b> ( <i>maximum</i> ):	650 lbs (295 kg)
<b>Scale Resolution:</b> <b>Scale Accuracy:</b> 30 - 100 lb (13.6 - 45.4 kg) 100 - 300 lb (45.4 - 136.1 kg) 300 - 500 lb (136.1 - 226.8 kg) 500 - 650 lb (226.8 - 294.8 kg)	0.2 lb (0.1kg)  ± 0.2 lb (0.1 kg) ± 0.4 lb (0.2 kg) ± 0.6 lb (0.3 kg) ± 0.8 lb (0.4 kg)
<b>Paper Roll</b> ( <i>maximum size</i> ):	(See <i>Paper Roll &amp; Tear Strap page</i> )
<b>Range of Motion &amp; Dimensions:</b>	(See <i>Range of Motion &amp; Dimensions page</i> )
<b>Weight of Table:</b> <b>w/upholstery</b> <b>w/packaging &amp; skid</b> ( <i>no uph.</i> ) <b>Uph. w/packaging</b> ( <i>shipped separately</i> )	475 lbs (216 kg) 492 lbs (224 kg) 45 lbs (20 kg)
<b>Power Cord Length:</b>	8 ft. (244 cm)
<b>Electrical Requirements:</b>	(See <i>Regulatory Compliance Charts</i> )
<b>Foot / Hand Control Voltage:</b>	14 VAC, SELV (Safety Extra Low Voltage)
<b>Simplex Receptacles</b> ( <i>maximum load</i> )	115 VAC, 5A, 50/60 Hz
<b>Fuses:</b> <i>IEC inlet fuses (2)</i> <i>F1 &amp; F2 (on main PC board):</i> <i>F3 (on main PC board):</i> <i>F4 (on main PC board)</i> <u><i>Models w/ drawer heaters</i></u> <i>distribution board fuses (2)</i>	6.3A, 250V, Type-T, 5 x 20 mm 10A, 250V, Type-T, 5 x 20 mm 160mA, 250V, Type-T, 5 x 20 mm 6.3A, 250V, Type-T, 5 x 20 mm  800mA, 250V, Type-T, 5 x 20 mm
<b>Duty Cycle</b> <b>(Motor Run Time):</b>	Intermittent Operation <i>[30 seconds ON / 5 minutes OFF]</i>
<b>Receptacles, Drawer Heater:</b>	<i>Continuous Operation</i>
<b>Classifications:</b>	Class 1, Type B Applied Part
<b>Protection against ingress of fluids:</b>	IPXO [ <i>Foot Control only: IPX1</i> ]

### **WARNING**

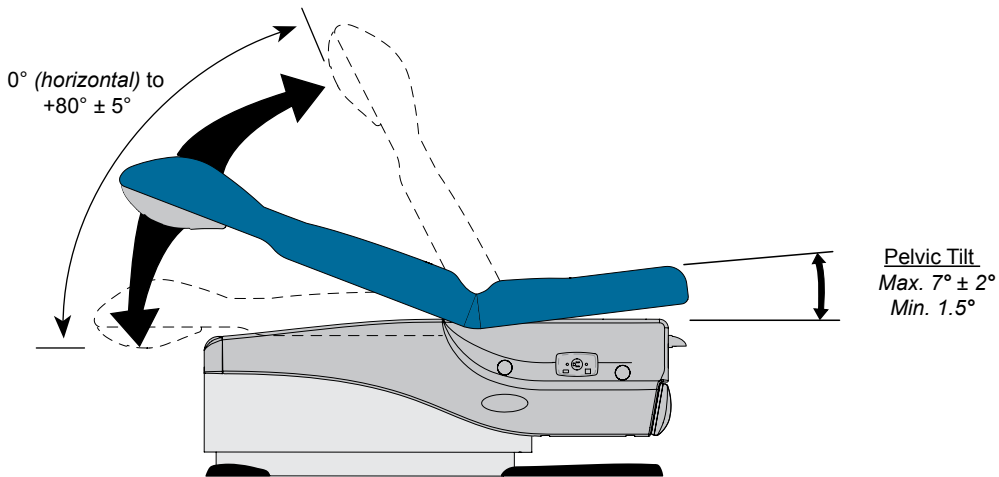
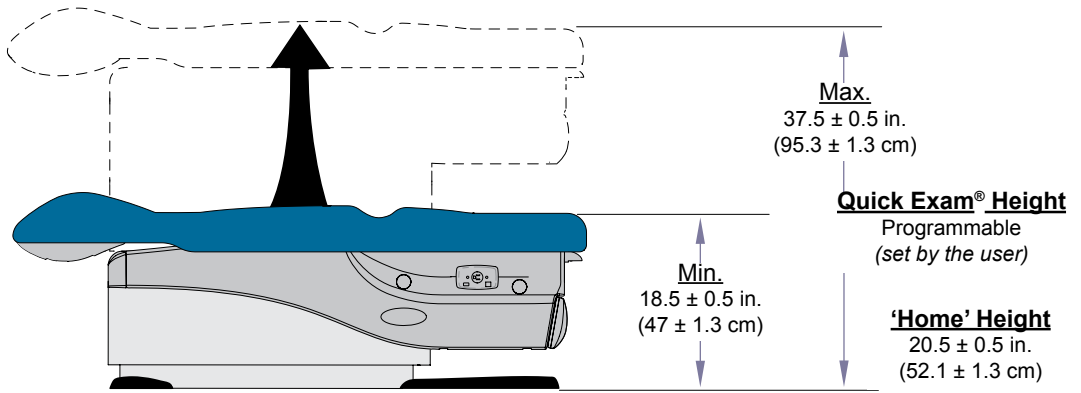
*Equipment not suitable for use in the presence of a flammable anesthetic mixture.*

# Model Identification / Compliance Chart

Model	Description	Complies To:				Electrical Ratings:		
		UL 60601-1	CAN / CSA 22.2, #601.1-M90	EN 60601-1-2 (EMC)	NFPA 99	VAC +/- 10%	Amps	Cycles (Hz)
625-004	Two-function table (Base / Back), w/ receptacles, pelvic tilt, drawer heater, wireless controls, IQscale®, and IQhub®.	•	•	•	•	115	12	50/60



Fire Code Ratings: All upholstery complies with California Bureau of Home Furnishing [Technical Bulletin 11Z](#) and California Code of Regulations, Sect. 93120-93120.12, Title 17. Optional upholstery is available that complies with California Bureau of Home Furnishing [Technical Bulletin 133](#).

# Range of Motion & Dimensions

MA82681

## Range of Motion & Dimensions (continued)

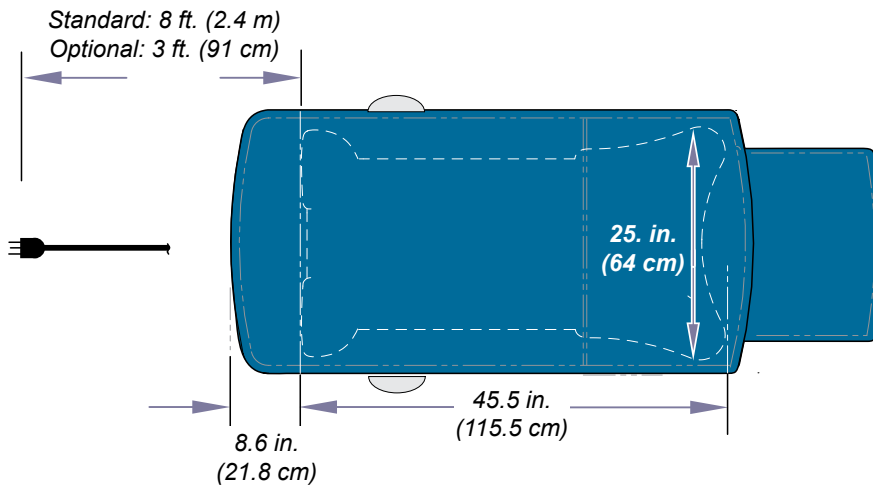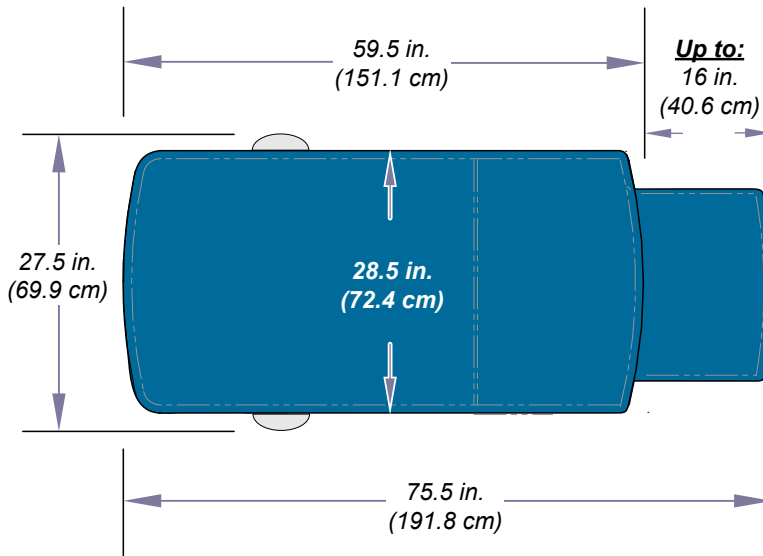MA8369i

# Warranty Information

**SCOPE OF WARRANTY** Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

**APPLICABLE WARRANTY PERIOD** The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components.

**OBTAINING WARRANTY SERVICE** Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at [www.midmark.com](http://www.midmark.com); by phone at 1-800-MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

**EXCLUSIONS:** This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

**SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.**

**EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER** MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

**STATUTE OF LIMITATIONS** No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

**NO AUTHORIZATION** No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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"This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device."

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### **Antennes**

"Under Industry Canada regulations, this radio transmitter may only operate using an antenna of a type and maximum (or lesser) gain approved for the transmitter by Industry Canada. To reduce potential radio interference to other users, the antenna type and its gain should be so chosen that the equivalent isotropically radiated power (e.i.r.p.) is not more than that necessary for successful communication."

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