625 Barrier-Free® Power Examination Table Setup Instructions

Caution

Heavy object. Use lifting aids and proper lifting techniques when moving table to minimize muscle strains or back injury.

Tools:

Battery Powered Drill / Impact Driver
Standard Wrenches
Standard Deep Well Socket
Box Cutter
Furniture Dolly (4 Wheel)

Language of origin: English

Equipment Alert

When moving the table, you must adhere to the following instructions.

ALWAYS: a) Transport table in full-down position with base shroud supports in place.

NEVER: a) Push on base shrouds.
b) Lift table by stirrups, back section, or upholstered top.

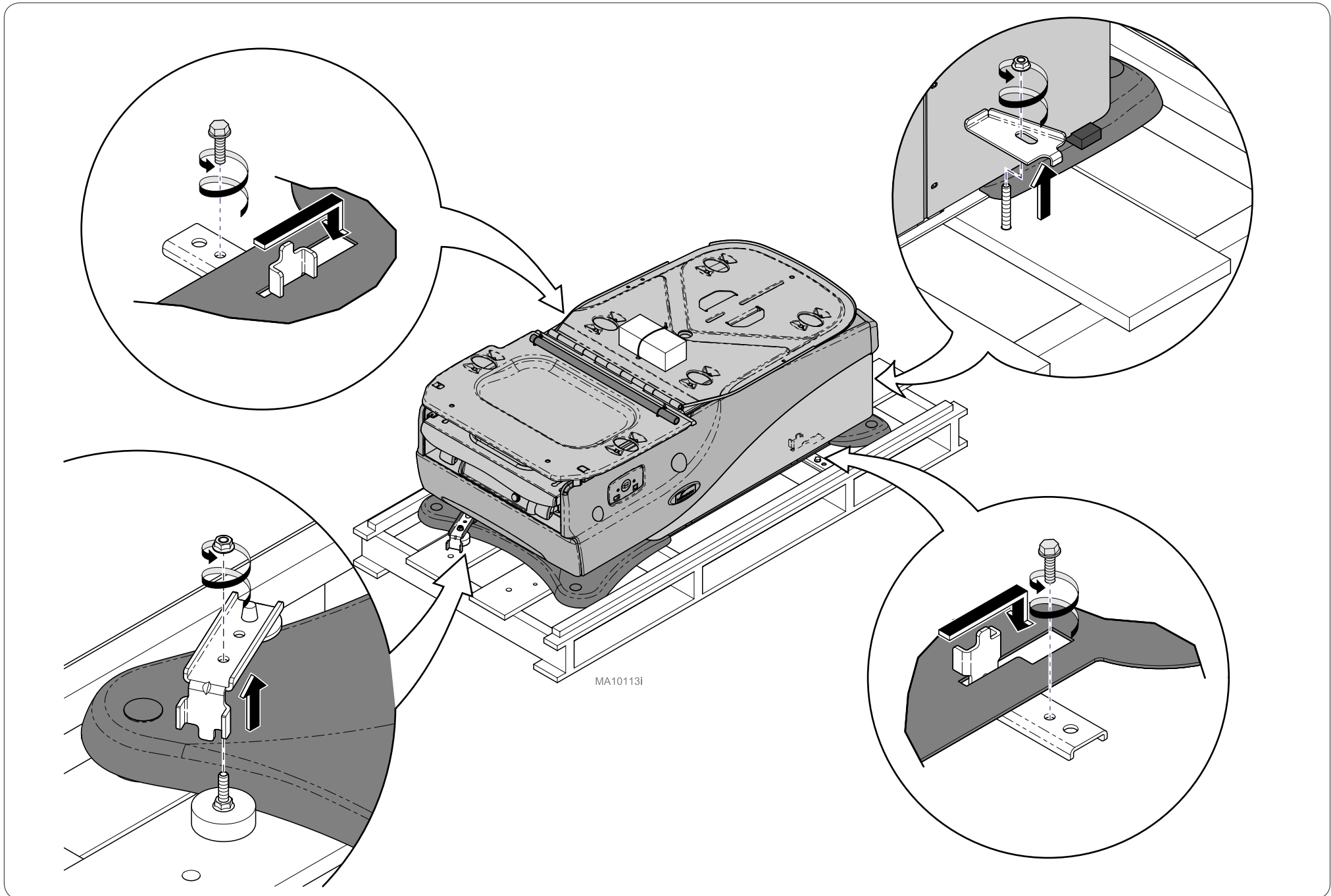
[English](#)

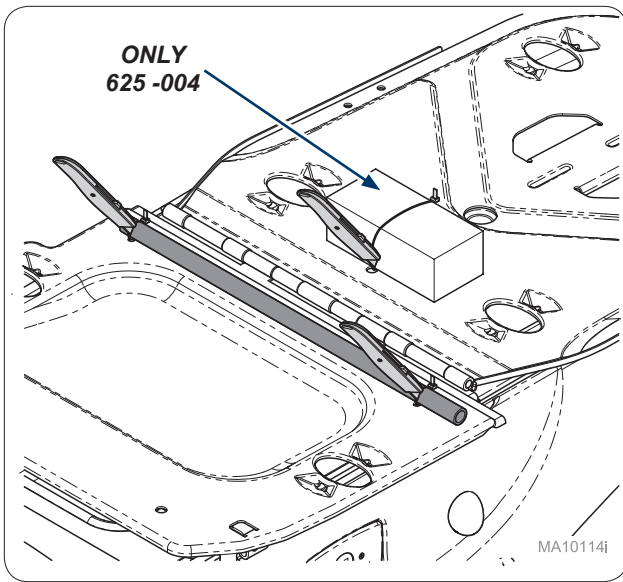
[Español](#)

[Français](#)


Step 1:
Remove brackets (4) from skid to release table.


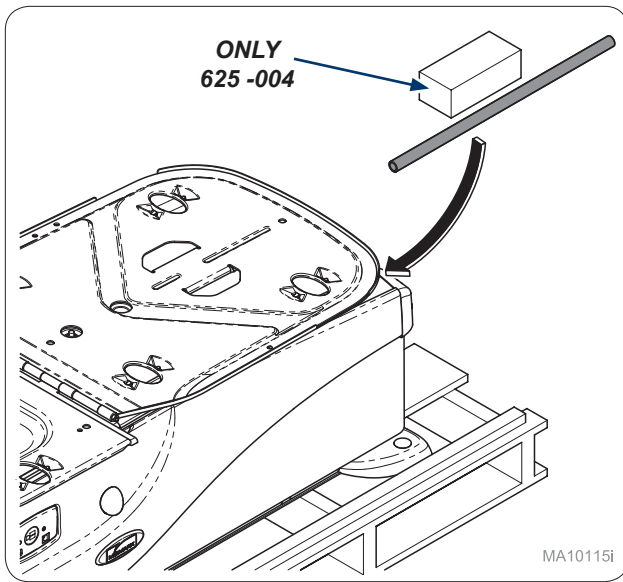
Step 2:
Cut zip ties for paper roll rod and box with table feet.

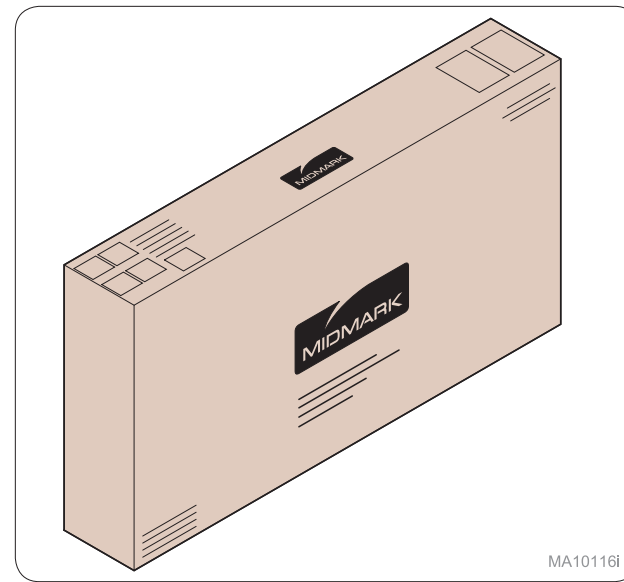
Step 3:
Place paper roll rod and box with table feet under table back.

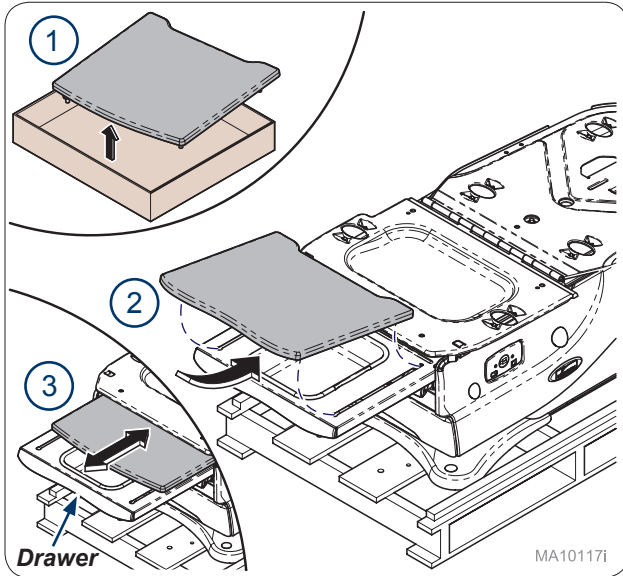
Step 4:
Locate box with upholstery.

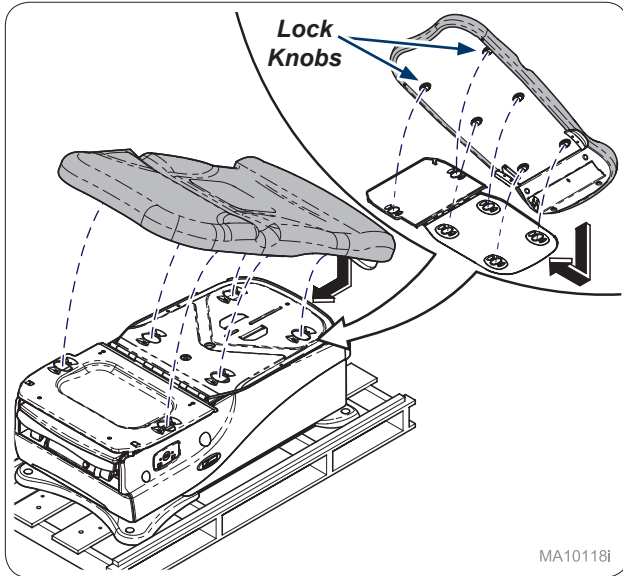
Step 5:
Remove footrest upholstery from box and install on table. Footrest should slide freely from drawer.

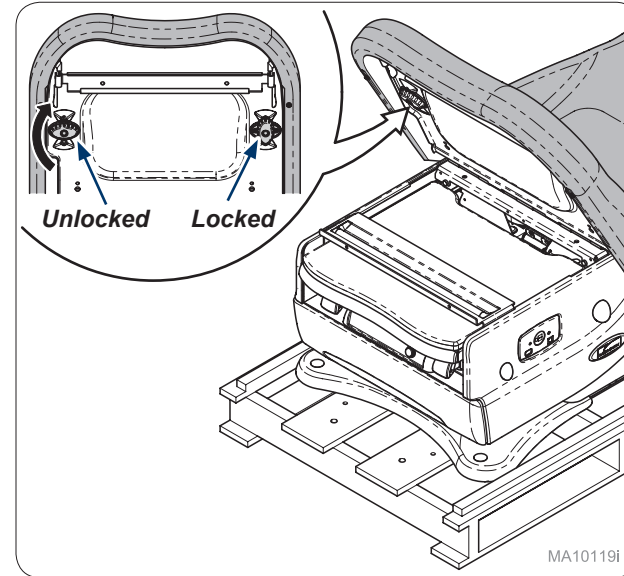Step 6:
Install upholstery onto table top aligning lock knobs (6) with holes (6).

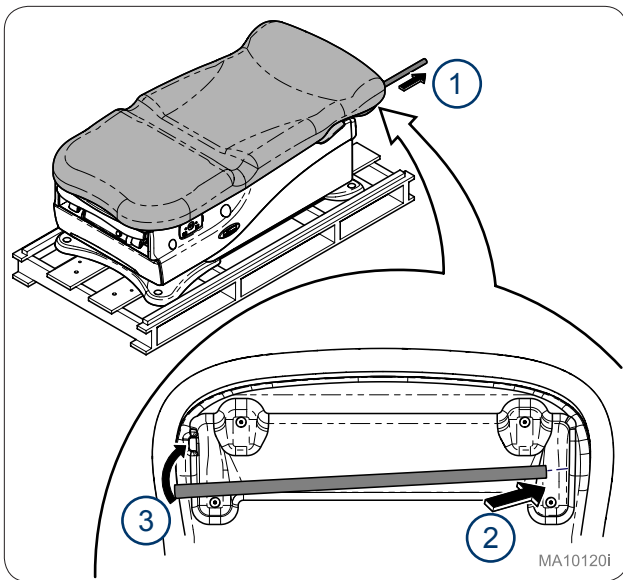
Step 7:
Lift table seat and latch locking knobs (2).


MA10120i

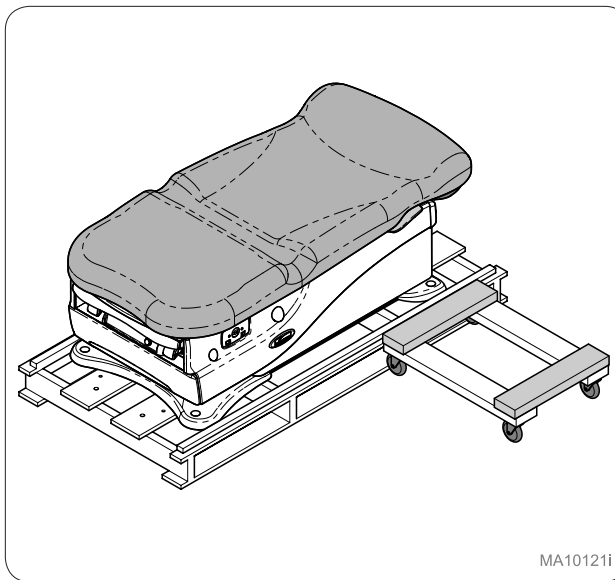Step 8:
Remove paper towel rod and install into back of upholstery top.


MA10121i

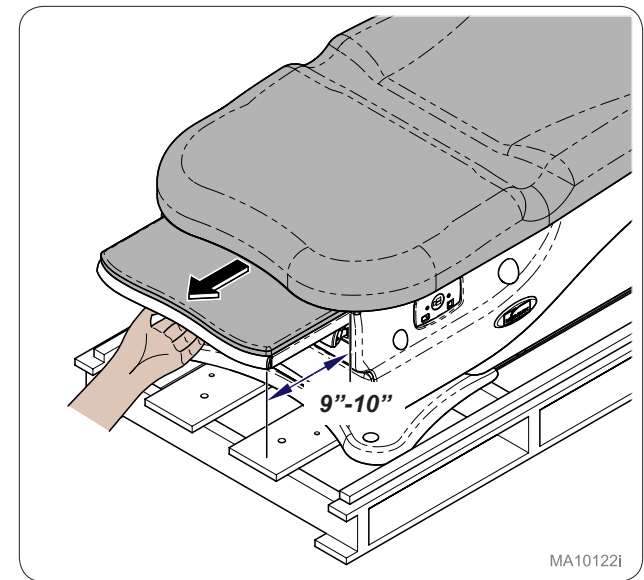Step 9:
Place dolly next to head end of table.

MA10122i

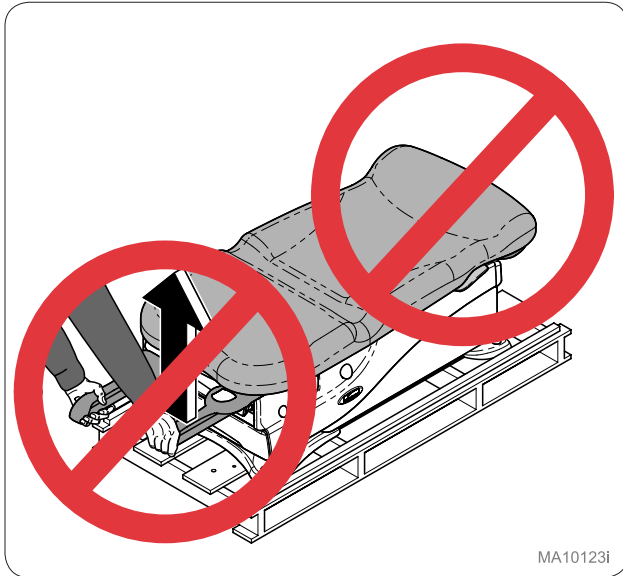Step 10:
Slide footrest 9"-10" out of table.


MA10123i


Equipment Alert

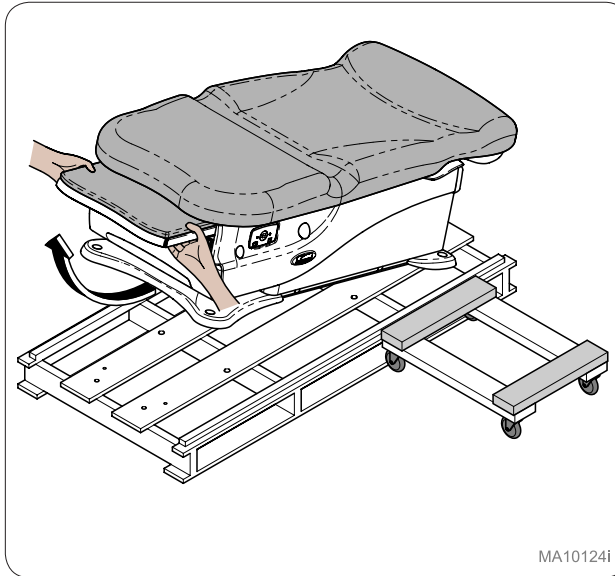Do **NOT** lift table by Upholstery or Stirrups.


MA10124i

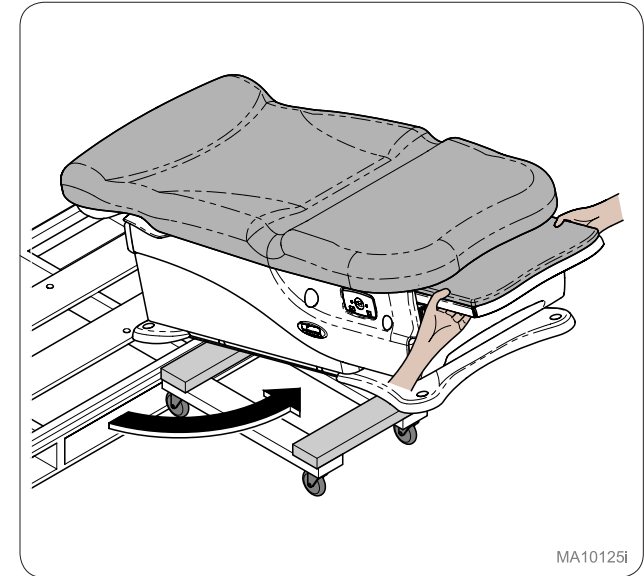Step 11:
Using proper lifting technique lift table off front of skid.


MA10125i

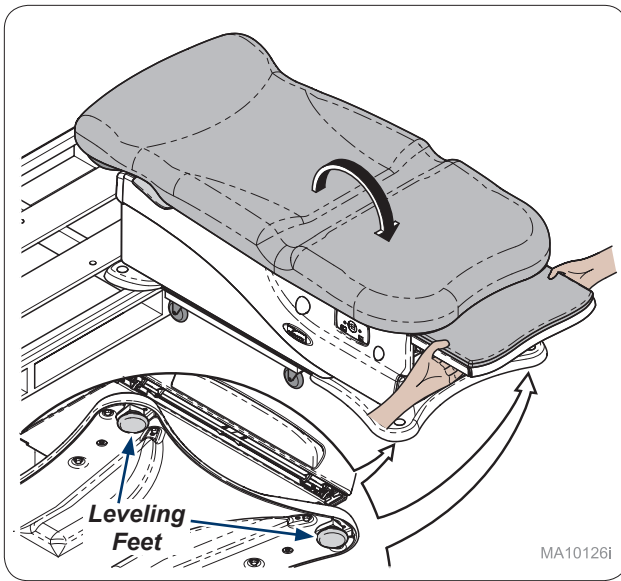
Step 12:
Rotate table to dolly.

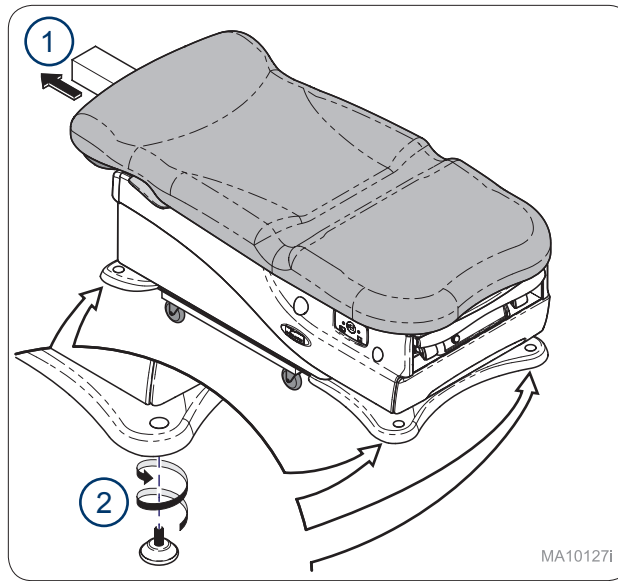
Step 13:
Lower table onto dolly. Look under base for leveling feet (4). **Note: Skip to step 15 if feet are installed.**

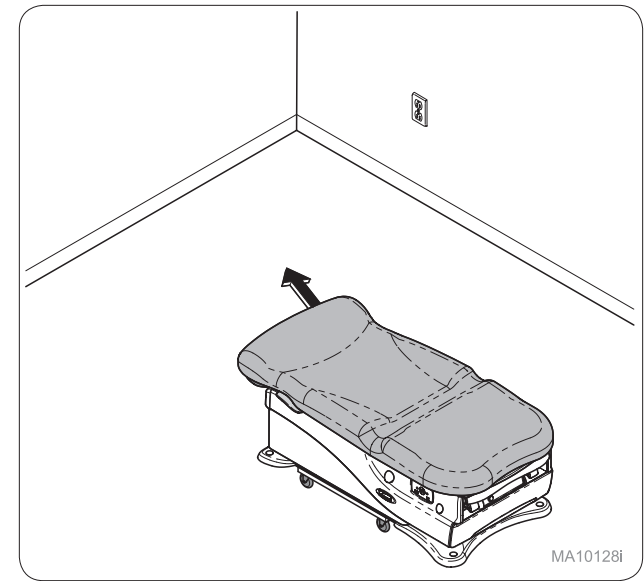Step 14: 625-004 (ONLY)
Remove box with feet (4) from under table back and install into base.

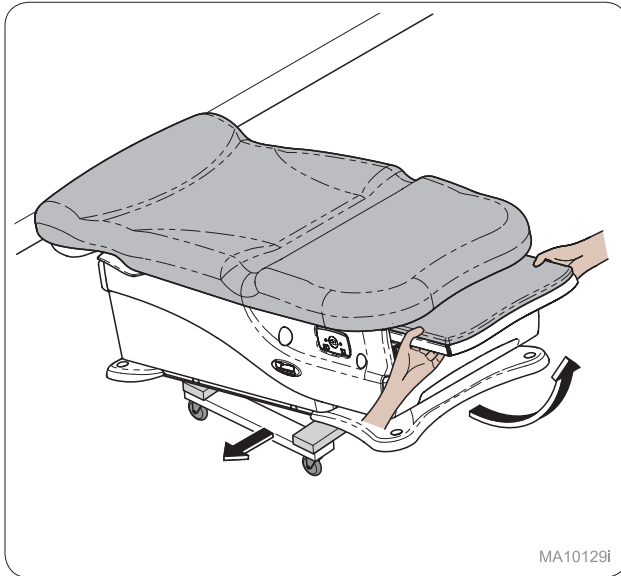
Step 15:
Move table to desired location.

Step 16:
Remove table from dolly with footrest.

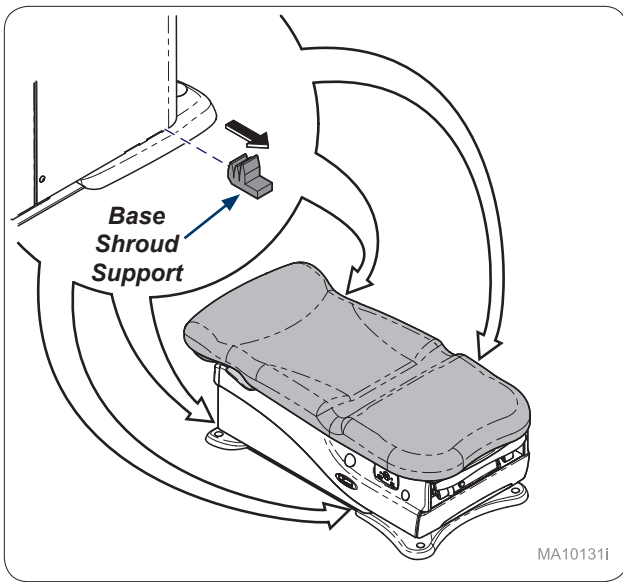
Step 17:
Remove power cord from treatment pan, connect to rear of table, and plug into wall or floor outlet.


MA10131i

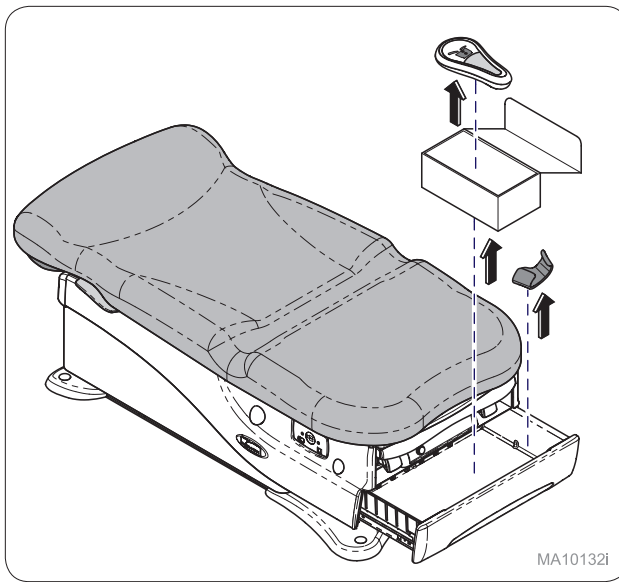Step 18:
Remove base shroud supports (4).


MA10132i

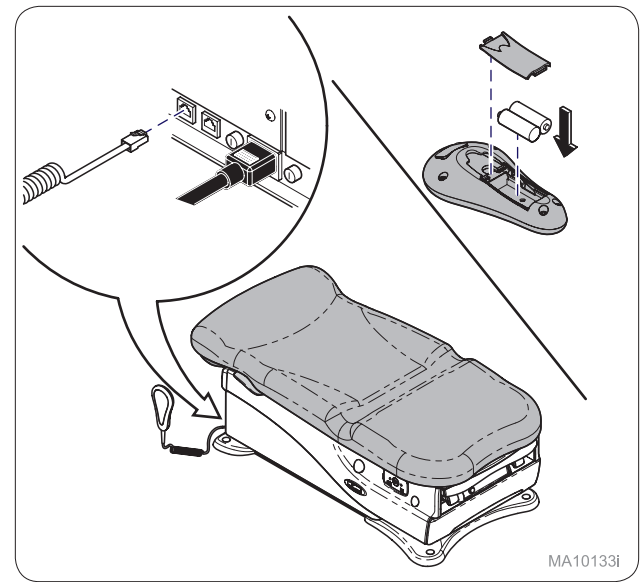Step 19:
Remove items from drawer.


MA10133i

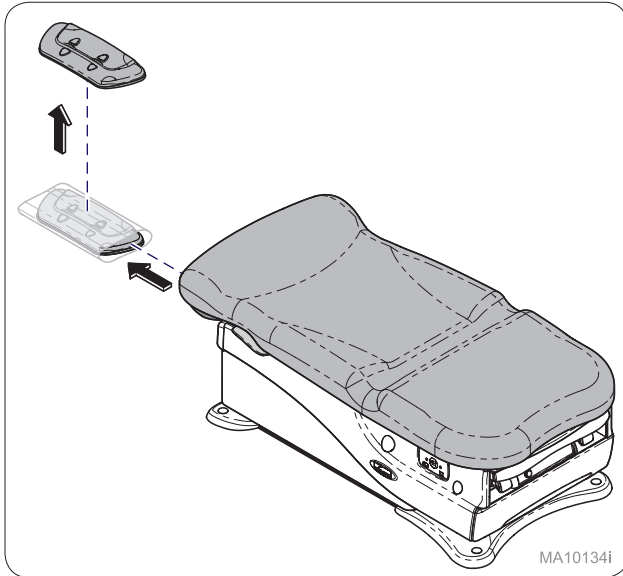Step 20:
Connect hand control to bottom of table. If hand control is wireless insert batteries.


MA10134i

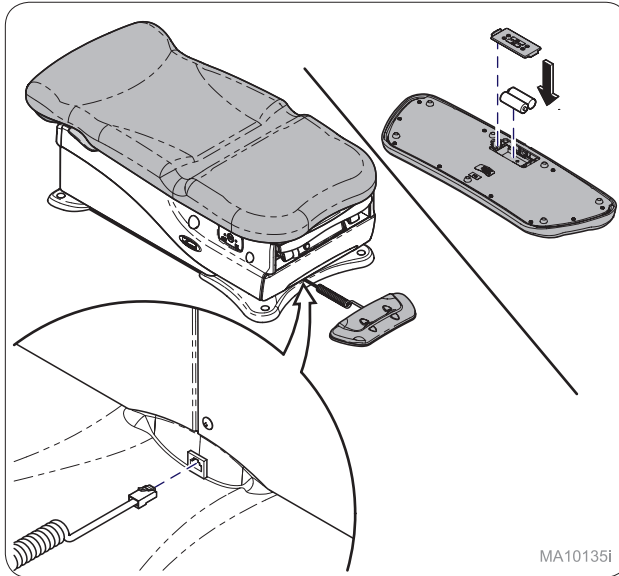Step 21:
Remove foot control from under table back.

MA10135i

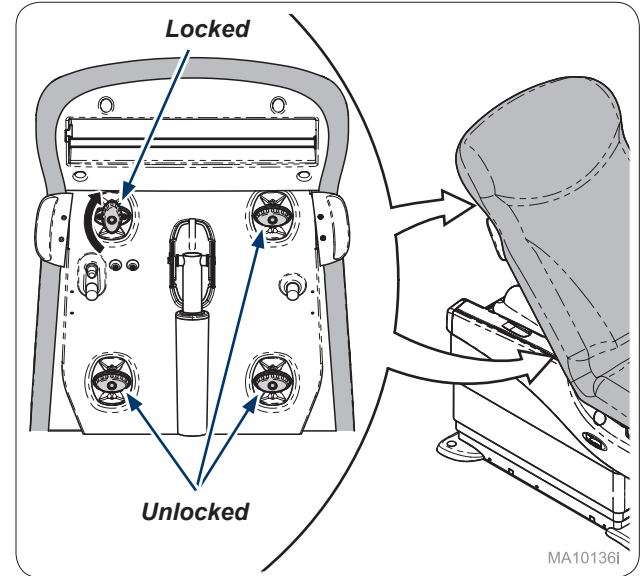Step 22:
Connect foot control to front base of table. If foot control is wireless insert batteries.


MA10136i

Step 23:
Use foot or hand control to raise table back and latch upholstery locking knobs (4).

Equipment Alert

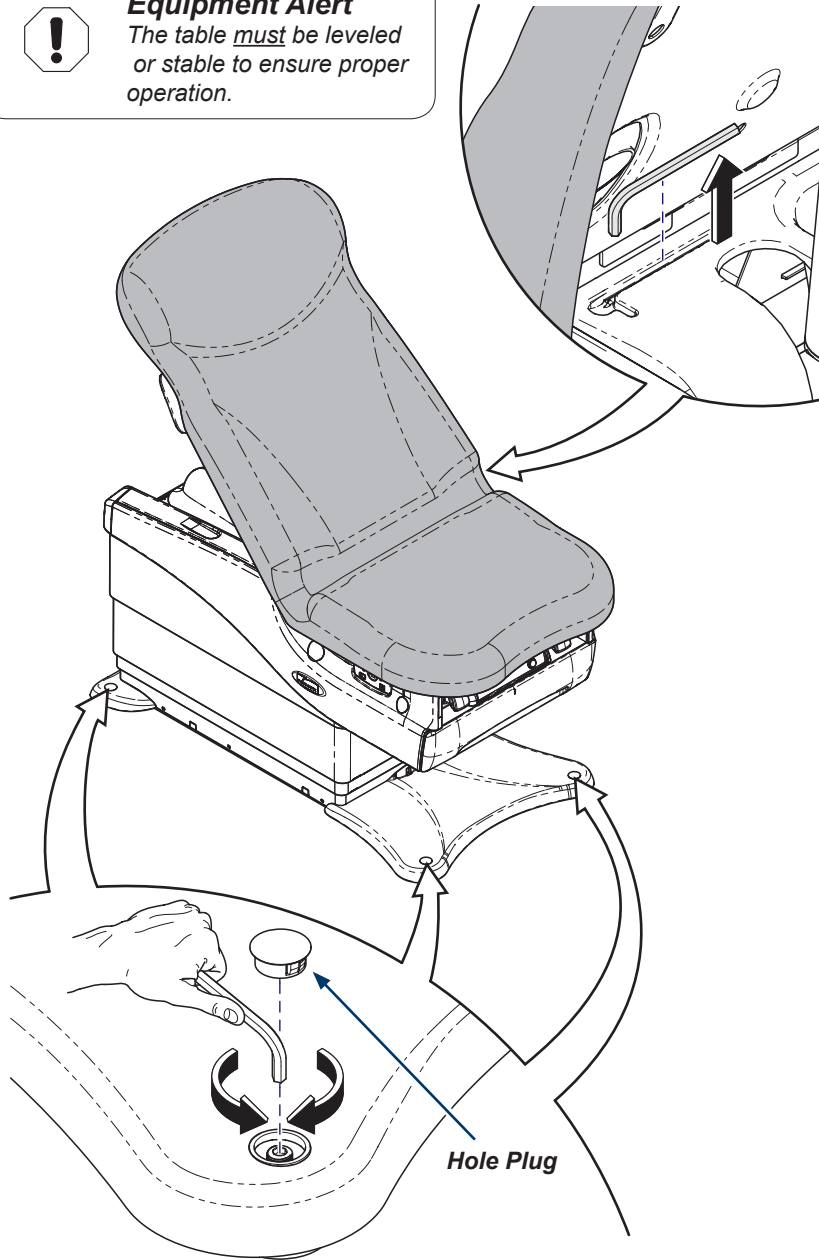The table must be leveled or stable to ensure proper operation.


Hole Plug

MA101371

Step 24:

Find Allen wrench under table back. Remove hole plug to level or balance table.

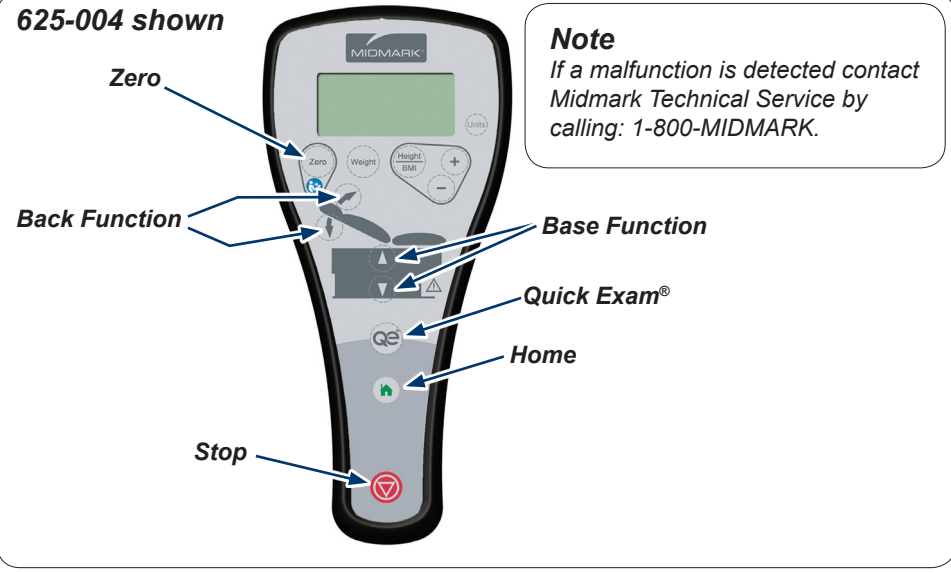625-004 shown

Note

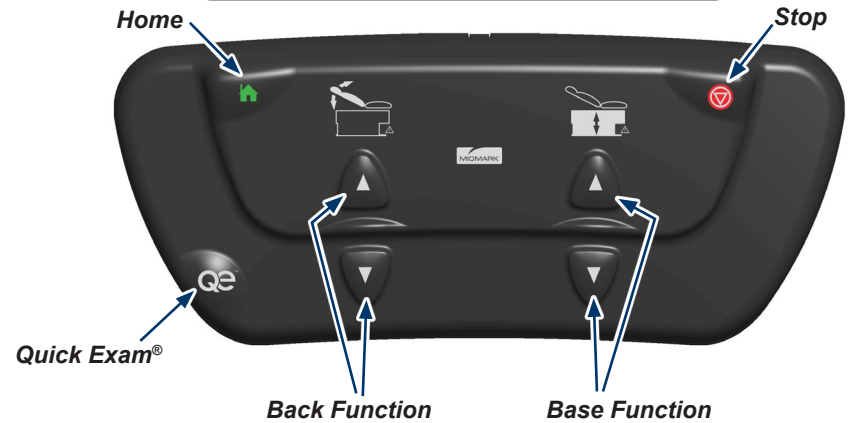
If a malfunction is detected contact Midmark Technical Service by calling: 1-800-MIDMARK.

Step 25:

Verify proper operation of hand control. Press the buttons for Base Up/Down, Back Up/Down, Quick Exam® and Home. Press home button, while table is moving press Stop. (All movement should stop)


Note

If a malfunction is detected contact Midmark Technical Service by calling: 1-800-MIDMARK.


Step 26:

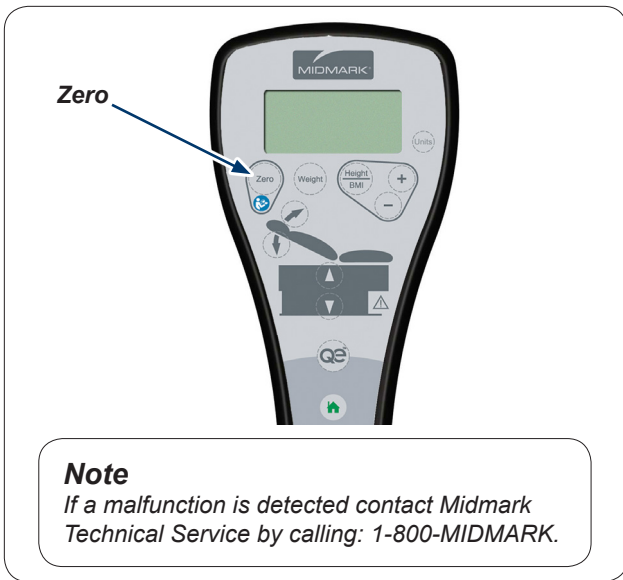Verify proper operation of foot control. Press the buttons for Base Up/Down, Back Up/Down, Quick Exam® and Home. Press home button, while table is moving press Stop. (All movement should stop)


Step 27: 625-004 (ONLY)

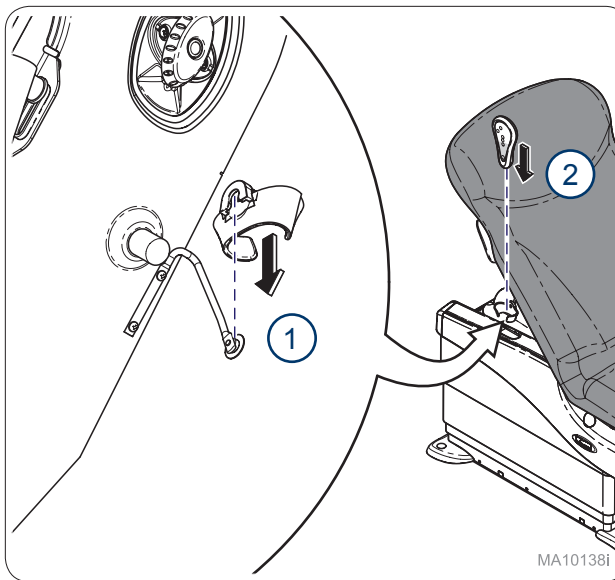Use hand control and press zero button to calibrate table.
Display should cycle and read "Scale Ready".


Step 28:

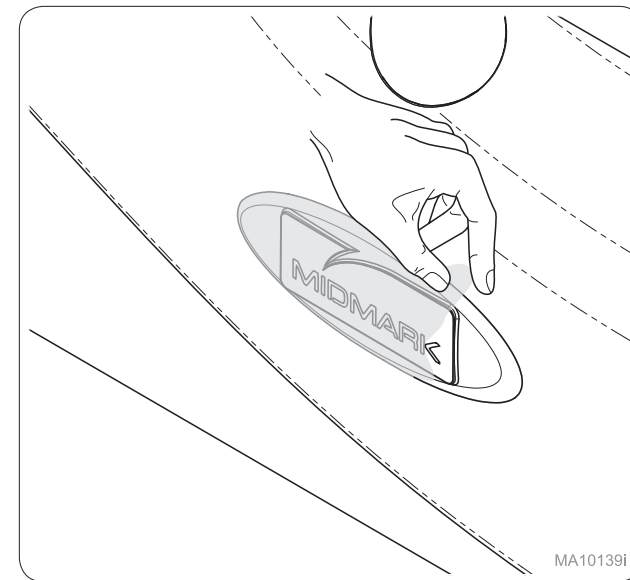Snap on holster to hanging bar and place hand control into holster.

Step 29:

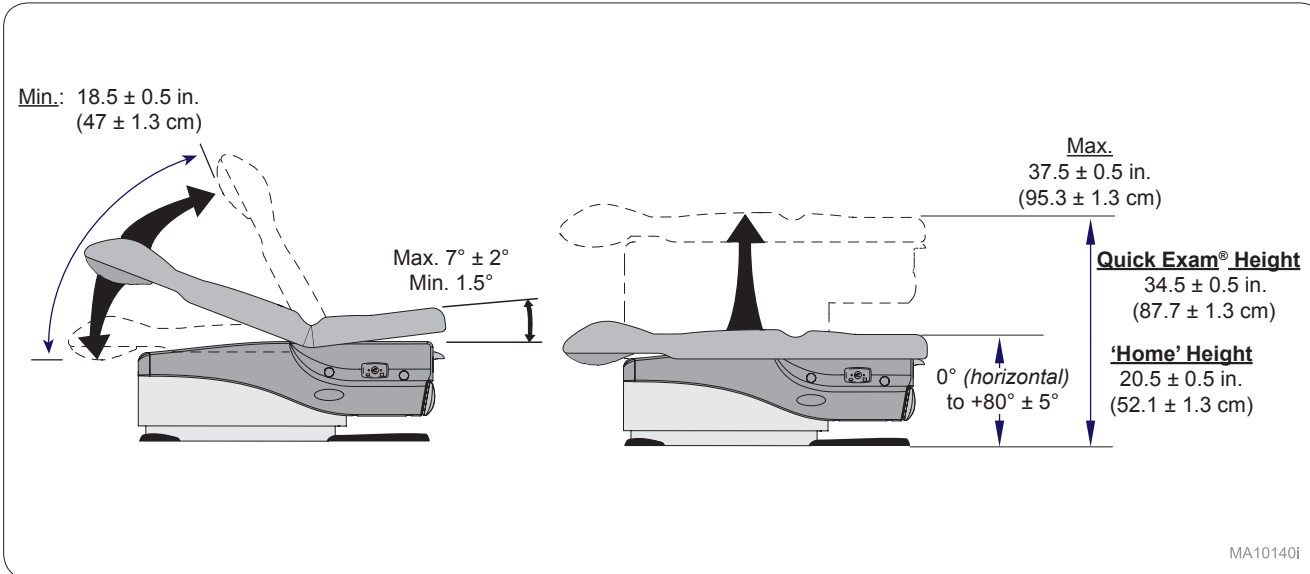Remove clear plastic covers (2) from Midmark name plate.

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