




# Scale / USB PC Board Kits [002-1512-00 / 002-1514-00 / 002-1515-00 / 002-1516-00]

**Components:**  
PC Board..... 1


**Special Tools:**  
none

**Step 1:** Remove cladding.

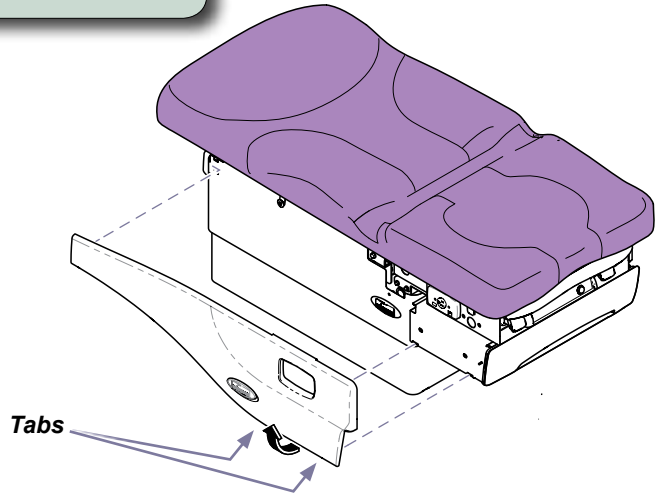


**Step 2:** Push LH side lever UP and RH side lever DOWN. Remove drawer.

*Note: Side levers are clear plastic.*

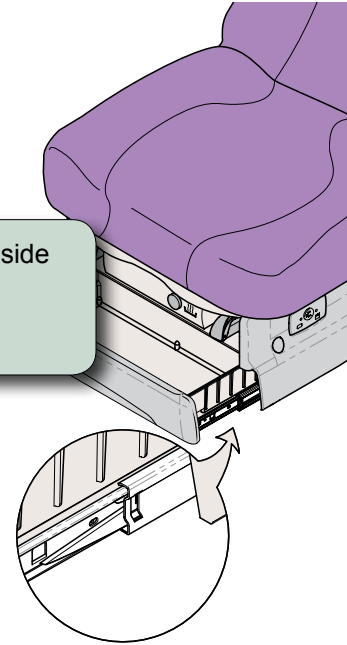


**Note**  
Lower table all the way down to remove shrouds.

**Step 3:** Remove eight screws.  
Remove outer shrouds.

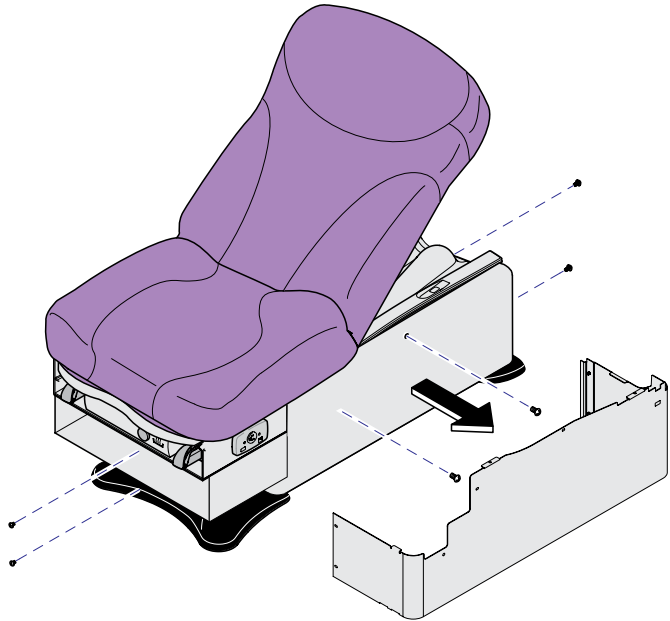

MA8398i

**Step 4:** Raise table to access PC Board.

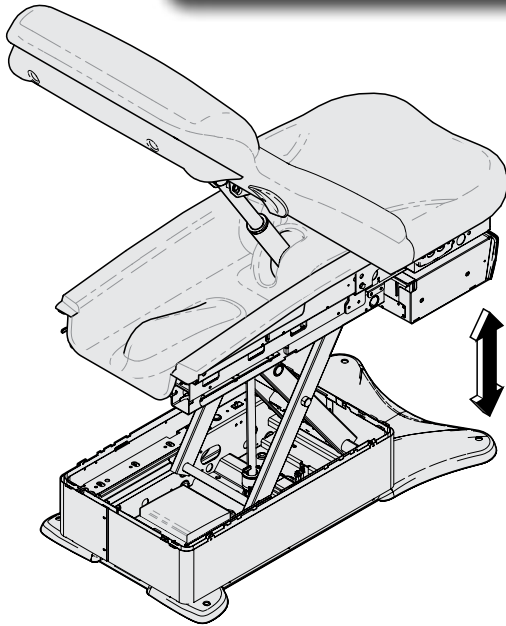


**Caution**

Unplug power cord before removing PC Board cover.

**Step 5:**

- A) Remove PC Board cover.
- B) Tag and disconnect wiring from PC Board.
- C) Remove PC Board.

*Tip:* Push latches back, then lift off of mounting studs.

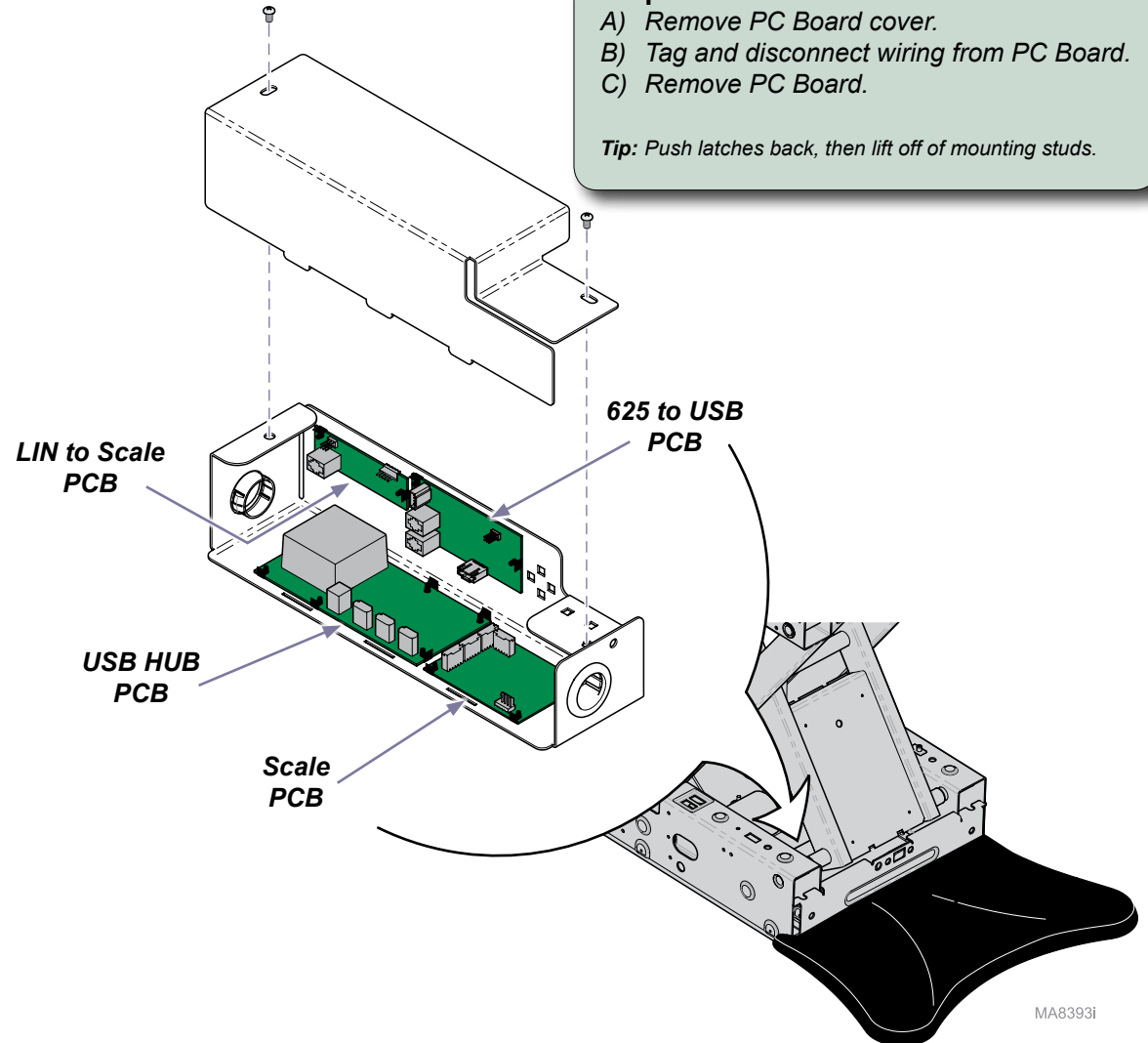



**Step 6:**

- A) Install new PC Board.
- B) Connect wiring to PC Board.
- C) Install PC Board cover.
- D) Install shrouds.

*Note:* Refer to removal procedure.

**Step 7:**

- A) *Insert tab at head-end of cladding into top shroud.*
- B) *Slide cladding over upperwrap.*
- C) *Pull bottom of cladding out, then push down on the top to engage the long tab onto top edge of upperwrap.*
- D) *Push bottom of cladding in, until two bottom tabs snap into place.*

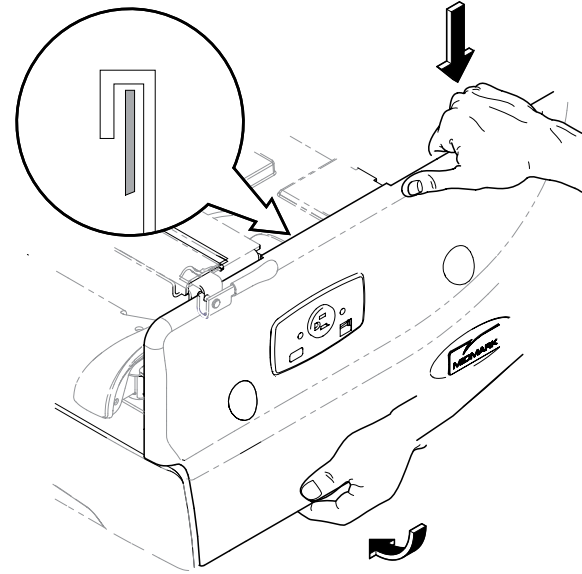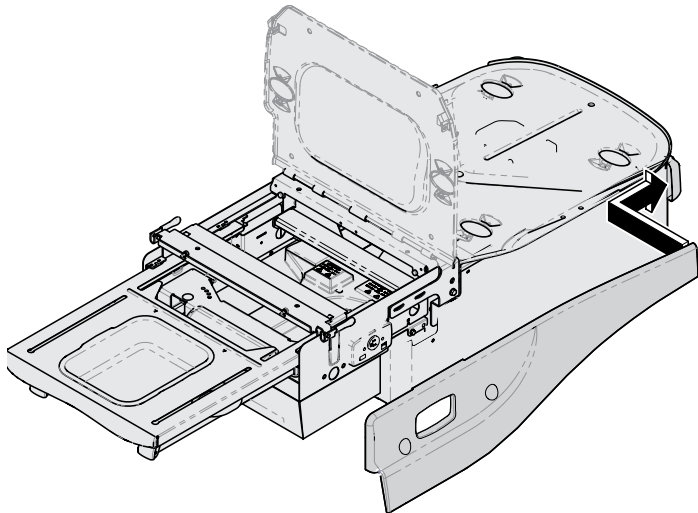

**Step 8:** Perform Scale Calibration Procedure.

*Note: For 002-1512-00 Scale PCB Only.*

## Scale Calibration Procedure

### Step 1:

- A) Remove any load from the patient surface.
- B) Press and hold the Weight and Plus buttons until "Pause" is displayed for at least one second, then release buttons.



### Step 2:

- A) When "CLrAL" is displayed, firmly press and release the plus button.
- B) After count down, "Ad50" will be displayed.




### Step 3:

- A) Place fifty pound calibrated weight on patient surface.
- B) Firmly press and release the plus button.
- Note: After count down "donE" will be displayed.



### Step 4:

- A) **Wait until display clears**, then firmly press and release the weight button.  
Note: "50 lbs" should be displayed.
- B) Remove calibrated weight.

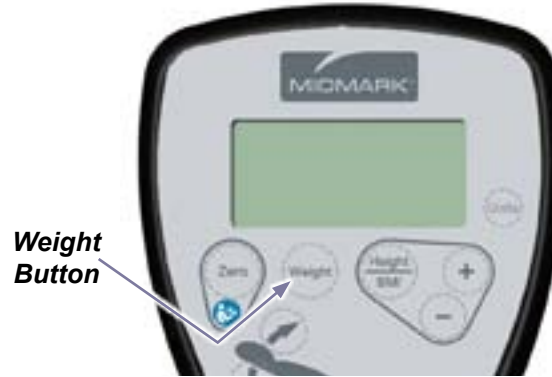

Because we care.