



# Wireless Basestation Kit [002-1564-00 / 002-1612-00]


**Components:**  
PC Board..... 1

**Special Tools:**  
none

**Compatible with serial numbers:**  
002-1564-00 [625-001 & 625-003 (S/N: V968528 thru V1266165)]  
002-1612-00 [625-001 & 625-003 (S/N: V1266166 thru Present)], [625-004 (All)]

**Caution**  
Unplug power cord before removing stirrup guide.

**Step 1:** Remove two screws and stirrup guide.  
*Note: If necessary, remove chair arm brace.*

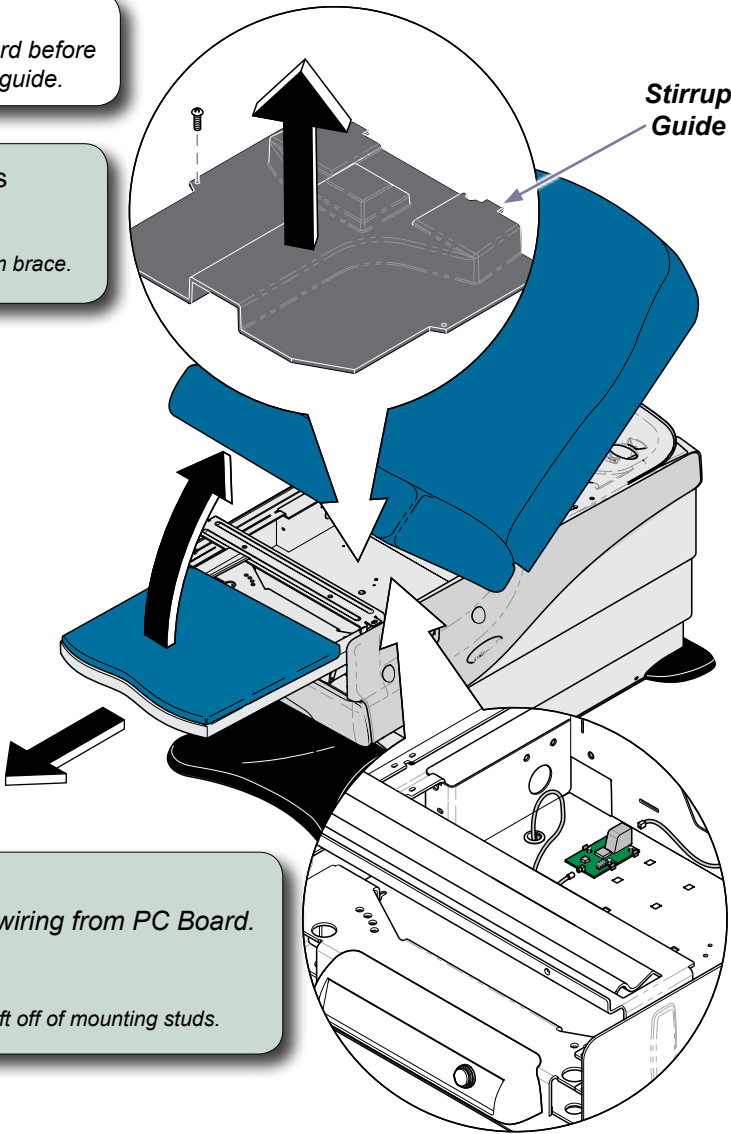

**Step 3:**  
A) Install new PC Board.  
B) Connect wiring to PC Board.  
C) Install stirrup guide.  
D) Perform Association Procedure.

**Step 2:**  
A) Tag and disconnect wiring from PC Board.  
B) Remove PC Board.  
*Tip: Push latches back, then lift off of mounting studs.*

**Association Procedure:**  
A) Disconnect power to the table for at least 3 seconds.  
B\*) Reconnect power, wait for the power up beep sequence from the table, then press & hold the **Stop & Home** buttons on the **wireless controller** until you hear three “beeps”.  
*\* Note: Step B must be performed within 10 seconds of the power up beep sequence from the table.*  
C) Check for proper operation of wireless controller.  
D) If associating a hand **and** foot control, repeat procedure using the other wireless controller.

MA8450i