



Installing Elevance Chair Shipping Pin Kit [002-1602-00 GN 1 and 002-10500-00 GN 3]

Applies to Models:


153810 (all)

Language of origin: English



Equipment Alert

Do not relocate chair without shipping pin.

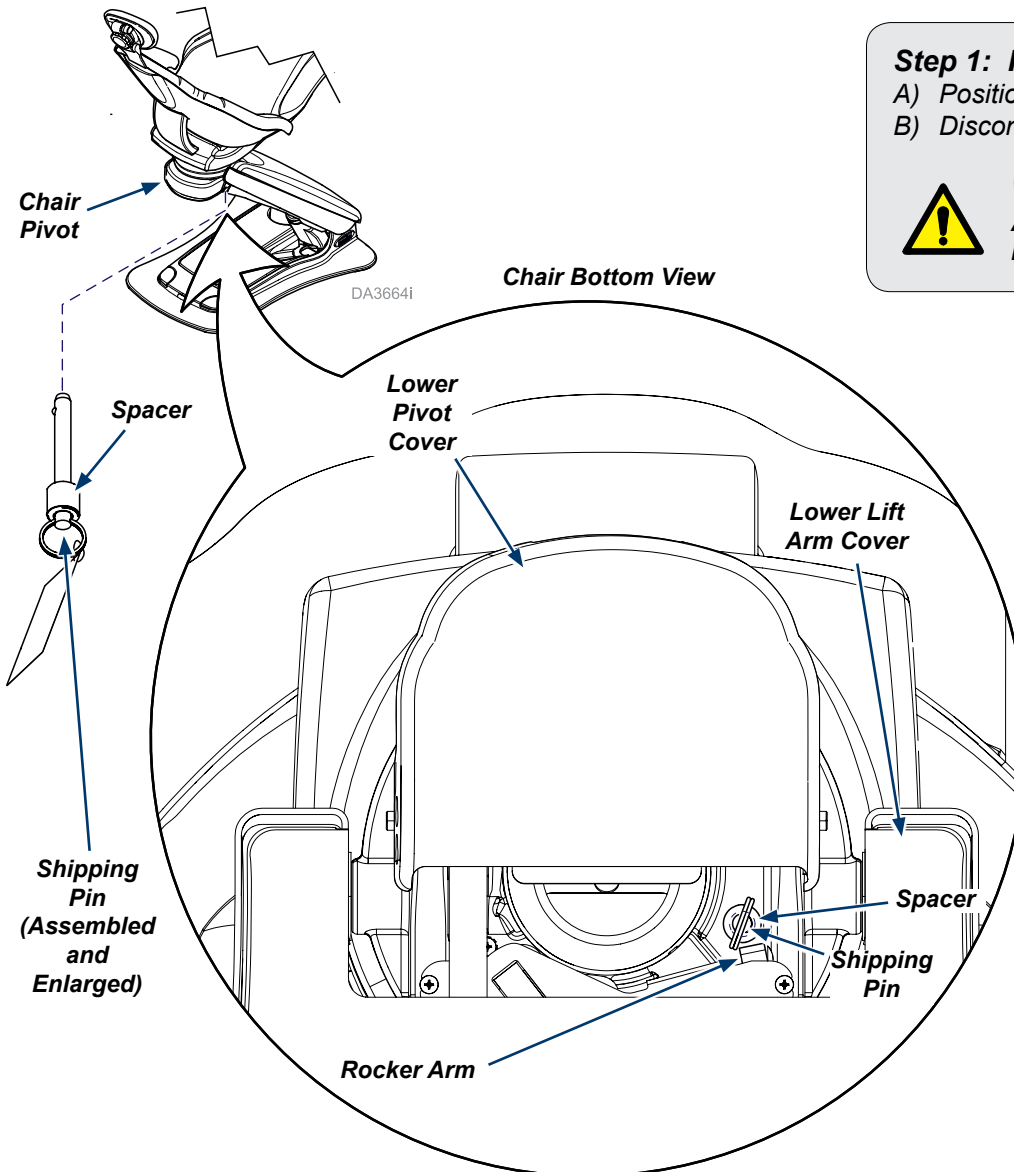

Step 1: Position chair.

- A) Position chair so lower lift arm cover position allows adequate space for the installation.
- B) Disconnect power from the chair.



WARNING

Always disconnect power from the chair before performing installations.
Failure to do so may result in personal injury.

Step 2: Install shipping pin.

- A) Secure tag to the pin.
- B) Install spacer to shipping pin (position near pin ring).
- C) Install the shipping pin fully into the pivot while moving the rocker arm as needed.

Note: The shipping pin prevents the chair from pivoting during transportation and shipment.

- D) Position spacer against pivot casting.
- E) Connect the power to the chair and verify operations.
- F) Position chair at lowest height before transportation or shipment.