




Midmark Digital Vital Signs Device Serial Cable Installation

[9A478]

Applies to Models:
625 (-004)
626 (-003, -004)
Special Tools:
none

WARNING
IEC 60950 approved Information Technology Equipment must be placed outside the "patient environment." Refer to safety instructions in the exam table user's guide.

Step 1: Connect serial cable to the exam table and the Midmark Digital Vital Signs Device.

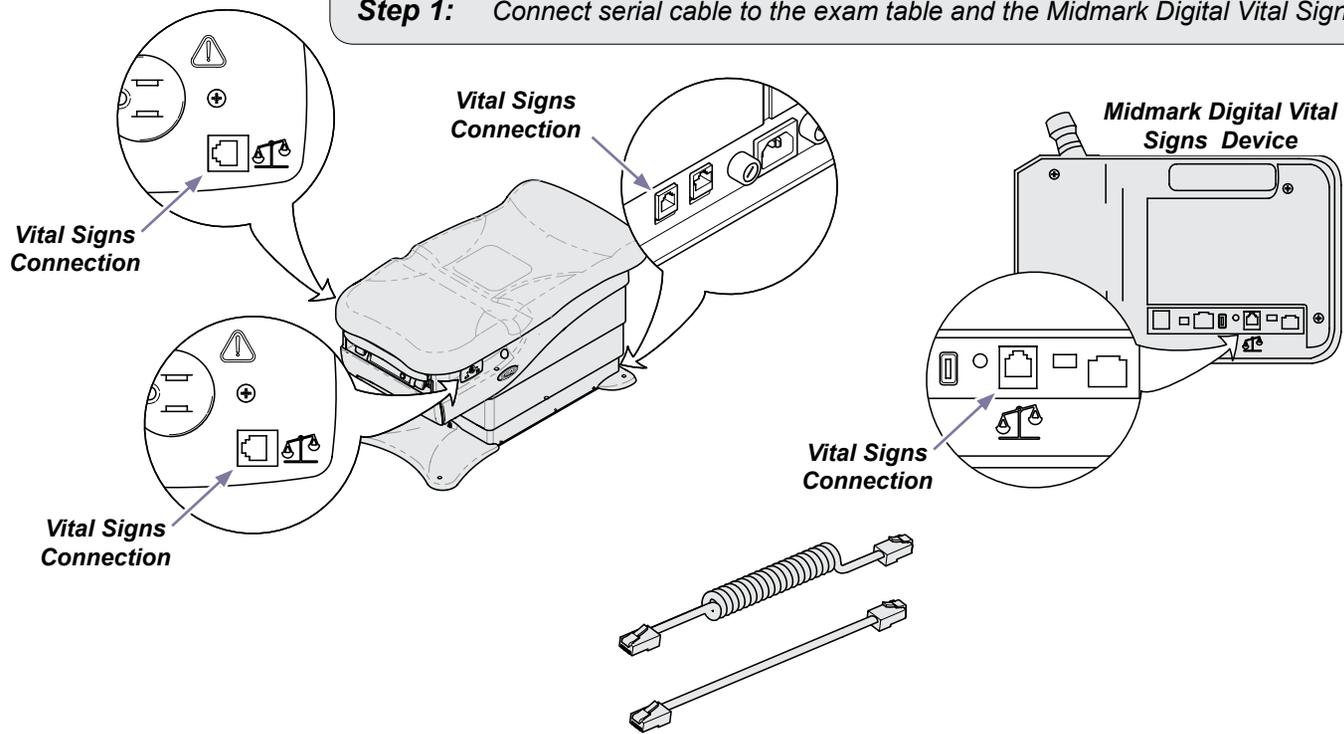


Note
The Midmark Digital Vital Signs Device may be connected to any of the three connection points on the exam table. However, only one Midmark Digital device may be connected at a time.

MA8532i

TP201 Rev. A

Step 2: Press the power button to turn on the Midmark Digital Vital Signs Device.

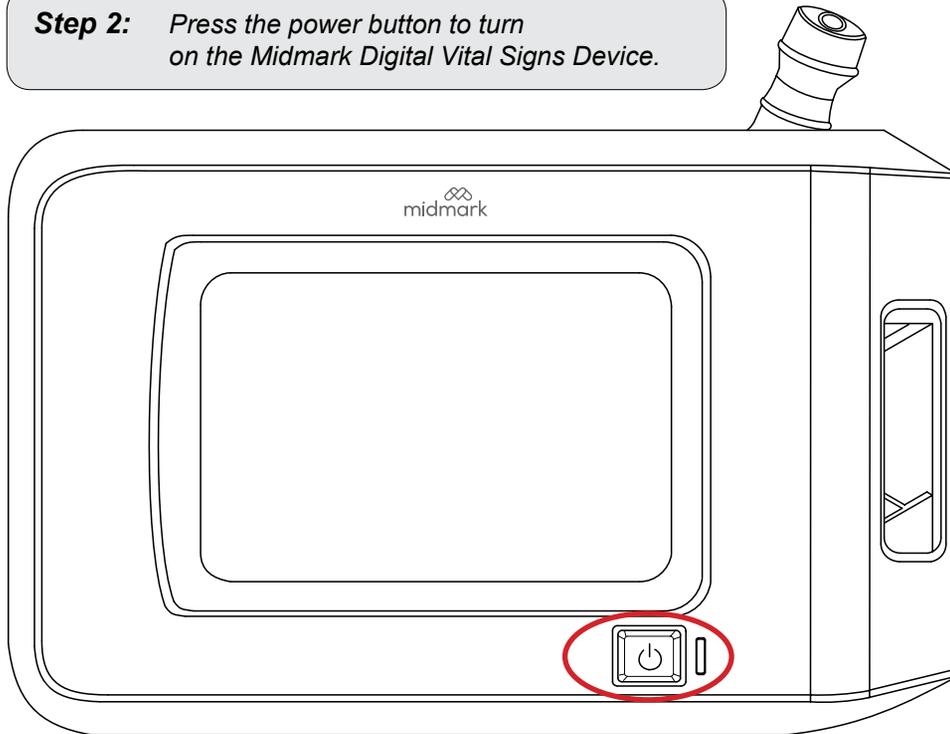

MA8553I

Note

Refer to the Midmark Digital Vital Signs Device operation manual for complete details regarding device set-up and usage.

Step 3: On the Midmark Digital Vital Signs Device Home screen, press the Tool icon to display the Password screen.



Midmark Digital Vital Signs Device Home Screen

Note

If the screen on your Midmark Digital Vital Signs Device does not look like the one shown in step 3, contact Midmark Support Services at 1-800-624-8950, option 2, to order an upgrade kit for your device.


Without this upgrade, the table controls shown in step 10 will not be available. However, sharing of patient weight between the exam table and the device will work properly without any further setup required.

Step 4: On the Password screen, enter your password (986 Default). Next, press OK to display the Settings screen.




Password Screen

Step 5: On the Settings screen, press MONITOR SETTINGS.

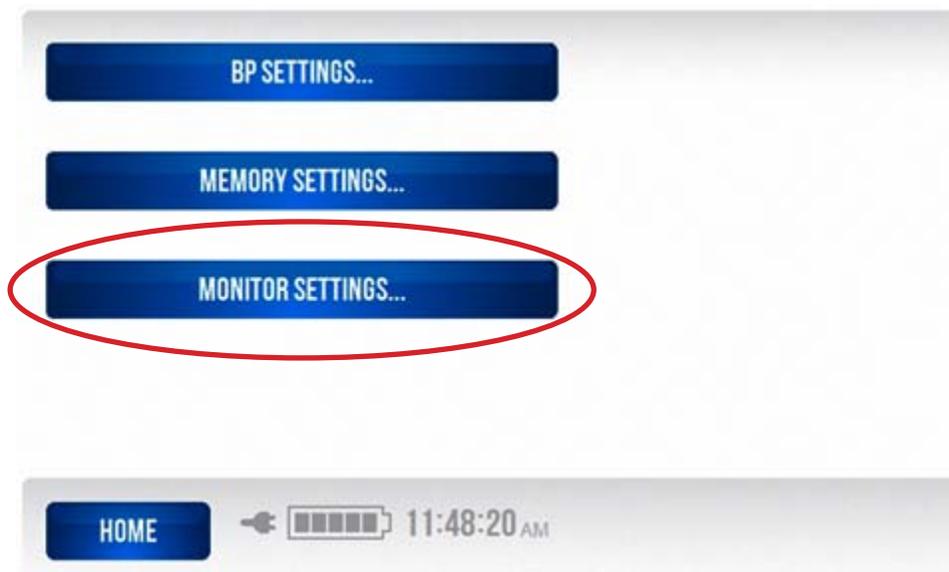

Settings Screen

Step 6: Once you have selected your monitor settings, press the MORE button to display the Units of Measure screen.

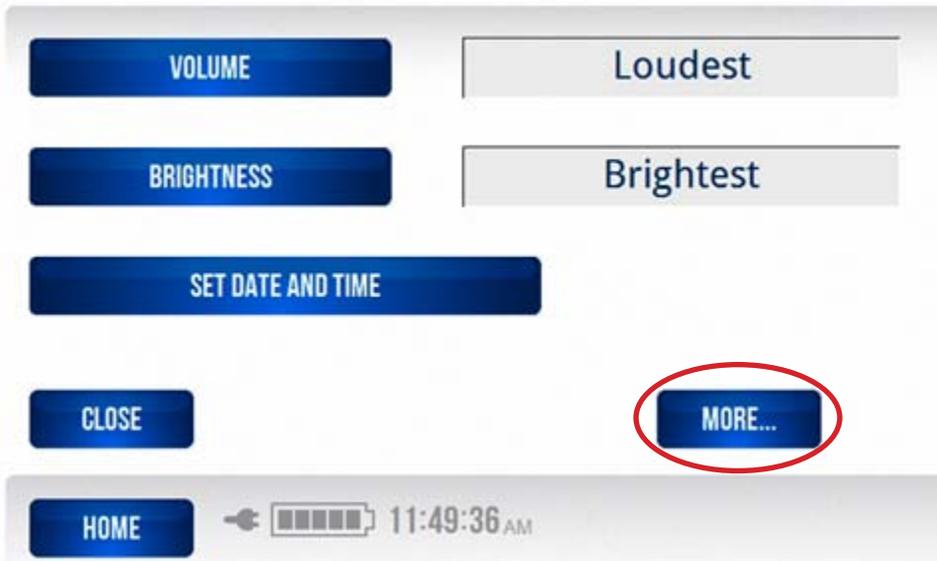

Monitor Settings Screen

Step 7: Once you have made your units of measure selections, press the TABLE button to display the Table Connection screen.



Units of Measure Screen

Step 8: On the Table Connection screen, select YES, then press OK to return to the Units of Measure screen.

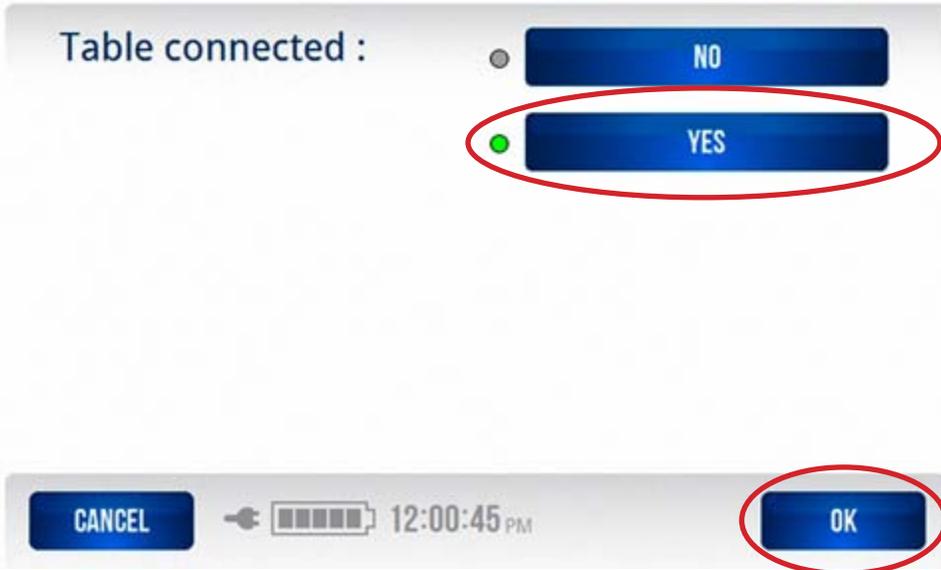


Table Connection Screen

Step 9: On the Units of Measure screen, press the HOME button to display the Midmark Digital Vital Signs Device Home screen.




Units of Measure Screen

Step 10: On the Midmark Digital Vital Signs Device Home screen, press the Table icon.



Midmark Digital Vital Signs Device Home Screen

When the Table icon is pressed on the Midmark Digital Vital Signs Device Home screen, the Midmark Digital Scale screen will be displayed.

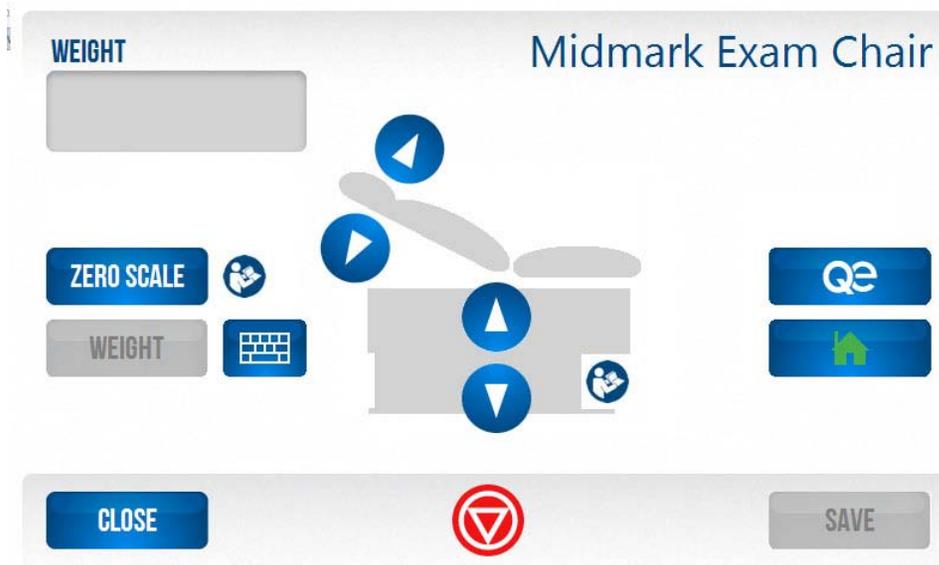

Midmark Digital Scale Screen