



Doctor's Heated Syringe [029-5710-00]




WARNING
Disconnect power, shut off air and water before performing this procedure.

Contents:

Heated Syringe	
w/ tubing	1
Syringe Tubing.....	1
Wire Harness.....	1
Clamp	1
Suppression Core.....	1
Valves.....	2
Knobs	2


Language of origin: English

- Step 1: Pre-installation.**
- A) Ensure parts are correct and undamaged.
 - B) Ensure correct wire and tubing lengths (see below graphic).
 - C) Disconnect syringe tubing and remove.

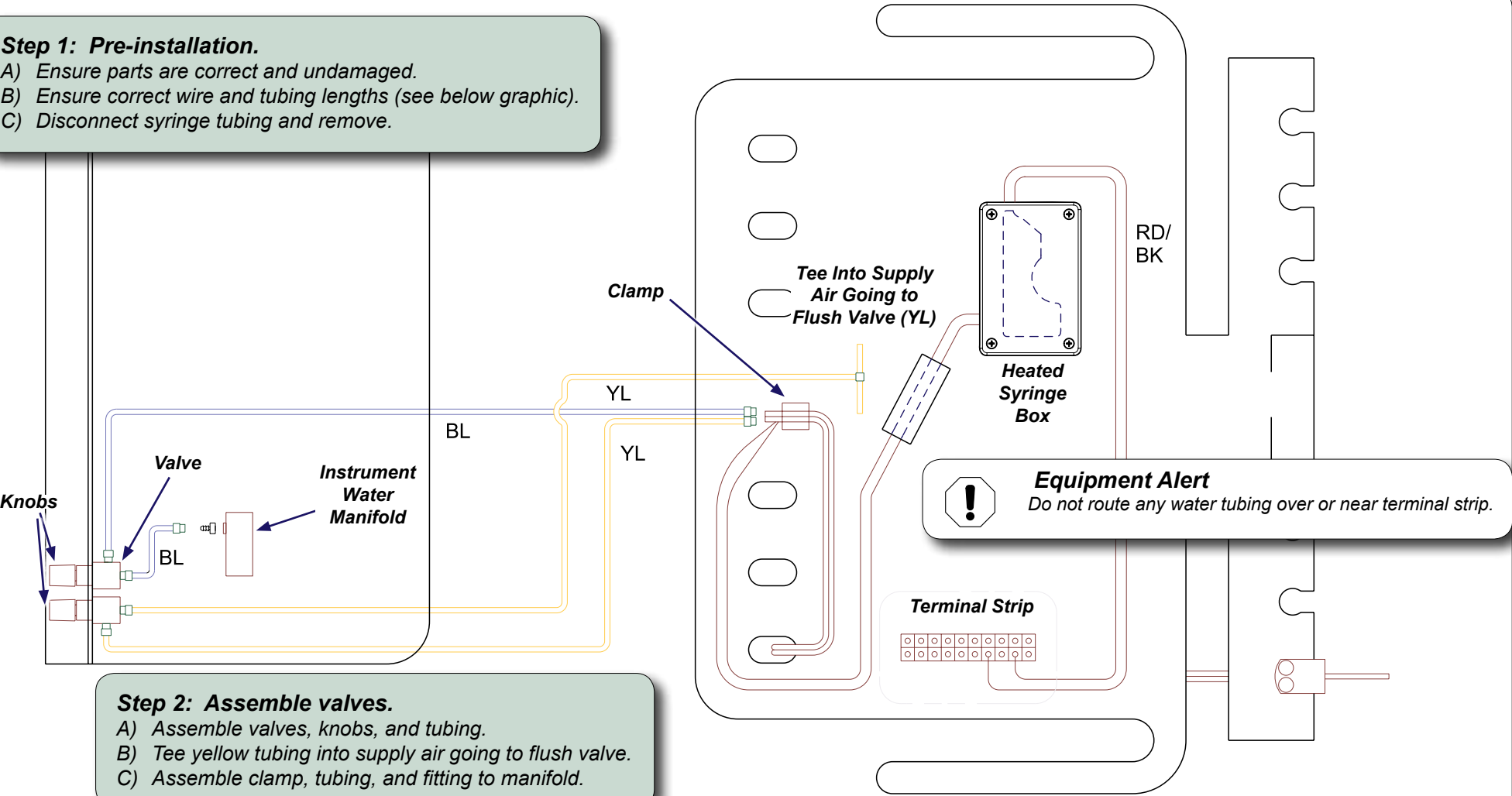


- Step 2: Assemble valves.**
- A) Assemble valves, knobs, and tubing.
 - B) Tee yellow tubing into supply air going to flush valve.
 - C) Assemble clamp, tubing, and fitting to manifold.

DA3699I

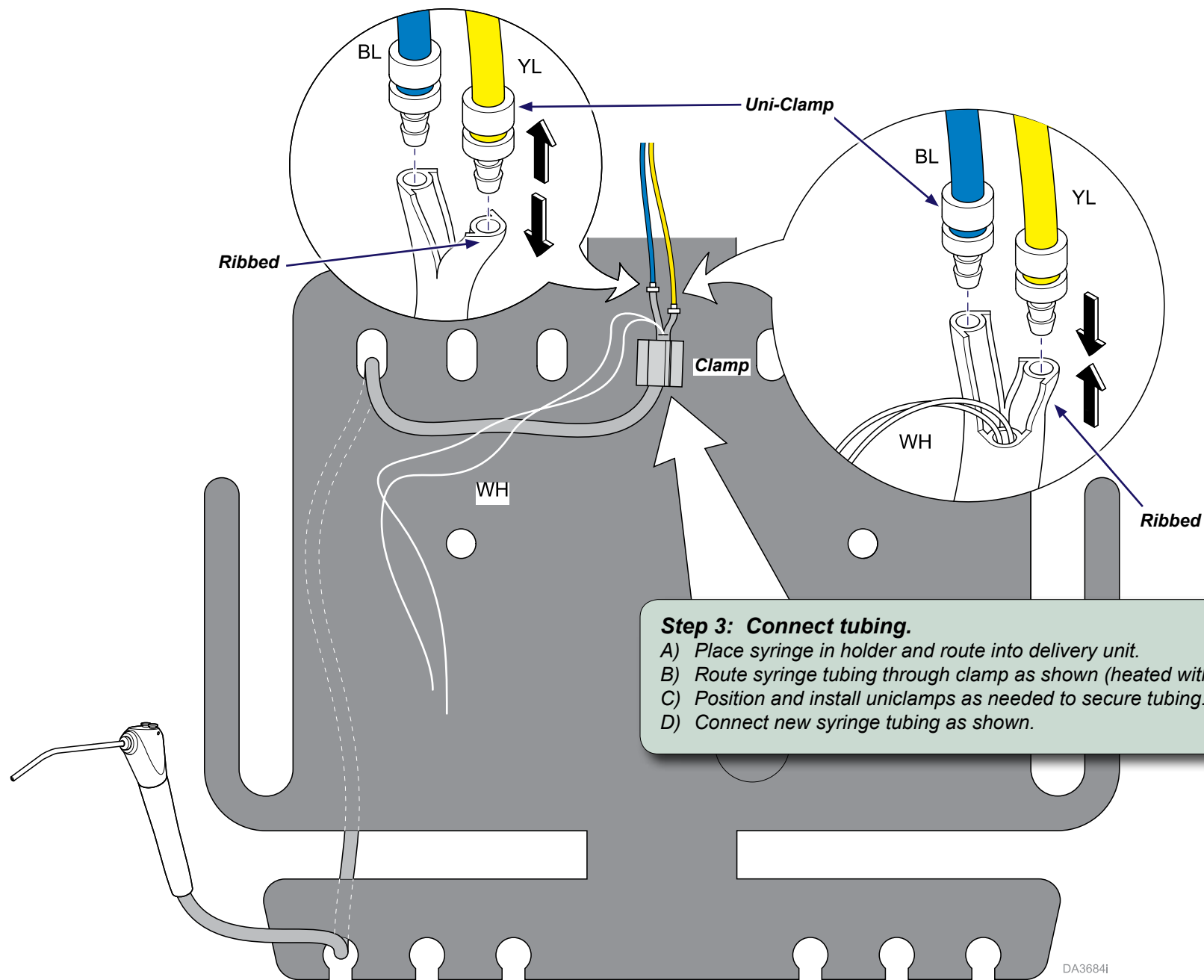

Step 4: Connect wiring.

- A) Connect red/black wires to terminal strip as shown.
- B) Route white wires through suppression core and connect.

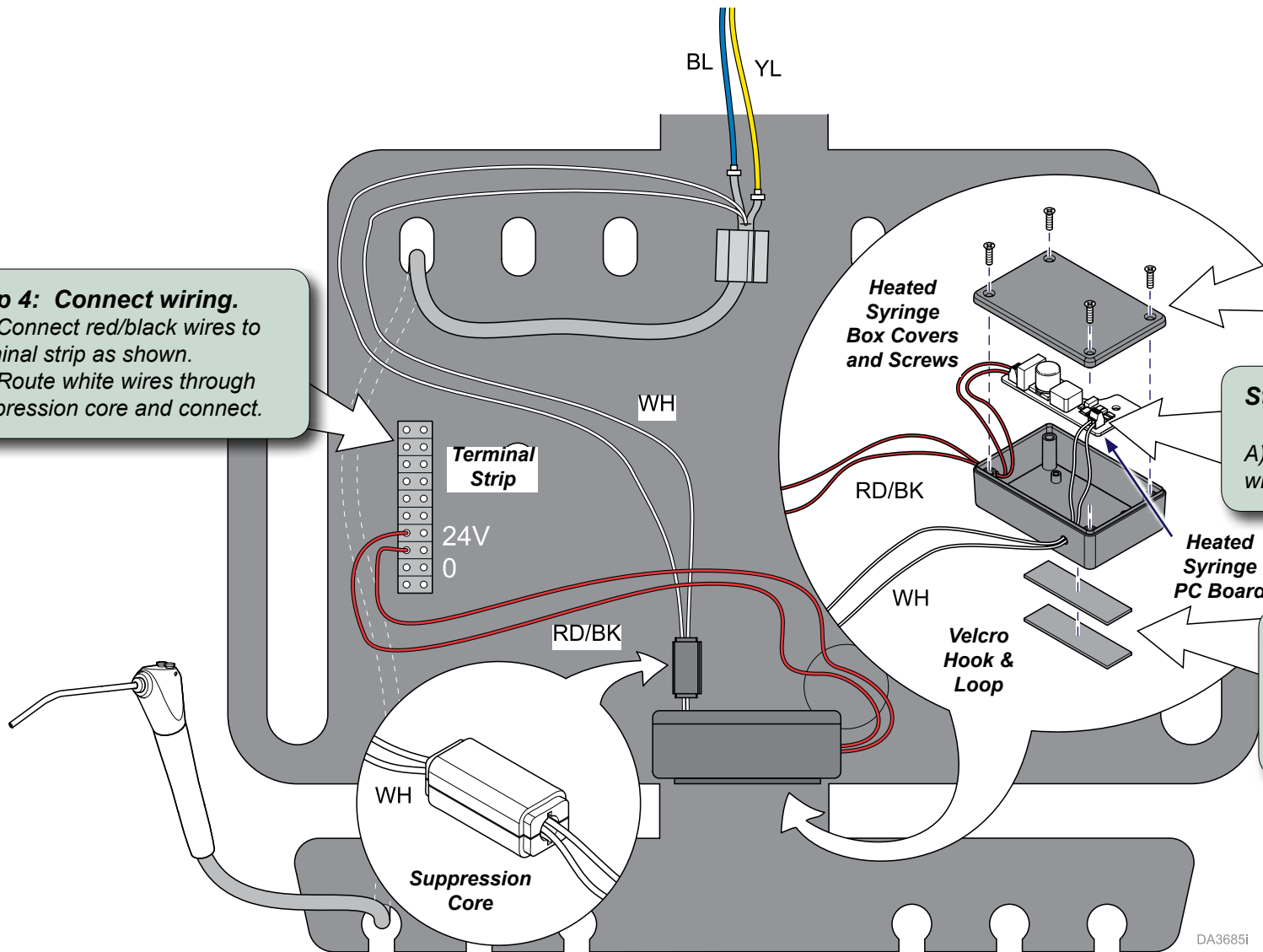

Step 6: Assemble box.

- A) Install covers as shown.
- B) Install screws to cover.

Step 5: Make wiring connections to PC Board.

- A) Connect red/black and white wires to syringe PC Board as shown.

Step 7: Secure box.

- A) Apply velcro to unit housing and box.
- B) Position box as shown and apply pressure to secure.

Step 8: Connect power and verify operation.

- A) Connect chair to power supply.
- B) Test syringe operation.



Because we care.