Pediatric Examination Table

[English](#)
[Español](#)
[Français](#)

For Models:
640 w/ Midmark Scale

User's Guide

Product Information

Dealer :


**Midmark® Authorized Service
Company:**

Table Model / Serial Number:

Date of Purchase:

Scale Model / Track Number:

**Digital Scale
Model / Track
Number Location**

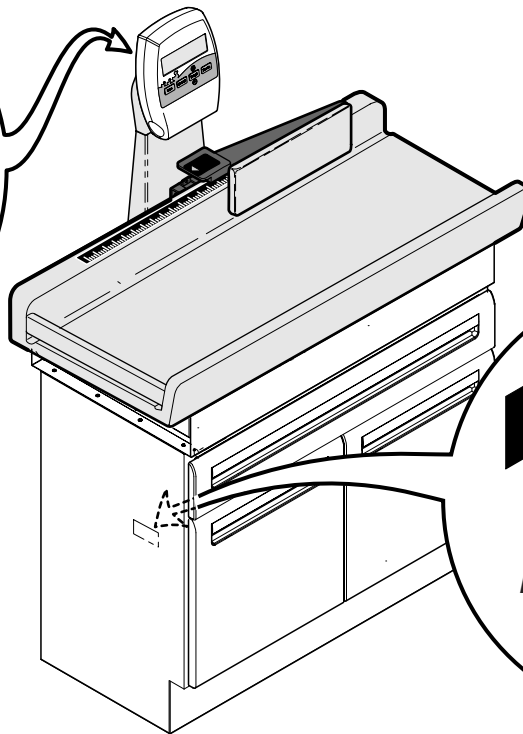**640 Table
Model / Serial
Number Location**

Product Registration

To register your product warranty go to Midmark.com.

Table of Contents

Important Information

<i>Safety Symbols</i>	4
<i>Digital Scale Accuracy</i>	5
<i>Intended Use</i>	6
<i>Electromagnetic Interference</i>	6
<i>Transportation / Storage Conditions</i>	7
<i>Disposal of Equipment</i>	7
<i>Authorized Representatives</i>	7
<i>Safety Instructions</i>	9

Installation

<i>Digital Scale: AC Adapter Cable Connection</i>	10
<i>Storage Shelf</i>	11

Operation

<i>Zero the Scale</i>	12
<i>Units of Measure</i>	13
<i>Weighing a Patient</i>	14
<i>Measuring Patient Length</i>	15
<i>Proper Weighing of Patients Guidelines</i>	17
<i>Paper Roll</i>	18

Maintenance

<i>Calling for Service</i>	19
<i>Cleaning</i>	19
<i>Periodic Maintenance</i>	19
<i>Leveling the Table</i>	20
<i>Door Adjustment</i>	20
<i>Measuring Arm Alignment</i>	21

Specifications

<i>Weights, Electrical Specifications, and Regulatory Compliance Information</i>	22
<i>Dimensions</i>	23

Warranty Information

<i>Limited Warranty</i>	24
-------------------------------	----

Important Information - Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a situation which could result in equipment damage.

Note

Amplifies a procedure, practice or condition.

Symbol Glossary

These symbols may appear on your equipment and/or in the manuals.

No Pushing

Refer to manual

Keep Dry

Fragile

Catalogue Number

Serial Number

Manufacturer


Type BF, Applied Part
(Upholstery)


Proper Shipping
Orientation

Maximum stacking height
(palletted units)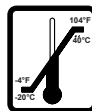

Temperature limits for
storage of product

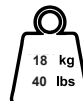Patient Weight Limit

Important Information - continued

Digital Scale Accuracy

Equipment Alert

Before using the digital scale, verify the scale is measuring accurately using a reference weight of known value. In addition, refer to the Preventive Maintenance Schedule for the recommended frequency of this check. If the scale is determined to be out of calibration, contact your authorized Midmark® service provider before using the scale.

Intended Use

The examination table is intended to provide positioning and support of patients for general examination procedures conducted by medical professionals. The intended patient weight is to be between 5lbs to 40lbs, and must be attended by medical professional or parent at all times. The digital scale is intended for regular monitoring to assess patient weight change, as well as measuring patient weight to assist medical diagnosis. The manual infantometer is intended for regular monitoring to assess patient length change, as well as measuring patient length to assist medical diagnosis. The storage drawers and shelves are intended to be used to store materials such as table paper and other typical medical examinations supplies.

Caution

The patient on the table must be attended by doctor / assistant or parent at all times.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room
- Plug table into isolated circuit
- Increase separation between table and interfering device
- Contact Midmark® if interference persists

Transportation / Storage Conditions

Equipment Alert

The Midmark® digital scale is a delicately adjusted precision instrument. Avoid excessive shock / vibration. Scale should not be exposed to direct sunlight or located near a heat source. If scale is moved to a location with a temperature difference of 68°F (20°C) or more, allow scale to acclimate for two hours before using.

Ambient Temperature Range:.....-25°C to 60°C (-13°F to 140°F)
Relative Humidity..... 5% to 95% (non-condensing)
Atmospheric Pressure0.49atm (500hPa) to 1.05atm (1060hPa)

Disposal of Equipment

At the end of product life, the table, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Saudi Arabia should direct all questions, incidents and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Authorized Representatives - continued

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,

Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800


Safety Instructions

WARNING

Risk of Electrical Shock Hazard: Accessory equipment connected to the SIP/SOP port must be certified according to the respective IEC standards, i.e., IEC 60950 for data processing equipment or IEC 60601-1 for electromedical equipment. All combinations of equipment must be in compliance with IEC 60601-1-1 systems requirements. Anyone connecting additional equipment to the SIP/SOP port configures a medical system, and therefore is responsible that the system complies with the requirements of the system standard IEC 60601-1-1.

IEC 60950 approved Information Technology Equipment must be placed outside the "patient environment." The patient environment is defined as an area 1.5 m (4.92 feet) from the patient.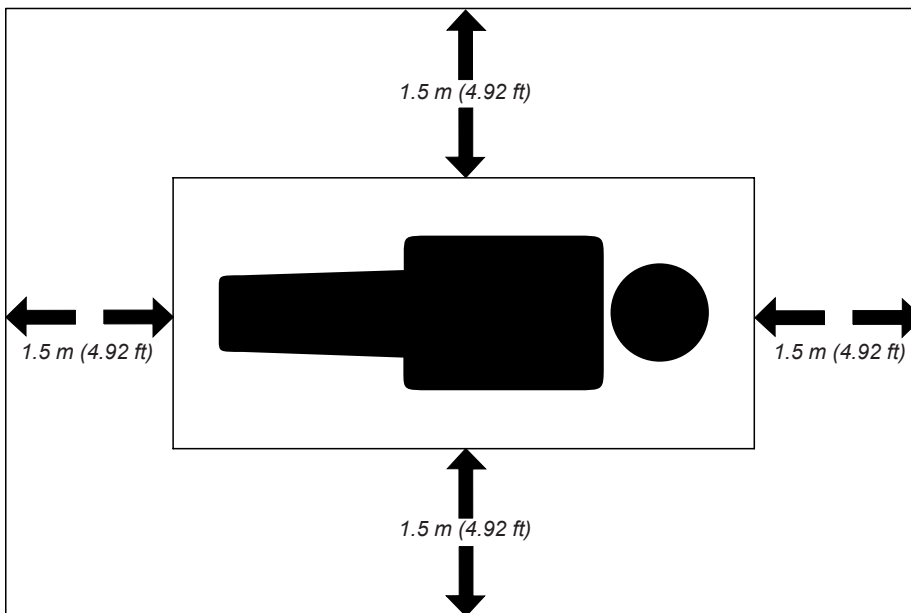

The following options are recommended if there is a need to remediate system leakage current:

- **Redundant protective earth connection shall be made to either the medical device or the other equipment.**
- **The other equipment shall be powered from a medical grade (IEC 60601-1 compliant) separating transformer.**

WARNING

No modification of this equipment is allowed. Table must be positioned in normal use such that the power cords can be easily accessed to unplug. Only transport table unloaded with doors and drawers closed.

Installation

Digital Scale: AC Adapter Cable Connection

Equipment Alert


Only use Midmark® supplied AC adapter, part number 015-3153-00.

Note

Communication ports are intended for Midmark service personnel only.

To connect AC adapter cable...

- Remove access cover from the back of display.
- Plug adapter cable into AC port on display.
- Connect adapter cable plug to outlet.
- Install access cover.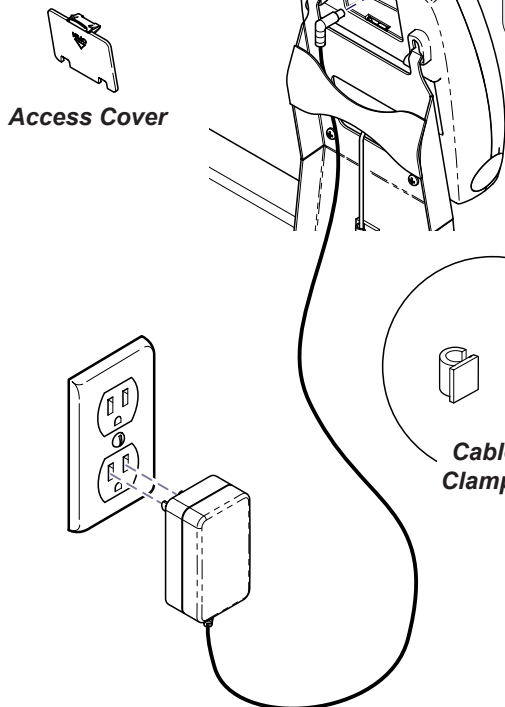


Access Cover

Cable Clamps

Note

Two adhesive-backed cable clamps are provided. These can be used to secure the adapter cable to the table, wall, etc. (see illustration below).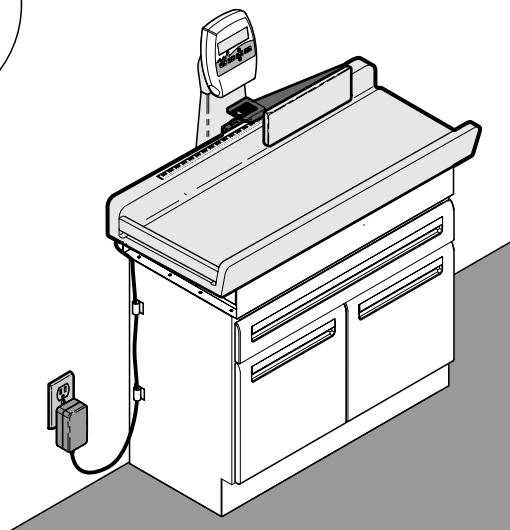


Installation - continued

Storage Shelf

To install storage shelf...

- A) Install four shelf clips.
- B) Latch rear lip of shelf into shelf tabs.
- C) Lower shelf onto shelf clips.

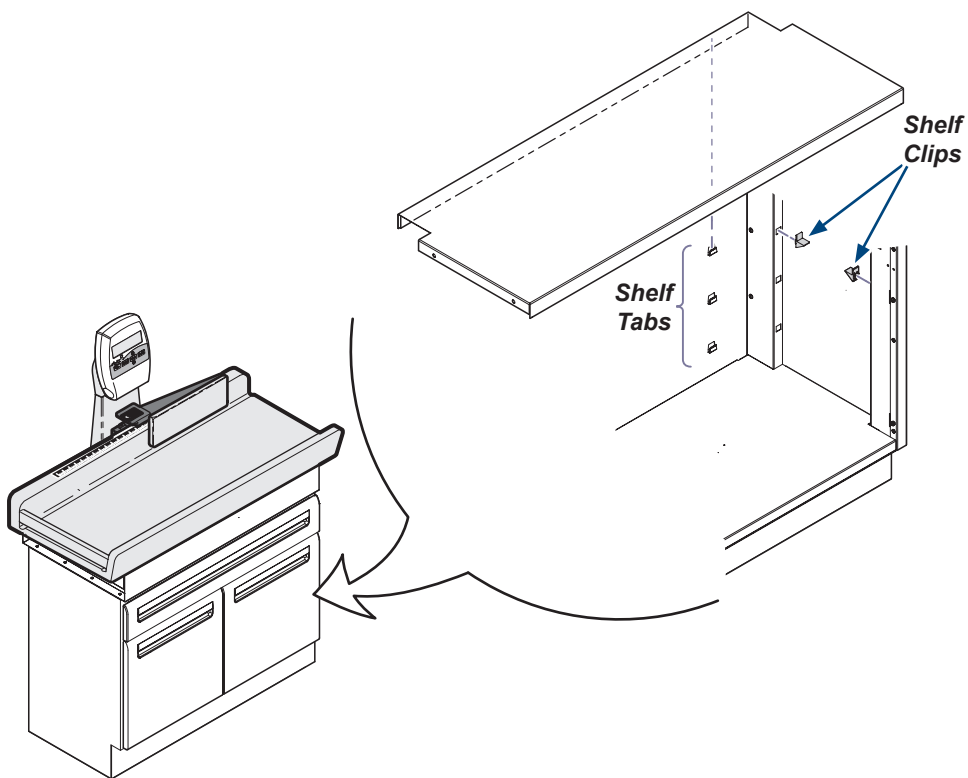Operation

Digital Scale:

Zero the Scale

Best practice is to always zero the scale prior to weighing the patient. For details, please refer to the weighing guidelines in this manual.

Caution

The maximum patient weight rating for the table is 18 kg (40 lbs).
The maximum patient length rating for the table is 91 cm (36 in.).
The patient on the table must be attended by the doctor / assistant or parent at all times.


Note

Before positioning patient on exam table, ensure the scale has been zeroed.
Failure to do so may affect the measurement of the patients weight.

To zero the scale...

A) Press and release **Zero/On** button.

Note: The display will flash **SCALE rEAdY** then display zeros.


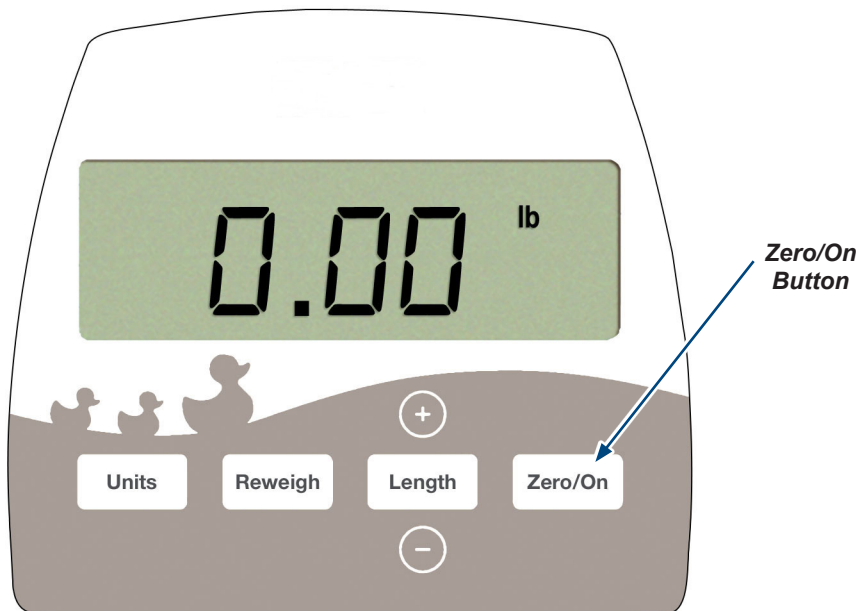


Operation - continued

Digital Scale:

Units of Measure

To change the unit of measure...
A) Press and release **Zero/On** button.
B) When Zeros are displayed, press the **Units** button to scroll through the available units of measure.

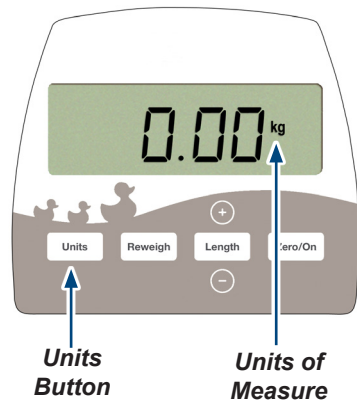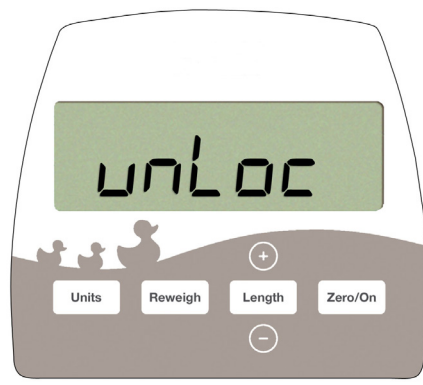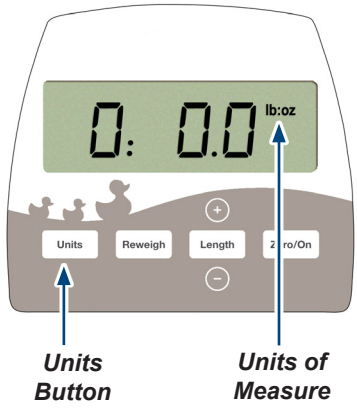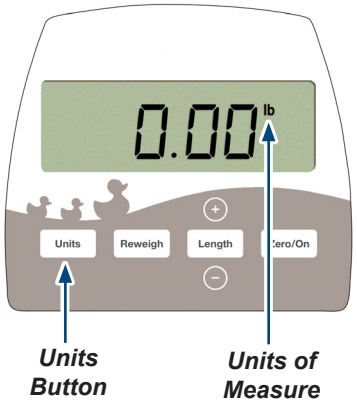To lock or unlock the unit of measure...
A) Press and hold **Units** button for five seconds.
B) The display will read **Loc** or **unLoc** when desired unit of measure is selected.

Note: The other units of measure can be toggled to but will only display for 5 seconds.

Equipment Alert
To turn off unwanted units of measure, contact your local service provider.

Operation - continued

Digital Scale:

Weighing a Patient


Caution

For accurate weight measurement, refer to *Proper Weighing of Patients* in this manual.

To weigh a patient...

- A) Press and release **Zero/On** button.
- B) With zeros displayed, position patient in the center of the table top.
- C) Limit movement until “Stable Hold” or “Average Hold” and patient weight are displayed.

Note: Depending on movement of the patient, “Stable Hold” or “Average Hold” will be displayed.
If patient motion is prolonged, the reading will average the weight over time for the final reading.

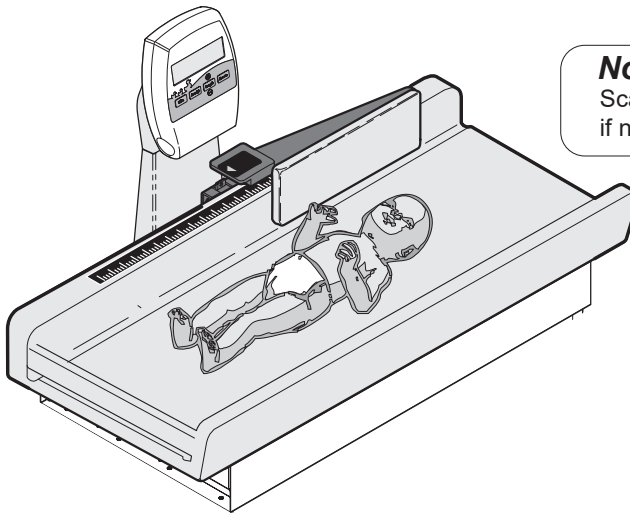Note

Scale will automatically go into *Sleep Mode* if no activity is detected for 30 seconds.

To reweigh a patient...

- A) Press and release the **Reweight** button.

Note: This will allow a new measurement to be taken without the patient having to be removed from table top.

**Reweight
Button**

Operation - continued

Measuring Patient Length

Caution


The maximum patient weight rating for the table is 18 kg (40 lbs).

The maximum patient length rating for the table is 91 cm (36 in.).

The patient on the table must be attended by the doctor / assistant or parent at all times.

To measure patient length...

- A) Lay patient on table with feet against measurement stop and legs extended.
- B) Pivot measuring arm out as shown.
- C) Carefully slide scale arm until it contacts patients head.
- D) Read the measurement at arrow on scale arm.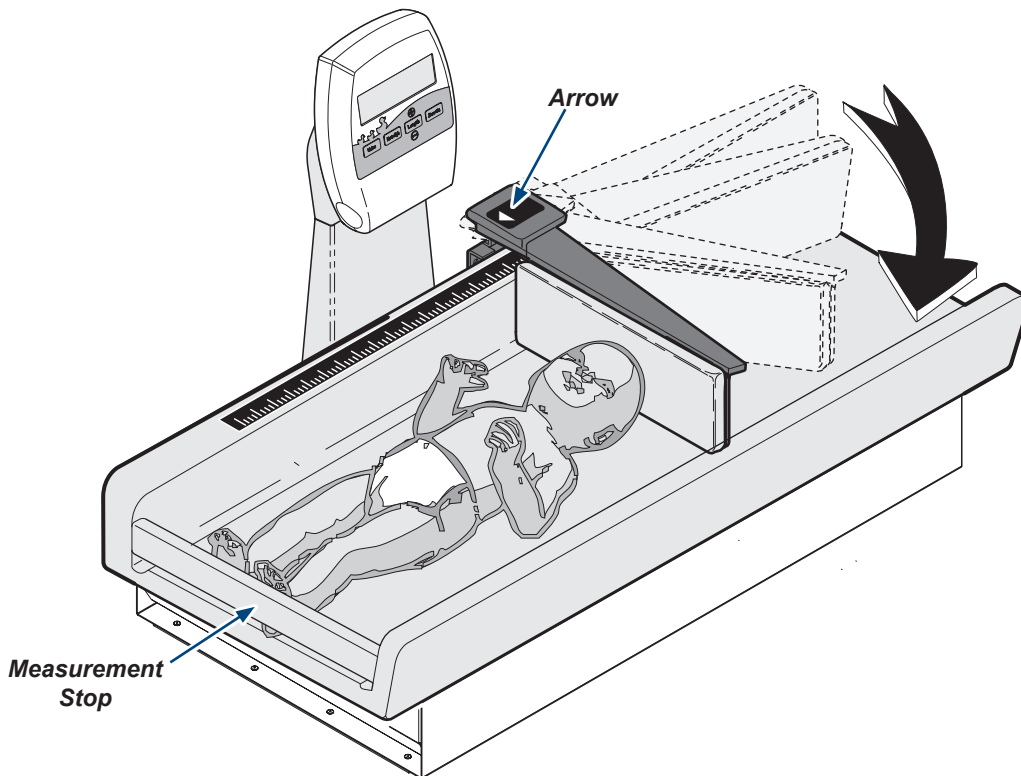

Operation - continued

Measuring Patient Length - continued


Note

This function is intended for length entry input into scale display.

To display patient / length...

- A) Press and release **Length** button.
- B) Press the **Plus** or **Minus** buttons to adjust reading.

Note: Select the length units by pressing the **Units** button to toggle between the length in inches or centimeters. Subsequent presses of Length button will result in toggling between length entry and weight display modes. The scale will power off if no buttons are pressed within a 30 second time period.

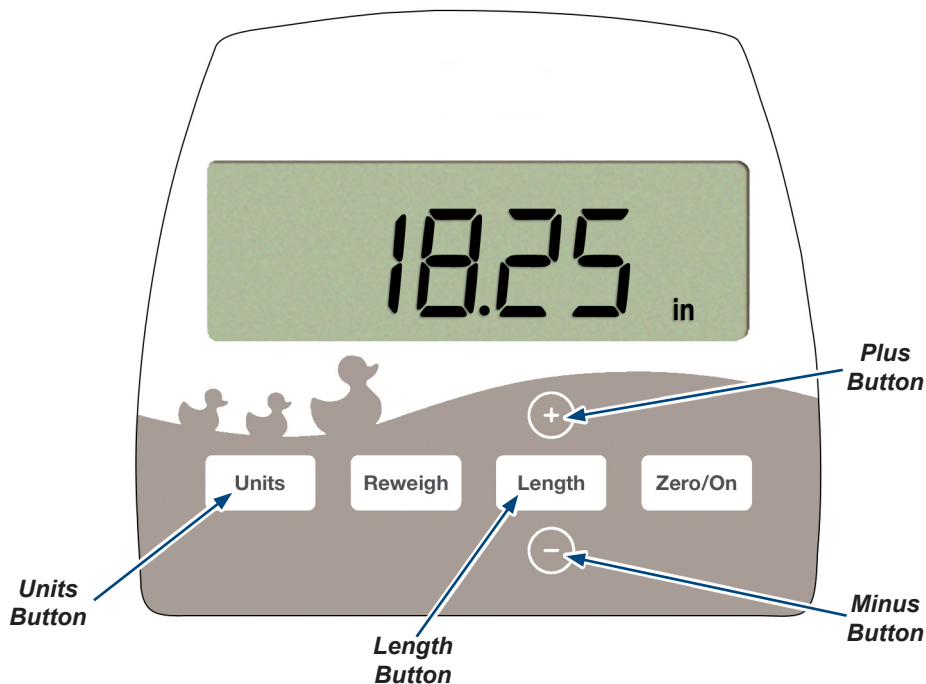
To ensure proper weighing of patients please review the following guidelines.

Proper Weighing of Patients Guidelines

Don't zero the scale while the patient is on the table.


Don't weigh patients with any items such as purses, coats etc. on table.

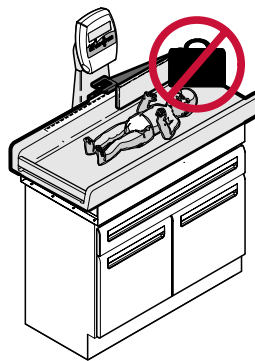Don't lean or otherwise apply load to table while measuring weight.

Do ensure the patient is within edges of table and not grasping scale tower.
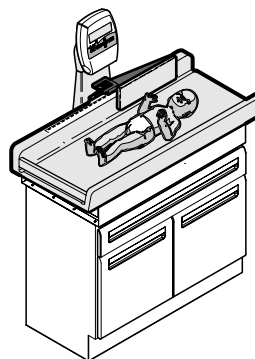


Paper Roll

Paper Roll Size (max.)..... 35.6 cm long x 7.6 cm diameter (14 in. x 3 in.)

To install the paper roll...

- A) Pivot paper roll cover down to open.
- B) Install paper roll, then pivot cover up to hold roll in place.
- C) Feed paper under measurement stop and over table top.
- D) Feed paper under tear strap at opposite end of table.

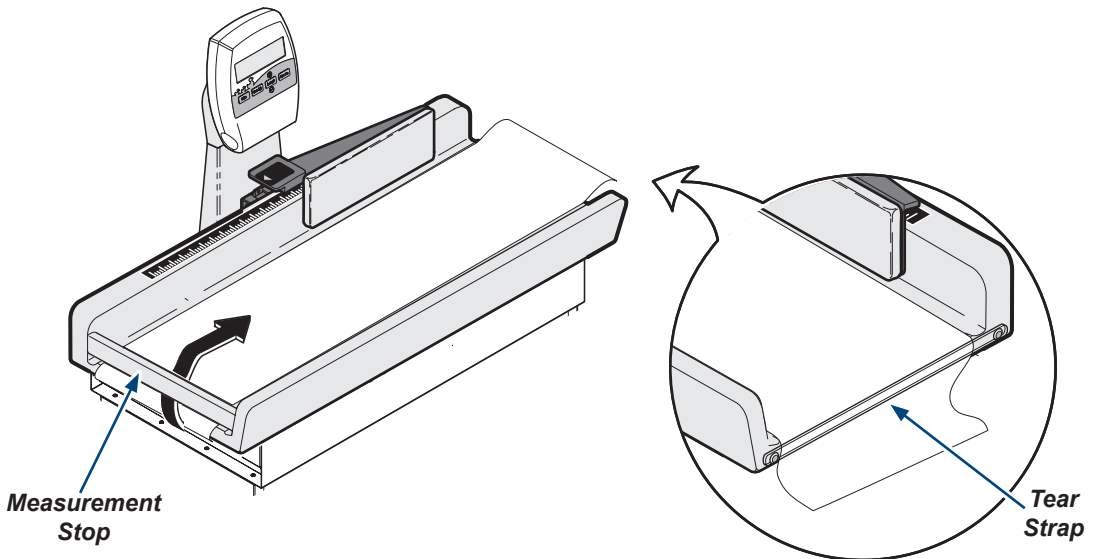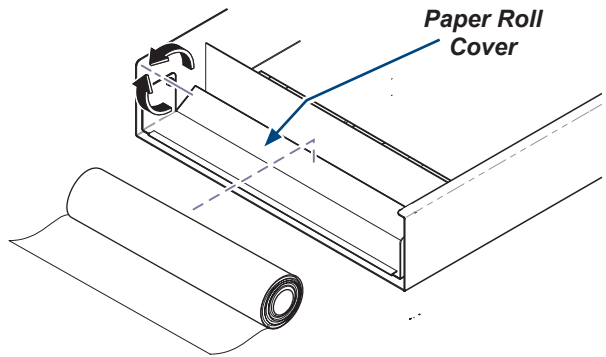Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM
Friday 8:00 AM until 5:00 PM (ET)

Cleaning

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

- Power cord(s) should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

Leveling the Table


Equipment Alert

To ensure proper operation,
the table must be level.

To level the table...

- A) Place a level on table top.
- B) Adjust leveling feet until table is level.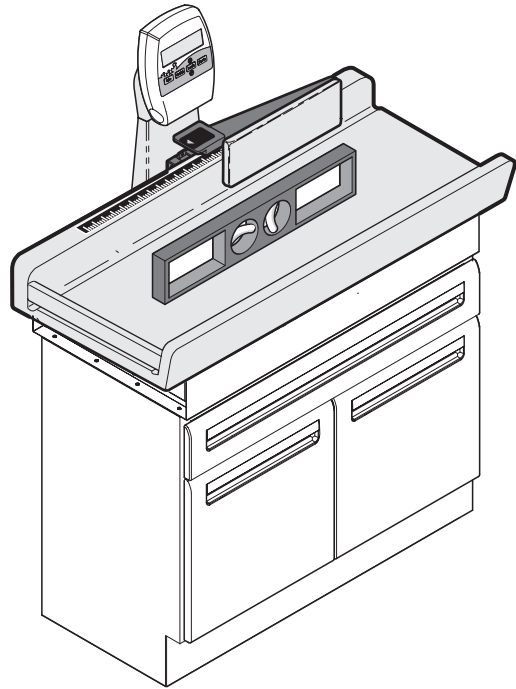


Leveling Feet

Qty: 4 (located at each corner)

Door Adjustment

Depth Adjustment

Turn this screw to move
panel forward / back.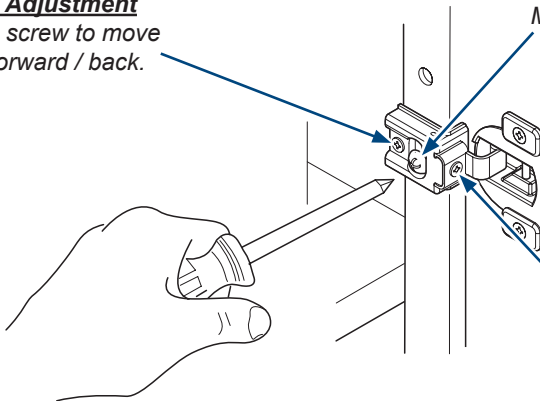

Vertical Adjustment

Loosen this screw on both hinges.
Move door up / down, then tighten screws.

Horizontal Adjustment

Turn this screw to move
panel side to side.


Measuring Arm Alignment

To check measuring arm alignment...

- Pivot measuring arm out as shown.
- Using a carpenter's square, check to see if the measuring arm is square with the back of the table top.
- If necessary, adjust arm until square.

To adjust measuring arm...

Using a 3/32" allen wrench, adjust setscrew in / out until measuring arm is square with the back of the table top.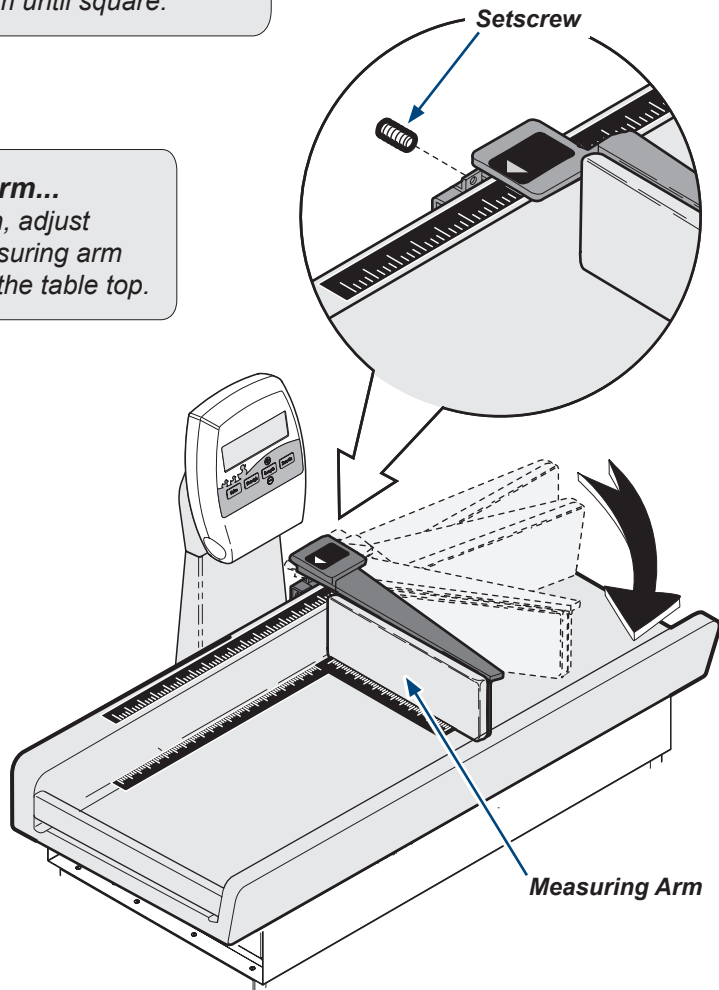

Specifications

Weights, Electrical Specifications, and Regulatory Compliance Information

Specifications Chart	
Regulatory Compliance:	ANSI/AAMI ES 60601-1:2005 CAN/CSA C22.2 NO. 60601-1-08
Patient Weight:	2 kg (4.5 lbs) - (40 lbs)
Paper Roll (maximum size)	35.6 cm long x 7.6 cm diameter (14 in. x 3 in.)
Weight of Table:	77 kg (170 lbs)
Digital Scale Accuracy:	+/- 28 g (1.0 oz) for weights < 10 kg (22 lbs) +/- 56 g (2.0 oz) for weights ≥ 10 kg (22 lbs)
Digital Scale Increments:	0.5 oz 0.02 lbs 0.01 kg
Power Source AC Adaptor:	Input: 100-240VAC, 50-60Hz, 0.1A
Power Consumption:	10 Watts max.
Display:	LCD w/ 25 mm (1 in.) numerals
Linear Scale (Measuring Arm) Accuracy: Minimum Increments:	± 0.5 cm (1/8 in.) 0.5 cm (1/8 in.)
Table Dimensions:	Refer to: <i>Dimensions</i>


WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air or nitrous oxide.

Dimensions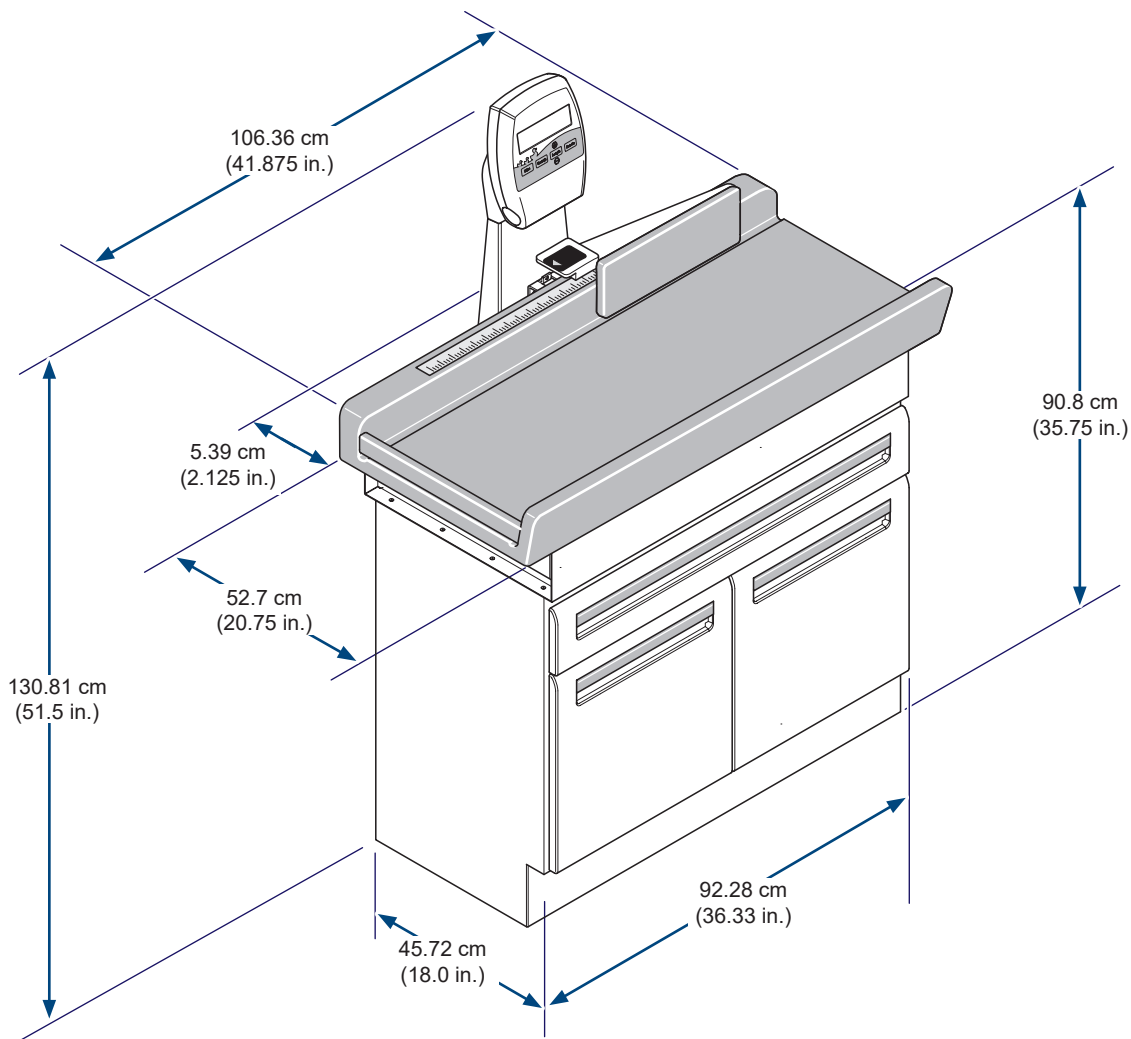

Midmark® + Ritter® Medical Equipment (Tables, Chairs, Physician Seating, Lights, Workstations and Sterilizers) Limited Warranty

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components. QuickClean® Ultrasonic Cleaners are warranted for a period of three (3) years.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at midmark.com; by phone at 1.800.MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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TP202 20-42-FO-00014 Rev A1 C2169 © Midmark Corporation 2015