



# Veterinary Folding Exam Table

For Models:

8012-00-001

8012-00-XXX

8012\*



## User Guide

TP202 Rev. A

Style P

003-2603-00 Rev. EA3 (10/13/23)

© Midmark Corporation 2013


# Product Information

**Dealer :**

**Date of Purchase:**

**Model / Serial Number:**

**Midmark Authorized Service Company:**

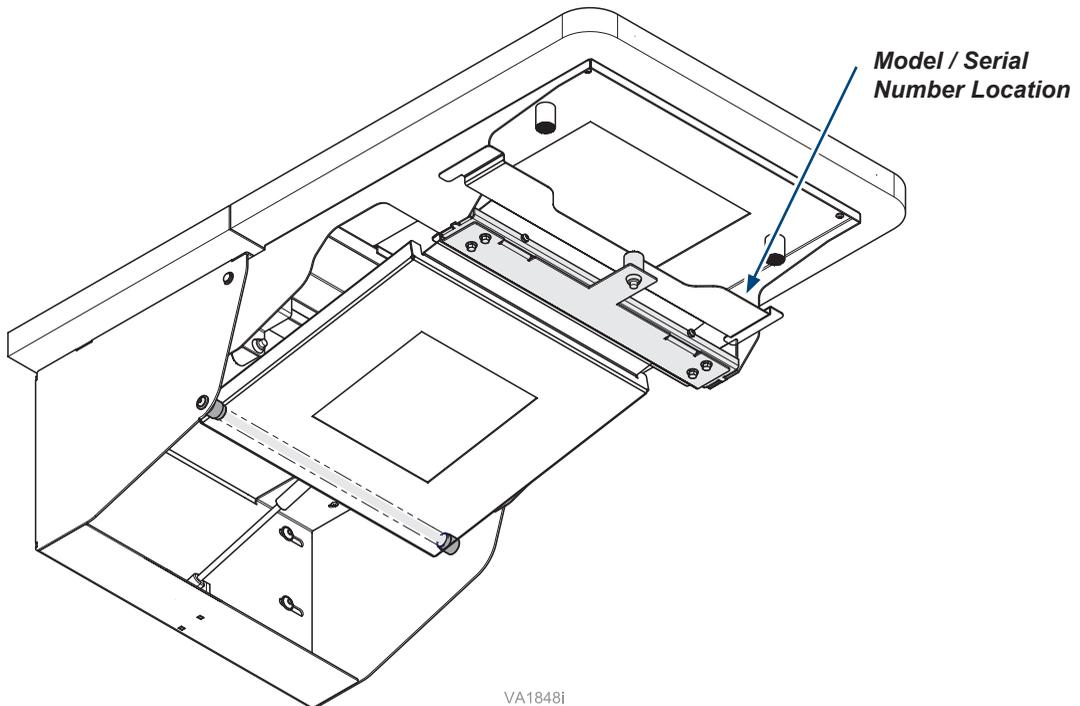

VA1848i

# Table of Contents

## **Important Information**

<u>Serial Number Location</u> .....	2
<u>Safety Symbols</u> .....	4
<u>Product Registration</u> .....	4
<u>Transportation / Storage Conditions</u> .....	4
<u>Intended Use</u> .....	5
<u>Disposal of Equipment</u> .....	5

## **Operation**

<u>Table Up / Down</u> .....	6
------------------------------	---

## **Maintenance**

<u>Calling for Service</u> .....	7
<u>Cleaning</u> .....	7
<u>Periodic Maintenance</u> .....	7

## **Specifications**

<u>Weights, Dimensions</u> .....	8
----------------------------------	---

## **Warranty**

<u>Limited Warranty</u> .....	9
-------------------------------	---

# Important Information

## Safety Symbols



### **DANGER**

Indicates an imminently hazardous situation which will result in serious or fatal injury. This symbol is used only in the most extreme conditions.



### **WARNING**

Indicates a potentially hazardous situation which could result in serious injury.



### **Caution**

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices



### **Equipment Alert**

Indicates a potentially hazardous situation which could result in equipment damage.

## Product Registration



## Transportation / Storage Conditions

Ambient Temperature Range:..... 41°F to 100°F (5°C to +38°C)  
Relative Humidity..... 10% to 90% (non-condensing)  
Atmospheric Pressure ..... 500hPa to 1060hPa (0.49atm to 1.05atm)

## ***Intended Use***

This product is intended to provide support for small to medium sized patients during general examinations, minor procedures and preparatory procedures conducted by qualified veterinary professionals in both office and hospital environments. This product is for veterinary use only.

## ***Disposal of Equipment***

At the end of product life, the table, accessories and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

# Operation

## Table Up / Down




### **Caution**

*Keep clear of table sections when moving table to prevent pinching.  
Do not disengage table with patient on board.*

### **To move the table up or down...**

- A) *Lift locking arm handle (on either side) to allow table to go down.*
- B) *Lift up the table top to raise table top.*

*Note: A minimal amount of force is required to push table down.*

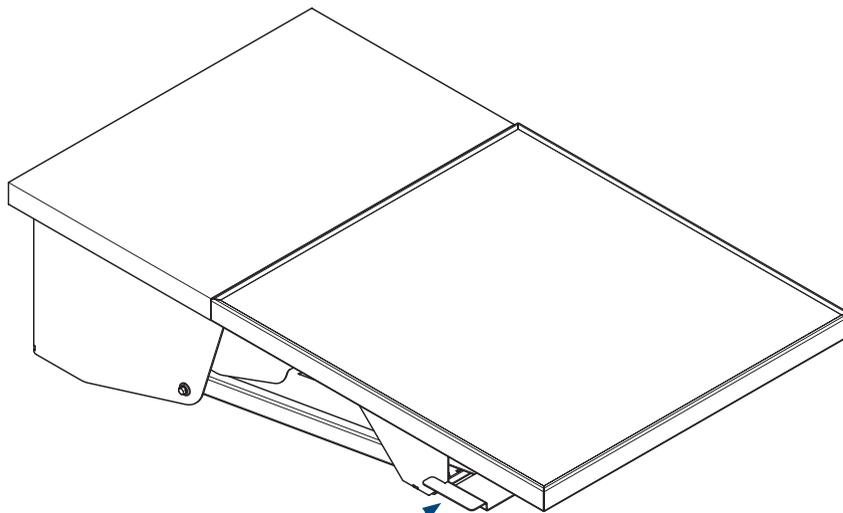

**Locking Arm**  
**Typical Both Sides**

VA18571

# Maintenance

## Calling for Service

### Note

*Model / serial number information is required when calling for service.*

If service is required, contact Midmark directly:

US 844.856.1232 / Canada 937.526.8585

Monday-Thursday 8:00 am until 6:00 pm

Friday 8:00 am until 5:00 pm (ET)

[midmark.com](http://midmark.com)

## Cleaning

### Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

### Stainless Steel Surfaces:

For best results, stainless steel should be cleaned weekly or whenever films or deposits become apparent. Use a mild cleaning agent and whenever possible rinse thoroughly and towel dry. More stubborn deposits or tightly adhering debris may require vigorous rubbing or brushing. If a commercial cleaner needs to be used, make sure it is approved for use on stainless steel surfaces.



### Equipment Alert

*Always use nylon or plastic bristle scrubbers. The use of steel wool or steel brushes may cause steel particles to become embedded in the surfaces and will cause rust.*

*Avoid prolonged surface exposure to chlorides, bromides, iodides and thiocyanates. Salty solutions allowed to evaporate on stainless steel surfaces may cause pitting, corrosion or metal discoloration.*

## Periodic Maintenance



Periodically inspect the following areas:

- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

# Specifications

## Specifications Chart

Patient Weight (max):	125 lbs. (56.7 kg)
-----------------------	--------------------

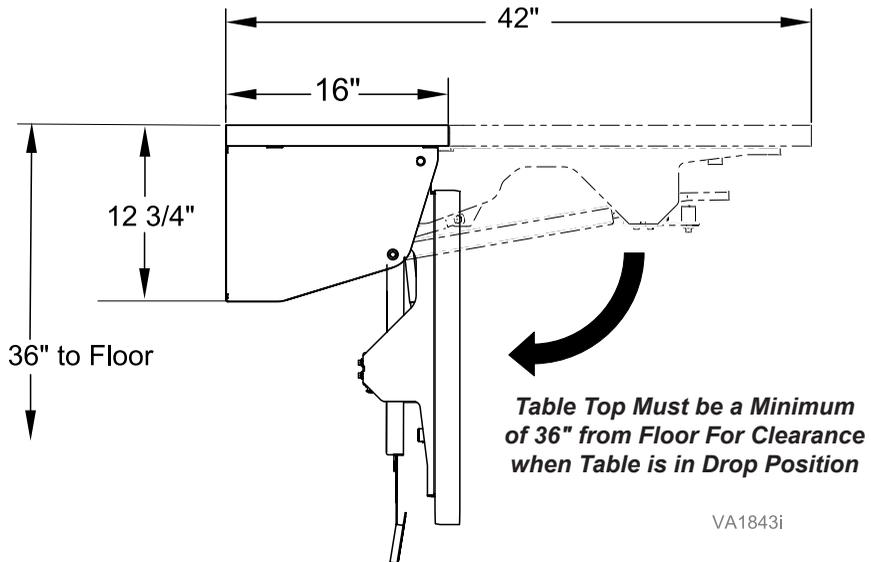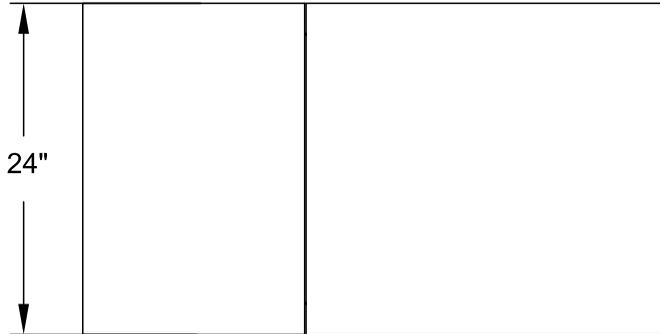

# Warranty Information



# Warranty Registration





**Notes:**

**Midmark Corporation**

60 Vista Drive

Versailles, OH 45380 USA

1.800.643.6275

1.937.526.3662

