

Install Guide Midmark 630 HumanFORM® Procedure Chair

Applies to Models:


630 (-010 thru -013)

630 (-020 thru -023)

Language of origin: English


To install seat / back upholstery...

- A) Position back section of chair flat.
- B) Position front of seat section against seat mounting board.
- C) Press seat section down firmly.
- D) Position top of back section against the top of the back mounting board.
- E) Check for a consistent gap between seat and back sections.
- F) Press back section down firmly.

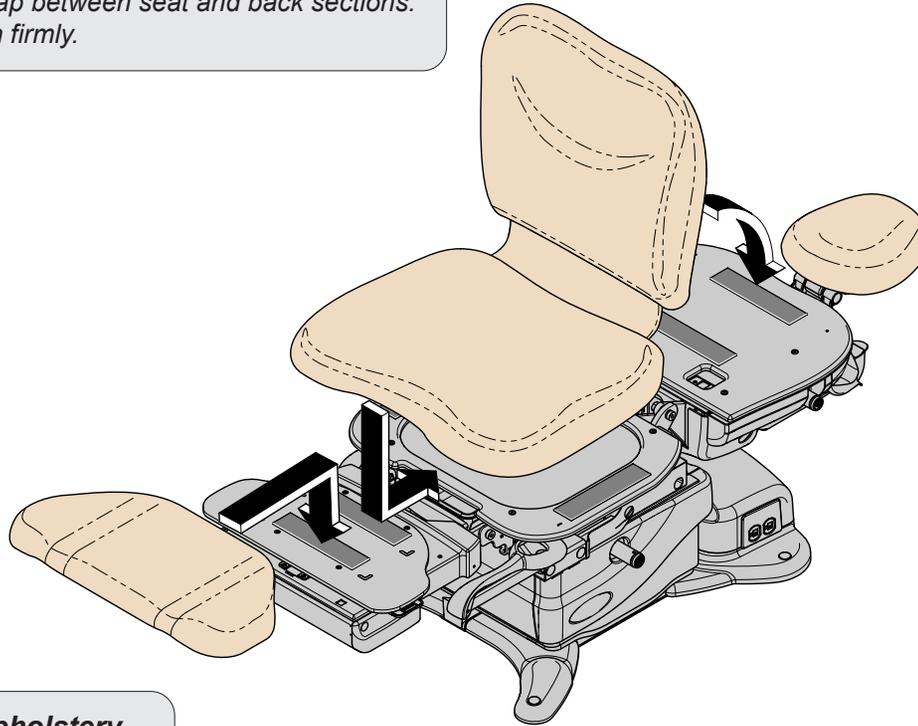

To install headrest assembly...

- A) Loosen headrest adjustment knob.
- B) Insert mounting stud into headrest fixture.
- C) Position headrest as desired.
- D) Tighten headrest adjustment knob.

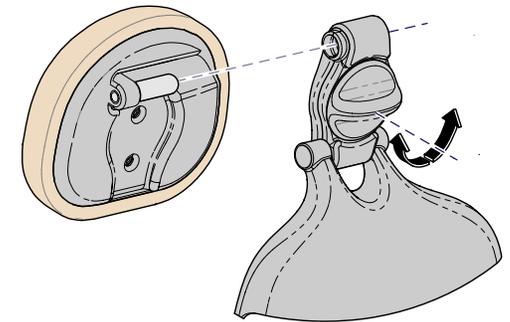

To install footrest upholstery...

- A) Align hook and loop fastener.
- B) Press down firmly.

MA8978I