

Quick Reference Guide Midmark 630 HumanFORM® Procedure Chair

Model Number:
Serial Number:

Dealer Information:

Purchase Date:

Upholstery Color:

003-2641-00 Rev FA3 (8/10/22)

This guide outlines basic operation of the chair. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



MA8954p

Maintenance

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner buildup.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue buildup, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner. Use of chlorine based disinfectant can cause damage to the hand control.

Calling for Service


If service is required, contact your Midmark dealer. To contact Midmark directly:
1.800.MIDMARK (1.800.643.6275)
Monday-Friday 8:00 am until 5:00 pm (EST)

- Français : Référez-vous à midmark.com
- Deutsch: Siehe midmark.com
- Español: Refiera a midmark.com

Midmark Corporation
60 Vista Drive
Versailles, OH 45380 USA
1.800.643.6275
1.937.526.3662
midmark.com

To adjust the headrest...


- Loosen knob.
- Move pivot points to desired positions.
- Tighten knob.

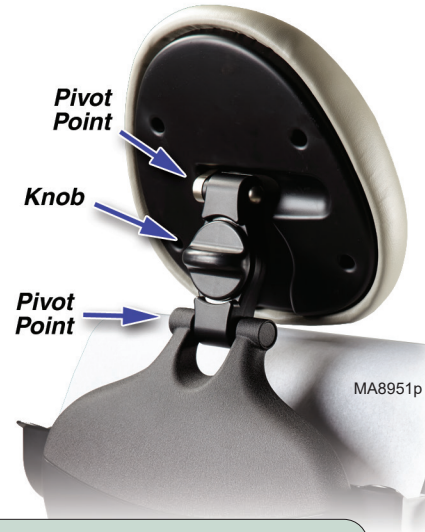

MA8951p

To adjust the foot rest...

- Pull up on both release handles.
- Slide foot upholstery to desired position.
- Release release handles.

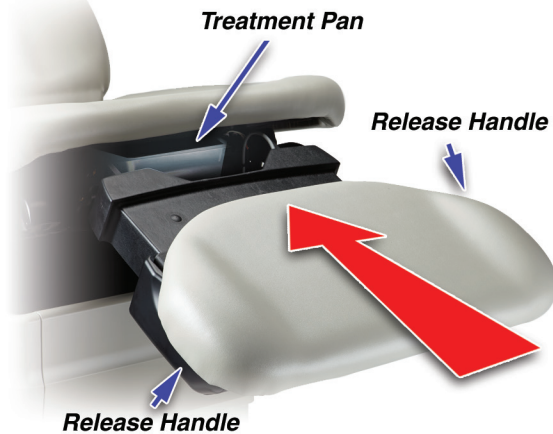

MA8952p

Note

The Foot Up and Down functions are disabled when the treatment pan is pulled out. (You will hear several "beeps" if Foot Up or Foot Down is activated with pan pulled out).

To resume Foot Up or Foot Down operation...
Push treatment pan all the way in, then press Foot Up or Foot Down button.

Rotational Base (optional)

This feature allows you to rotate the chair base 350°.

To rotate the chair base...

- Press and release brake pedal.
- Rotate the chair to desired position.

To lock base in position...

- Press and release brake pedal.

Note: The base will lock automatically after one minute.



MA8953p


Brake Pedal

To operate the Base / Back / Tilt / Foot functions...


Press and hold the appropriate button.

ATTENTION: The Foot Up and Down functions are disabled when the treatment pan (under seat) is pulled out.

Example: The Foot Up button is called out in the illustration.

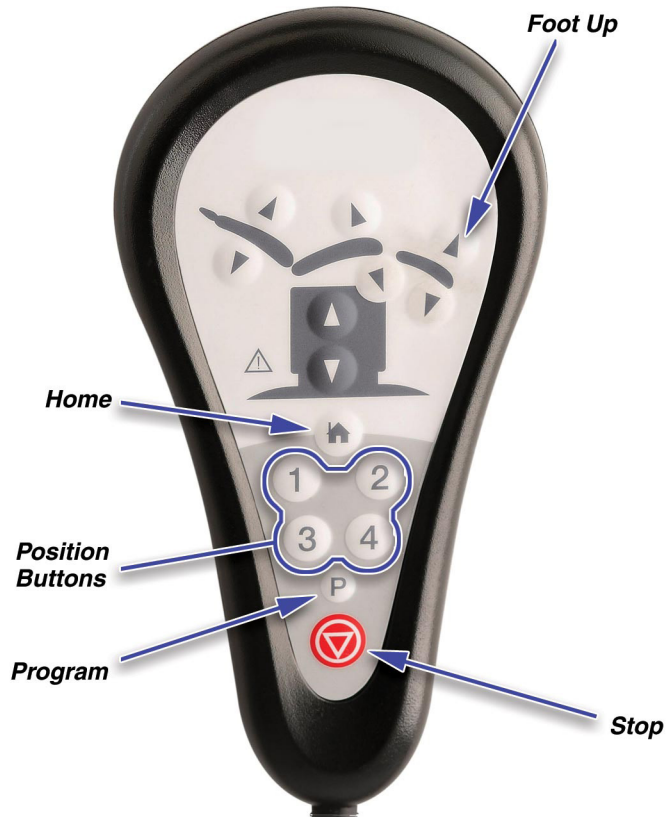

MA8955p

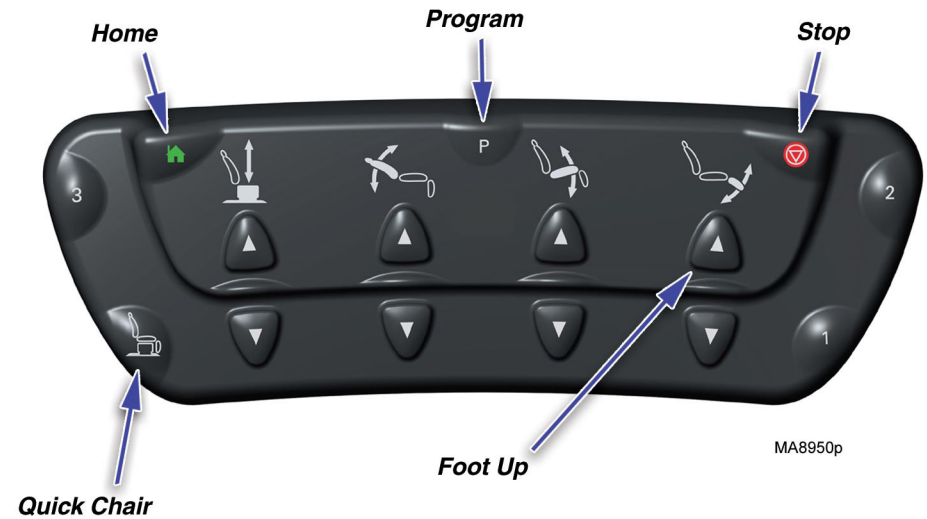

MA8950p

Programming Chair Positions (programmable models only)

This feature allows you to easily recall frequently used chair positions.

To program a chair position...

- A) Move the chair to desired position.
- B) Press the Program button. [single "beep"]
- C) Press the desired Position button (1, 2, 3 or 4). [three "beeps"]

Tip: You must press the Position button within 5 seconds of pressing the Program button.

To recall a programmed position...

Press the desired Position button (1, 2, 3 or 4).

Note

To stop movement of any automatic chair function...
Press the Stop button on the hand or foot control.

Home Function

This function automatically lowers the base so that your patient may easily mount or dismount the chair.

To lower the base to 22 in. (56 cm)...

Press, then immediately release the Home button.

To lower the base to 18 in. (46 cm)...

Press and hold the Home button until you hear a single "beep".

Quick Chair Function (w/foot control only)

This function activates the Base Down, Tilt Down, Foot Down and Back Up functions at the same time.

To activate the Quick Chair function...

Press and hold the Quick Chair button on the foot control.