

## Accessory Rail [9A557 / 9A558]

### Applies to Models:


630 (-010 thru -013)  
630 (-020 thru -023)  
631-001

Language of origin: English

### To remove existing hand control mount...


- Loosen knob.
- Remove existing hand control mount.

**Note:** If socket has protective cap installed, remove it.

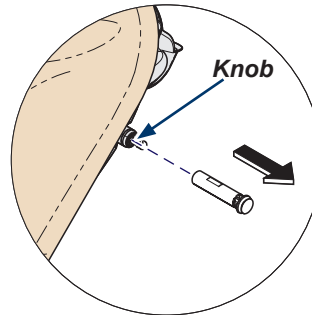

### Note

The spline shaft is keyed.  
Rotate accessory rail until it slides into socket.

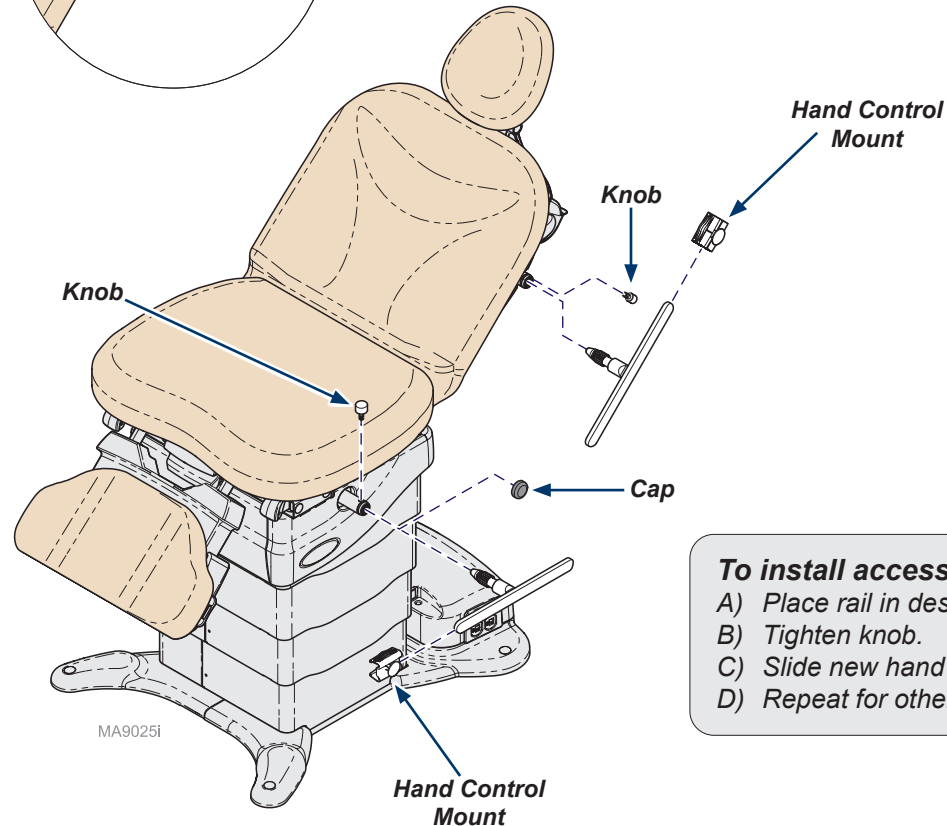

### To install accessory rail...

- Place rail in desired location.
- Tighten knob.
- Slide new hand control mount onto rail.
- Repeat for other side.