

Rail Mount System [9A564 / 9A626]

Intended Use:

To mount 253 exam light
to Midmark® 630 and 631-001 chair.

Restrictions:

Requires Accessory Rails
9A557001 (28" Upholstery)
9A557002 (32" Upholstery)

Applies to Models:

630 (-010 thru -013)
630 (-020 thru -023)
631-001


Language of origin: English

To Install rail mount assembly...

- Slide rail mount weldment onto accessory rail as shown.
- Install locking knob and tighten to secure weldment in place.


To Install light and cord restraint...

- Align light with holes in mounting plate.
- Secure light by installing two screws thru the top holes.
- Position cord restraint, then install two bottom screws.

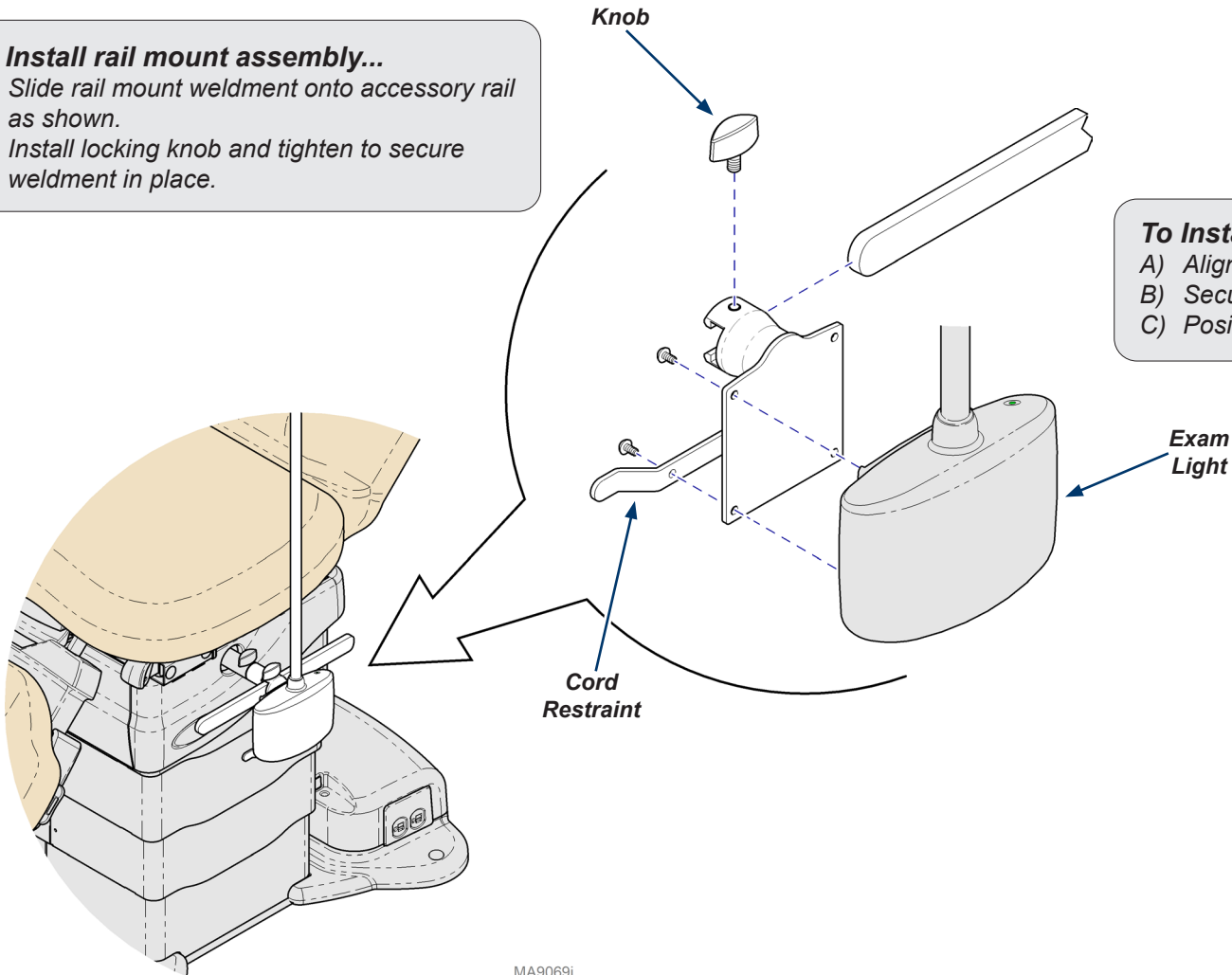

Before using light:

Remove protective film from lens.

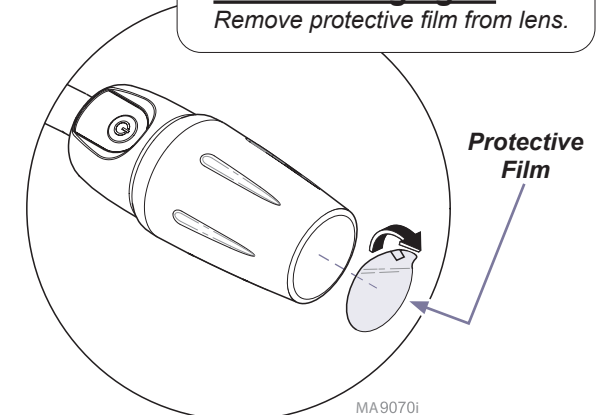

MA9069i

MA9070i