

Articulating Chair Arm [9A550]

Applies to Models:

630 (-010 thru -013)
630 (-020 thru -023)
631-001

Language of origin: English

Intended Use:

To support the patient's arms during procedures.

Restrictions:

Do not exceed maximum weight rating for the chair arms.
Maximum weight rating (each arm): 225 lb (102 kg)

Note: The spline shaft is keyed.
Rotate chair arm until it slides into socket.


To install chair arms...

- Remove cap from arm socket.
- Position arm as shown.
- Insert arm into socket until it locks into place.



Caution

Be sure the patients arms and hands are properly placed on top of chair arms during the reclining of the back section. Failure to do so could result in personal injury.

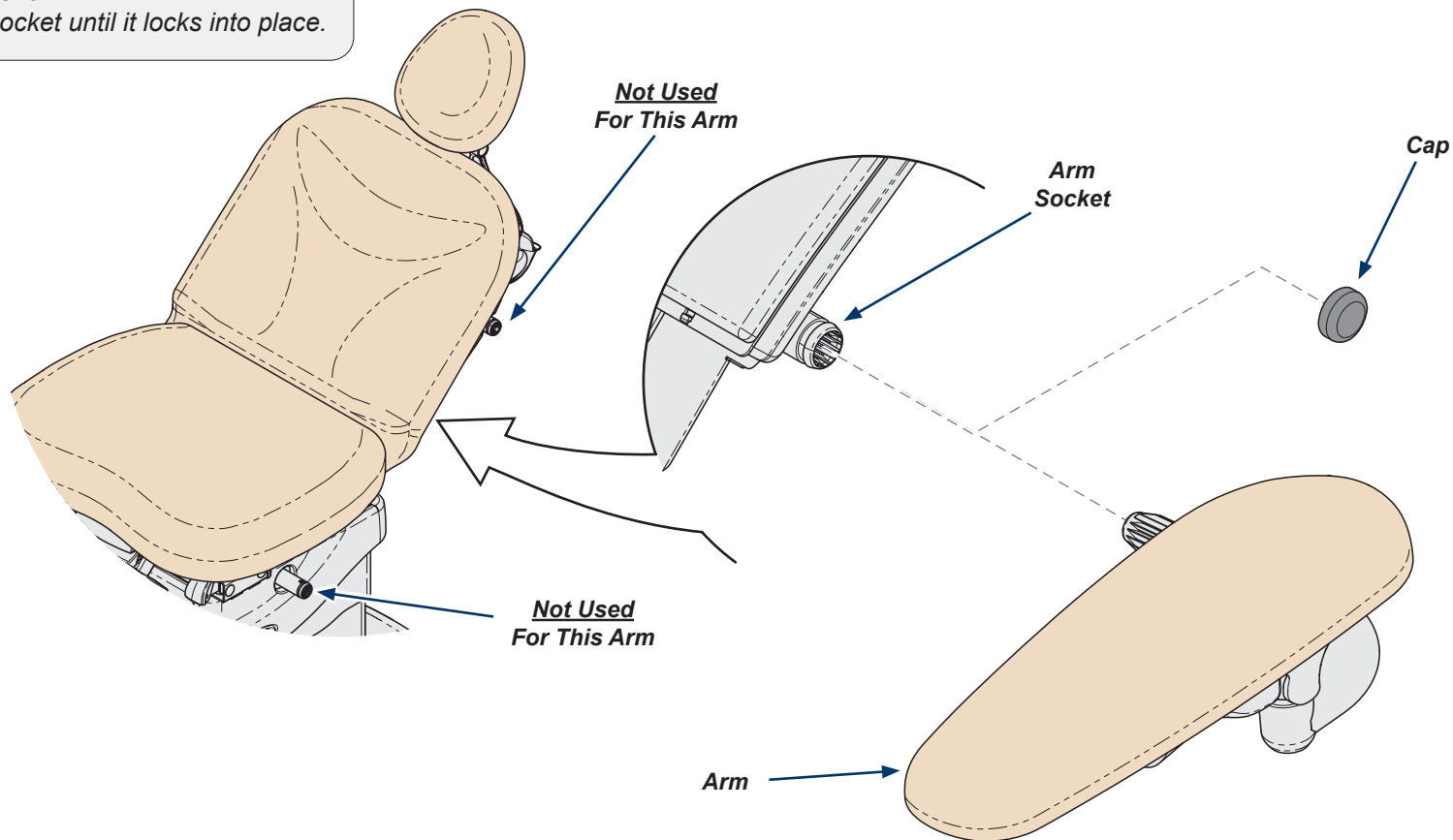

MA9027i