



PCB w/Photo Interrupter Installation [002-1752-00 (115V) & 002-1752-01 (230V)]

Components:
PC Board 1
Photo Interrupter 1
Pressure Interlock
Assembly 1

Special Tools:
none


Note

This kit is required for M3 sterilizers having a Serial Numbers V811650 thru Vxxxxxx.



Caution

Always disconnect power and allow sterilizer to cool before removing cover.


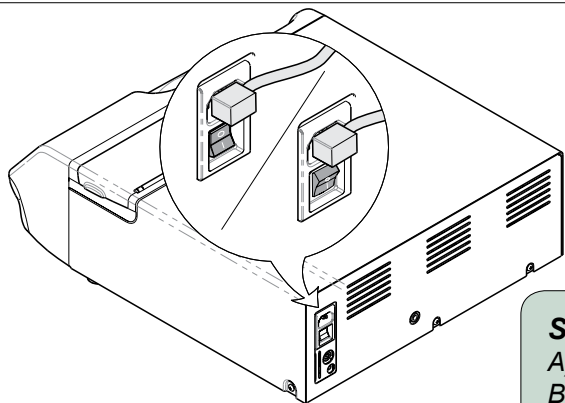


Step 1: Remove Cover.

- A) Disconnect power to the sterilizer.
- B) Remove screws.
- C) Slide cover back and up to remove.

Step 2: Remove PC Board assembly.

- A) Disconnect all wire connections from PC Board.
- B) Disconnect transducer tube.
- C) Remove two nuts.
- D) Lift PCB assembly straight up to remove.
- E) Repeat this step in reverse to install the new PC Board Assembly.

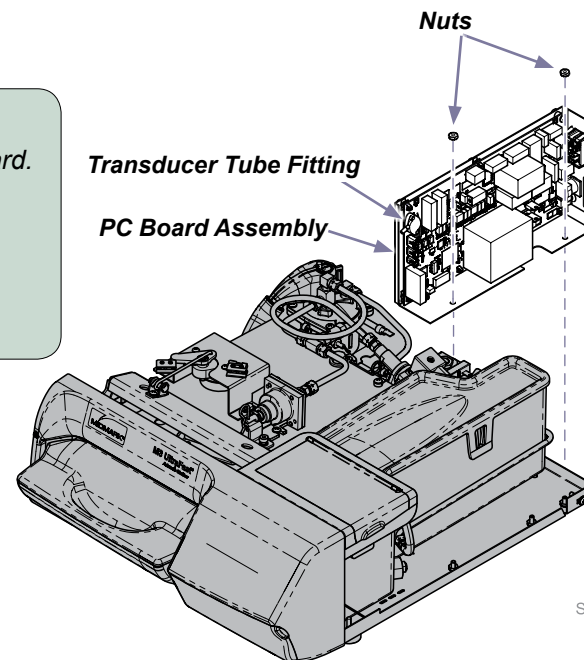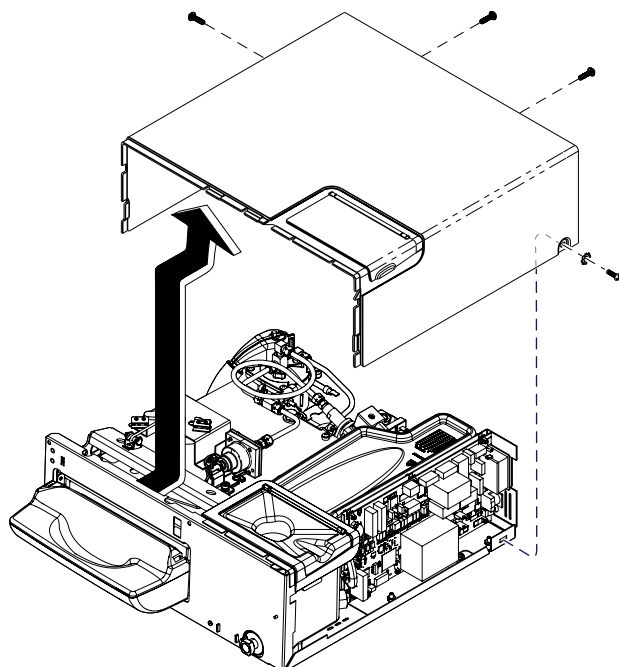


SA1785i

Step 3: Install Photo Interrupter.

- A) Remove cotter pin.
- B) Remove old lock bracket screw and bracket.
- C) Remove inside screw from interlock assembly.
- D) Install the inside screw through photo interrupter mounting bracket and interlock assembly.
- E) Install new lock bracket with tab centered into photo interrupter and aligned in groove on interlock assembly.
- F) Install cotter pin.
- F) Connect photo interrupter wires to J12 on the PC Board.

Pressure Interlock with Photo Interrupter

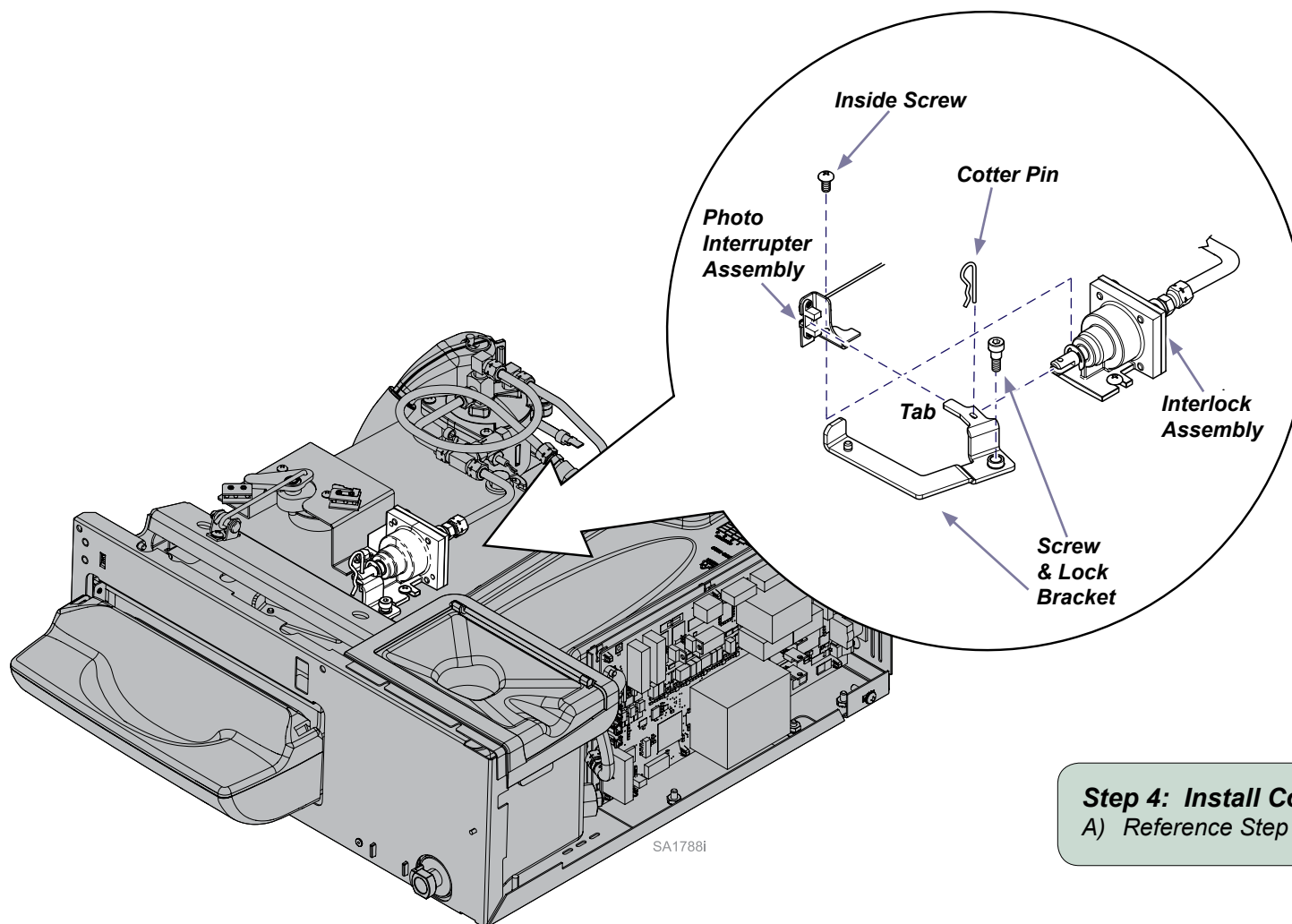

Step 4: Install Cover and Connect Power.

- A) Reference Step 1.