

# Socket Accessory Installation

*Applies to: Patient Support Spline Adaptor [9A582]*


**Components:**  
Adaptor..... 2

**Special Tools:**  
none



## Step 1: Install extension.

- Place adaptor to receiver assembly (9A585).
- Tighten adaptor using included allen wrench and combination wrench.
- Repeat for other side.

*Note: Align notches so adaptor fits completely into receiver assembly.*

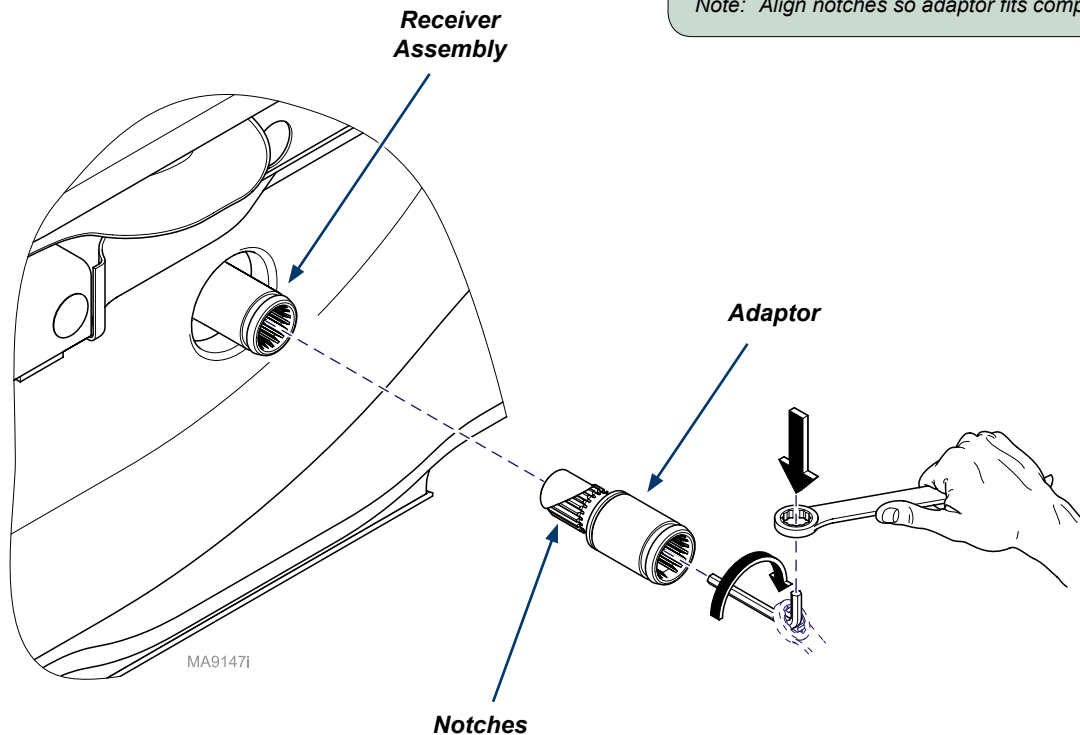

TP200 Rev. A

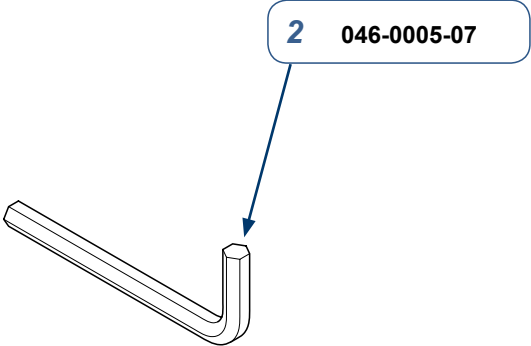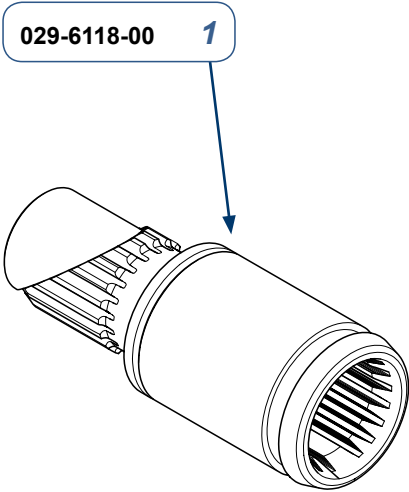

MA9148I

|                         |     |  |
|-------------------------|-----|--|
| Serial Numbers / Dates: | all |  |
|-------------------------|-----|--|

| 9A582 Wide Spline Adaptor Assembly<br>(includes items 1 - 3) |                                      |      |
|--------------------------------------------------------------|--------------------------------------|------|
| Item                                                         | Description                          | Qty. |
| 1                                                            | Adaptor                              | 2    |
| 2                                                            | 1/4 Allen Wrench                     | 1    |
| 3                                                            | Instructions (003-2728-00 not shown) | 1    |