Installation: Power Supply Model 153941

Applies to Models:
153941-001 (115V with USB)
153941-002 (230V with USB)
153941-003 (115V)
153941-004 (230V)

Special Tools:
none

Language of origin: English

Note

Power supply units are available in 115V~ and 230V~ models.
Ensure installed model matches the electrical mains supply power.

WARNING



Before servicing, disconnect power from the power supply.
Replace top cover after servicing (e.g. replacing fuses,
accessing the accessory power terminals, etc.)

Step 1: Unpack power supply and identify connection ports.

- A) Remove power supply from box.
- B) Remove bubble wrap from power supply.
- C) Locate connection ports using the illustration below.

Note

153941-002 model shown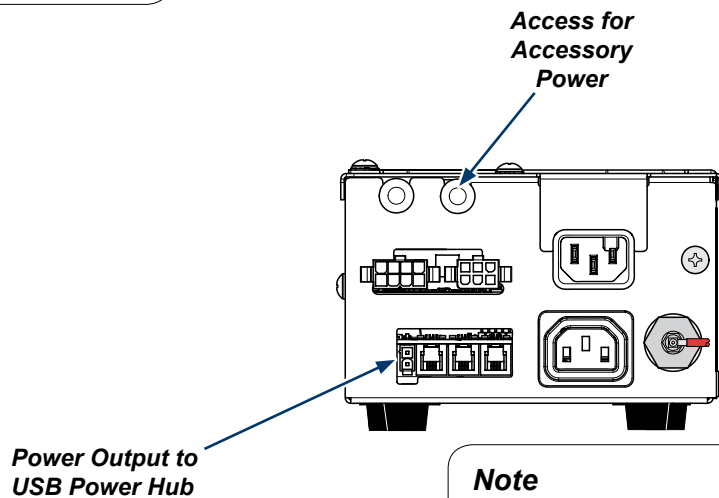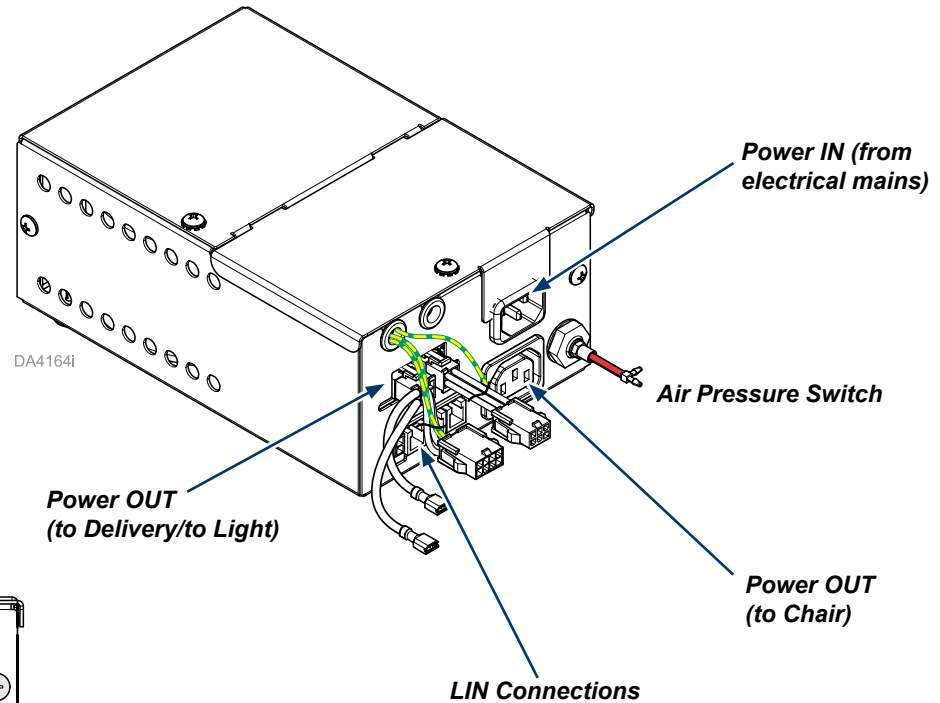

Note

USB Power Hub only shown for 153941-001/-002 models.

Intended Use

The Power Supply is intended to be a non-user-accessible part. It is designed to deliver electricity at predetermined voltage levels (AC and/or DC) to a Midmark Dental Chair, Unit, and/or Light.

Electromagnetic Interference

Midmark dental operatory components are designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this operatory, remove the interfering device from room and / or plug product into an isolated circuit.

Transportation / Storage / Operating Conditions

Transportation / Storage Temperature:.....23°F to 100°F (-5°C to 38°C)

Relative Humidity..... 10% to 90% (non-condensing)

Atmospheric Pressure 7.2 PSI to 15.3 PSI (50 kPa to 106 kPa)

Operating Temperature Range:59°F to 95°F (15°C to 35°C)

Disposal of Equipment

At the end of this product's life, the unit, accessories and other consumable goods may be contaminated from normal use. Consult local codes and ordinances for proper disposal of this equipment and other consumable goods.

Calling For Service

Direct all service inquiries to your authorized Midmark dealer. When calling for service, you must provide the following information:

Model / serial number
Date of purchase
Symptom(s) of malfunction

Ordering Parts

The following information is required when ordering parts:

Serial number & model number
Part number for desired part
(Refer to Exploded Views and Parts at midmark.com)

Non-warranty parts orders may be faxed to Midmark using the Fax Order Form in the back of this manual.


For warranty parts orders, call Midmark's Technical Service Department with the required information.

Hours: 8:00 am to 5:00 p.m. EST (Monday thru Friday)
Phone: 1-800-Midmark

WARNING

Do not modify this equipment without authorization of the manufacturer.

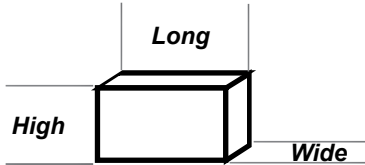Electrical Requirements:	115 V~ , 60Hz, 10A
	Fuses F1, F2 and F10: F375mAL, 125V~
	Fuses F3, F4 and F8: T10AH, 250V~
	Fuses F5 and F6: T7AL, 250V~
	Fuses F7, F9, F11 and F12: T6.3AL, 250V~
	230V~ , 50/60Hz, 6.3A
	Fuses F1, F2 and F10: F375mAL, 125V~
	Fuses F3, F4 and F8: T6.3AH, 250V~
	Fuses F5 and F6: T7AL, 250V~
	Fuses F7, F9, F11 and F12: T6.3AL, 250V~
Air Requirements:	Maximum 80 -100 PSI (5.52 - 6.9 bar)
Classifications:	Class I, Ordinary Equipment (IPX0), Continuous Operation
Certifications:	ES/IEC/EN 60601-1, CAN/CSA C22.2 No. 60601-1 (Safety Standards) EN/IEC 60601-1-2 (EMC Standards)
Dimensions:	
	8.2 in. (20.8 cm.) Long 5.5 in. (14.0 cm.) Wide 3.4 in. (8.6 cm.) High

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture with oxygen, air, or nitrous oxide.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury if not avoided.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury if not avoided. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

NOTE

Amplifies a procedure, practice, or condition.

Consult Manuals


**Type T
Fuse**

**Protective
Earth Ground**


Step 2: Splice the power supply's air pressure switch line into the master pilot shut-off lines.

- A) Connect air pressure switch line to the air supply master pilot shut-off line.
B) Connect air pressure switch line to the water supply master pilot shut-off line.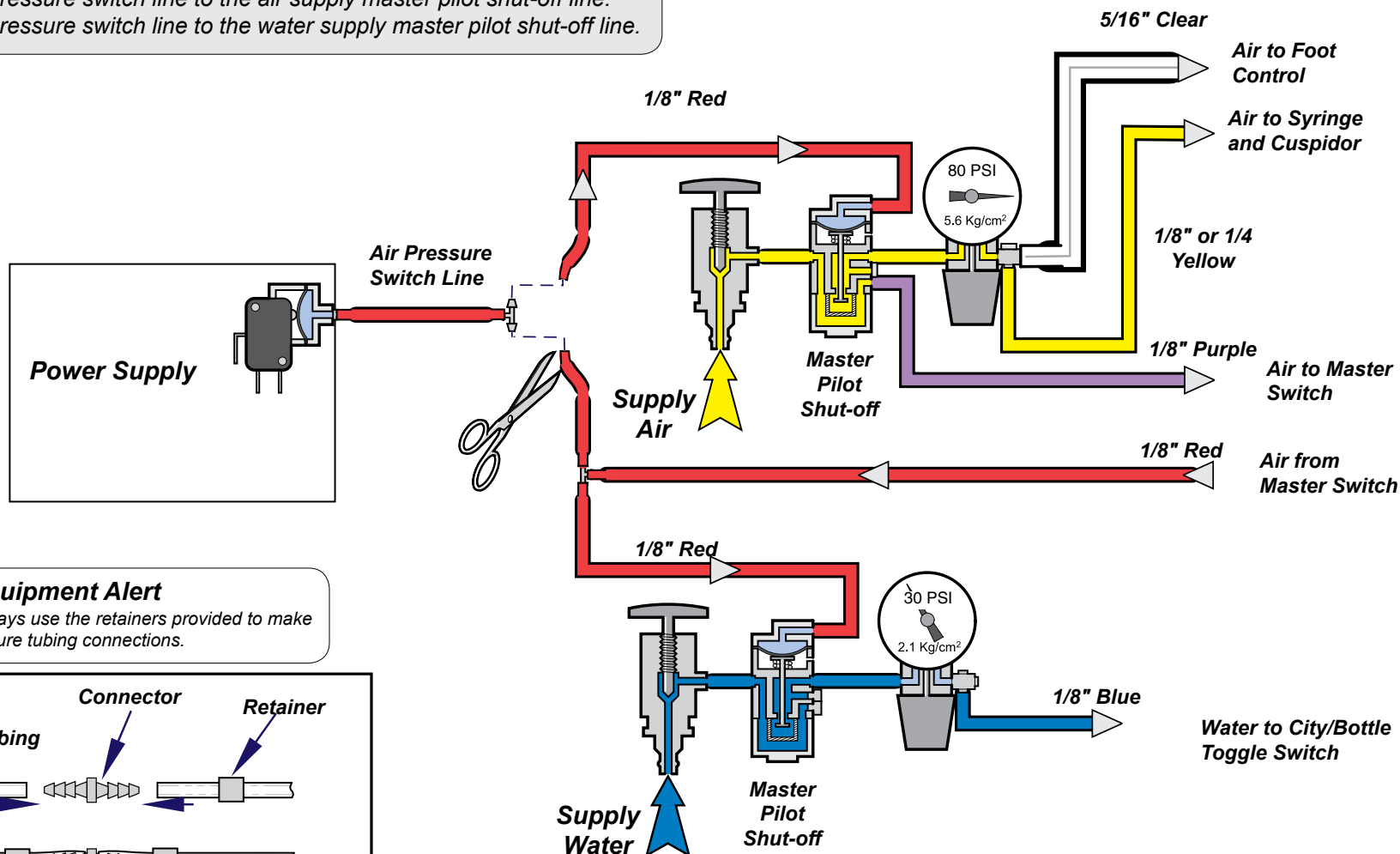

Equipment Alert

Always use the retainers provided to make secure tubing connections.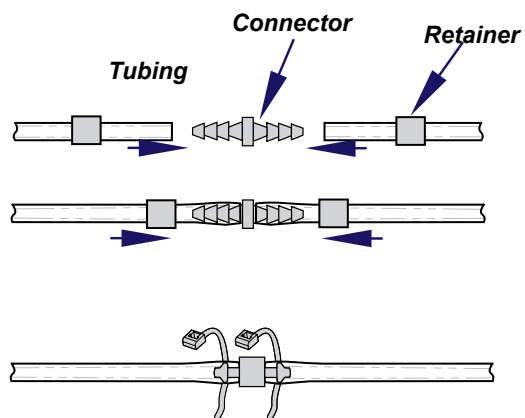

**WARNING**

Electrical power must be supplied through an electrical mains receptacle compliant w/local codes. If no local code, then install with these requirements:

115 VAC Model: 115V~, 60Hz, 10A

230 VAC Model: 230V~, 50/60Hz, 6.3A

Note

USB Power Hub only shown for 153941-001/-002 models.

**Equipment Alert**

Accessory power connection must be a medical device, maximum power 24 V~, 6.3 A.

**WARNING**


Before servicing, disconnect power from the power supply. Remove the top cover if needed (one screw) then install top cover after servicing (e.g. replacing fuses, accessing the accessory power terminals, etc.)

For 153941-001/-002 Models Only:**Step 3a: Make electrical connections to power supply.**

- A) Connect delivery power to jumper harness shown below.
- B) Connect chair power cord.
- C) Connect light power (optional) to jumper harness shown below.
- D) Connect LIN connector(s).
- E) Connect, then plug supply power cord into mains supply receptacle.
- F) Secure harnesses with ties as needed.

**Equipment Alert**

230 V~ model requires air pressure to activate chair. If air pressure is unavailable, then plug chair power **1** into supply power cord **2**.

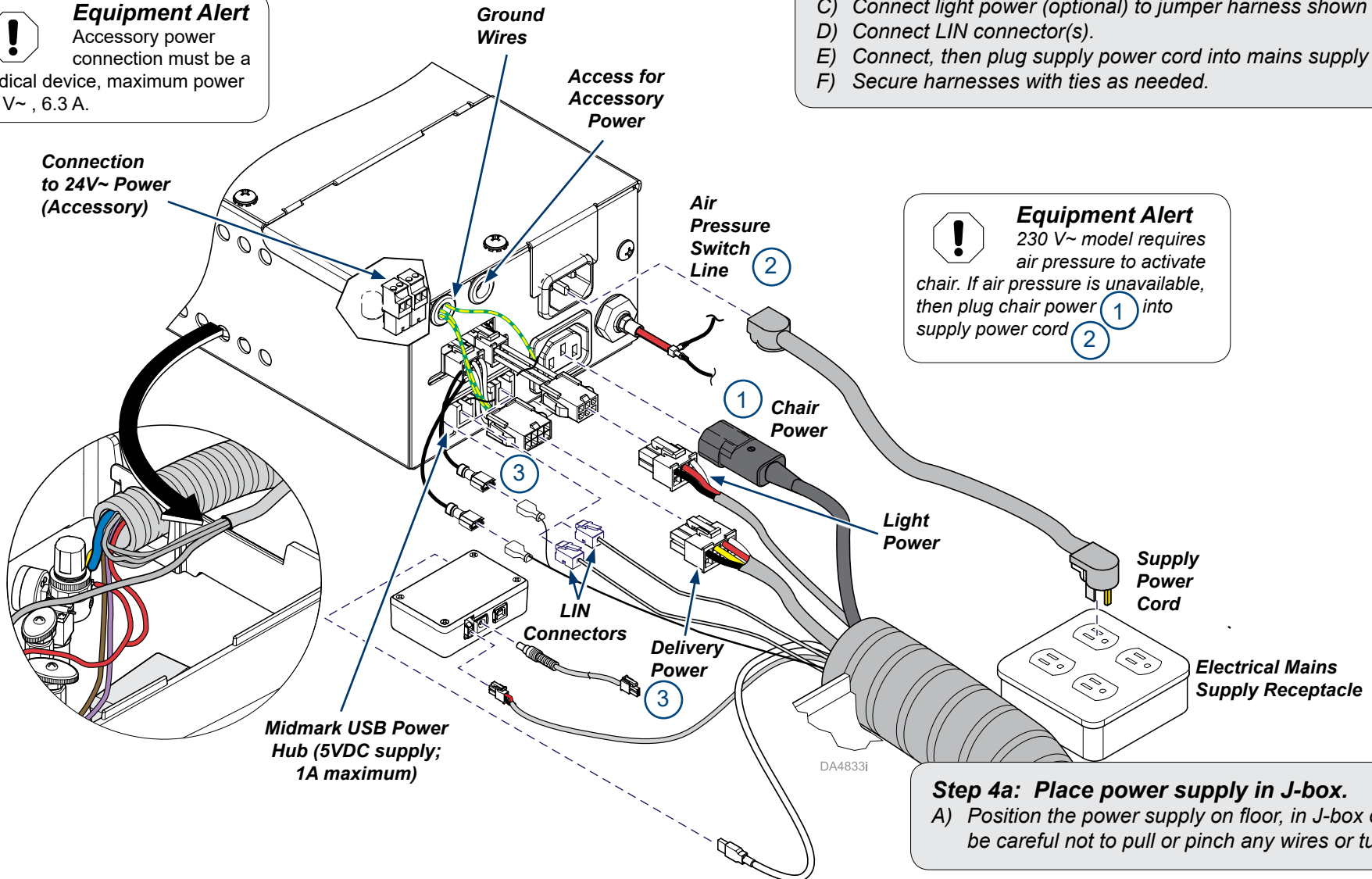**Step 4a: Place power supply in J-box.**

- A) Position the power supply on floor, in J-box or cabinet, be careful not to pull or pinch any wires or tubing.

**WARNING**

Electrical power must be supplied through an electrical mains receptacle compliant w/local codes. If no local code, then install with these requirements:

115 VAC Model: 115V~, 60Hz, 10A

230 VAC Model: 230V~, 50/60Hz, 6.3A

**Equipment Alert**

Accessory power connection must be a medical device, maximum power 24 V~, 6.3 A.

Connection to 24V~ Power (Accessory)

Ground Wires

Access for Accessory Power

Air Pressure Switch Line

1 Chair Power

Light Power

Delivery Power

Supply Power Cord

Electrical Mains Supply Receptacle

DA4834i

**WARNING**

Before servicing, disconnect power from the power supply. Remove the top cover if needed (one screw) then install top cover after servicing (e.g. replacing fuses, accessing the accessory power terminals, etc.)

For 153941-003/-004 Models Only:**Step 3b: Make electrical connections to power supply.**

- Connect delivery power to jumper harness shown below.
- Connect chair power cord.
- Connect light power (optional) to jumper harness shown below.
- Connect LIN connector(s).
- Connect, then plug supply power cord into mains supply receptacle.
- Secure harnesses with ties as needed.

**Equipment Alert**

230 V~ model requires air pressure to activate chair. If air pressure is unavailable, then plug chair power 1 into supply power cord 2

Step 4b: Place power supply in J-box.

- Position the power supply on floor, in J-box or cabinet, be careful not to pull or pinch any wires or tubing.

Warranty Information

SCOPE OF WARRANTY

Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will at Midmark's option repair or replace components of the dental products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. Midmark's obligation under this limited warranty is limited to the repair or replacement of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which, upon examination by Midmark, prove to be defective. This warranty extends only to the first retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

APPLICABLE WARRANTY PERIOD

The applicable warranty period, measured from the date of invoice to the original user, shall be as follows:

1. OPERATORY PRODUCTS
 - (a) Two (2) years for all products (except for the items in (b) through (d)).
 - (b) "KINK-VALVE" module carries a ten (10) year warranty.
 - (c) The original light bulb on a new light carries a one (1) year warranty.
 - (d) Accessories not manufactured by Midmark are excluded including but not limited to Bien Air handpiece systems, Dentsply Cavitron scaler, Satelec scaler and curing light, and Sopro cameras.
2. ORAL SURGERY PRODUCTS are warranted for a period of one (1) year.
3. STERILIZER PRODUCTS are warranted for a period of one (1) year.
4. ULTRASONIC CLEANER PRODUCTS are warranted for a period of two (2) years.
5. AIR AND VACUUM PRODUCTS
 - (a) PowerAir® oil-less compressors – Five (5) years or 3,500 hours of use, whichever occurs first.
 - (b) PowerVac® and PowerVac®G dry vacuums – Five (5) years or 10,000 hours of use, whichever occurs first (except that the vacuum pump warranty term is ten (10) years or 20,000 hours of use, whichever occurs first).
 - (c) Classic Series® wet-ring vacuums – Five (5) years or 10,000 hours of use, which ever occurs first.
 - (d) PowerMax surgical suction – Two (2) years.
 - (e) Hg5 Series Amalgam Separator - One (1) year.
 - (f) Midmark manufactured accessories - One (1) year.
6. INTEGRA™ DENTAL CASEWORK AND ARTIZAN® EXPRESSIONS PRODUCT
 - (a) Five (5) years for all products and components including door and drawer fronts, casters and slides, except for the items in (b), (c) and (d).
 - (b) Three (3) years for electrical components such as task lights/LED lights, cords, controls and accessories.

- (c) Two (2) years for sliding track monitor mount and components and upholstery.
- (d) One (1) year for countertops and resin, including accessories.

7. IMAGING PRODUCTS are warranted for a period of two (2) years except for the ClearVision CR reader which is warranted for a period of one (1) year.
8. MIDMARK Replacement Parts and Accessories carry a ninety (90) day warranty.

EXCLUSIONS

This limited warranty does not cover and Midmark shall not be liable for the following;

(1) defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner; (2) products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environment conditions and electrical power requirements; (3) products considered to be of a consumable or sterile nature; (4) accessories or parts not manufactured by Midmark; (5) charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark; (6) costs and expenses of routine maintenance and cleaning; (7) representations and warranties made by any person or entity other than Midmark; (8) matching of color, grain or texture except to commercially acceptable standards; (9) changes in color caused by natural or artificial light; (10) custom manufactured products; (11) alterations or modifications to the product by any person or entity other than Midmark; and (12) Products that would otherwise be covered under Sections 1 and 2 of this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical, animal health or dental market, as the case may be, in which purchaser intends to use the product.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER

MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

WARRANTY DISCLAIMER

THIS LIMITED WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS

No action may be brought against Midmark for breach of this limited warranty, or implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

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