

Wet or Dry Treatment Table Installation

Applies to Models:
VT

Special Tools:
none

Language of origin: English


Note:

This install guide covers multiple mounting options. Refer to appropriate pages for installation steps.


Attention:

Midmark recommends two installers and using a licensed professional to install required plumbing.


Flush Mount (page 4)

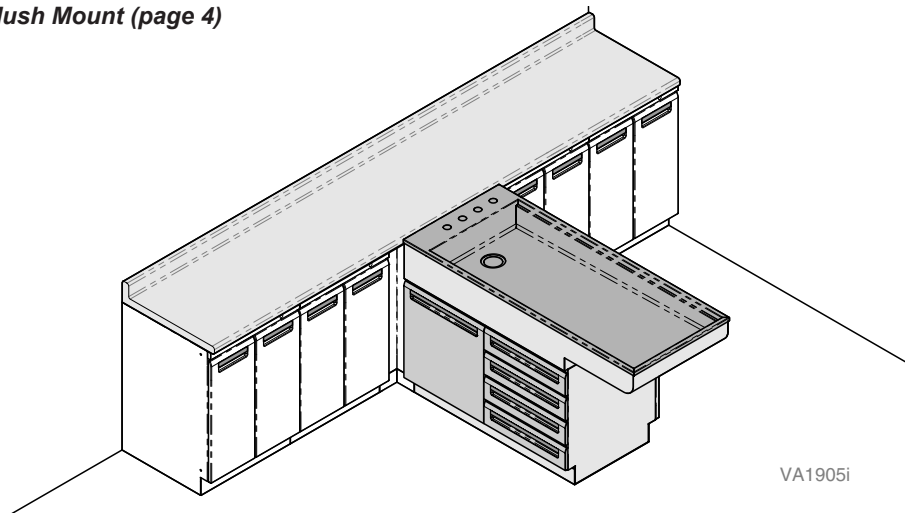


Free Standing Mount (page 3)

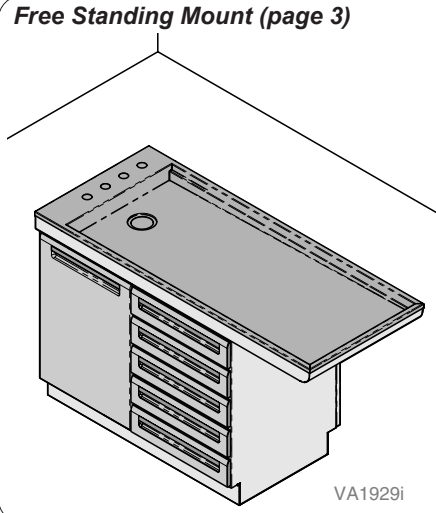


Wall Mount (page 5)

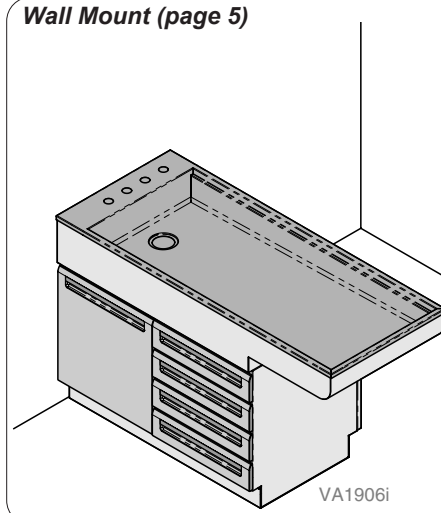

Chase Mount (page 6)

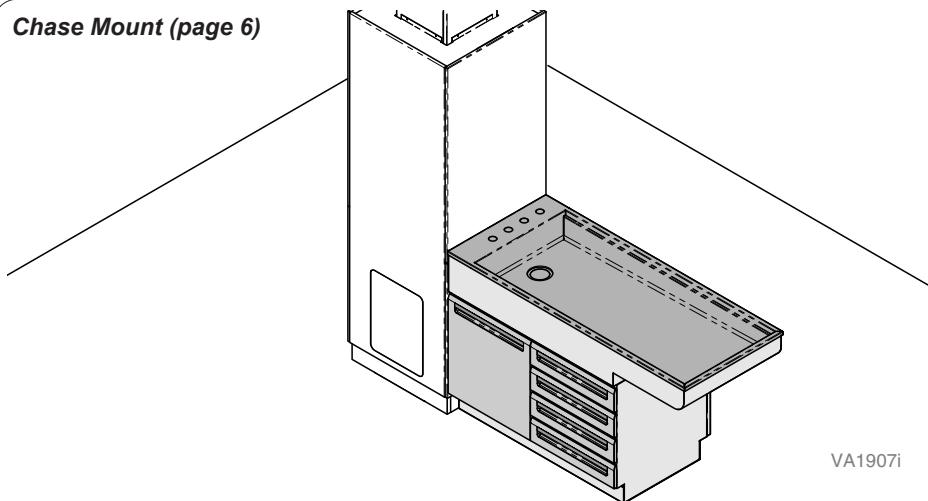

Recessed Mount (page 7)

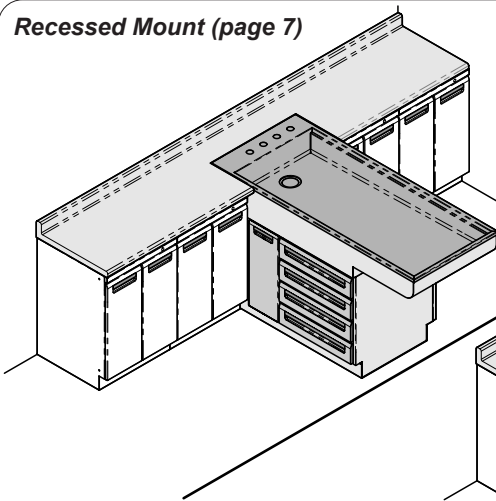

Recessed Raised Mount (page 9)



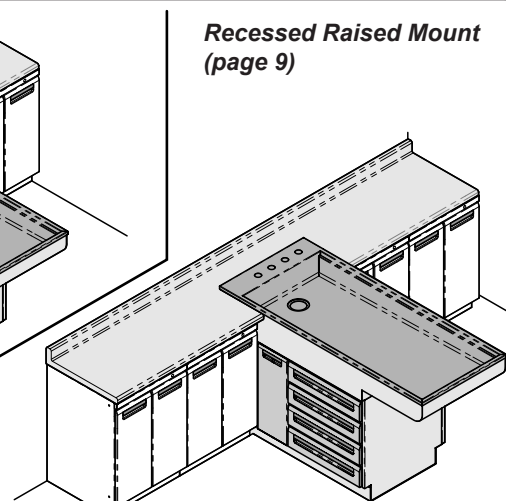


Table Dimensions


Standard
Wet Table and Plumbing Cutout
Dimensions

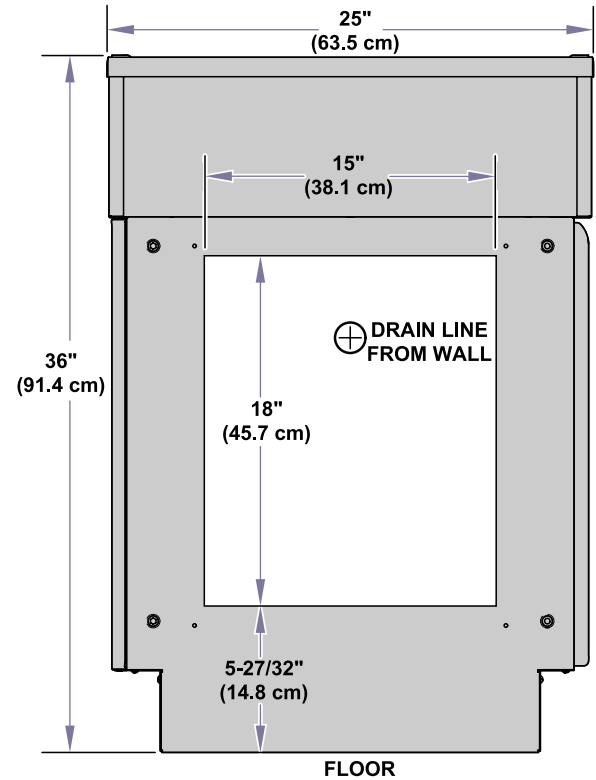

Shallow
Wet Table and Plumbing Cutout
Dimensions

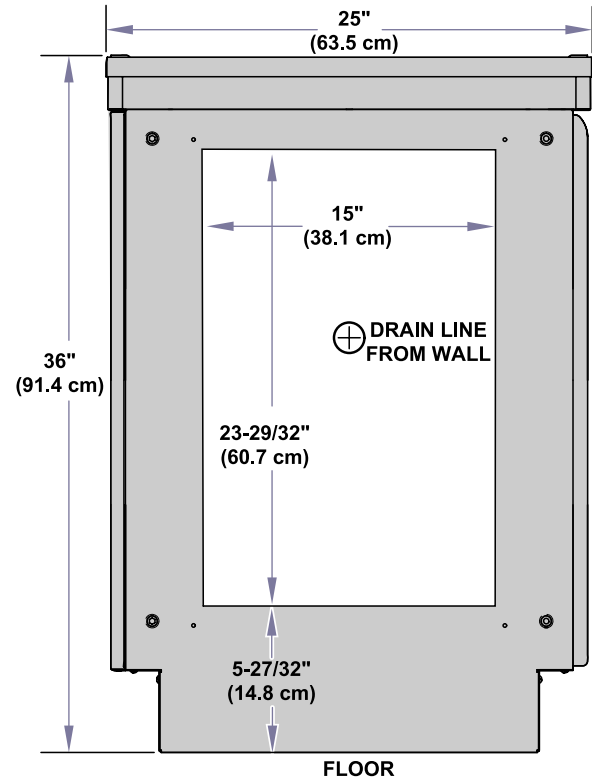

Dry Table Dimensions

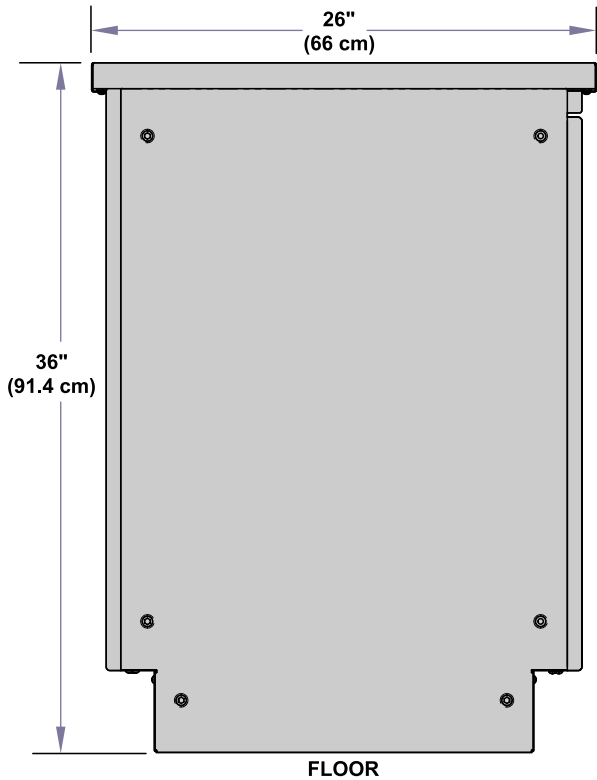

TP200 Rev. A

VA1903



Free Standing Mount Installation




Equipment Alert

Midmark requires anchoring table to the floor when using in the middle of office.

Shallow Wet Table

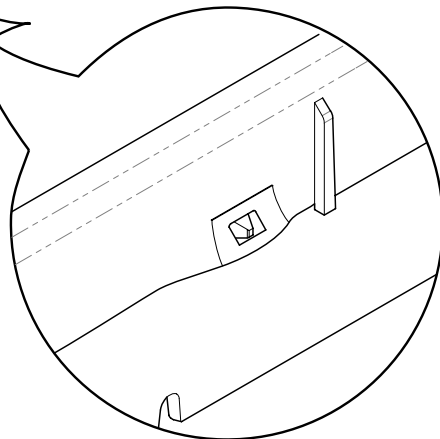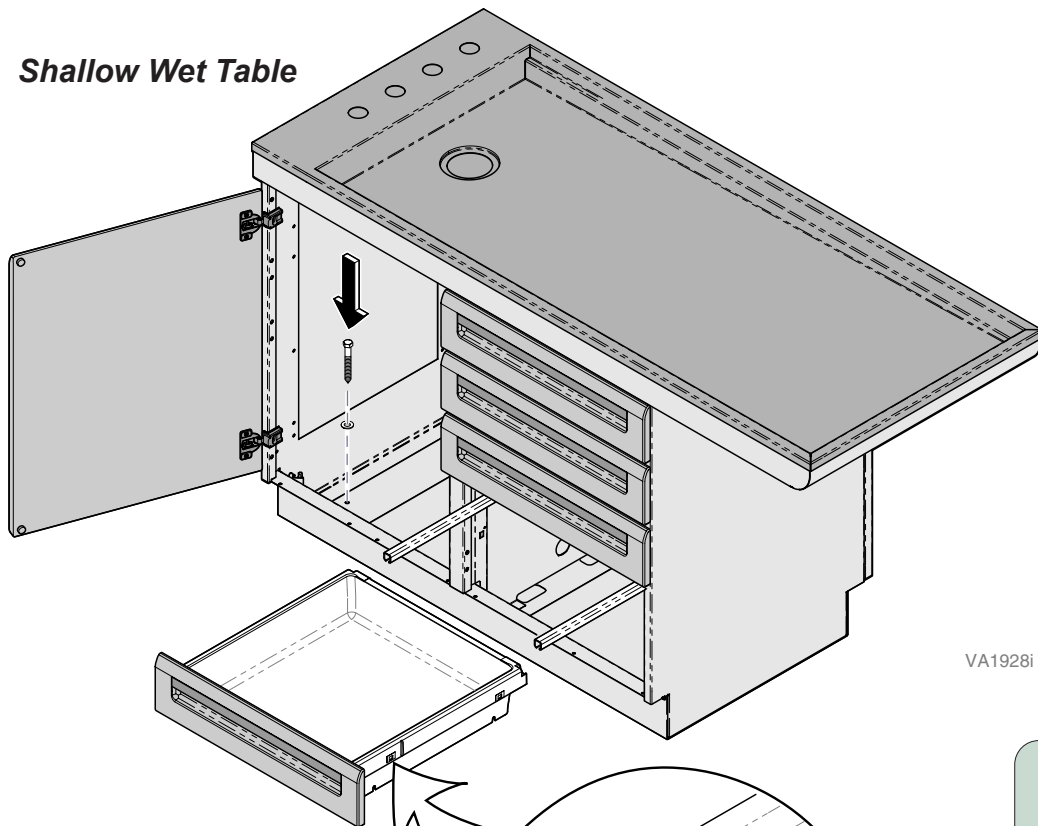

Dry Island

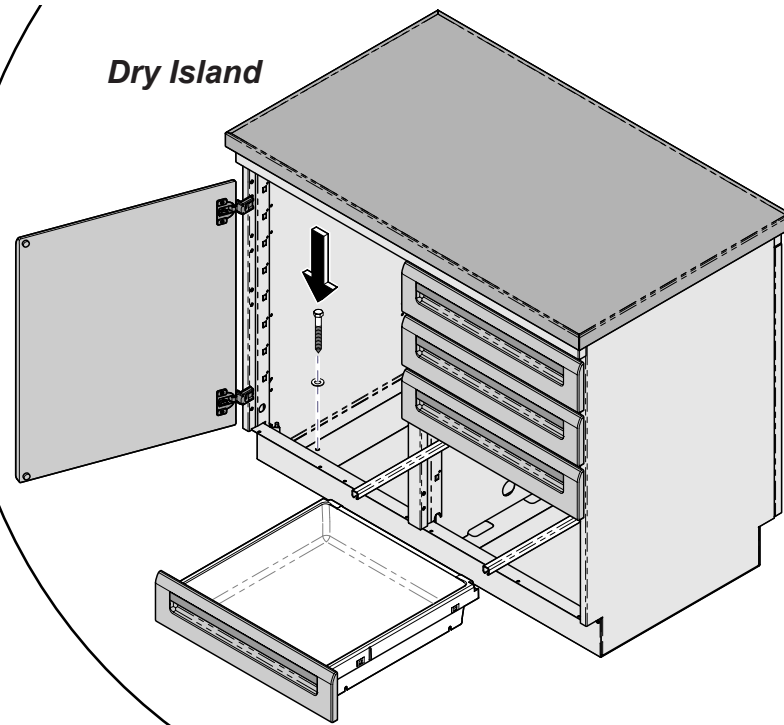

VA1928i


Step 1: Remove table from skid

- A) Pull tabs on drawer outward to release from slides.
- B) Drill four holes into floor through mounting holes in bottom of table. Use appropriate drill for floor materials.
- C) Prior to installing mounting fasteners, table may need to be leveled. Shims may be required.
- D) Install mounting hardware at four locations.

Note:

Refer to later page for Table Plumbing Installations.

Flush Mount Installation

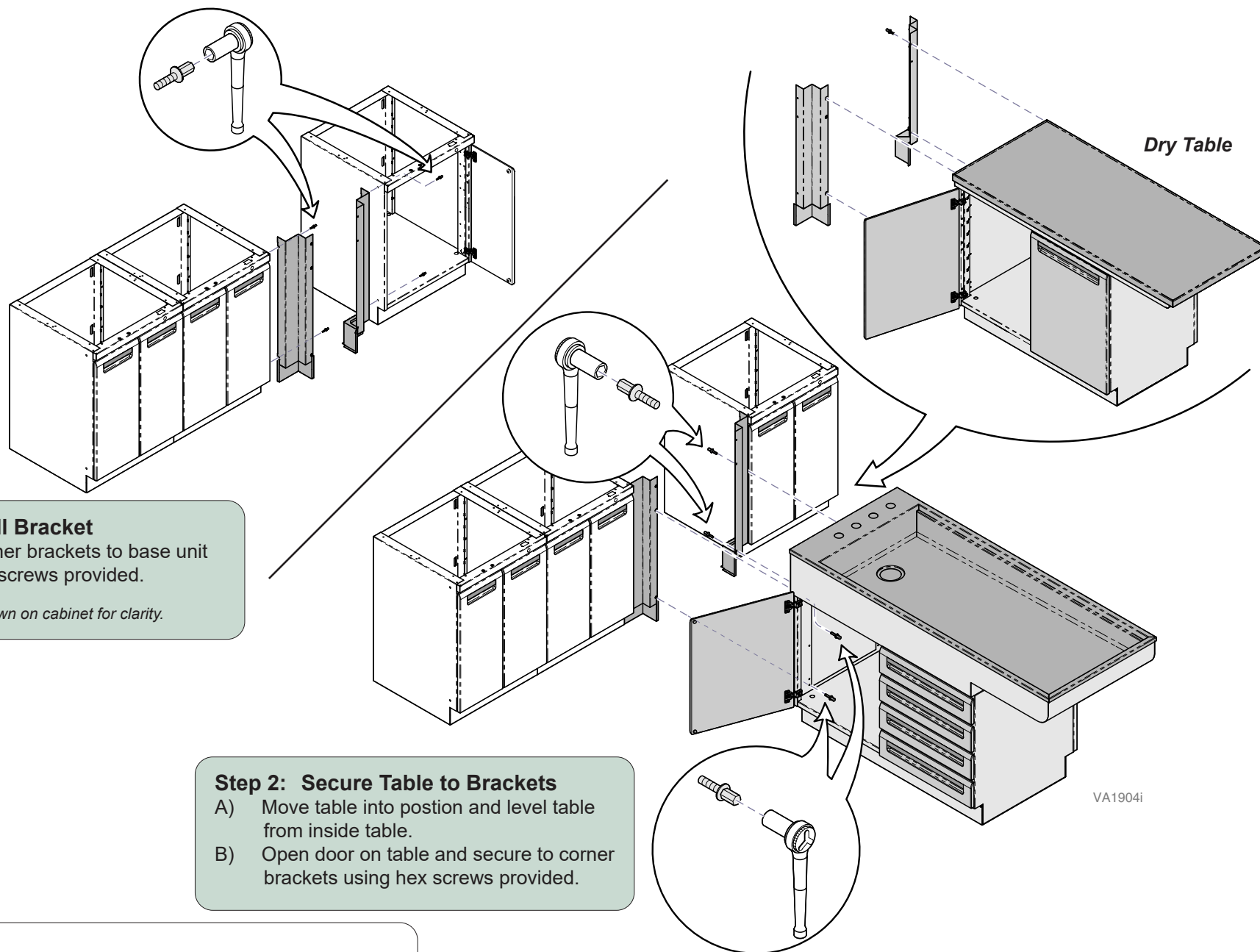

Step 1: Install Bracket

- A) Mount corner brackets to base unit using hex screws provided.

Note: Door not shown on cabinet for clarity.

Step 2: Secure Table to Brackets

- A) Move table into position and level table from inside table.
B) Open door on table and secure to corner brackets using hex screws provided.


Note:

Refer to later page for Table Plumbing Installations and full leveling adjustments.

Wall Mount Installation


Step 1: Mount to wall with studs

- A) Find center of each stud to be used for mounting.
- B) Slide table into position and level.
- C) Secure to wall using washers and screws being sure to hit studs.

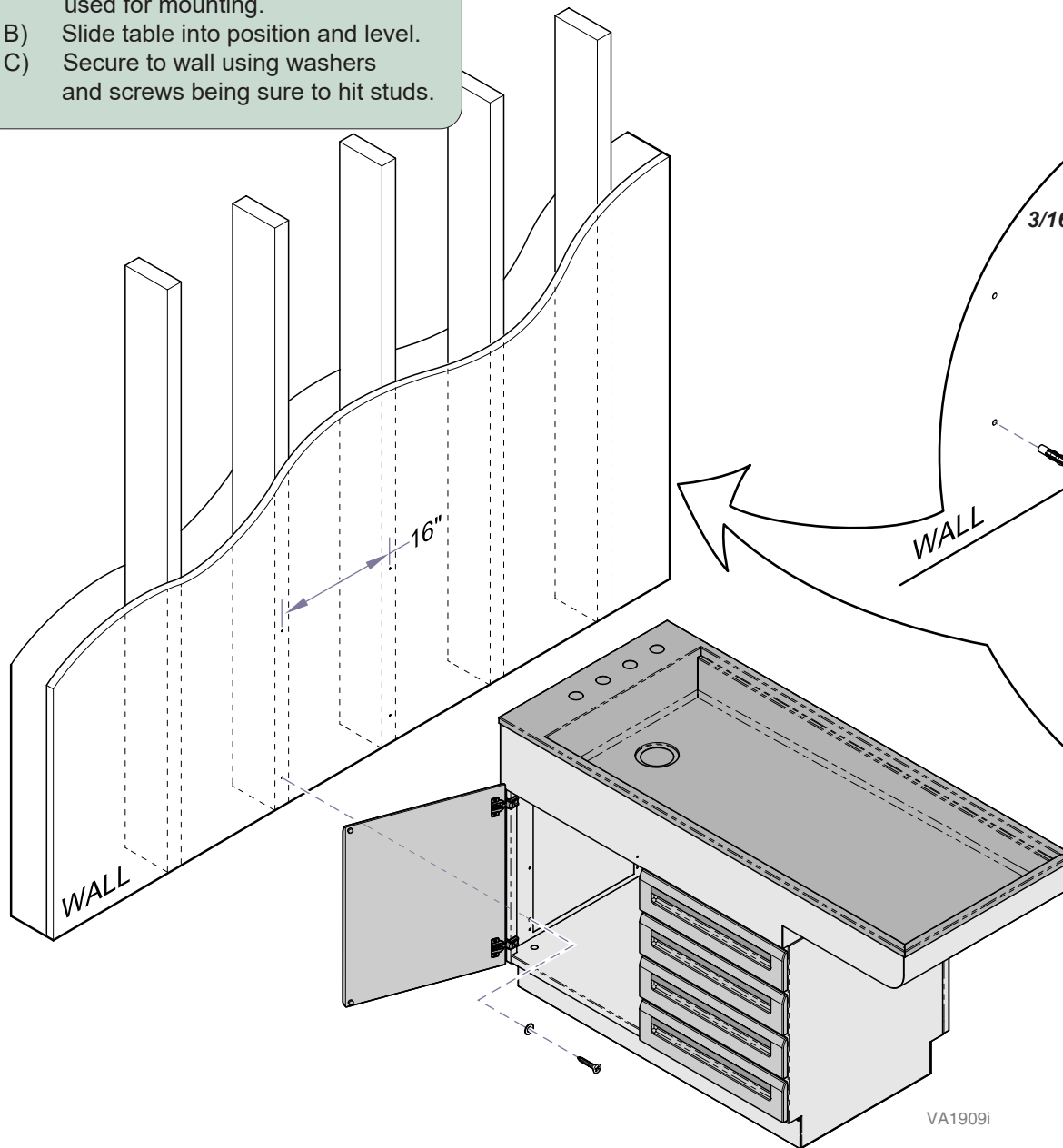

Step 2: Mount to wall without studs

- A) Slide table into position, level and mark hole locations on wall.
- B) Slide table away from wall and drill holes using 3/16" bit.
- C) Insert anchors into wall.
- D) Slide cabinet back to wall and tighten screws and washer to anchors.

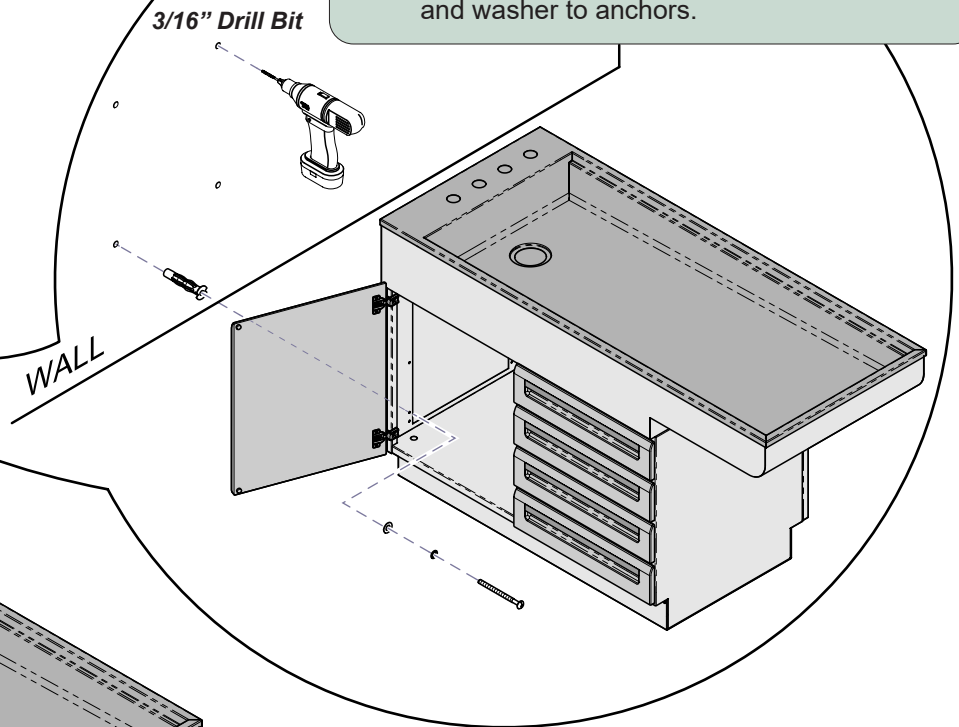

Note:

Refer to later page for Table Plumbing Installations and full leveling adjustments.

TP200 Rev. A

*Refer to chase installation manual
for mounting hole locations.*




- A) Move table into position and level.
- B) Secure Table to Chase using mounting hardware provided with the Chase (#10 screws, washers and nuts).


Refer to later page for Table Plumbing Installations and full leveling adjustments.

Recessed Mount Installation

**Standard
Wet Table Dimensions**

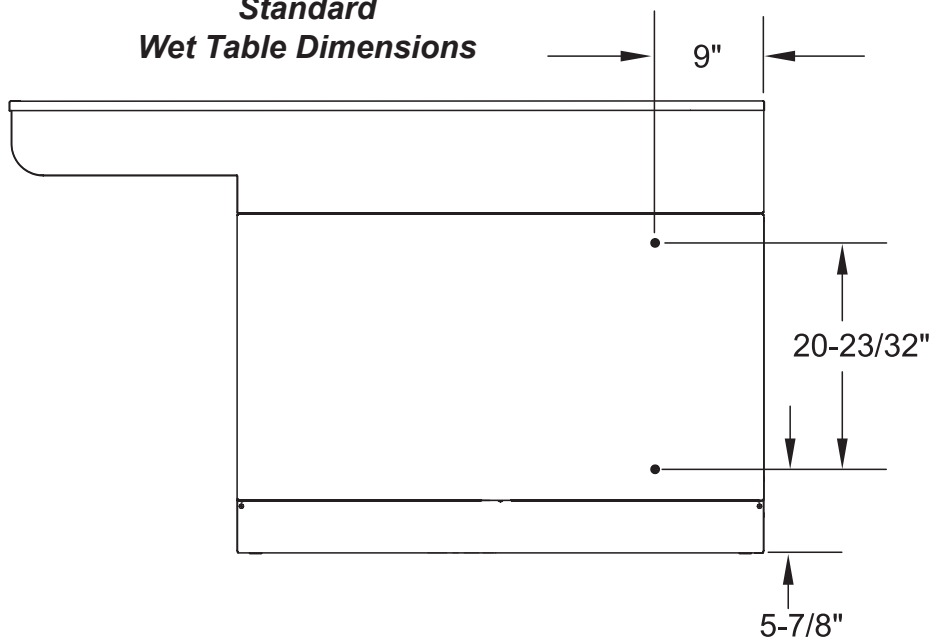


**Shallow
Wet Table Dimensions**

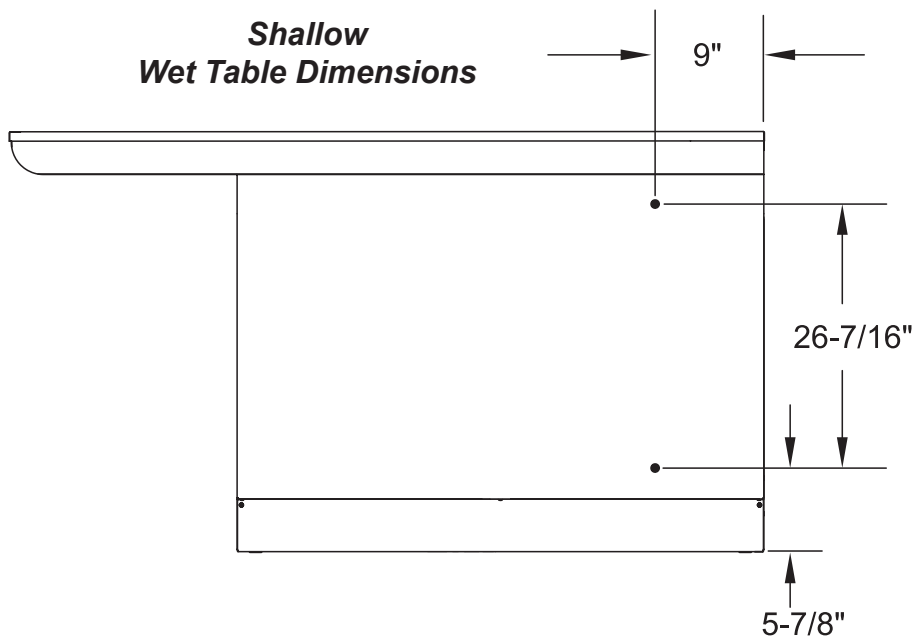

Step 1: Drill for Bracket

- A) Use dimensions shown to mark top and bottom hole locations on back panel of table.
- B) Drill 1/4" holes in marked locations.

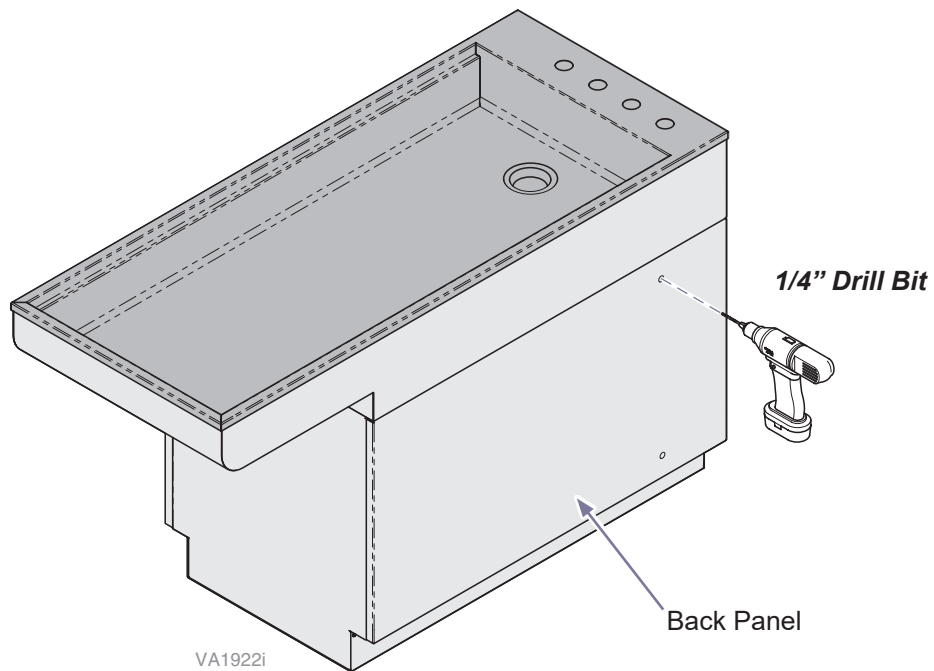

Recessed Mount Installation (cont'd)

Step 2: Install Brackets

- A) Secure brackets to base units using hex screws provided.

Note: Door not shown on cabinet for clarity.

Note:

Verify plumbing connections will reach proper location before performing **Step 3**.

Step 3: Secure Table to Bracket

- A) Move table into mounting position and level.
B) Install table to brackets through door opening.

Note: Install table bracket to cabinet bracket on door/drawer side. Install through drilled holes on back panel to cabinet bracket.


Note:

Refer to later page for Table Plumbing Installations and full leveling adjustments.


VA1910i

Recessed Raised Mount Installation

**Standard
Wet Table Dimensions**

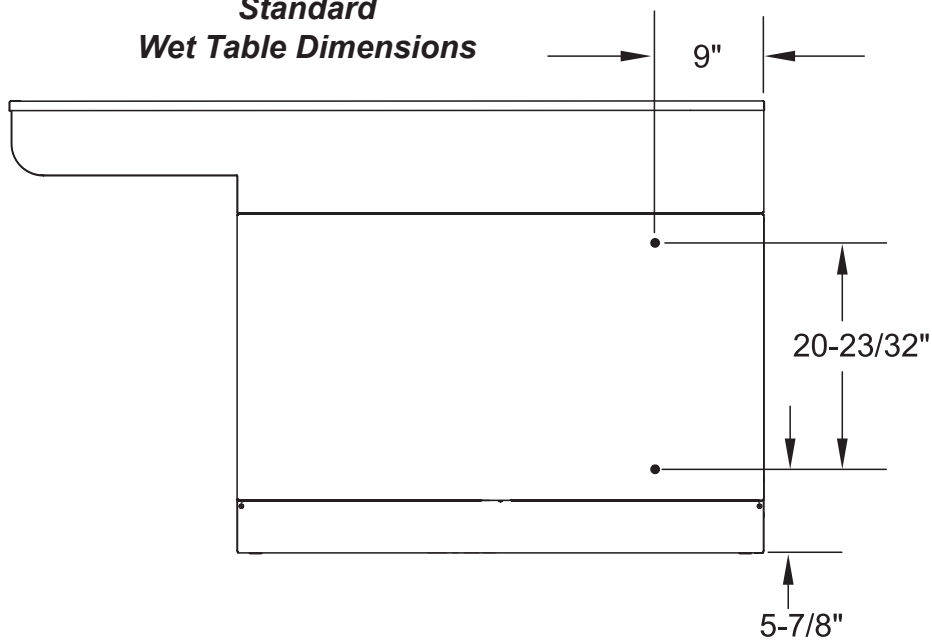


**Shallow
Wet Table Dimensions**

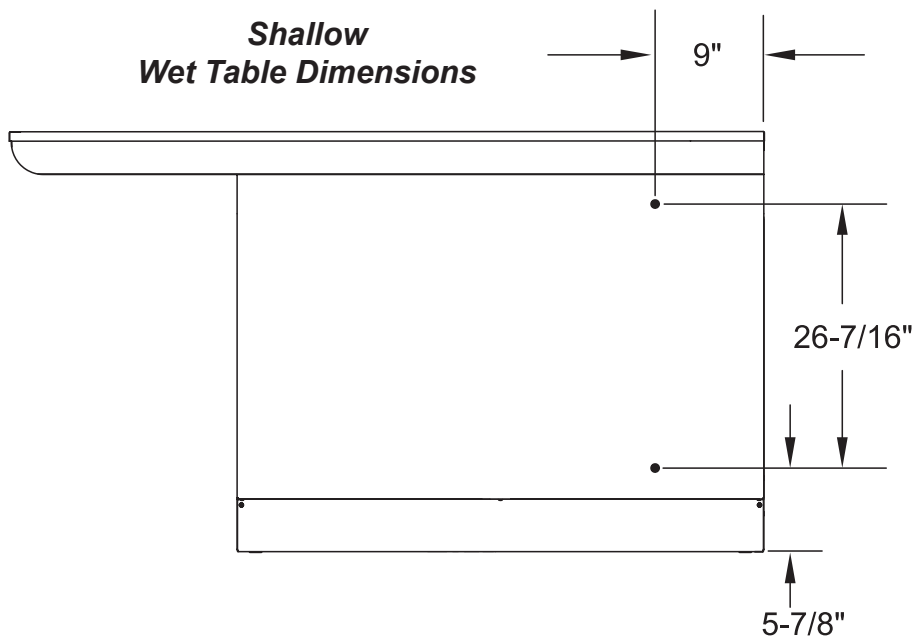

Step 1: Drill for Bracket

- Use dimensions shown to mark top and bottom hole locations on back panel of table.
- Drill 1/4" holes in marked locations.

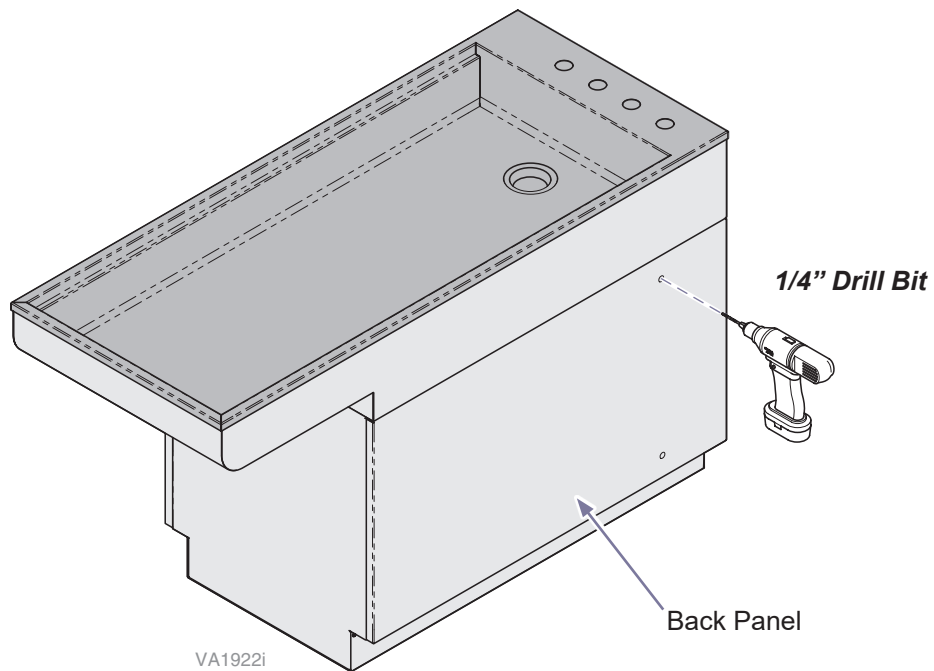

Step 2: Level Table

- Adjust leveling feet to adjust height of table to 36.5".

Recessed Raised Mount Installation (cont'd)

Step 3: Install Brackets

- A) Secure right bracket to table and left bracket to cabinet as shown using hex screws provided.


Note:

Verify plumbing connections will reach proper location before performing **Step 4**.

Step 4: Secure Table to Bracket

- A) Move table into mounting position.
B) Install table to base units through door openings.

Note: Install table to base unit with hex screws (2) as shown. Install phillip head screws (2) inside base unit as shown.

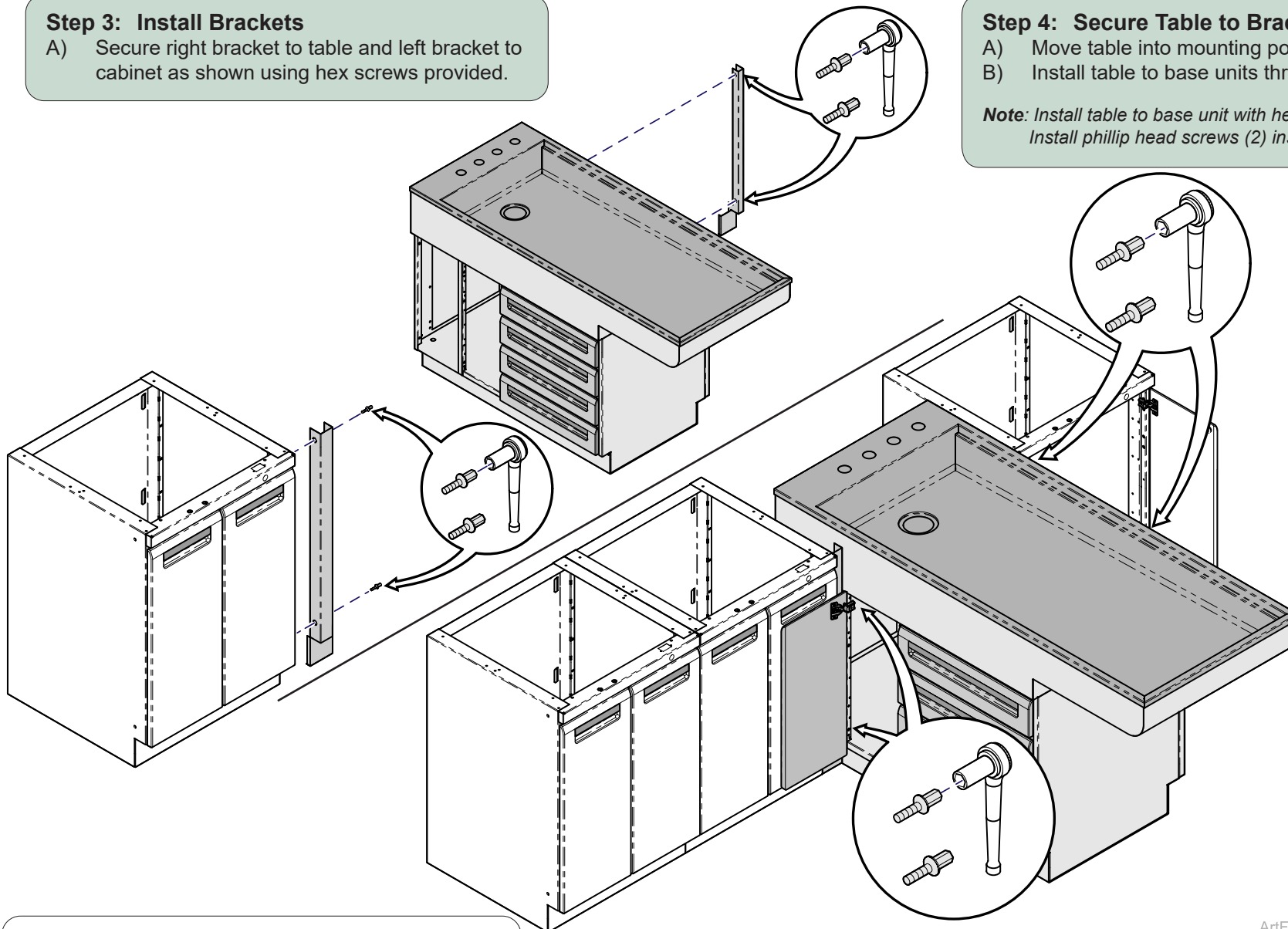

Note:


Refer to later page for Table Plumbing Installations and full leveling adjustments.

ArtFull

Plumbing Installation

Step 2: Install Faucet and Sprayer

- A) Follow the manufacturer's instructions to install faucet and sprayer.
- B) If sprayer is not applicable, use hole cap to plug the hole.

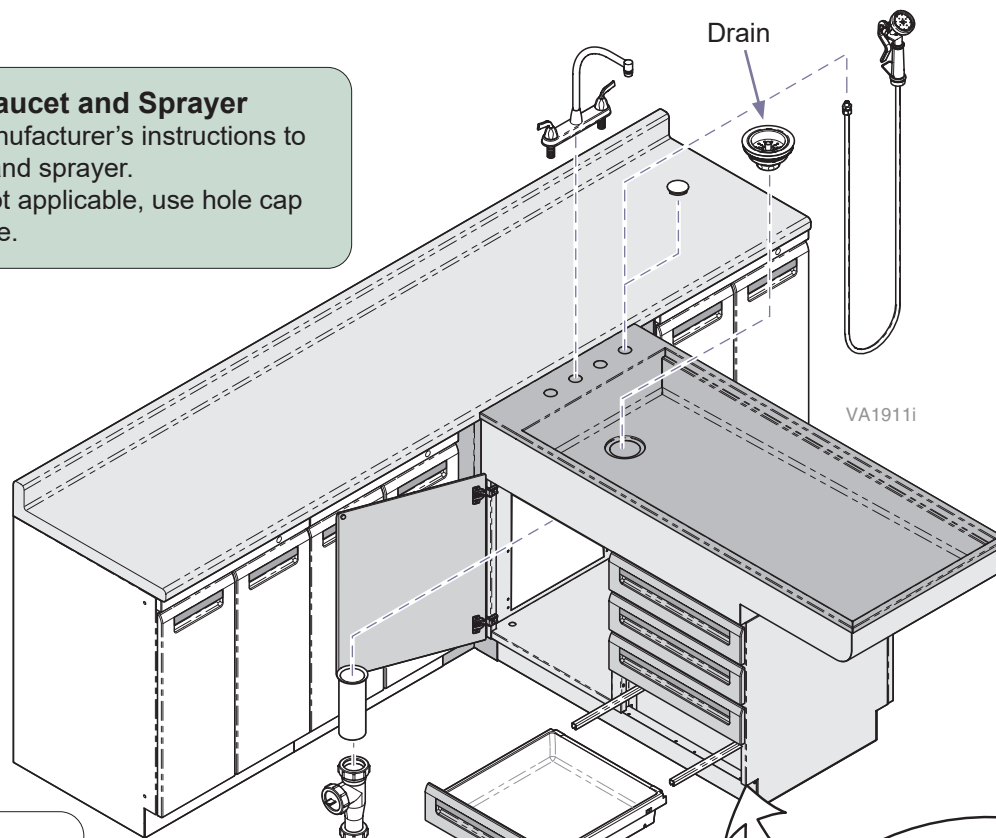


Step 1: Level Table (if necessary)

- A) Pull tabs on drawer outward to release from slides.
- B) Level Table in necessary in all four corners using leveling feet.
- C) Reinstall drawer.

Attention:


Midmark recommends to reach out to your local municipality and / or plumbing inspector for any backflow prevention needs.

**P-Trap
NOT Supplied**

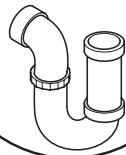

Step 3: Install Drain, Tail Pipe, and Hair Trap

Note: The debris trap is not intended to replace a P-trap. The debris trap is a plumbing device that has a removable strainer intended for installation between the tub drain and before the standard P-trap.

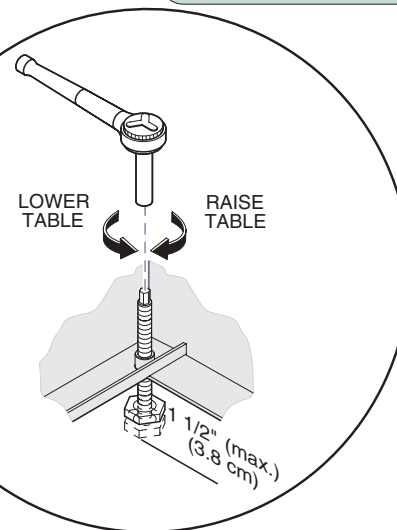

Dental Suite Installation


Step 1: Install Wet Peninsula Dental Table

Step 2: Install VetPro™ 5000 Table Mount


Note: Refer to VetPro™ 5000 Install Manual 003-2000-00.

Step 3: Install VetPro™ Solo unit or Junction Box


Note: Refer to Solo Installation Manual 003-1996-00, or Junction Box installation manual 003-1914-00.

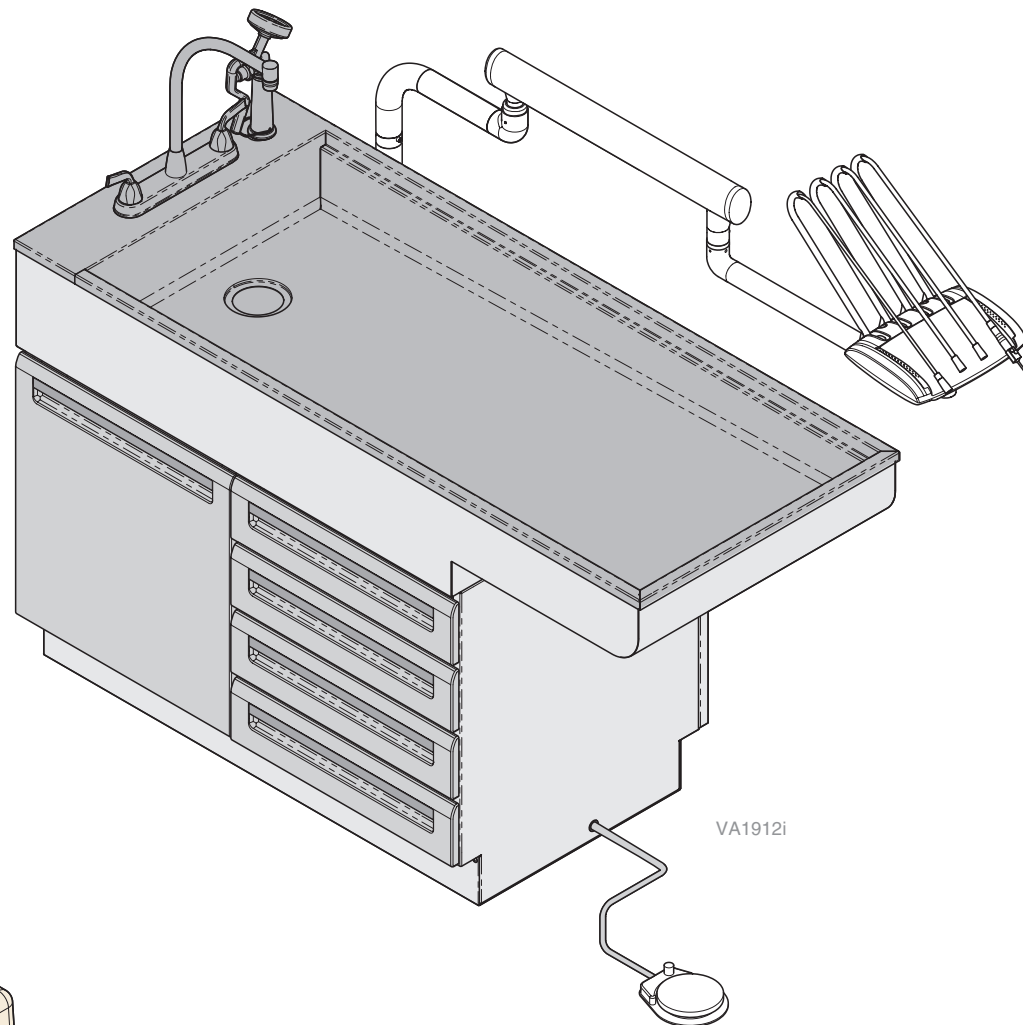

VetPro™ Solo

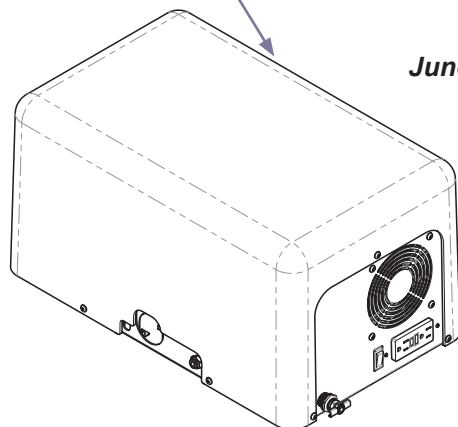


Junction Box

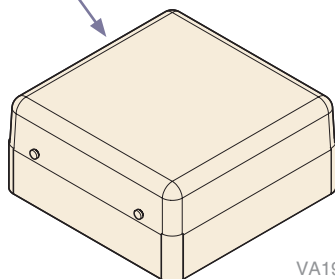


VA1913i

Wet Table Grate Installation

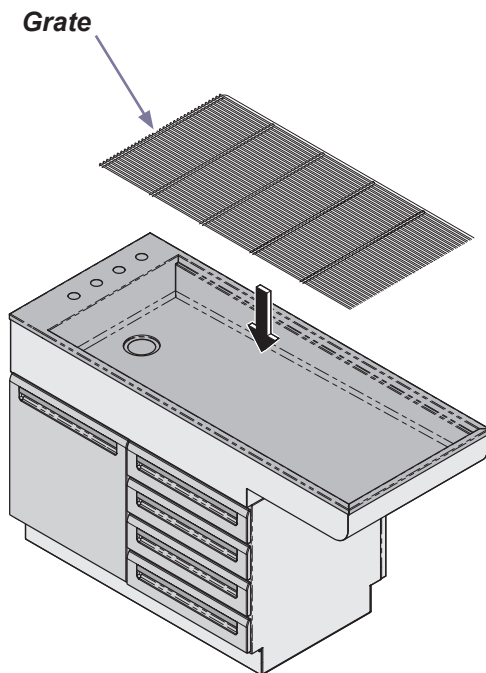

Step 1: To Install Grate

A) Place grate on top of grooves in table.

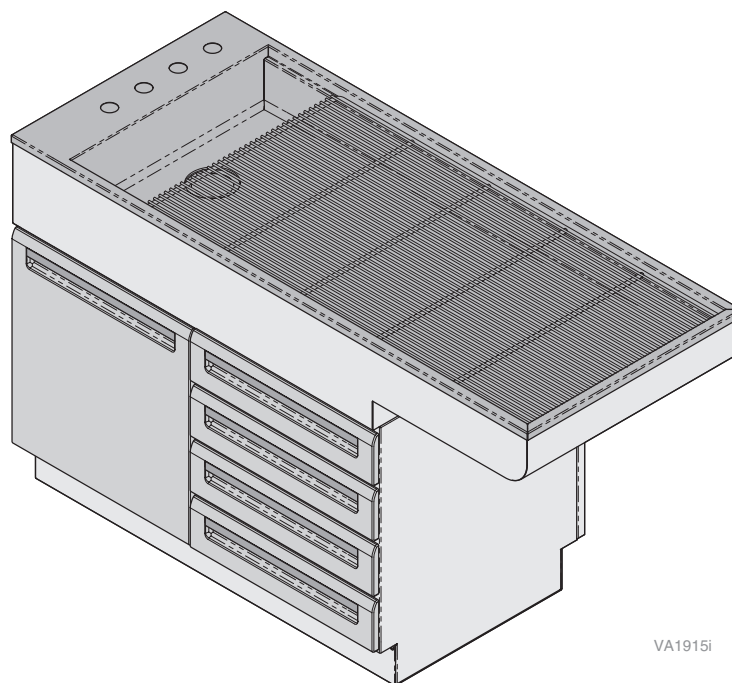

VA1915i