

# 255 LED Light Driver Board Installation [002-1891-00]

## Components:

Driver Board

## Special Tools:

none

Language of origin: English



### Warning

Always disconnect power before removing covers.  
Failure to do so may result in personal injury.



### Equipment Alert


Do not damage harnesses when removing cover.

### Step 1: Access driver board.

- Turn light OFF and disconnect power.
- Remove cover and components from back of lighthead assembly shown.



Replace Driver Board

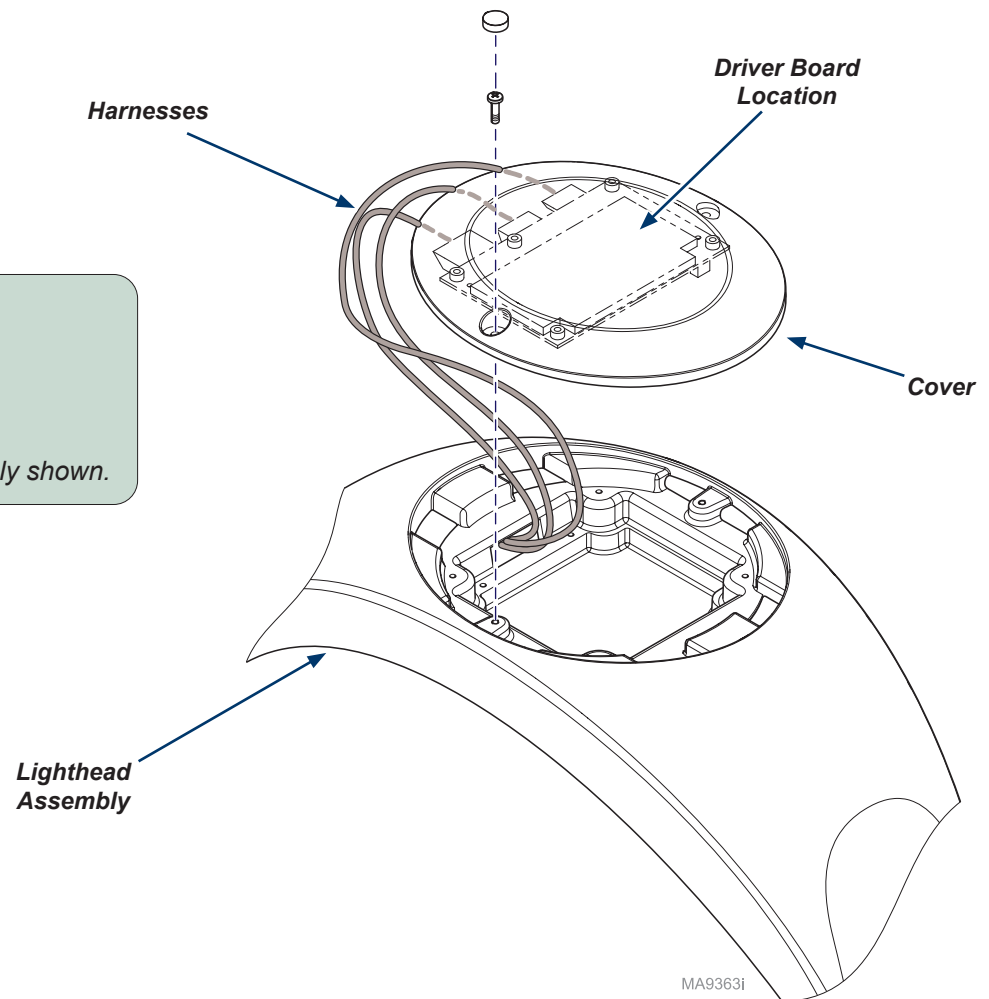

MA9363i

**Step 2: Remove driver board.**

A) Remove connectors and screws from cover shown.



**Step 3: Install new driver board.**

- A) Peel off film and secure driver board.
- B) Connect wire harnesses.
- C) Complete steps 1 through 2 in reverse order.
- D) Connect power and verify operation.