


Monitor Harness Replacement Kit [002-1893-00]

Applies to Models:

153829 (-000 thru -008)
153830 (-000 thru -008)
153906 (-010 thru -018)
153907 (-010 thru -018)

Language of origin: English

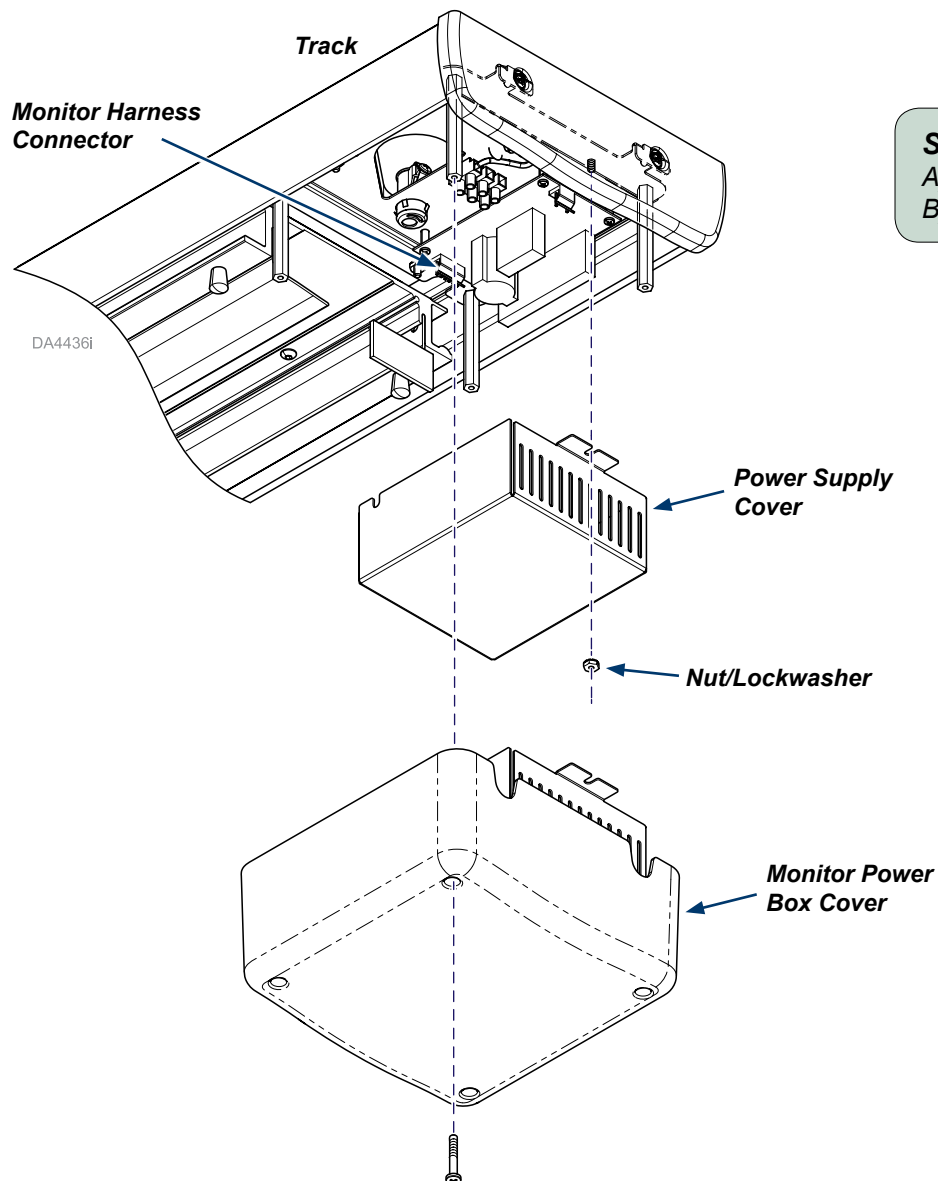

Step 1: Access monitor power harness.


- A) Remove screws and monitor power box cover from track.
- B) Remove nut, lockwasher and power supply cover.

Step 2: Disconnect monitor power harness.

- A) Disconnect monitor power harness end only from PC Board.

Step 3: Finalize installation.

- A) Cut existing monitor power harness with enough room for new harness.
- B) Crimp monitor power harness together in track as shown.
- C) Place and install new monitor power harness as shown.
- D) Complete Steps 1 to 2 in reverse order ensuring proper connections.
- E) Connect all power and verify operation.

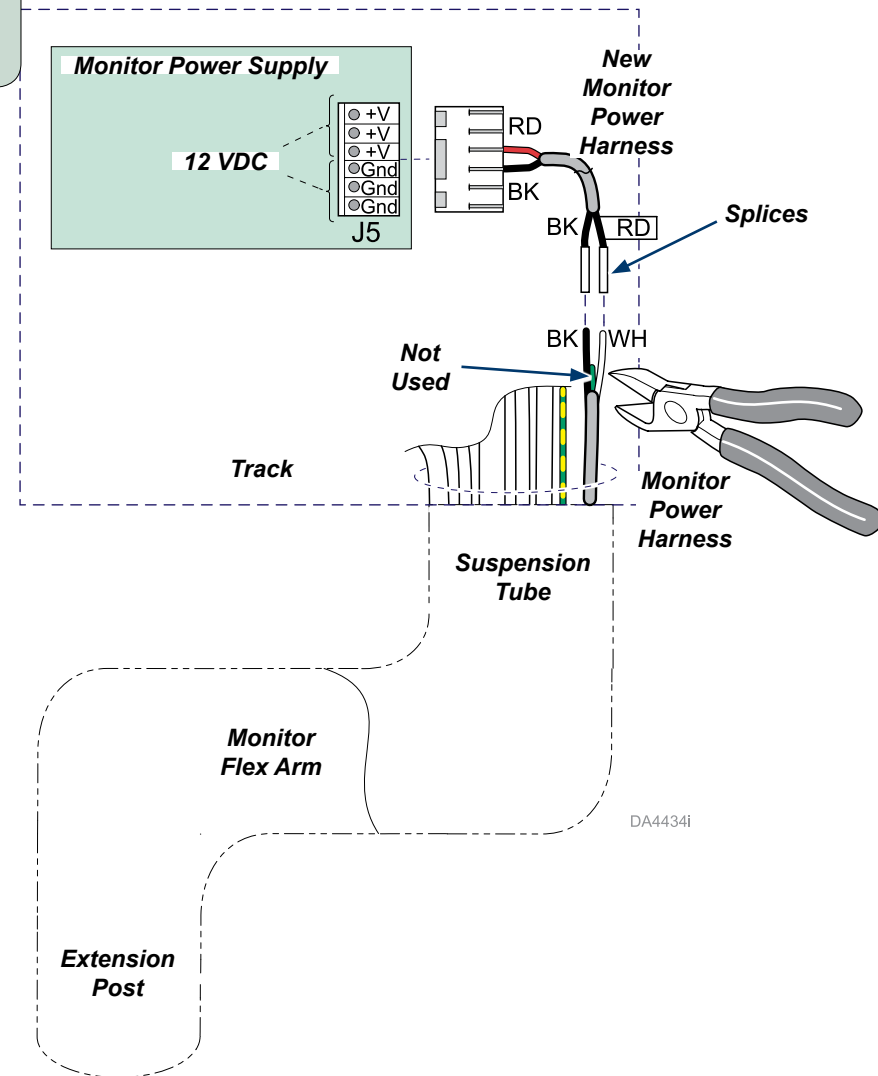

Because we care.