255

LED Procedure Light

[English](#)

[Español](#)

[Français](#)

For Models:

Ceiling: 255 (-021 to -024)

Mobile: 255 (-025 / -027)

Wall: 255 (-026)

User Guide

Style P


Product Information

Dealer :

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

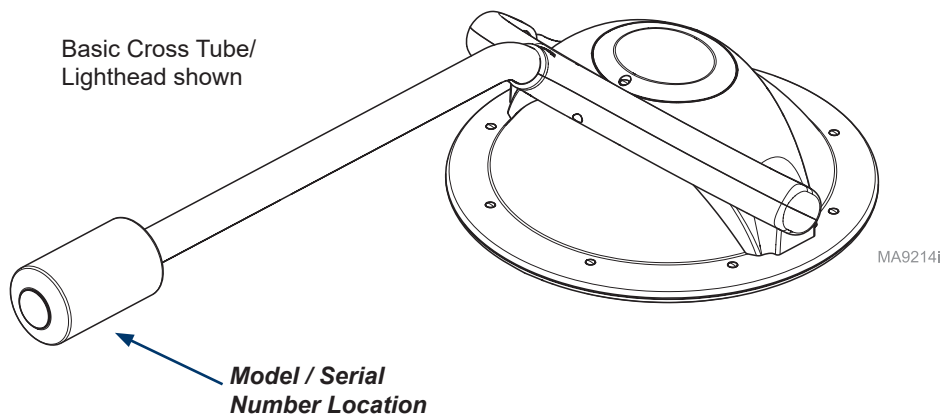Product Registration

To register your product, go to midmark.com.

Table of Contents

Important Information

Safety Symbols.....4

Intended Use.....5

Electromagnetic Interference.....5

Disposal of Equipment.....5

Transportation / Storage Conditions.....5

Operation

Power & Adjustment Switches.....6

Positioning the Lighthead.....7

Maintenance

Preventative Maintenance.....8

Cleaning & Disinfecting9

Sterilizing the Handle.....11

Replacing the Fuses:

Ceiling Mounted Lights12

Wall Mounted Lights.....13

Mobile Lights.....14

Ball Pivot Tension Adjustment.....15

Cross Tube Counterbalance Adjustment.....16

Specifications

Specifications Chart.....17

Dimensions / Range of Motion:

Ceiling Mounted Lights20

Wall Mounted Lights.....21

Mobile Lights.....22

Model Identification / Compliance

Model Identification / Compliance Chart.....23

Warranty Information

Limited Warranty25

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices


Equipment Alert
Indicates a situation which could result in equipment damage.

Note
Amplifies a procedure, practice or condition.

Symbol Glossary

These symbols may appear on your equipment and/or in the manuals.

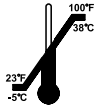	Type B, Applied Part		Keep Dry
	Refer to instruction manual/booklet		Maximum stacking height (Refer to “n” number on package.
	Fragile		Transportation Orientation
	Proper Shipping Orientation		Relative Humidity
	Protective Earth Ground		Atmospheric Pressure
	Fuse Rating Specification		Transportation & Storage Temperature
	Catalogue Number		
	Serial Number		
	Manufacturer		

Intended Use

This product is intended to be used in medical environments where illumination is required for medical use during external examinations and procedures.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room
- Plug light into isolated circuit
- Increase separation between light and interfering device
- Contact Midmark if interference persists

Disposal of Equipment

At the end of product life, the light, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Transportation / Storage Conditions

Ambient Temperature Range:.....-30°C (-22°F) to 60°C (140°F)
Relative Humidity..... 10% to 90%
Atmospheric Pressure500 hPa (15 in.Hg) to 1060 hPa (31 in.Hg)

Operation Conditions

Ambient Temperature Range:..... 10°C (50°F) to +40°C (104°F)
Relative Humidity.....30% to 75%
Atmospheric Pressure700 hPa (20 in.Hg) to 1060 hPa (31 in.Hg)

Operation

Power and Adjustment Switches


To turn the light ON / OFF...

Press the power button ().

To adjust the light...

Press the light intensity controls to increase / decrease (+ / -).


Note: Indicator lights show current settings.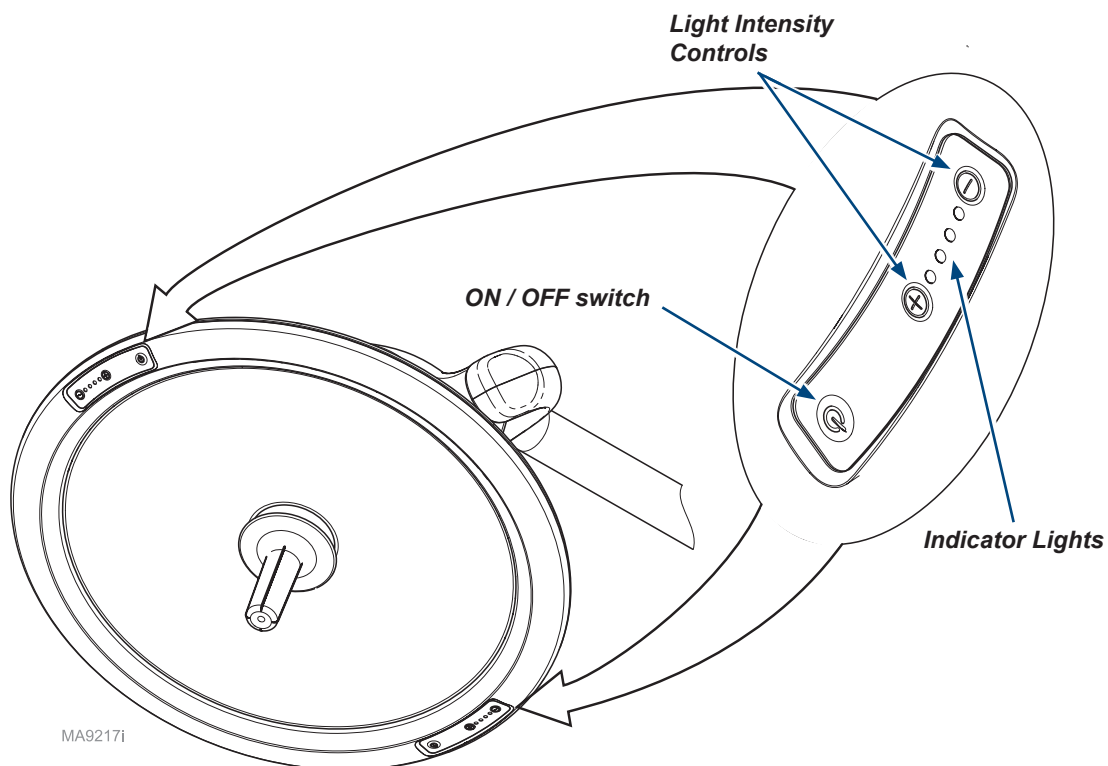

Positioning the Lighthead

To position the lighthead...

Grasp the sterilizable handle, then rotate the arm / lighthead to desired position.

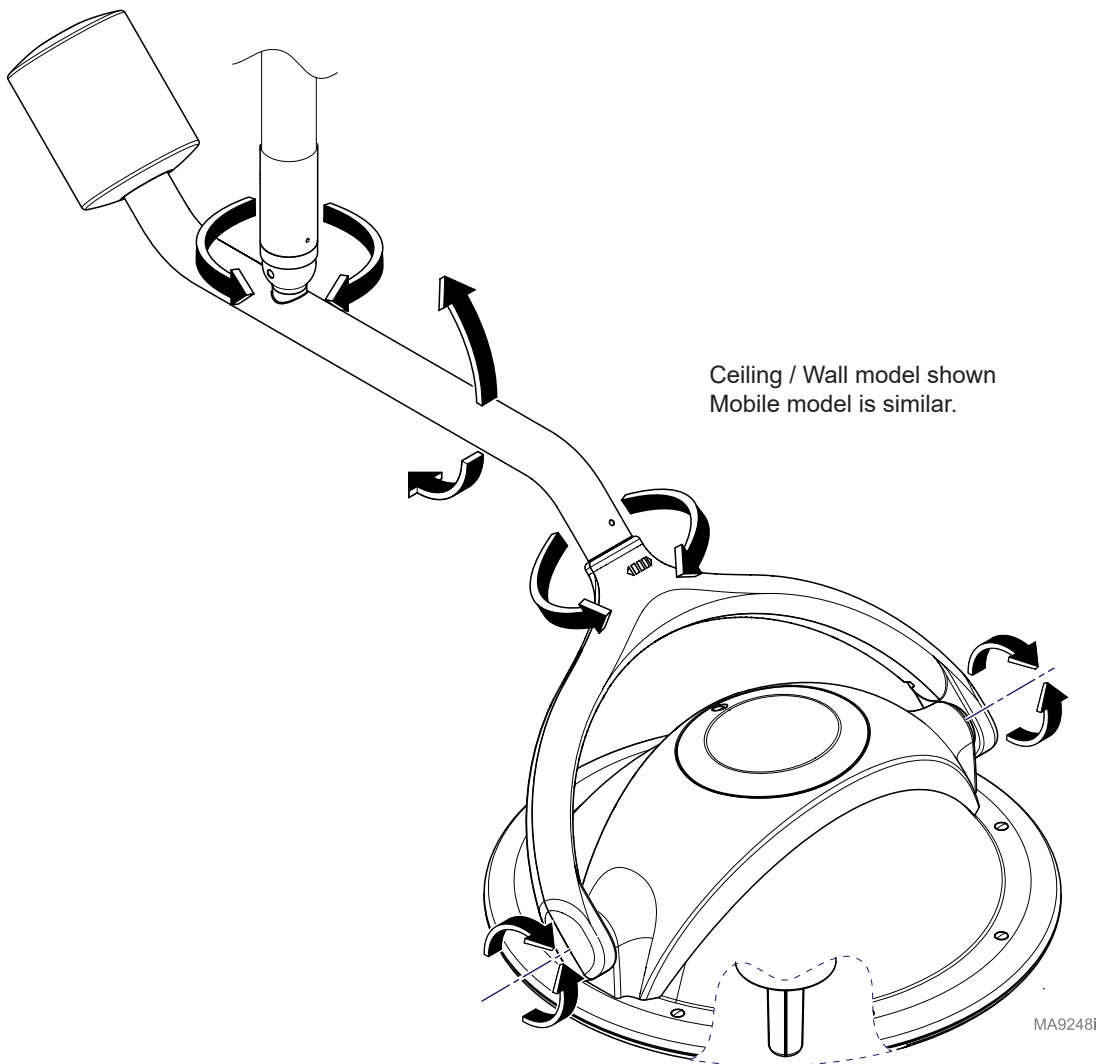Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact your authorized Midmark dealer.

To contact Midmark directly:

1.800.MIDMARK (1.800.643.6275)
8:00 am until 5:00 p.m. Monday - Friday (EST)
midmark.com

Preventative Maintenance

Periodically inspect the following areas:

- Power cord(s) should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

Have an authorized service technician inspect your equipment every six months.

Frequency	Activity
Semi-annually	Inspect rotation of lighthead / arms at all pivot points. Arms should move freely, without noise. Arms should not drift when released. Check for excessive rotation. (<i>This would indicate internal stops may be broken</i>). Use the Range of Motion section in this manual as a reference.
	Turn light ON. Rotate lighthead / arms at all pivot points. Light should not flicker or flash at any time.
	Inspect the faceplate and inside of the lighthead for signs of damage.
	Inspect lighthead for excessive dust or dirt buildup inside lighthead. <i>Do <u>not</u> attempt to clean interior - contact an authorized service technician.</i>

Maintenance

Cleaning & Disinfecting

WARNING

- *Always disconnect power before cleaning / disinfecting.*
- *Prevent fluids from leaking into interior or onto electrical contacts.*
- *Clean EXTERNAL SURFACES ONLY (arm assemblies / lighthouse).*
- *Do NOT attempt to clean / disinfect internal surfaces. Contact an authorized service technician.*

Caution

The light and surrounding components may be HOT! Allow the light to cool for at least five minutes before contacting surfaces. Never operate the light with faceplate removed.

Equipment Alert

Follow current National Guidelines for Disinfection and Sterilization in Healthcare Facilities.

Equipment Alert

Use only quaternary disinfectants on light. Staining, pitting, discoloration, or softening could occur if phenolic, iodophor or glutaraldehyde-based solution is used on plastic surfaces of the lighthouse. Also, using alcohol or aerosol spray cleaners / disinfectants containing substantial amounts of alcohol in the formula may damage the lens.

Cleaning and Disinfecting (continued)


Equipment Alert

*Do not use alcohol or abrasive compounds on faceplate.
Wring excess water / solution from cloth before wiping.*

To clean external surfaces...

- A) Clean faceplate with an antistatic acrylic cleaning solution using a soft, clean, cloth.
- B) Wipe external surfaces of arms / lighthouse with a mild detergent and water solution.
- C) Rinse all surfaces with a soft cloth and clean water.
- D) Wipe all surfaces dry.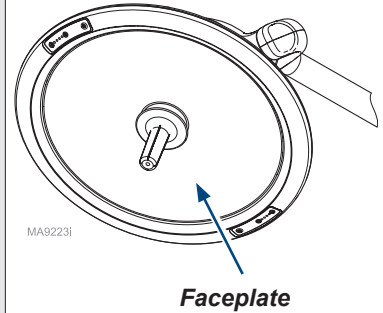

Faceplate

To disinfect external surfaces...

- A) Apply appropriate disinfecting solution (see EQUIPMENT ALERT) to soft cloth.
- B) Wring excess solution from cloth, then wipe external surfaces of arms / lighthouse.
- C) Do not rinse or dry. Allow solution to air dry.

To clean faceplate...

Use an antistatic acrylic cleaning solution (ex: Brilliance) or an LCD screen cleaner (ex: Klear Screen). The solution should be free of ammonia, carbon tetrachloride, and acetone. After softly blowing away free debris and dust, apply the cleaning product with a non-abrasive, lint-free cloth or microfiber cloth.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

Sterilizing the Handle

Caution



The handle is not sterile when shipped. If your application requires the handle to be sterile, it must be sterilized before use or covered with a sterile protective barrier.

Equipment Alert

Use only steam sterilization on the handle.

To sterilize, process the handle in a dynamic air removal steam sterilizer (sterilizer that uses pressure pulses and / or vacuum purges to remove air from the chamber) at a temperature of 270° F (132° C) for four minutes minimum. If using a Midmark M9 or M11 sterilizer, use the “POUCHES” cycle. It is recommended that the handle be pouched or wrapped for sterilization to maintain its sterility until placed on the light. When pouching or wrapping the handle, use only sterilizer pouches or wraps that have been cleared by the FDA and labeled for use with the steam sterilization cycle parameters (e.g. temperature and exposure time) of the sterilization cycle being used. Place the handle on the sterilizer tray so steam can enter the opening and air and any steam condensate can drain from the handle (see illustration below).

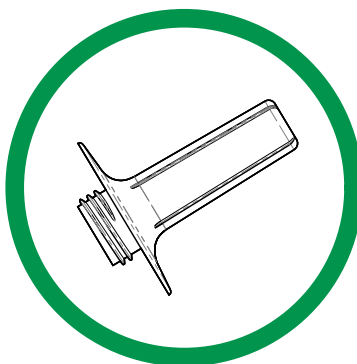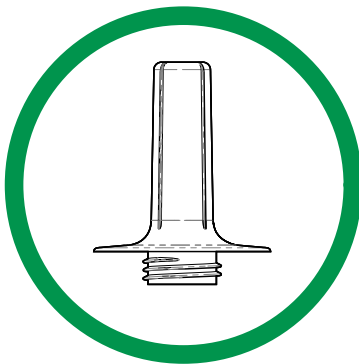MA8156i

Replacing the Fuses

Ceiling Mounted Lights

Warning

Always disconnect power before replacing fuses.

To install fuses...

- A) Turn light OFF (●).
- B) Disconnect power from light.
- C) Remove four screws, then lower ceiling cover.
- D) Using a flat-bladed screwdriver, rotate fuse cap counter-clockwise 1/4 turn, while pushing upward.
- E) Pull fuse out of fuse cap. See Note.
- F) Insert new fuse into fuse cap.
- G) Using a flat-bladed screwdriver, rotate fuse cap clockwise 1/4 turn, while pushing into fuse socket.
- H) Install ceiling cover.
- I) Connect power and verify operation.


Caution

To completely isolate the light from electrical mains supply, the circuit breaker must be turned off.

Note

There are two fuses on the dual light and one fuse on the single light. After removing the fuse, inspect it to determine whether it is faulty.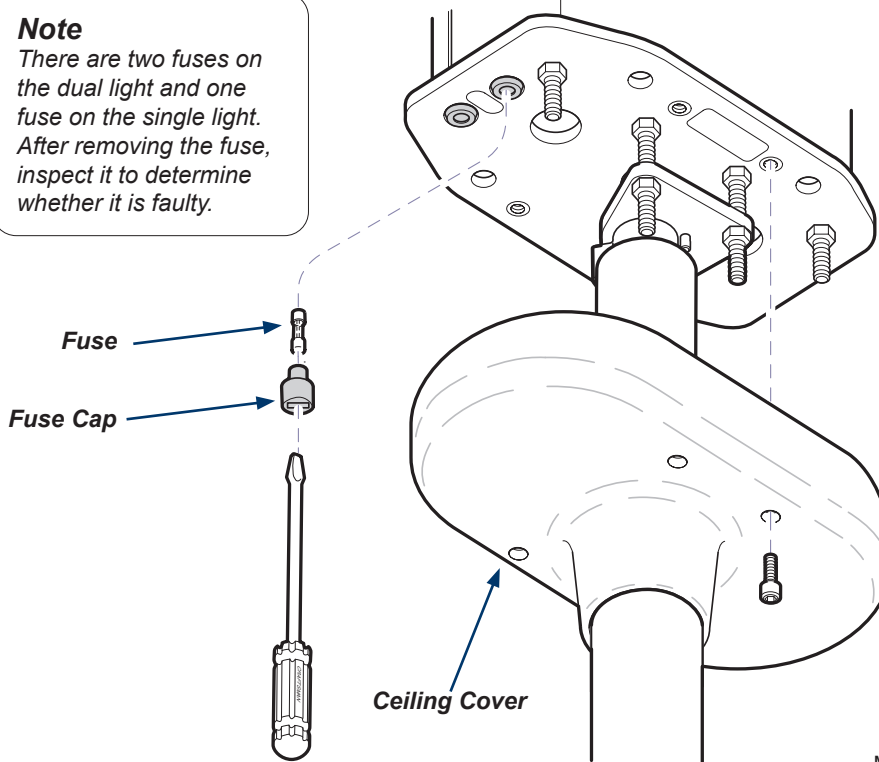

MA8474i

Replacing the Fuses - continued

Wall Mounted Lights

Warning

Always disconnect power before replacing fuses.

To install light...

- A) Turn light OFF (●).
- B) Disconnect power from light.
- C) Remove bracket cover.
- D) Using a flat-bladed screwdriver, rotate fuse cap counter-clockwise 1/4 turn, while pushing upward.
- E) Pull fuse out of fuse cap.
- F) Insert fuse into fuse cap.
- G) Using a flat-bladed screwdriver, rotate fuse cap clockwise 1/4 turn, while pushing into fuse socket.
- H) Install bracket cover.
- I) Connect power and verify operation.


Caution

To completely isolate the light from electrical mains supply, power cord must be unplugged.

Equipment Alert

Do not block the electrical outlet where light connects into.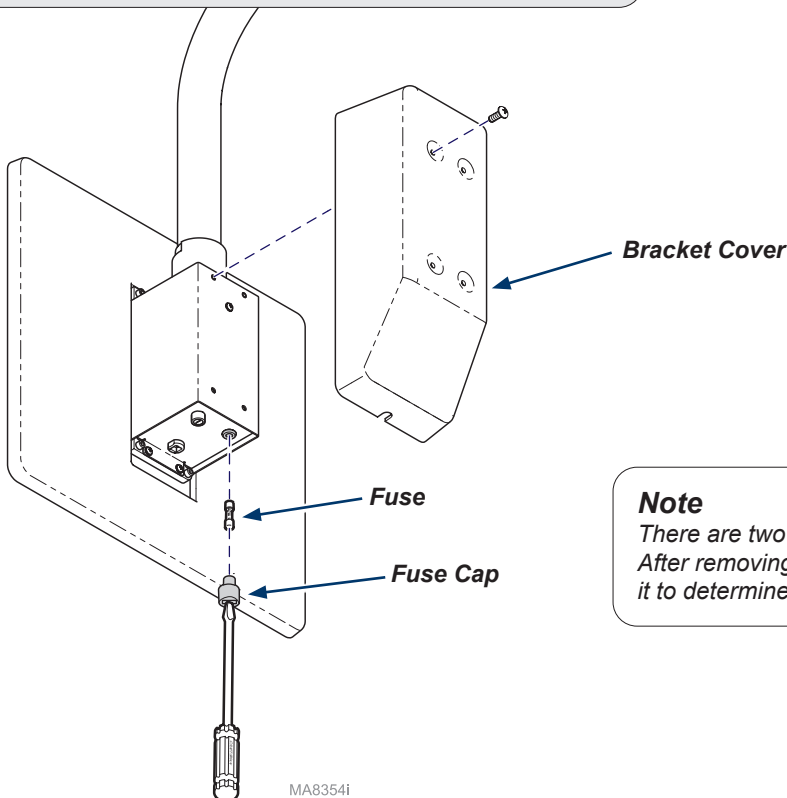

Note

There are two fuses on this light. After removing the fuse, inspect it to determine whether it is faulty.

MA8354i

Replacing the Fuses - continued

Mobile Lights

Caution

To completely isolate the light from electrical mains supply, power cord must be unplugged.


Equipment Alert

Do not block the electrical outlet where light connects into.

To replace fuses...

- A) Turn light OFF (⏻).
- B) Disconnect power from light.
- C) Using a flat-bladed screwdriver, push in on locking tab, then pull fuse drawer out.
- D) Pull fuses out of fuse drawer to inspect.
- E) Insert fuses into fuse drawer.
- F) Insert fuse drawer into AC inlet.
(Push in until you hear a clicking sound).
- G) Connect power and verify operation.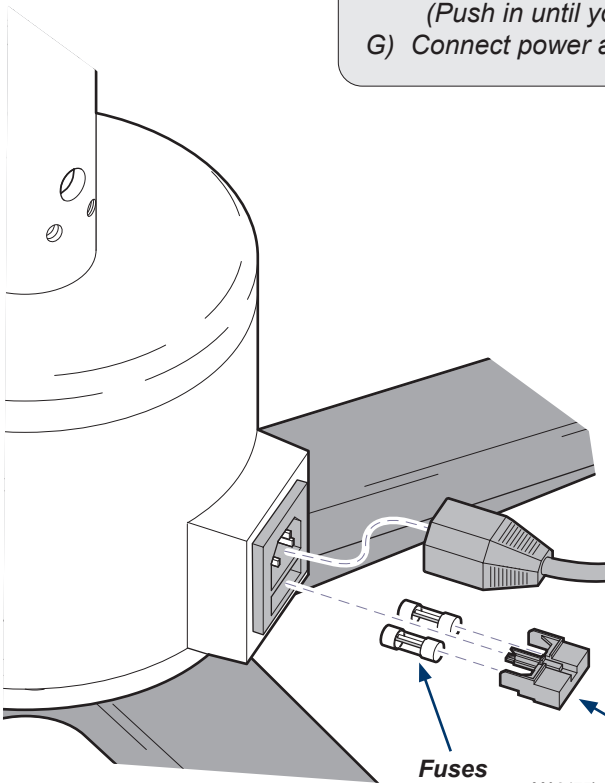


Note

There are two fuses on this light. After removing the fuse, inspect it to determine whether it is faulty.

Ball Pivot Tension Adjustment

To adjust ball pivot tension...


- A) Remove screw, then slide sleeve up to expose opening.
- B) Insert screwdriver into adjustment hole.
- C) Move the screwdriver LEFT / RIGHT to adjust tension setting
(LEFT: **decreases** tension / RIGHT: **increases** tension)
- D) Remove screwdriver, and check for desired tension.
Repeat Step C until desired tension is achieved.
- E) Slide sleeve down, then install screw.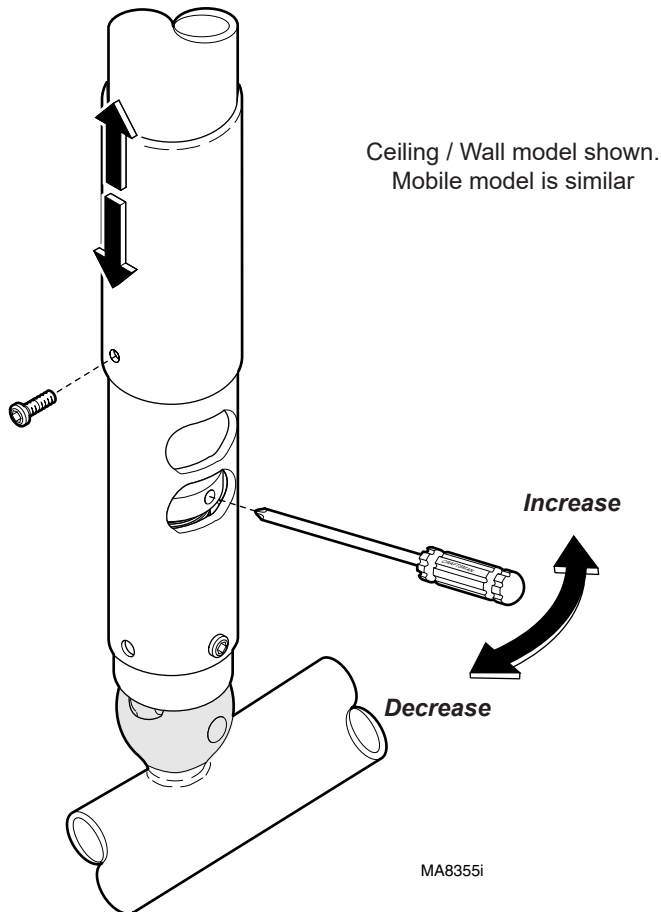

Cross Tube Counterbalance Adjustment

To adjust cross tube counterbalance...

- A) Remove cap from counterbalance.***
- B) Loosen / tighten the adjustment screw until the cross tube balances in horizontal position.***
- C) Install cap onto counterbalance.***

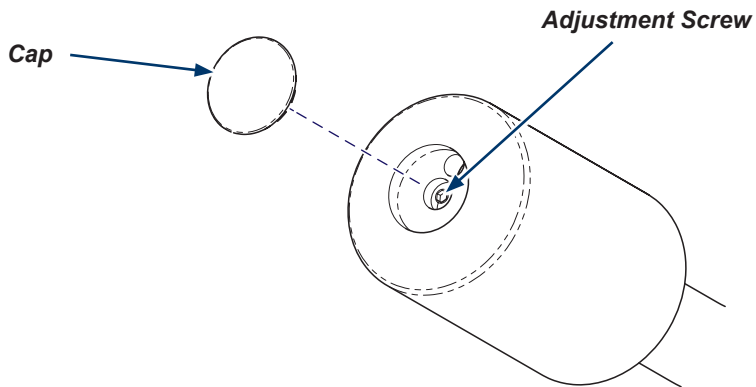MA9221i

Specifications

Specifications	
Illumination @ 1 m (39 in):	7500 fc (80,729 lux)
Dim Setting Central Illuminance	
Setting 1 (highest)	7500 fc (80,729 lux)
Setting 2	6200 fc (66,736 lux)
Setting 3	4400 fc (47,361 lux)
Setting 4 (lowest)	3400 fc (36,597 lux)
LED Light Life:	>30,000 hours @ $E_c \geq 70$
Color Temperature:	4,350K (± 300)
Color Rendering Index (CRI):	98
Total Irradiance:	300 w/m ²
Lighthead Field Diameter:	7.5 in. (19 cm)
D50 / D10 ratio:	0.5
Electrical Requirements:	
<i>115 VAC models</i>	115 VAC, 50/60 Hz, 0.7 A, single phase
<i>230 VAC models</i>	230 VAC, 50/60 Hz, 0.5 A, single phase
Fuse Rating:	
<i>115 VAC models</i>	1.6A, 250 VAC, 5mm x 20mm, Type T, Slo-Blo, Time Lag, High Breaking Capacity (HBC)
<i>230 VAC models</i>	1.6A, 250 VAC, 5mm x 20mm, Type T, Slo-Blo, Time Lag, High Breaking Capacity (HBC)
Range of Motion / Height:	Refer to <i>Dimension / Range of Motion</i> section
Usable Range:	.76 m to 1.5 m (30 in to 59 in)
Focal Distance:	1 m (39 in)
Power Cord (<i>Mobile Models only</i>):	120 in. (3 m) long, Hospital Grade grounding plug

Specifications - continued

Specifications - continued	
Classifications:	FDA Class 1, Type B, continuous operation (refer to Model Identification / Compliance Chart)
Certifications:	
All Models	Listed by TUV
Weight:	
Mobile Models	65.8 lbs (29.8 kg)
Wall Mounted Model	85 lbs (38.6 kg)
Ceiling Mounted Models:	
single light (8 ft. ceiling)	64 lbs (29.1 kg)
single light (9 ft. ceiling)	65 lbs (29.5 kg)
double light (8 ft. ceiling)	116 lbs (52.7 kg)
double light (9 ft. ceiling)	118 lbs (53.6 kg)

Warnings

WARNING

Do not modify this equipment without authorization of the manufacturer.

WARNING

Should damage or loose parts be identified, the equipment should be removed from use and serviced. Failure to do so may result in injury.

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air or nitrous oxide.

Caution

To completely isolate the light from electrical mains supply:

For lights provided with a power cord, the power cord must be unplugged.

For lights wired directly to mains supply, the wiring must be connected to an appropriately sized circuit breaker and the circuit breaker must be turned off.

WARNING

No modification of this equipment is allowed.

WARNING

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

WARNING

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 inches (30 cm) to any part of the Procedure Light, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.


WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov


Dimensions / Range of Motion

Ceiling Lights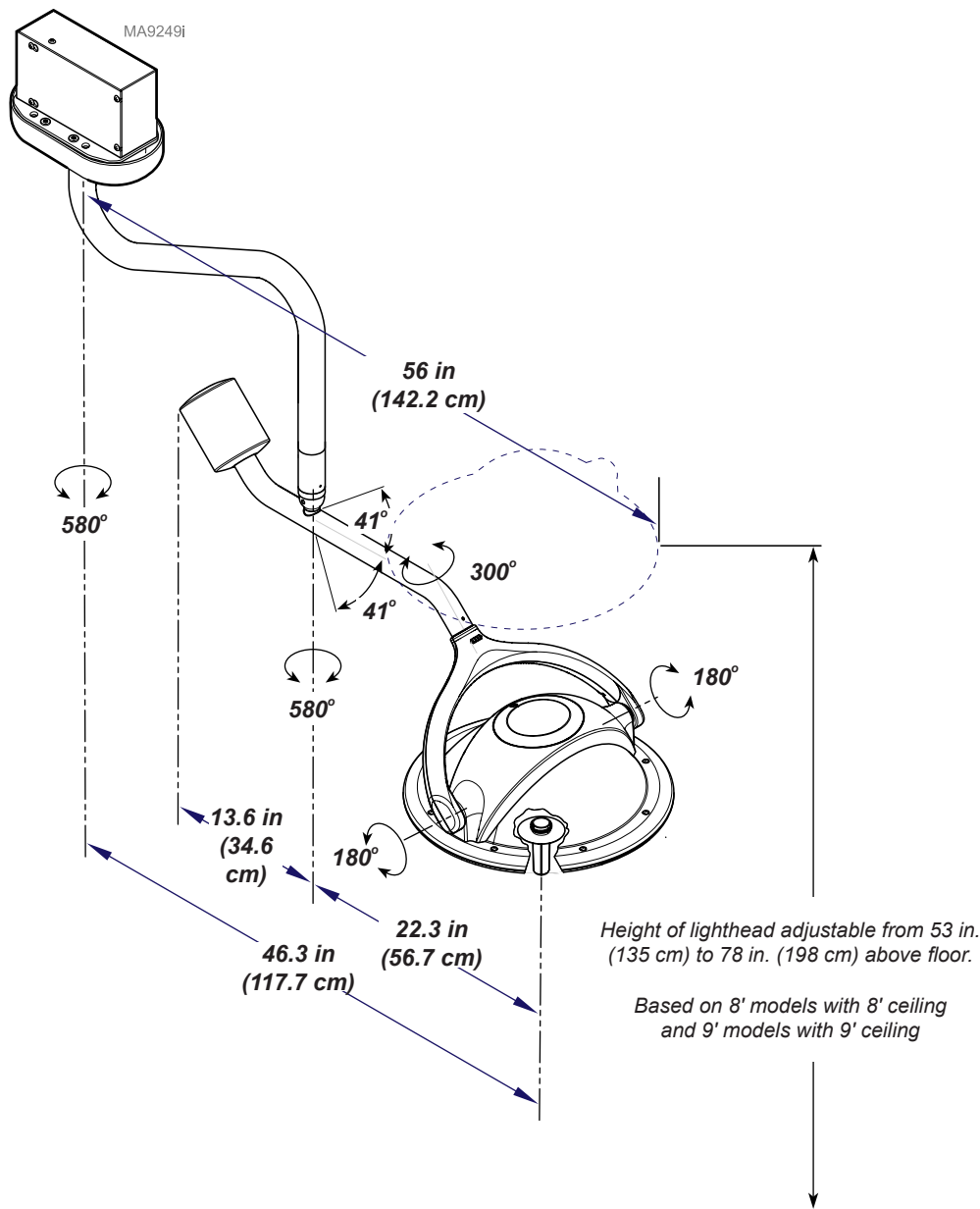


Dimensions / Range of Motion - continued

Wall Mounted Lights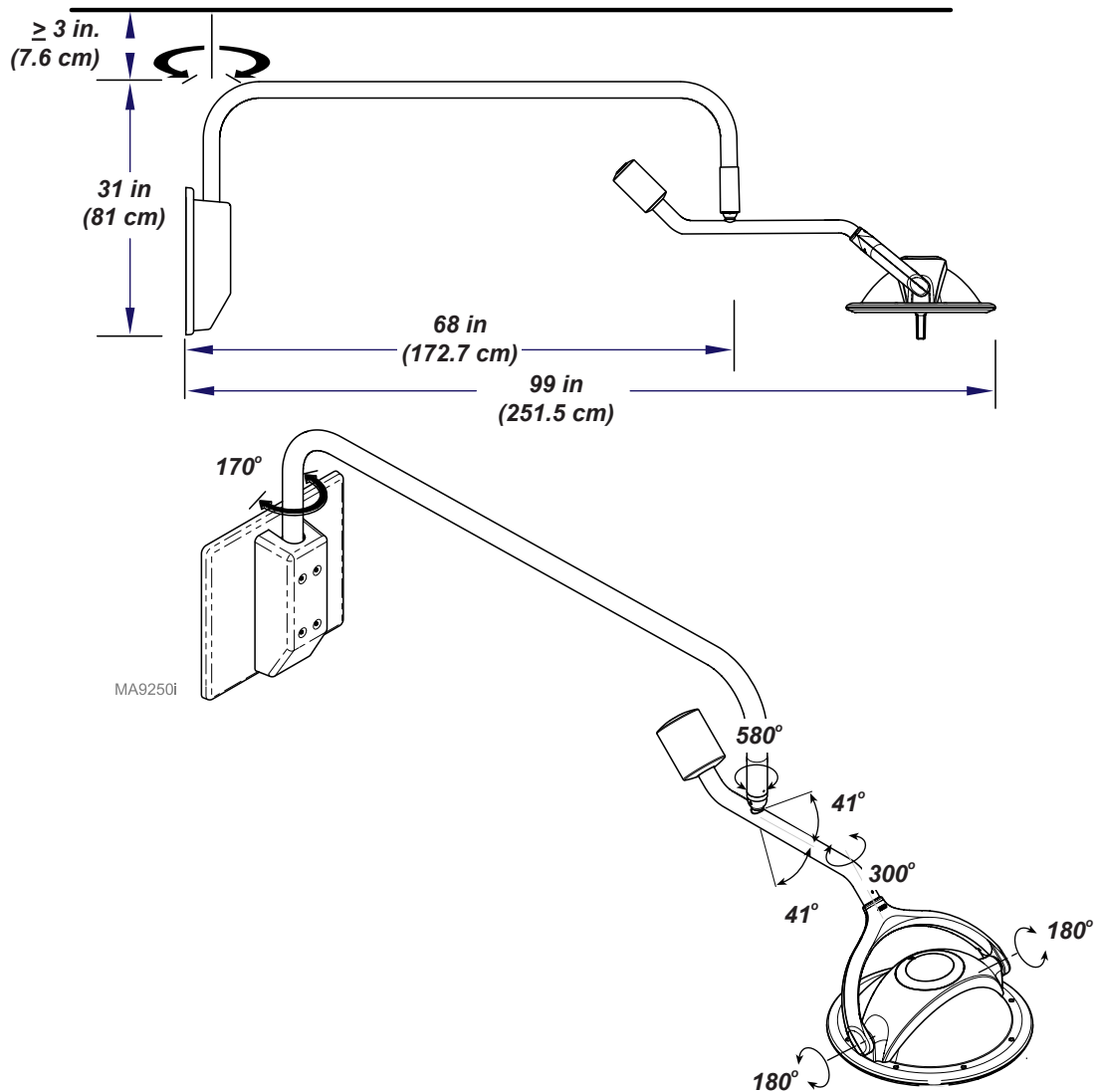

Dimensions / Range of Motion - continued

Mobile Lights

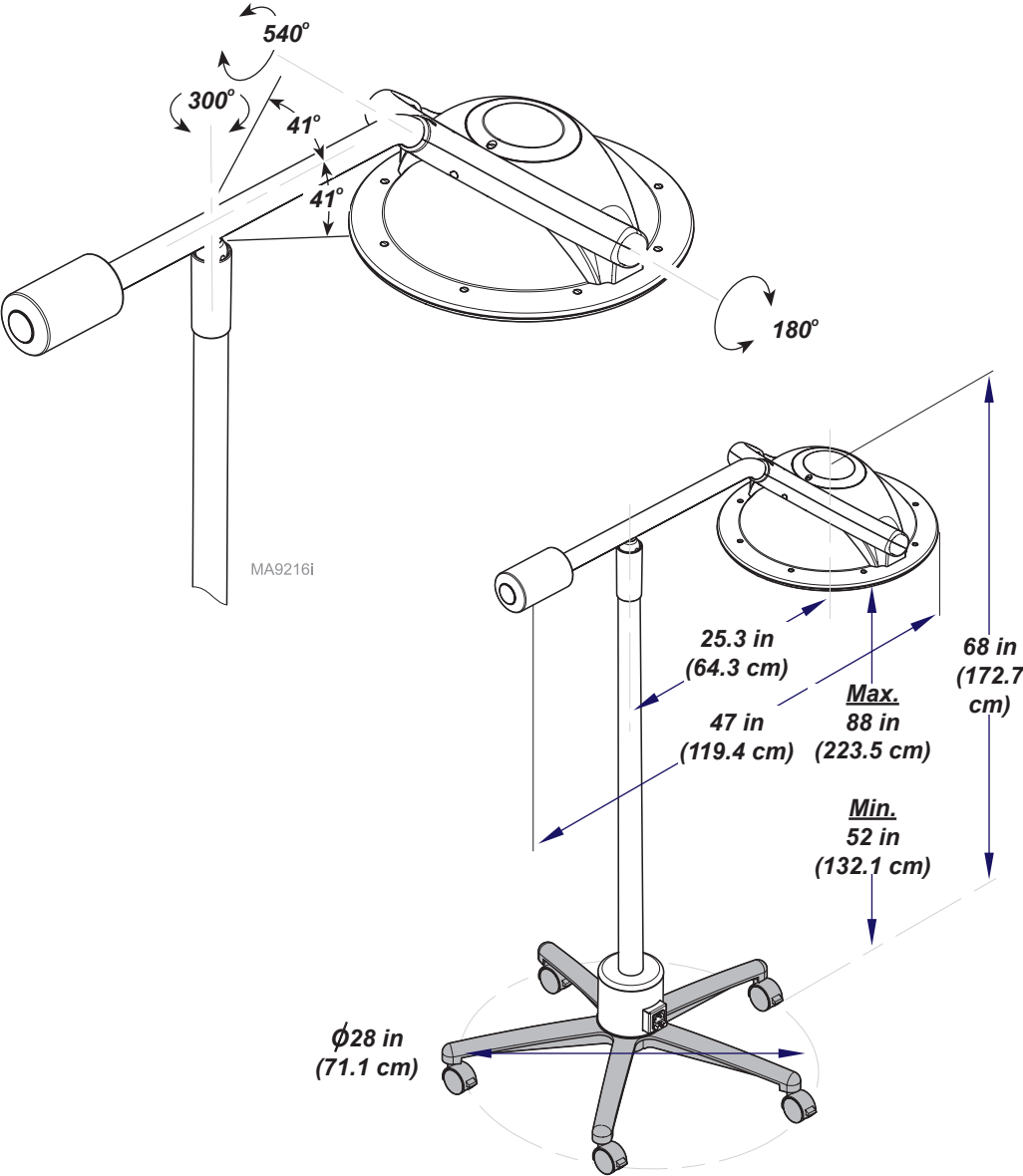Model Identification / Compliance Chart

Model	Description	Complies To:						Electrical Ratings:		
		ES 60601-1	CAN / CSA C22.2 #60601-1	60601-1-2 (EMC)	IEC 60601-1	60601-2-41 (Luminaires)	CE	VAC +/- 10%	Amps	Cycles (Hz)
255-021	8' single light, ceiling, Midmark	•	•	•	•	•	•	Configurable		50 / 60
								115	0.7	
								230	0.5	
255-022	9' single light, ceiling, Midmark	•	•	•	•	•	•	Configurable		50 / 60
								115	0.7	
								230	0.5	
255-023	8' double light, ceiling, Midmark	•	•	•	•	•	•	Configurable		50 / 60
								115	0.7	
								230	0.5	
255-024	9' double light, ceiling, Midmark	•	•	•	•	•	•	Configurable		50 / 60
								115	0.7	
								230	0.5	
255-025	Mobile light, Midmark	•	•	•	•	•	•	Configurable		50 / 60
								115	0.7	
								230	0.5	
255-026	Wall Mount light, Midmark	•	•	•	•	•	•	Configurable		50 / 60
								115	0.7	
								230	0.5	
255-027	Mobile Light w/o cord, Midmark	•	•	•	•	•	•	Configurable		50 / 60
								115	0.7	
								230	0.5	

EMC - Manufacturer's Declaration and Guidance

Note

The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals. The procedure light is not intended to be used in a residential environment.

Immunity Test	Immunity Test Level
Electrostatic Discharge (ESD) IEC 61000-4-2	±15 kV Contact, ±15 kV Air
Radiated RF EM Fields IEC 61000-4-3	3V/m 80MHz-2.7 GHz 80% AM at 1kHz
Rated Power Frequency Magnetic Fields IEC 61000-4-8	30 A/m
Electrical Fast Transients IEC 61000-4-4	±2 kV at 100kHz repetition frequency
Surges IEC 61000-4-5	Line to Line: ±1 kV Line to Ground: ±2 kV
Conducted RF IEC 61000-4-6	3V: 0.15MHz -80MHz 6V in ISM Bands between 0.15MHz - 80MHz 80% AM at 1kHz
Voltage Dips IEC 61000-4-11	0 % UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0°
Voltage Interruptions IEC 61000-4-11	0 % UT; 250/300 cycle

Emissions Test	Emissions Test Level
Conducted and Radiated RF Emissions	CISPR 11 Group 1 Class A

Midmark Warranty - Dental Products

1. SCOPE OF WARRANTY

Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will at Midmark's option repair or replace components of the dental products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. Midmark's obligation under this limited warranty is limited to the repair or replacement of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which, upon examination by Midmark, prove to be defective. This warranty extends only to the first retail purchaser of a product through an authorized Midmark distributor and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

2. APPLICABLE WARRANTY PERIOD

Effective March 1, 2018, these applicable warranty periods, measured from the date of invoice to the original user, shall be as follows:

(1) OPERATORY PRODUCTS

- (a) 5 years for all products (except for the items in (b) through (g)).
- (b) 2 years for upholstery (chairs and stools).
- (c) "KINK-VALVE" module carries a 10 year warranty.
- (d) The original light bulb on a new light carries a 1 year warranty.
- (e) SerenEscape® Heat and Massage System carries a 3 year warranty.
- (f) Accessories not manufactured by Midmark are excluded including but not limited to Bien-Air® handpiece systems, Exorvision Monitors, Dentsply® Cavitron® scaler, Satelec® scaler and curing light, and Sopro® cameras.
- (g) Light retrofit kits carry a 2 year warranty.

(2) ORAL SURGERY PRODUCTS are warranted for a period of 1 year.

(3) STERILIZER PRODUCTS are warranted for a period of 1 year.

(4) QUICKCLEAN ULTRASONIC CLEANERS are warranted for a period of 3 years.

(5) MECHANICAL ROOM PRODUCTS

- (a) PowerAir® oil-less compressors – 5 years or 3,500 hours of use, whichever occurs first.
- (b) PowerVac® and PowerVac®G dry vacuums – 5 years or 10,000 hours of use, whichever occurs first (except that the vacuum pump warranty term is 10 years or 20,000 hours of use, whichever occurs first).
- (c) Classic Series® wet-ring vacuums – 5 years or 10,000 hours of use, whichever occurs first.
- (d) PowerMax high performance vacuum – 2 years.
- (e) NXT Hg5 Series Amalgam Separator – 1 year.
- (f) Midmark manufactured accessories – 1 year.

(6) SYNTHESIS® AND ARTIZAN® EXPRESSIONS CABINETRY

- (a) 5 years for all products and components including door and drawer fronts, casters and slides, except for the items in (b), (c) and (d).
- (b) 3 years for electrical components such as task lights/LED lights, cords, controls and accessories.
- (c) 2 years for sliding track monitor mount and components and upholstery.
- (d) 1 year for countertops and resin, including accessories.
- (f) Midmark manufactured accessories – 1 year.
- (e) 1 year for Airglide® System.

(7) IMAGING PRODUCTS are warranted for a period of 2 years, except for the Extraoral Imaging Systems (EOIS) which are warranted for a period of 5 years, including the tubehead and sensor. The EOIS workstation computer is warranted for a period of 2 years.

(8) MIDMARK Replacement Parts and Accessories carry a 90 day warranty.

- (a) Classic Series twin pump replacement cartridges – 3 year warranty

(9) Software is warranted for 1 year.

3. OBTAINING WARRANTY SERVICE

Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at www.midmark.com, by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380, or by phone at: 1.800.MIDMARK. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within 30 days after purchase in order to obtain the benefits of this limited warranty.

4. EXCLUSIONS

This limited warranty does not cover and Midmark shall not be liable for the following;

- (1) defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environment conditions and electrical power requirements;
- (3) products considered to be of a consumable or sterile nature;
- (4) accessories or parts not manufactured by Midmark;
- (5) charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) costs and expenses of routine maintenance and cleaning;
- (7) representations and warranties made by any person or entity other than Midmark;
- (8) matching of color, grain or texture except to commercially acceptable standards;
- (9) changes in color caused by natural or artificial light;
- (10) custom manufactured products;
- (11) alterations or modifications to the product by any person or entity other than Midmark; and
- (12) products that would otherwise be covered under Sections 1 and 2 of this limited warranty, but are acquired:
 - (i) from a person or entity that is not Midmark or one of its authorized dealers; or
 - (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical, animal health or dental market, as the case may be, in which purchaser intends to use the product.

5. SOFTWARE

With respect to software that is a product or component thereof, Midmark does not warrant that the software: (1) is error free; (2) can be used without problems or interruptions; or (3) is free from vulnerability to intrusion or attack by viruses or other methods.

6. EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER

Midmark's only obligation under this limited warranty is the repair or replacement of defective parts. Midmark shall not be liable for and hereby disclaims any direct, special, indirect, incidental, exemplary or consequential damages or delays, including, but not limited to, damages for loss of profits or income, loss of use, downtime, cover and employee or independent contractor wages, payments and benefits.

7. WARRANTY DISCLAIMER

This limited warranty is Midmark's only warranty and is in lieu of all other warranties, express or implied. Midmark makes no implied warranties of any kind including any implied warranties of merchantability or fitness for a particular purpose. This warranty is limited to the repair or replacement of defective parts.

8. STATUTE OF LIMITATIONS

No action may be brought against Midmark for breach of this limited warranty, or implied warranty, if any, or for any other claim arising out of or relating to the products, more than 90 days following expiration of the limited warranty period.

9. NO AUTHORIZATION

No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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