# Integration Control Module® (ICM)

## Software Versions v1.3.x & v2.0.x

[English](#)

[Español](#)

[Français](#)

### Note


The ICM consists of control pad buttons and a screen area. The screens available depend on the software version, named in the convention: **v1.0.x**, **v1.1.x**, etc., where **x** can be any number. The **x** indicates a minor version update that does not require a change to this document.

### Contents:

Buttons .....	1
Screen Types.....	2
Home Screen.....	3
User Setup Menu.....	4
Configuration Menu .....	6
Using Controls Screens...7	
Editing Control Screens...8	
Ratio Selection .....	9
Flush Operations.....10	

## Buttons

Most buttons invoke a number of different actions, depending on the screen displayed. The **basic** functions of each button are shown here.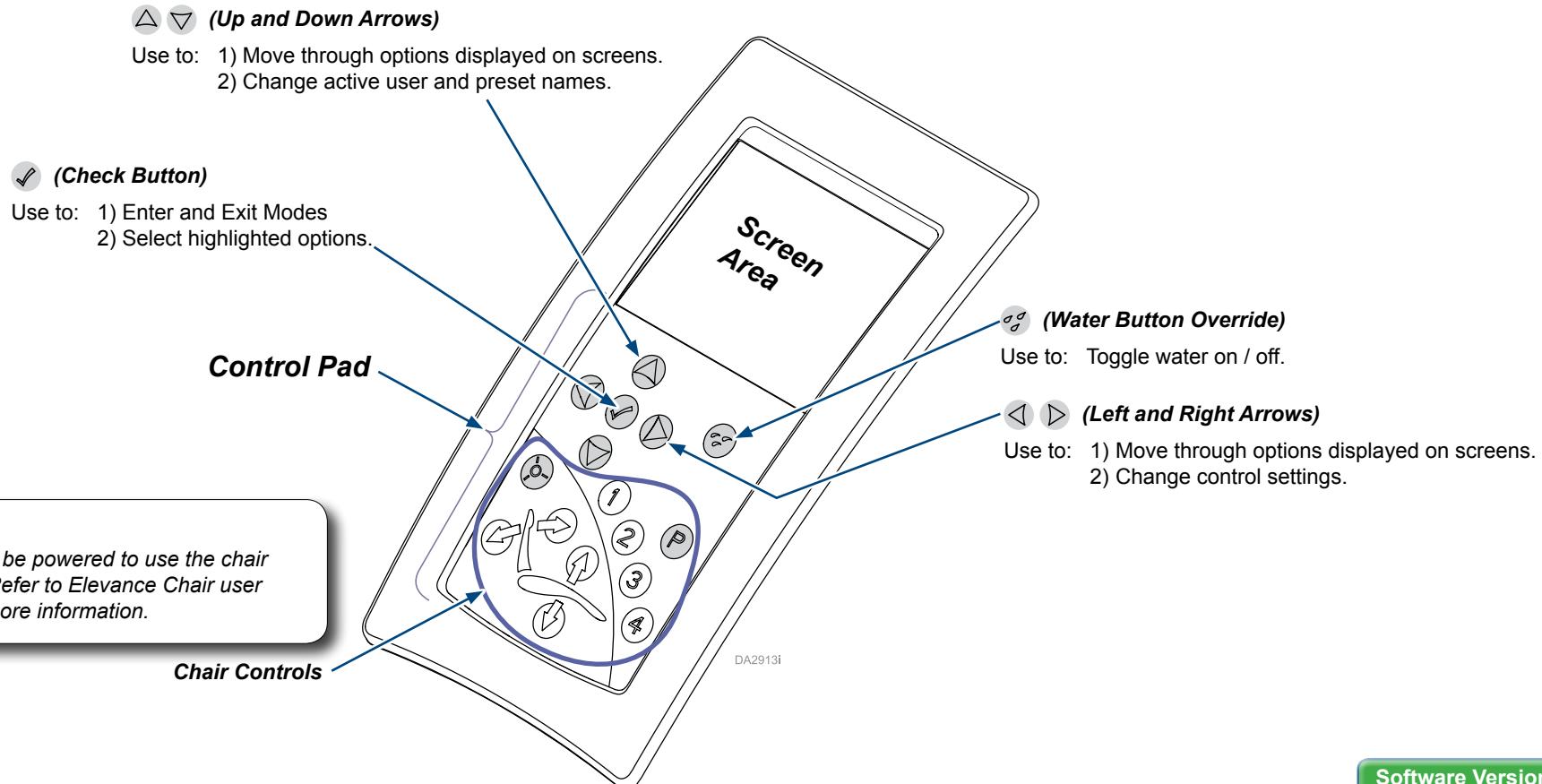

Software Versions  
v1.3.x & v2.0.x

## Screen Types

Variations of the Home Screen appear as the Elevance delivery system is powered on and completes an initialization process. As users interact with the control pad buttons, the footpedal or pull (remove from the holder) smart handpieces, a screen relevant to that user action appears.


Below are the screen types you will see appearing on the ICM.

**Home**  
Appears as the unit powers up.

### Flush Operations


Access by pulling two or more smart handpieces.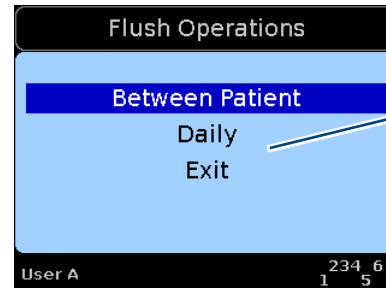


### Helpers

Always look for the helpers! They tell you which buttons are active and what pressing them will do.


Buttons can be used differently from screen to screen so **always look at the helpers on the active screen.**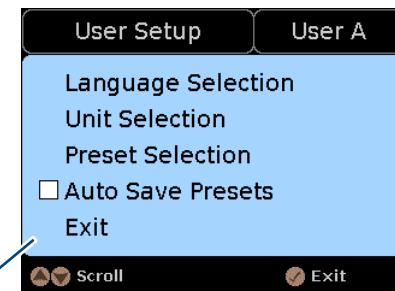


**User Setup**  
Access by pressing +

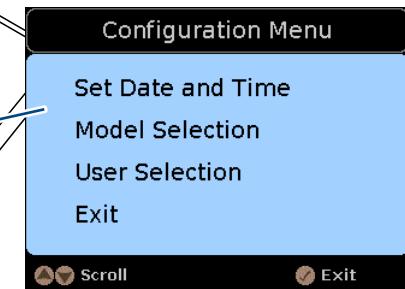**Configuration Menu**  
Access by pressing +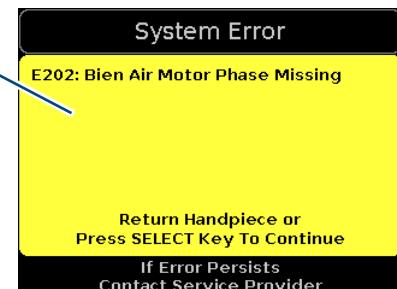

**Error**  
Appears only when an error occurs.

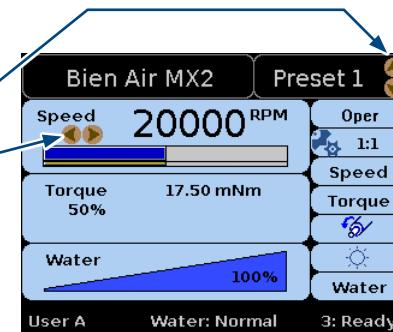
**Control**  
Access by pulling a smart handpiece.

**Note:**  
Screen appearances may vary. Exact screens cannot be shown.

## Home

The Home screen displays when the master switch is turned ON.  
The bottom half of the screen changes as the software initializes...

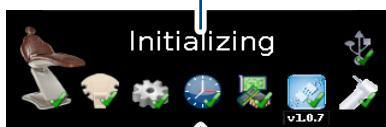Components with status icons appear in the bottom half of the screen, as the software communicates with each one.

Press   to display active User Name. Handpiece control settings and flush timers are saved uniquely to each user. Continue to press   to scroll through /make active another User Name.

Press  to display the control software version.

Press  to display the serial number of the delivery unit.

After a period of inactivity, the screen automatically will go into sleep mode, which appears as a totally black screen (not shown).

## Component Icons

	<b>Chair</b>		<b>Configuration</b>
	<b>Assistant's Unit</b>		<b>Clock</b>
	<b>Electric Motor</b>		<b>Interface Board</b>
		<b>v1.0.7</b>	Software Version Number for the Interface Board
	<b>Extended I/O</b>		<b>USB</b>

**Note:**

Software v1.3.x allows users to enter customized User Names of up to 20 characters in length. Access this feature via the Configuration Menu/User Selection screen.

## Status Icons

	Component ready.		Component NOT communicating.
	Component communicating but it has a problem.		Optional component is not connected.

## User Setup Menu


Press + to access the User Setup Menu. (Each user can have a different Setup.)

Press to highlight a listed setup.

Press to Go to the options for the highlighted setup.

Highlight and set the active option(s) using the buttons indicated by the helpers on each Selection screen that appears.

OR highlight Exit and press to return Home.

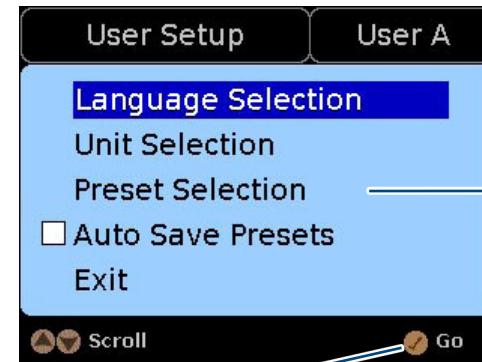
[see next page](#)

Press to highlight the language you want to make active.

Press to Select and return to the User Setup Menu.

OR press to go to the Unit Selection screen.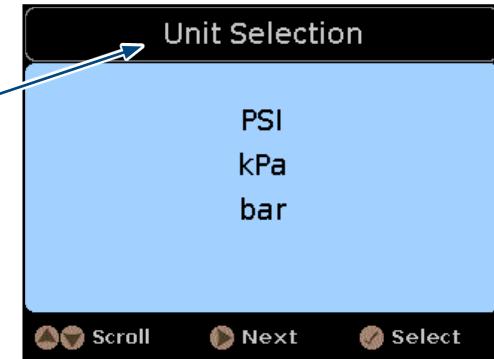

## Auto Save Presets


ON ( = ON (default) ) / OFF ( = OFF )

When Auto Save Presets is ON, every control setting you modify is automatically saved to the active User Name /Preset.

When Auto Save Presets is OFF, you can change settings repeatedly without saving them to that User/Preset Name, or you can respond to the prompt to save any new setting independently.

## User Setup Menu...continued - Preset Selection

Software Versions  
v1.3.x & v2.0.x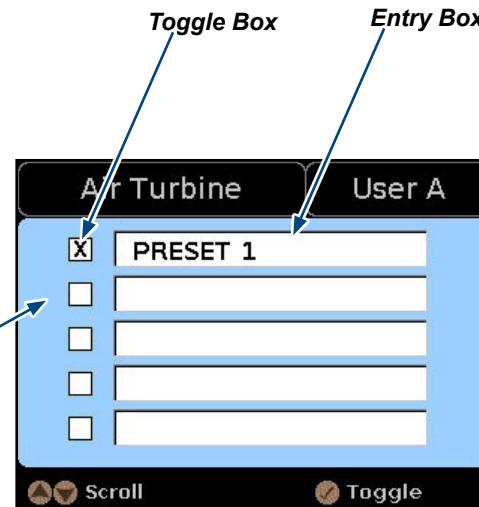

Available Characters*				
Presets can have 10 Characters				
User Names have 20 Characters				
	N	q	Ç	ì
'	O	r	È	í
,	P	s	É	î
-	Q	t	Ê	ï
.	R	u	Ë	ñ
0	S	v	ì	ò
1	T	w	í	ó
2	U	x	î	ô
3	V	y	ï	ö
4	W	z	Ñ	ù
5	X	€	Ò	ú
6	Y	"	Ó	û
7	Z	Œ	Ô	ü
8	a	"	Ö	ÿ
9	b	"	Ù	
:	c	œ	Ú	
A	d	Ý	Û	
B	e	í	Ü	
C	f	£	ß	
D	g	«	à	
E	h	¬	á	
F	i	°	â	
G	j	»	ä	
H	k	¿	æ	
I	l	À	ç	
J	m	Á	è	
K	n	Â	é	
L	o	Ä	ê	
M	p	Æ	ë	

With Preset Selection highlighted press to Go to the Preset Selection screen.

Press to highlight a listed handpiece.

Press to go to the Preset Selection screen for that handpiece.

Press and to position cursor in the desired Toggle box and press to Toggle an X in it.

Press to move cursor into the Entry Box, then press to scroll through the Available Characters\* until the desired one appears.

Press to move to the next character space and press to scroll through the Available Characters\* until the desired one appears.

Repeat until the desired Preset Name appears in the Entry Box.

Press to go back to the Toggle Box, THEN press repeatedly to scroll through all the Preset Names to the Save / Cancel screen (not shown).



Highlight Save or Cancel as desired and press to return to the Preset Selection screen.

Tips: Having an X in the Toggle Box makes the Preset Name an available choice on the Control Screen. Preset names cannot be deleted but they can be overwritten or made unavailable by toggling the X out of the toggle box.

From the Preset Selection screen you could press to return to the User Setup screen.

# Configuration Menu - Entering User's Names

Software Versions  
v1.3.x & v2.0.x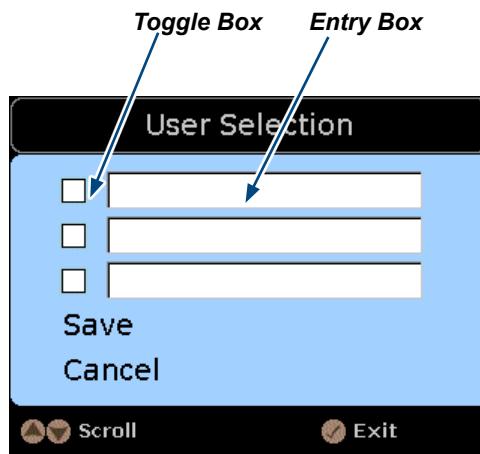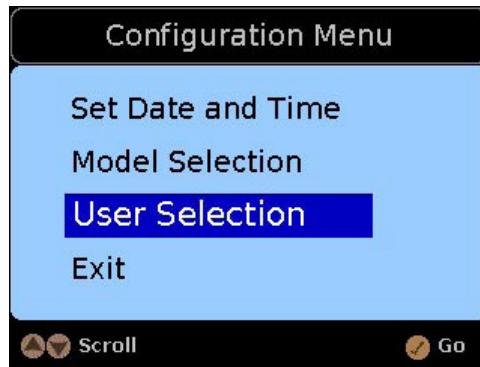

Press + to access the Configuration Menu.

Press to highlight the User Selection option, as shown here. Press to Go to the User Selection screen.

Press and to position cursor in the desired Toggle Box and press to Toggle an X in it.

Press to move cursor into the Entry Box, then press to scroll through the Available Characters (shown on previous page) until the desired one appears.

Press to move to the next character space and press to scroll through the Available Characters until the desired one appears.

Repeat until the desired User Name appears in the Entry Box.

Press to go back to the Toggle Box, THEN press repeatedly to highlight the Save or Cancel option.

Press to Cancel or Save as desired and return to the Configuration Menu.

**Tips:** Having an X in the Toggle Box makes the User Name an available Selection from the Home Screen.  
User Names cannot be deleted but they can be overwritten or made unavailable by toggling the X out of the Toggle Box.

## Note:

The Configuration Selection screens shown below exist for service technicians and should not require modifications.


Reference the *Troubleshooting Guide: Elevance® Delivery Systems* at [midmark.com](http://midmark.com) for more information.

Press to highlight a Model option.

Press to go to the Handpiece Assignment Screen.

Or press to return to Configuration Menu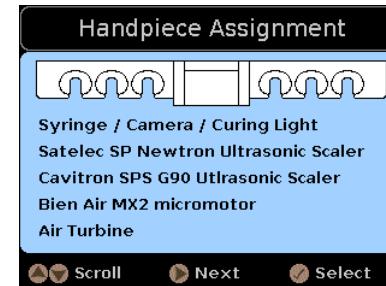

You must modify one of the date/time values to start/re-start the real time clock.


Always press to save the changes you have made and exit to the Configuration Menu.

## Interacting with Control Screens

Software Versions  
v1.3.x & v2.0.x

A control screen appears when a **smart** handpiece is pulled from the holder. See table. Each handpiece has unique control settings and screens.

“Smart” Handpieces (ones with control screens)	Handpieces without control screens
Electric Motor (Bien Air)	Cameras
Air Turbine	Curing Light
Scaler	Syringe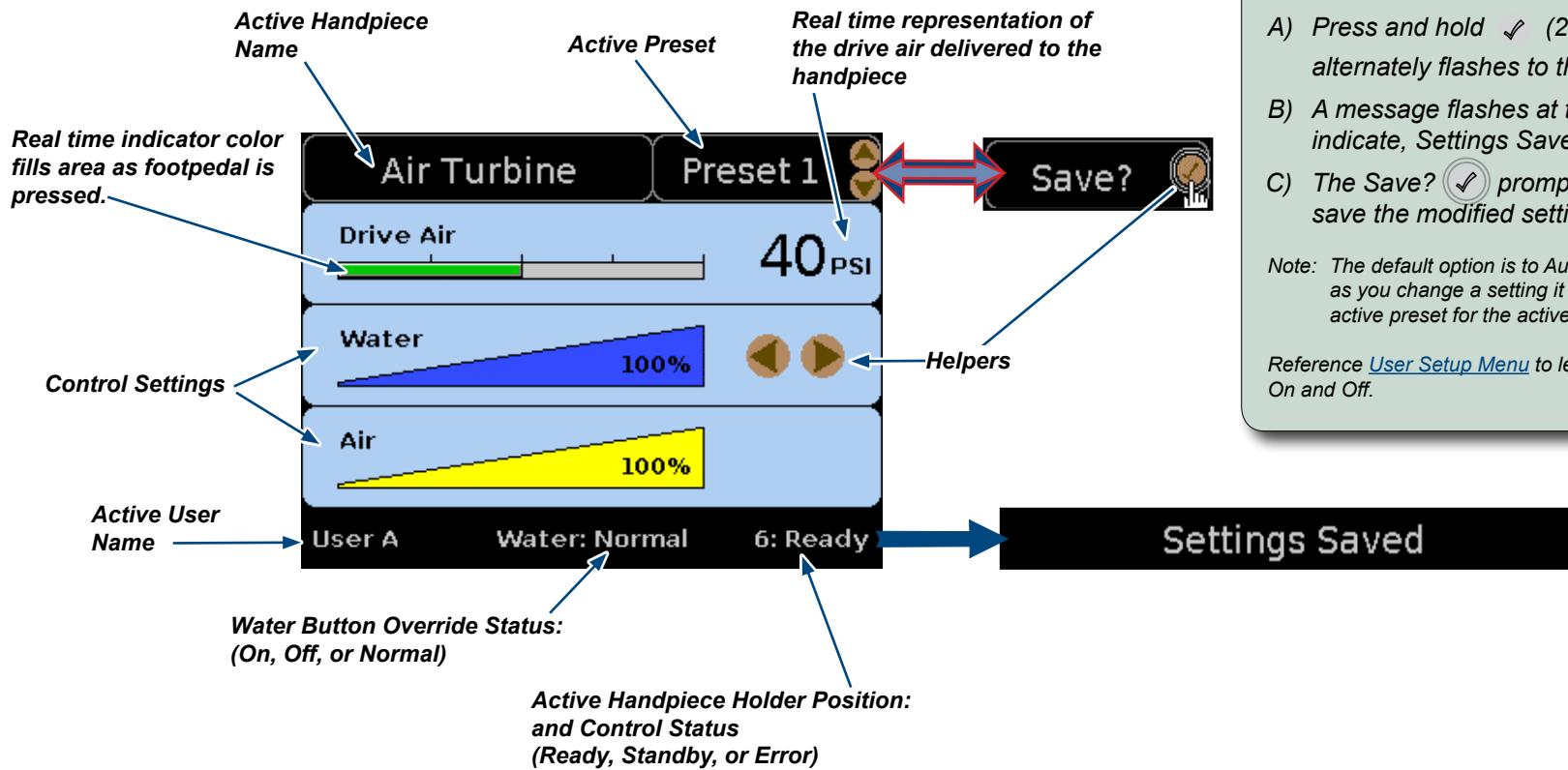

### How to save modified control settings when Auto Save Presets are turned off...

- A) Press and hold (2 sec.) as the Preset Name alternately flashes to the Save? prompt.
- B) A message flashes at the bottom of the screen to indicate, Settings Saved, then returns to normal.
- C) The Save? prompt continues to flash until you save the modified settings.


Note: The default option is to Auto Save Presets, meaning as soon as you change a setting it is automatically saved as the active preset for the active user.

Reference [User Setup Menu](#) to learn how to turn Auto Save Presets On and Off.

A control screen initially appears in Normal Mode (shown) where can be used to change the Preset and can be used to change the active control setting (water in screen shown).

Helpers appear beside the active control settings and indicate which control pad buttons are active for the setting.

Pressing puts the control screen into Edit Mode (not shown). When in Edit mode use to move from control setting to control setting.

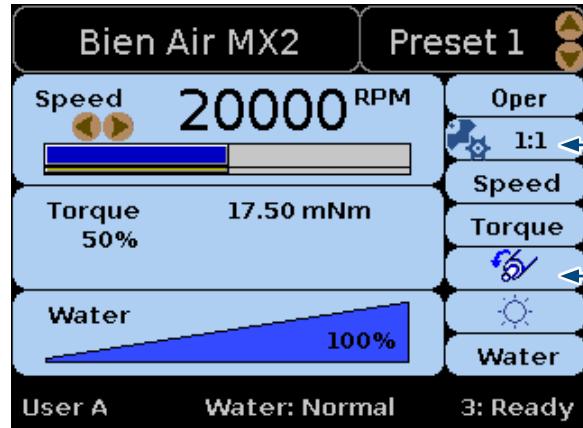**Normal Mode Screen**

**Note:**

The motor direction icon displays a blue arrow to run in the forward direction and a red arrow to run the motor in reverse.

The normal mode control screen displays when a handpiece is pulled.

Use to change Presets. The Preset Name displayed is active.

Helpers indicate the active controls (Preset and Speed control setting in example shown).

Use to change the speed control setting.

**Equipment Alert**


The color and the ratio on the control screen contra angle setting should always match the ratio and color etched on the contra angle being used.

Increasing

Direct

Reducing

Press to change mode.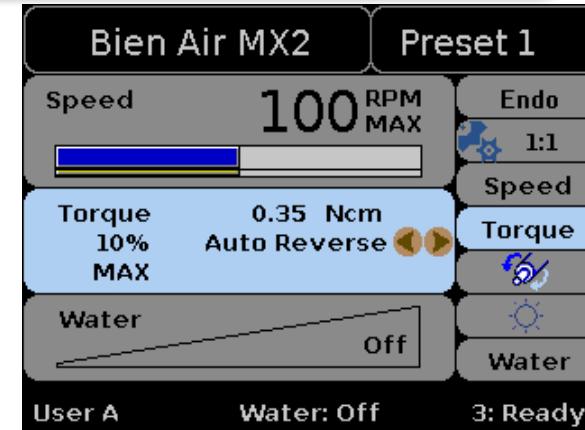

**Edit Mode Screen**

In Edit mode, the helpers appear beside the new active setting and it becomes highlighted (Auto Reverse setting in example shown).

In Edit mode, are used to move through the available control settings, listed in the right column.

Use to change the active control setting.

See the previous pages for information on the [Auto Save Presets](#).

The Bien Air MX2 handpiece includes an endodontic mode of handpiece operation.

Use to toggle between Endodontic/Operative operating modes when it is the active control setting on the Edit Screen. (Endo mode is active in the Edit Mode Screen shown.).


**Equipment Alert**

For Endodontic Operations using Bien Air MX2 handpiece:  
Ensure the bypass toggles are in the NORMAL position.

## Ratio Selections Screen

The Ratio Selections screen displays when in the Edit mode  1:1 is active and  +  are pressed.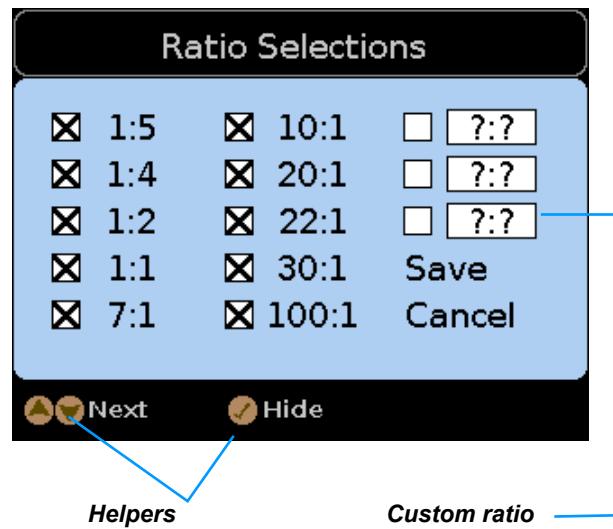

### **View and choose which ratios to display in edit mode**

Press  to scroll through ratios on the screen.

Press  to Hide/Show (remove or add a ratio to the display list).

Ratios with an X are the ones that will be available from the Contra Angle Ratio options you can scroll through in edit mode.

### **How to Add a Custom Ratio**

Scroll  to one of the custom ratio boxes.

Press  to change values in the field.

Press  to Show a custom ratio (X appears and field becomes active).

Press  to choose the Save or Cancel option.

Press  to invoke the chosen option and return to Edit mode screen.


When you return to the Edit Mode screen, the active ratio will still be the one displayed when you left the edit screen. ( in the example shown). Use  to change to the custom ratio.

## Flush Operation - Syringe

Software Versions  
v1.3.x & v2.0.x

### Note:

Pulling multiple smart handpieces is the ONLY way to access the Flush Operations Menu.

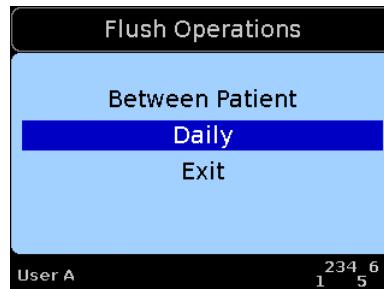
### Step 1 - Flush Operations Menu

- Pull two or more smart handpieces at the same time and the Flush Operations Menu appears.
- Use △ ▽ to move highlight to desired option.
- Press ✓ to select the highlighted flush operation.

### Step 2 - Flush Selection Screen appears....

- Use △ ▽ to move highlight to the SYRINGE FLUSH option.
- Use ◁ ▷ to change the flush time, if desired.
- Place the syringe(s) over a drain or container.
- Press ✓ to select to start the Syringe Flush operation.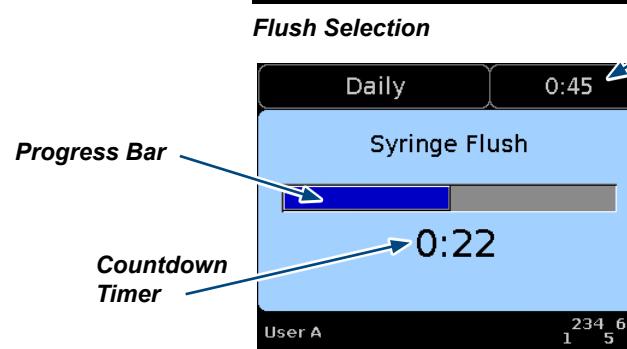


### Flush Time

NOTE: The Syringe Flush Process screen serves as a timer for this manual syringe flush process.

### Step 3 - Flush Process Screen appears....

- Press and hold the water button on the syringe(s) for the entire flush time as it displays on the screen.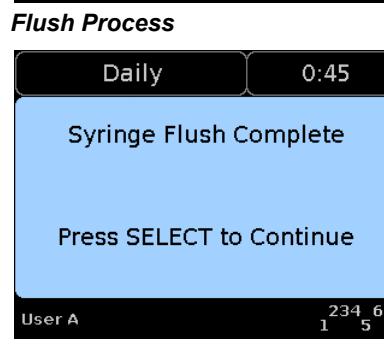


### Step 4 - Flush Complete Screen appears...

- Syringe Flush Complete appears when the flush process is finished.
- Return syringe(s) to the holder.
- Press ✓ (SELECT) to return to the Flush Selection screen.

Flush Operations Continue....

## Flush Operation - Handpiece


Software Versions  
v1.3.x & v2.0.x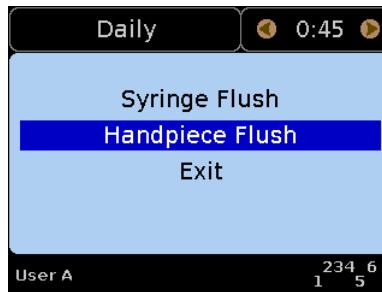

### Step 5 - Flush Selection Screen appears....

- Use  $\triangle \nabla$  to move highlight to HANDPIECE FLUSH option.
- Use  $\triangleleft \triangleright$  to change the flush time, if desired.
- Pull the desired handpieces, place them over a drain or container and press  $\checkmark$  to start the flush process.

Flush Selection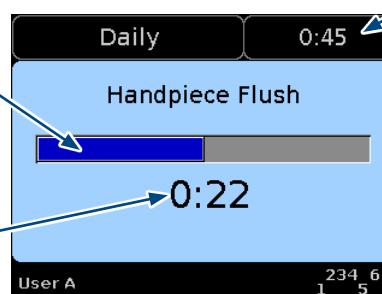

### Step 6 - Flush Process Screen appears...

The Flush Process screen appears, displaying an animated progress bar and countdown timer as the automated handpiece flush process occurs.


Flush Process

### Step 7 - Flush Complete Screen appears...

- Flush Complete message appears when the process is finished.
- Return all handpieces to correct holders.
- Press  $\checkmark$  (SELECT) to return to the Flush Selection screen.

Flush Complete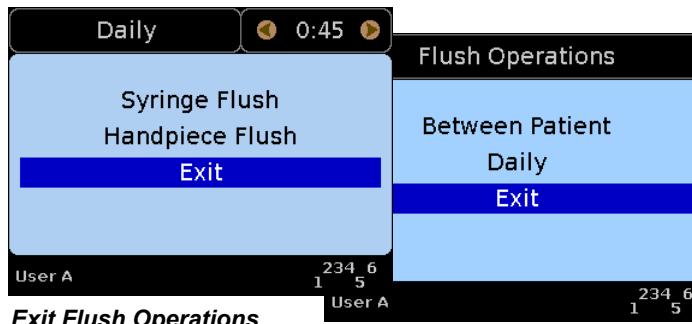

Exit Flush Operations

### Step 8 - Exit Flush Operations

- Use  $\triangle \nabla$  to move highlight to EXIT option.
- Press  $\checkmark$  (SELECT) to return to the Flush Operations Menu.
- Use  $\triangle \nabla$  to move highlight to EXIT option.
- Press  $\checkmark$  (SELECT) to return to the HOME screen.

**Midmark Corporation**  
60 Vista Drive  
Versailles, OH 45380 USA  
1-800-643-6275  
1-937-526-3662

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