



## Veterinary Dental Suction Tips

**Applies to Models:**

9A560001

Language of origin: English

### **Intended Use:**

*This product is intended to provide suction for dental procedures being conducted by qualified veterinary professionals in both the office and hospital environments. This product is for veterinary use only.*



### **Equipment Alert**



Tips are single-use and disposable only, and are **not** autoclaveable.



Tips are **not** sterile and should be properly cleaned and disinfected prior to use. Suggested means of disinfection would be by submersion into a chemical disinfectant to reach the inside channel of the tip.

002-1814-00  
qty 25



002-1815-00  
qty 25



VA1962i